
Invacare® Cascata

H720T

en	Mobile Toilet Chair User Manual	2
de	Mobiler Toilettenstuhl Gebrauchsanweisung.....	5
fr	Chaise roulante pour toilettes Manuel d'utilisation.....	8
it	Sedia WC con ruote Manuale d'uso	11
es	Silla de inodoro móvil Manual del usuario	14
pt	Cadeira sanitária móvel Manual de utilização.....	17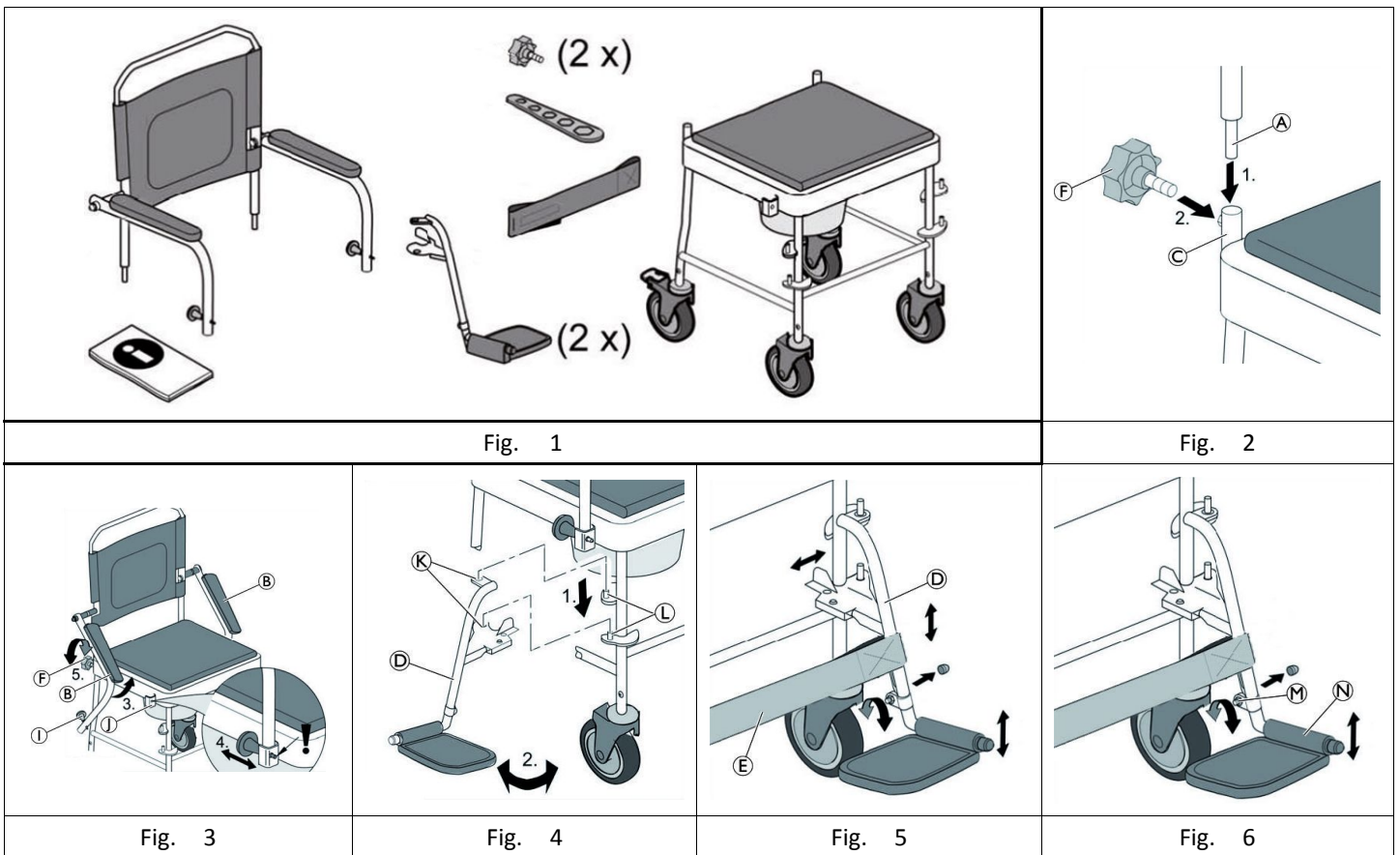

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

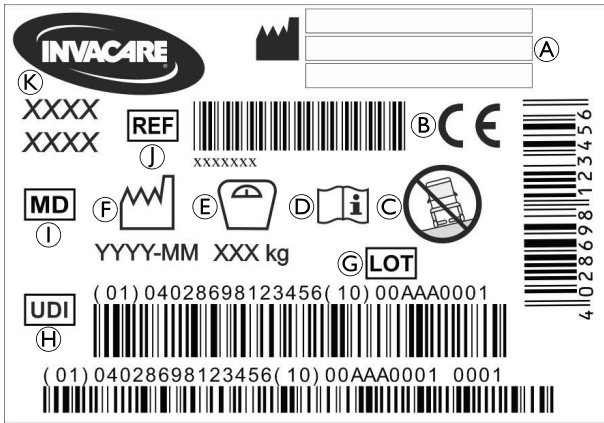
Fig. 7

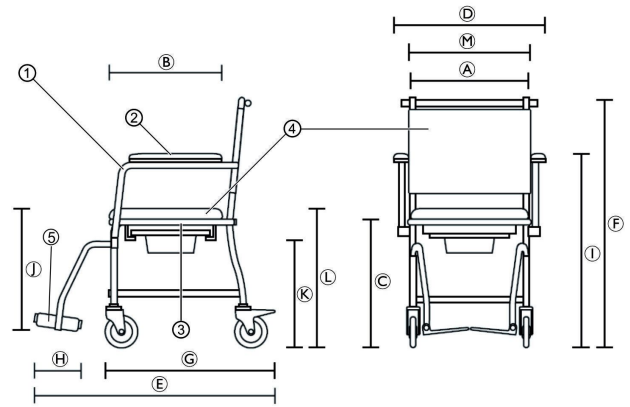
Fig. 8

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards. For further information about local standards and regulations, contact your local Invacare representative.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use in shower.
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.
- Do not stand on this product.
- Do not lift the product when under load.
- Only use the toilet chair on a level ground and indoors.

**CAUTION!****Risk of Burning Skin!**

The product components can heat up when exposed to external sources of heat.

- Do not expose the product to direct sunlight before use.

**WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

2.2 Labels and Symbols on the Product (Fig. 7)

Ⓐ	Manufacturer	Ⓒ	Lot number
Ⓑ	European Conformity	Ⓓ	Unique Device Identifier
Ⓒ	For use on level ground only	Ⓔ	Medical Device
Ⓓ	Read user manual	Ⓕ	Reference number
Ⓔ	Maximum user weight	Ⓖ	Product designation
Ⓕ	Production date		

The identification label is attached to the left rear tube of the product.

3 Product Overview**3.1 Intended Use**

The Invacare H720T is a mobile toilet chair composed of two flip down plastic padded armrests, a padded backrest, swing-away footrest, a seat plate with toilet cut-out and with two castors with brakes. A toilet pan with lid can be placed under the seat. This product is intended to be used as a toilet aid providing stable seating position while going to toilet.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 8 *Technical Data, page 4*.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup (Fig. 2 – 6)**CAUTION!****Risk of Finger Trapping**

- Take care during assembly.

1. Guide the backrest tube Ⓐ on both sides, from the top into the seat frame. (Fig 2) Ⓒ.
2. Tighten the star head screw Ⓔ(hand-tight). (Fig 3)
3. Swivel armrests Ⓑ on both sides upwards until the spring pin Ⓓ latches into the hole of the lock Ⓔ on the seat frame. (Fig 3)

**CAUTION!****Risk of Falling**

- Make sure that the spring pin latches on both sides with an audible click.

4. Place the footrests Ⓓ on both sides through the holes at the sides Ⓖ into the attachment pins Ⓕ. (Fig 4)
5. Swivel both footrests to the front.
6. Position the calf strap around the footrest tubes at the desired height and fasten it at the rear side with the hook and loop band.

Adjust the height of the footrests (Fig. 6)

1. Loosen nut Ⓖ and move the lower part of the footrests Ⓓ up and/or down. Tighten the nut again.

The device is dismantled by following the assembly procedure in reverse order.

5 Usage**Releasing the Armrests**

1. Pull the spring pin knob Ⓓ. The armrest is released from its fixation Ⓔ and swivels downward. (Fig. 3)

Transfer to and from the Chair**CAUTION!****Risk of Trapping/Squeezing**

All castors locks are foot-operated.

- Do not operate the castor locks by hand.

1. Lock the castor lock by stepping on the rear end of the pedal until it engages.
2. Flip up the footrests.
3. Alternative to 2.: Remove the calf strap Ⓔ then press the black lever to swivel the footrests to the side. (Fig. 5)

**WARNING!****Risk of Slipping and Rolling**

- Lock all castors with levers using the foot levers before sitting down on the toilet chair or standing up from it, as well as in the parking position.

**WARNING!****Risk of Tipping**

- Do not lean excessively forwards.
- Never stand on the footrests.
- Swivel the footrests to the side or fold up the foot plates before sitting down or standing up.

Inserting/Removing the Toilet Pan

The toilet pan can be inserted/removed with or without cover.

1. Insert/remove the toilet pan from the rear side between the seat and the bracket under the seat.

6 Maintenance

6.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

6.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning

NOTICE!

Maximum cleaning temperature:

- 60 °C by hand

1. Remove detachable parts.
2. Clean the product by wiping down all generally accessible surfaces.
3. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

7 After Use

7.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *6 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7.2 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

8 Technical Data

8.1 Dimensions, Weights and Materials (Fig. 8)

Dimensions			
Ⓐ	450 mm	Ⓛ	755 mm
Ⓑ	450 mm	Ⓜ	345 – 460 mm
Ⓒ	480 mm	Ⓨ	420 mm
Ⓓ	575 mm	Ⓩ	525 mm
Ⓔ	810 mm	ⓐ	470 mm
Ⓕ	945 mm	Width, cut out in seat plate	220 mm
Ⓖ	545 mm	Depth, cut out in seat plate	260 mm
Ⓗ	155 mm	∅ Castors	127 mm (5")

Weights	
Toilet Chair	12.8 kg
Maximum user weight	135 kg

Materials			
Ⓛ	Steel, powder coated		
ⓂⓎⓐ	PP	ⓓ	PVC

All product components are corrosion resistant and do not contain natural rubber latex.

8.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	

Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Poirier SAS
Invacare AG
Invacare GmbH