


Customer :  
 Customer No. :  
 Phone :

Contact :  
 Commission :  
 Serial No. :

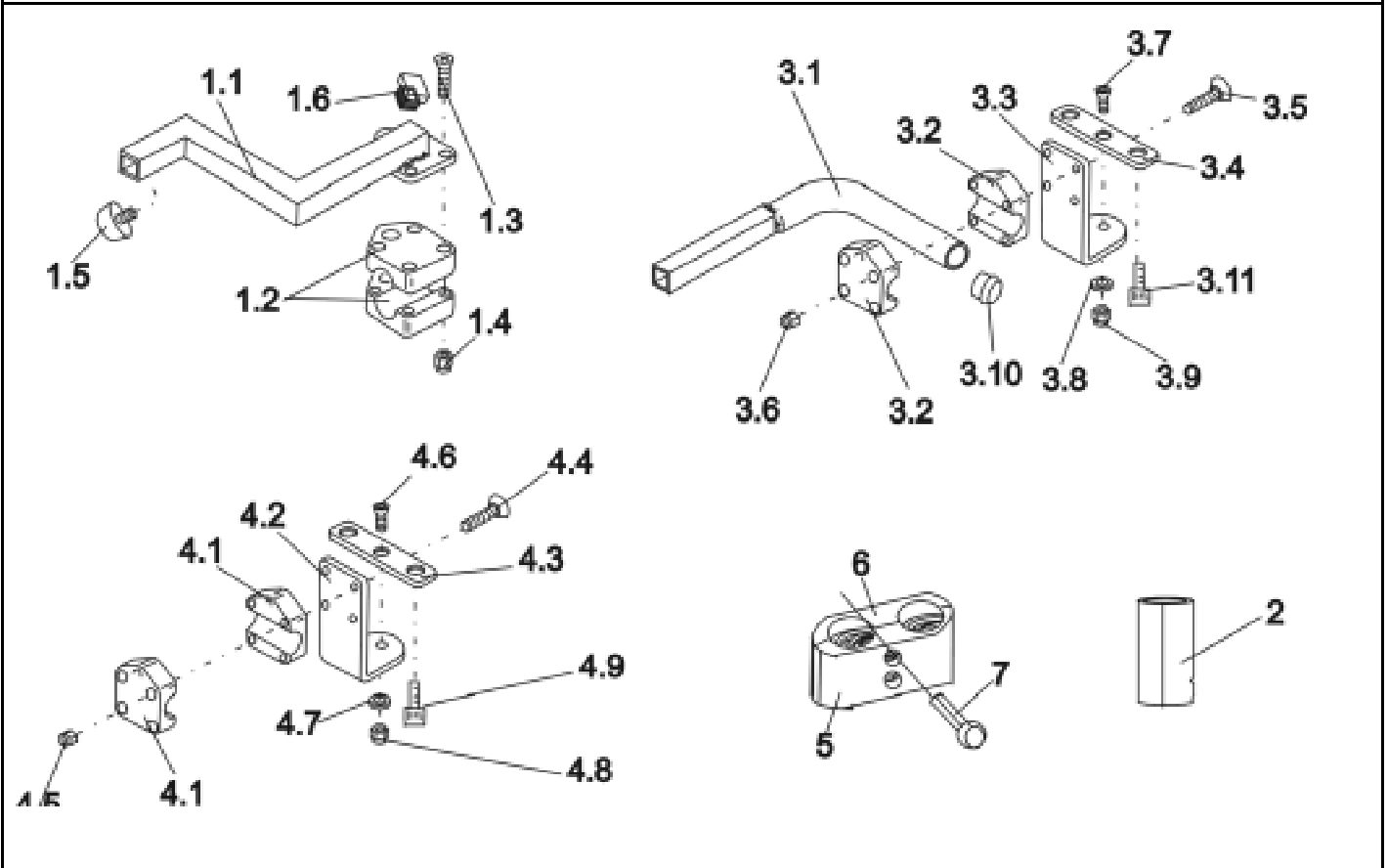


Item	Qty	Part-No.	Part-Designation
-	1	SP1422937	<b>ASSEMBLY KIT F. STD. CARER SEAT CONTROL LH .</b>
-	1	SP1422936	<b>ASSEMBLY KIT RH F. STD. CARER SEAT CONTROL</b>
1.1	1		ATTENDANT CONTROL SUPPORT TUBE LH; RAL 9005 BLACK
1.1	1		ATTENDANT CONTROL SUPPORT TUBE RH; RAL 9005 BLACK
1.2	2	Z12008A.P40	GRIPPING CLIP W. HEXAGON HOLE
1.3	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 35
1.4	4		NYLOC NUT M6
1.5	1	L31001	HAND KNOB M6 X 10
1.6	1		TUBE END PLUG E 15 X 15
-	1		ATTENDANT JOYSTICK BOX SUPPORT TUBE LH BLACK ->02/04
-	1		ATTENDANT JOYSTICK BOX SUPPORT TUBE RH BLACK ->02/04
-	1	1440438-9005	ATTENDANT JOYSTICK BOX SUPPORT REM 24SD 03/04->
2	4	Z183264.99	BUSH RD. 22 X 1,5 X 40
5	2	0572Y02646	TUBE CLAMP DIA 22 MM; BLACK
6	2	0572Y02647	TUBE CLAMP DIA 22 MM; BLACK
7	4		SOCKET HEAD CAP SCREW M6 X 25
-	1	SP1422819	<b>ASSEMBLY KIT F. SWITCHABLE CARER RECARO CONTROL</b>
3.1	1	F383065-9005	HOLDER FOR CARER RECARO CONTROL; RAL 9005, DEEP BLACK
3.2	2	Z12008A.P40	GRIPPING CLIP W. HEXAGON HOLE
3.3	1		ATTENDANT CONTROL, SECURING BRACKET; RAL 9005 BLACK
3.4	1		BRACKET; RAL 9005 BLACK
3.5	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 35
3.6	4		NYLOC NUT M6
3.7	1		COUNTERSUNK SOCKET HEAD SCREW M6 X 16

### Storm 3



Customer :  
 Customer No. :  
 Phone :

Contact :  
 Commission :  
 Serial No. :

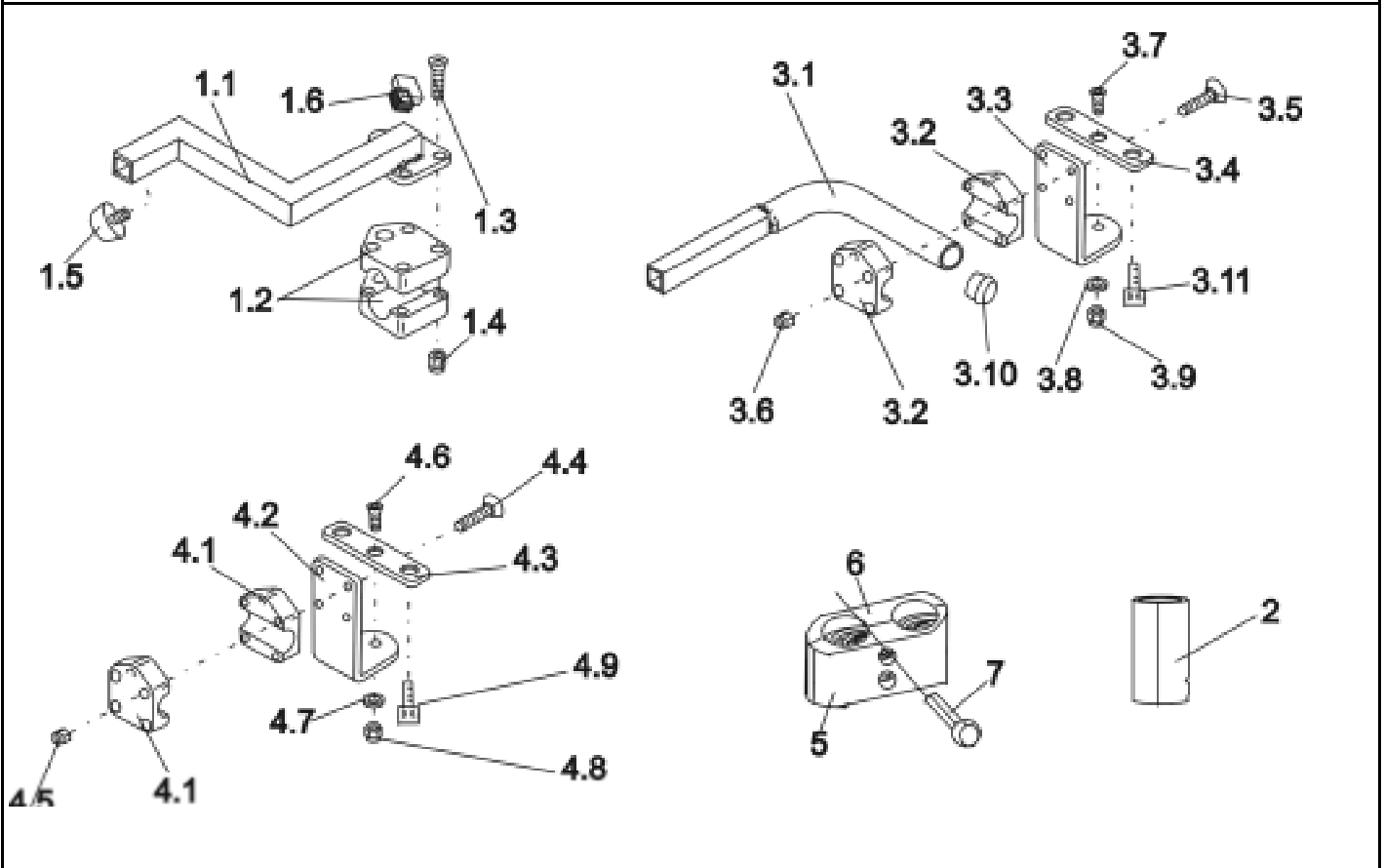


Item	Qty	Part-No.	Part-Designation
3.8	3		WASHER 6,4 X 12,5 X 1,6
3.9	1		DOMED NYLOC NUT M6, SELF-LOCKING
3.10	1		TUBE END PLUG SL 22
3.11	2		HEX HEAD SCREW M6 X 20
-	1	<b>SP1422818</b>	<b>ASSEMBLY KIT F. SWITCHABLE CARER CONTROL CPL.</b>
4.1	2	Z12008A.P40	GRIPPING CLIP W. HEXAGON HOLE
4.2	1		ATTENDANT CONTROL, SECURING BRACKET; RAL 9005 BLACK
4.3	1		BRACKET; RAL 9005 BLACK
4.4	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 35
4.5	4		NYLOC NUT M6
4.6	1		COUNTERSUNK SOCKET HEAD SCREW M6 X 16
4.7	3		WASHER 6,4 X 12,5 X 1,6
4.8	1		DOMED NYLOC NUT M6, SELF-LOCKING
4.9	2		HEX HEAD SCREW M6 X 20

### Storm 3



Customer :  
 Customer No. :  
 Phone :

Contact :  
 Commission :  
**Serial No. :**

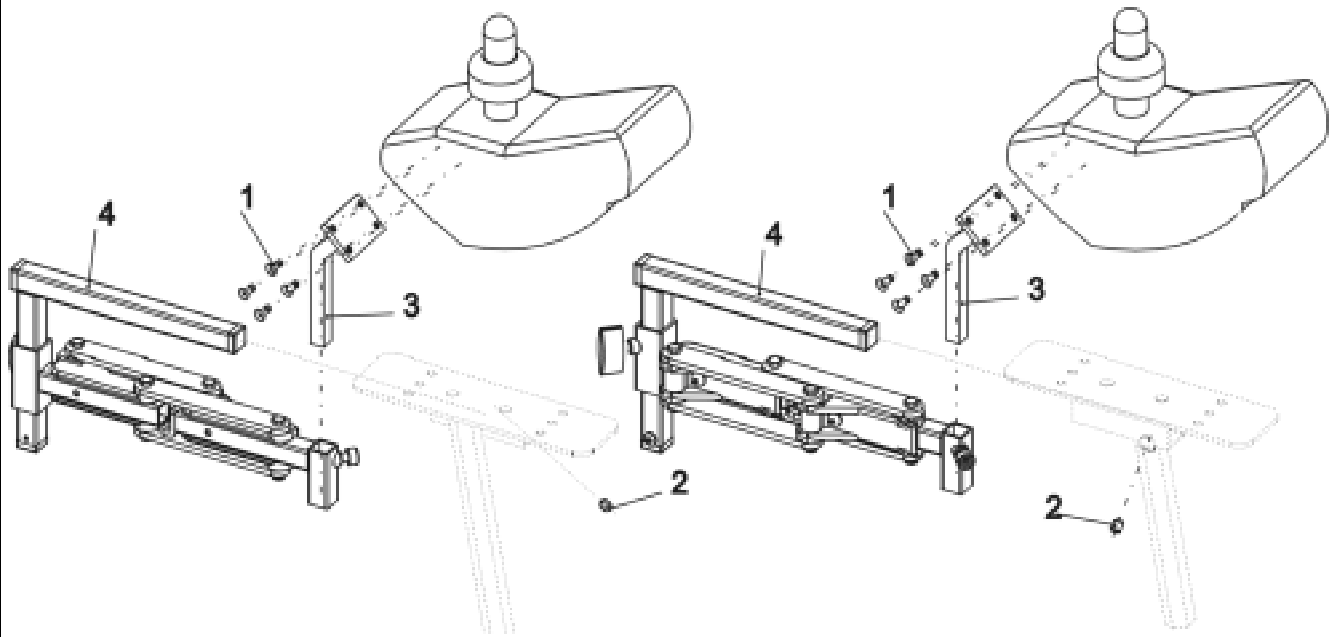


<b>Item</b>	<b>Qty</b>	<b>Part-No.</b>	<b>Part-Designation</b>
4	1	1421795	SWING-OUT ACS REMOTE HOLDER RH
4	1	1421796	SWING-OUT ACS REMOTE HOLDER LH
-	1	<b>SP1423434</b>	<b>SWING-OUT HEIGHT ADJUSTABLE REM 24SC HOLDER ASSE --&gt;02/04</b>
1	4		COUNTERSUNK CROSS HEAD SCREW M4 X 10
2	1		GRUB SCREW M6 X 8
3	1		ADAPTER F. SWING-OUT ACS REMOTE HOLDER
-	1	<b>SP1436543</b>	<b>SWING-OUT REM 24 SD AND SHARK REMOTE HOLDER 03/04--&gt;</b>
-	1	1440441-9005	REMOTE HOLDER SWING AWAY REM 24SD
-	2		COUNTERSUNK CROSS HEAD SCREW M5 X 16

**Storm 3**



Customer :  
 Customer No. :  
 Phone :

Contact :  
 Commission :  
**Serial No. :**

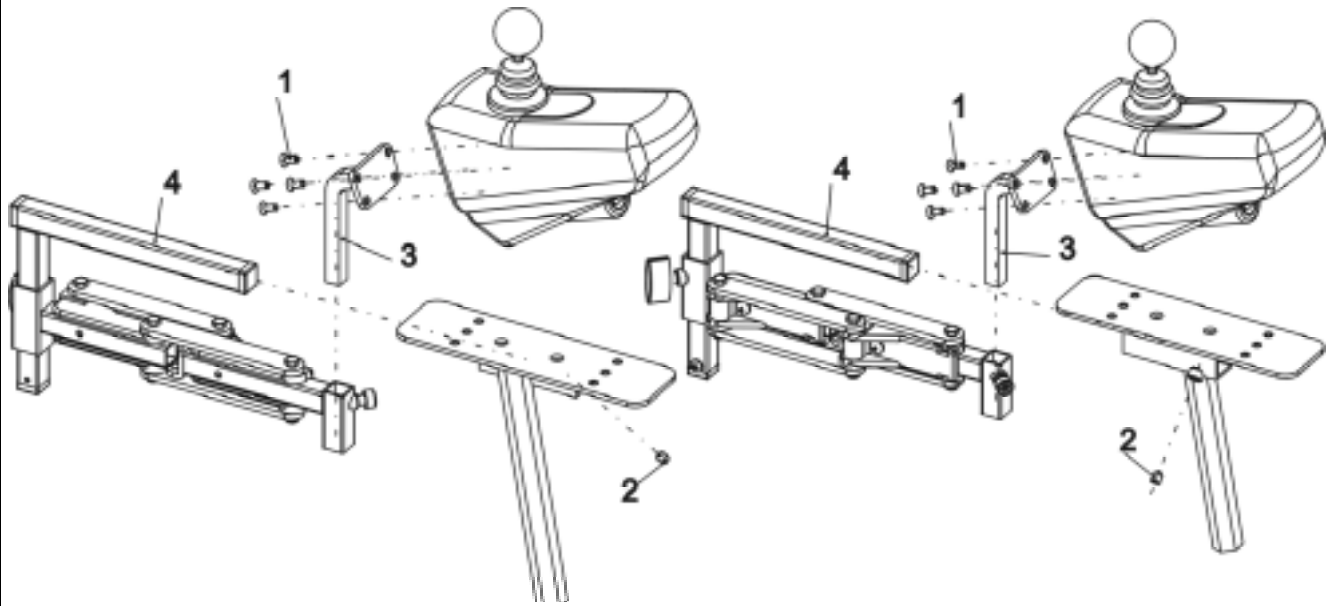

<b>Item</b>	<b>Qty</b>	<b>Part-No.</b>	<b>Part-Designation</b>
-	1	SP1422879	HEIGHT ADJUSTABLE & SWING-OUT REMOTE HOLDER RH CPL. (G8...
-	1	SP1422880	HEIGHT ADJUSTABLE & SWING-OUT REMOTE HOLDER LH CPL. (G8...
-	1	SP1423704	SWING-OUT HEIGHT ADJUSTABLE REMOTE HOLDER ASSEMBLY KIT
1	4		COUNTERSUNK CROSS HEAD SCREW M5 X 12
2	1		GRUB SCREW M6 X 8
3	1		ADAPTER F. SWING-OUT & HEIGHT ADJUSTABLE ACS REMOTE HOL...
4	1	1421795	SWING-OUT ACS REMOTE HOLDER RH
4	1	1421796	SWING-OUT ACS REMOTE HOLDER LH

### Storm 3

