



Seat Support/ Plate

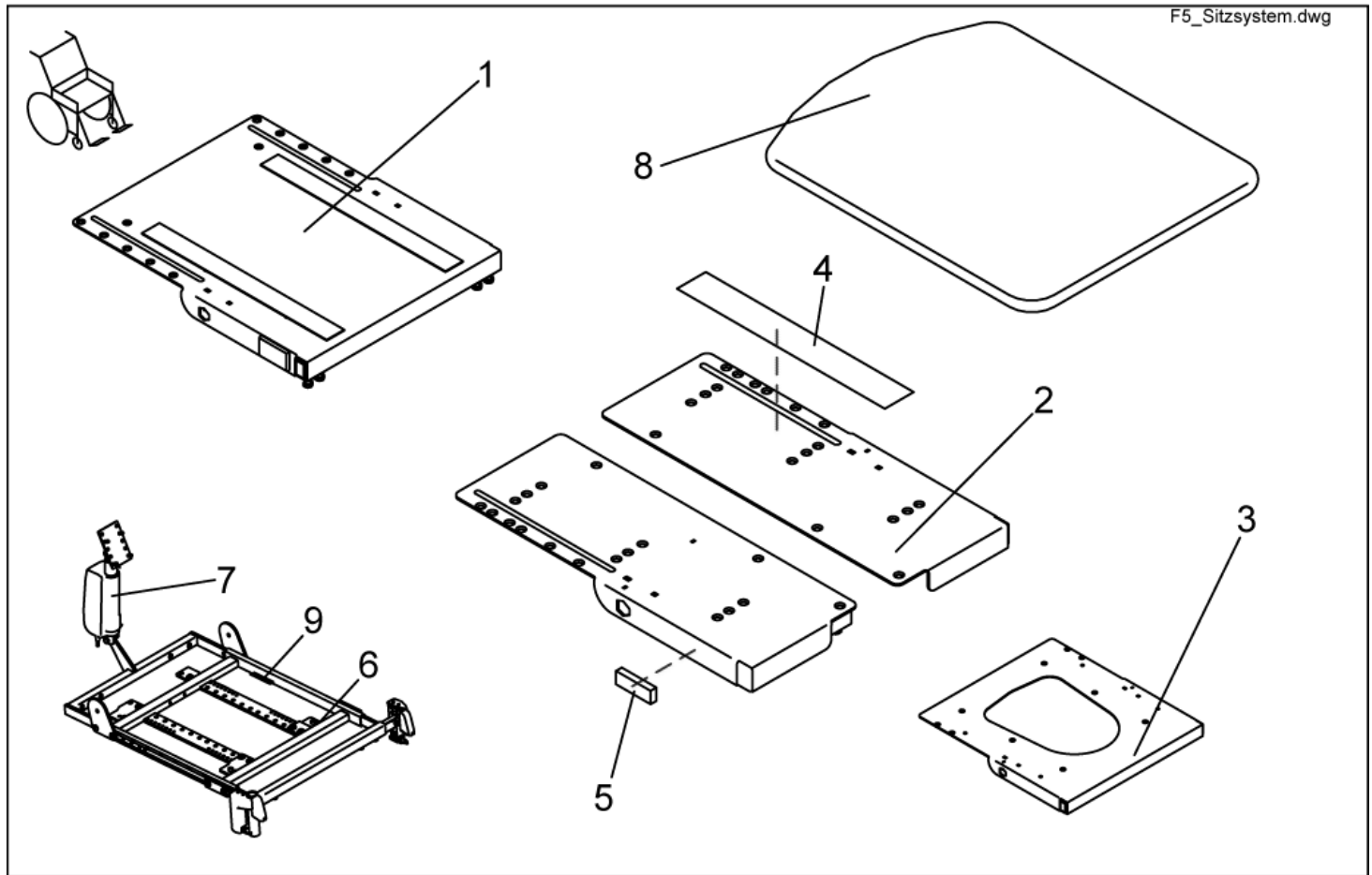


item	ordercode	description	quantity
1	SP1422060	SEAT PLATE (ONE PIECE) COMPLETE	1
1	1419999-9005	SEAT PLATE	1
1	SP1423205	SEAT MOUNTING KIT (ONE-PIECE VERSION)	1
2	SP1422121	SEAT PLATE, FLEX F. SB 39 CPL.	1
2	SP1422064	SPLIT SEAT SB 43 CPL.	1
2	SP1422065	SPLIT SEAT SB 48 CPL.	1
2	1420458-9005	SEAT PLATE LH CPL.	1
2	1420457-9005	SEAT PLATE RH CPL.	1
2	SP1423206	SEAT (SPLIT) ASSEMBLY KIT	1
2	1444732	ASSEMBLY KIT FOR SPLIT SEAT SB 43cm	1
2	1444733	ASSEMBLY KIT FOR SPLIT SEAT SB 48cm	1
2	1436355	INSTALLATION KIT BRACKET SEAT REINFORCEMENT (RH + LH)	1
3	SP1423183	SEAT PLATE RECARO CPL.	1
3	1423174-9005	RECARO SEAT PLATE	1
4	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 400	2
5	P31298	REFLECTOR, YELLOW	2
6	SP1428110	SEAT SUPPORT Optimist, Seat width 45cm, complete	1
7	1419417	MOTOR LA 28.1-100-24-001	1
8	5314532	SEAT PLATE 410 X 520 MM	1
8	5314533	SEAT PLATE 460 X 520 MM	1
9	S182007.P40	ATTACHMENT SHEET METAL FOR SRB PLUG	1

Mounting kit seat plate

1423205.dwg

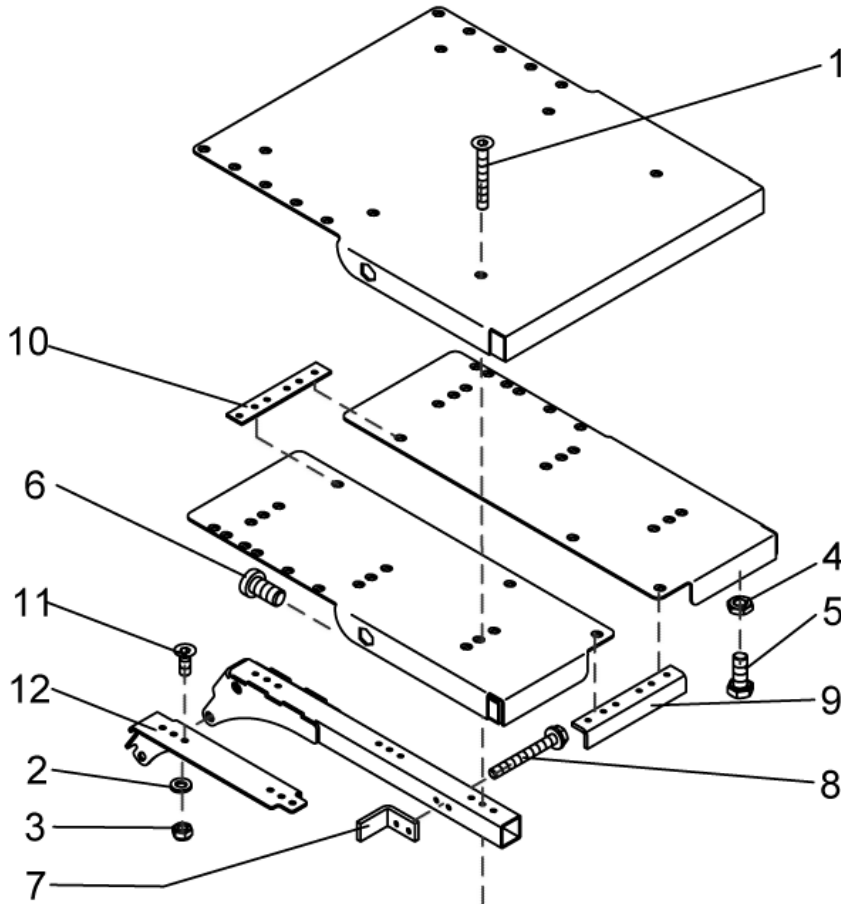


item	ordercode	description	quantity
1-8	SP1423205	SEAT MOUNTING KIT (ONE-PIECE VERSION)	1
1,2,3,4,5,6,7,8,9,10,11	SP1423206	SEAT (SPLIT) ASSEMBLY KIT	1
1,2,3,4,5,6,7,8,9,10,11	1444732	ASSEMBLY KIT FOR SPLIT SEAT SB 43cm	1
1,2,3,4,5,6,7,8,9,10,11	1444733	ASSEMBLY KIT FOR SPLIT SEAT SB 48cm	1
2,3, 11,12	1436355	INSTALLATION KIT BRACKET SEAT REINFORCEMENT (RH + LH)	1
1	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9	6
2	NA	WASHER 6,4 X 12,5 X 1,6	10
3	NA	NYLOC NUT M6	10
4	NA	HEX NUT M8	4
5	NA	HEX HEAD SCREW M8 X 20	4
6	NA	BOLT M10 X 20	2
7	1421700-9005	ANGLE BRACKET RH, SB 43	1
7	1421705-9005	SEAT ANGLE LH SB 48	1
7	1421701-9005	ANGLE BRACKET LH, SB 43	1
7	1421706-9005	SEAT ANGLE RH, SB 48	1
8	NA	BOLT M6 X 50	4
9	NA	ANGLE BRACE	1
10	NA	REINFORCING PLATE	1
11	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 16	4
12	NA	BRACKET SEAT REINFORCEMENT LH	1
12	NA	BRACKET SEAT REINFORCEMENT RH	1

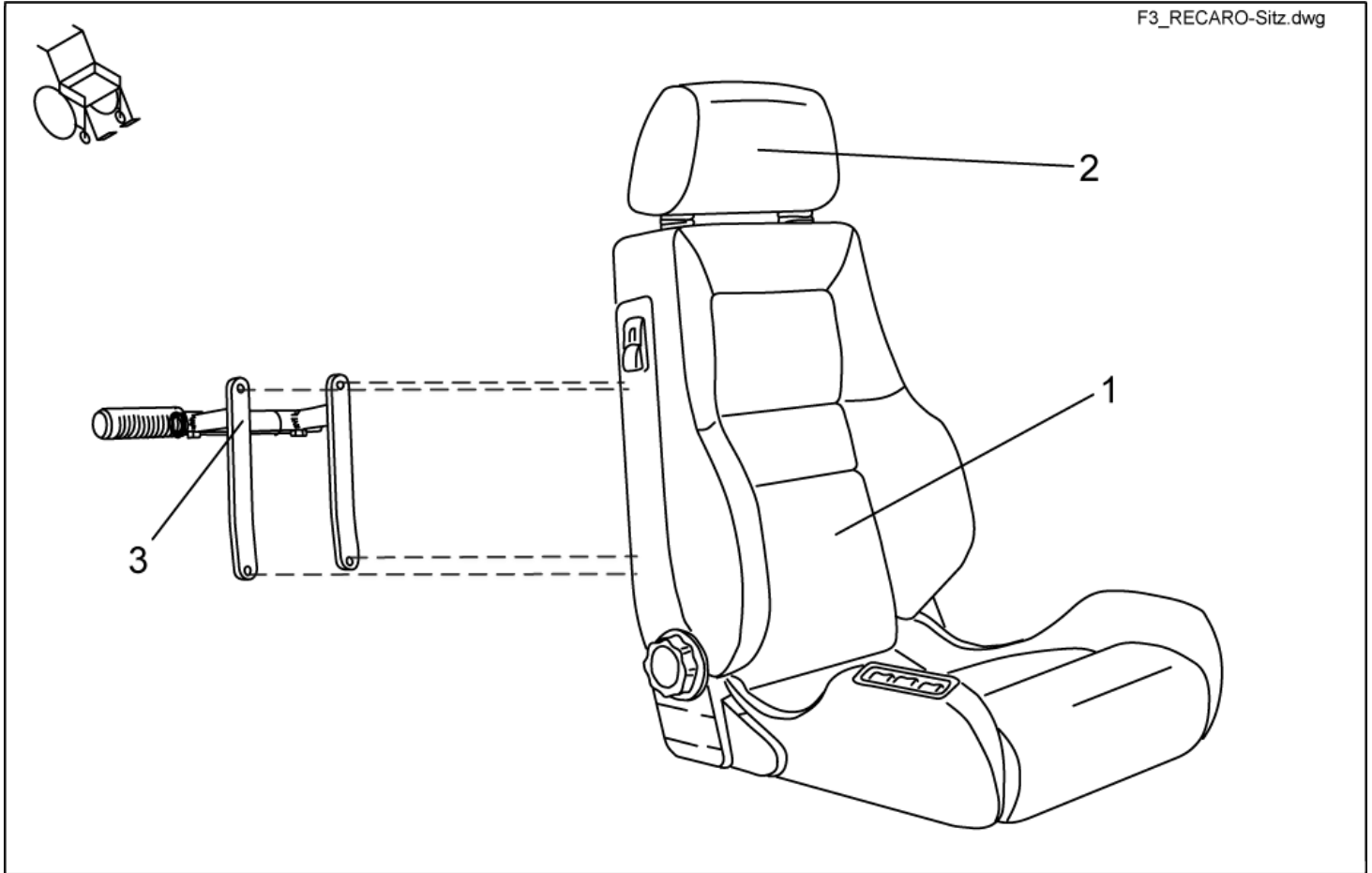

item	ordercode	description	quantity
1	SP1422546	RECARO SEAT N-JOY WITH MOUNTING KIT	1
1	SP1420495	RECARO SEAT MOBILITY WITH MOUNTING KIT	1
-	SP1423173	RECARO SEAT MOUNTING KIT	1
-	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO	1
-	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO	1
2	N89999	HEADREST - SPORT NARDO	1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	





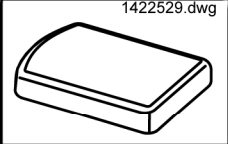




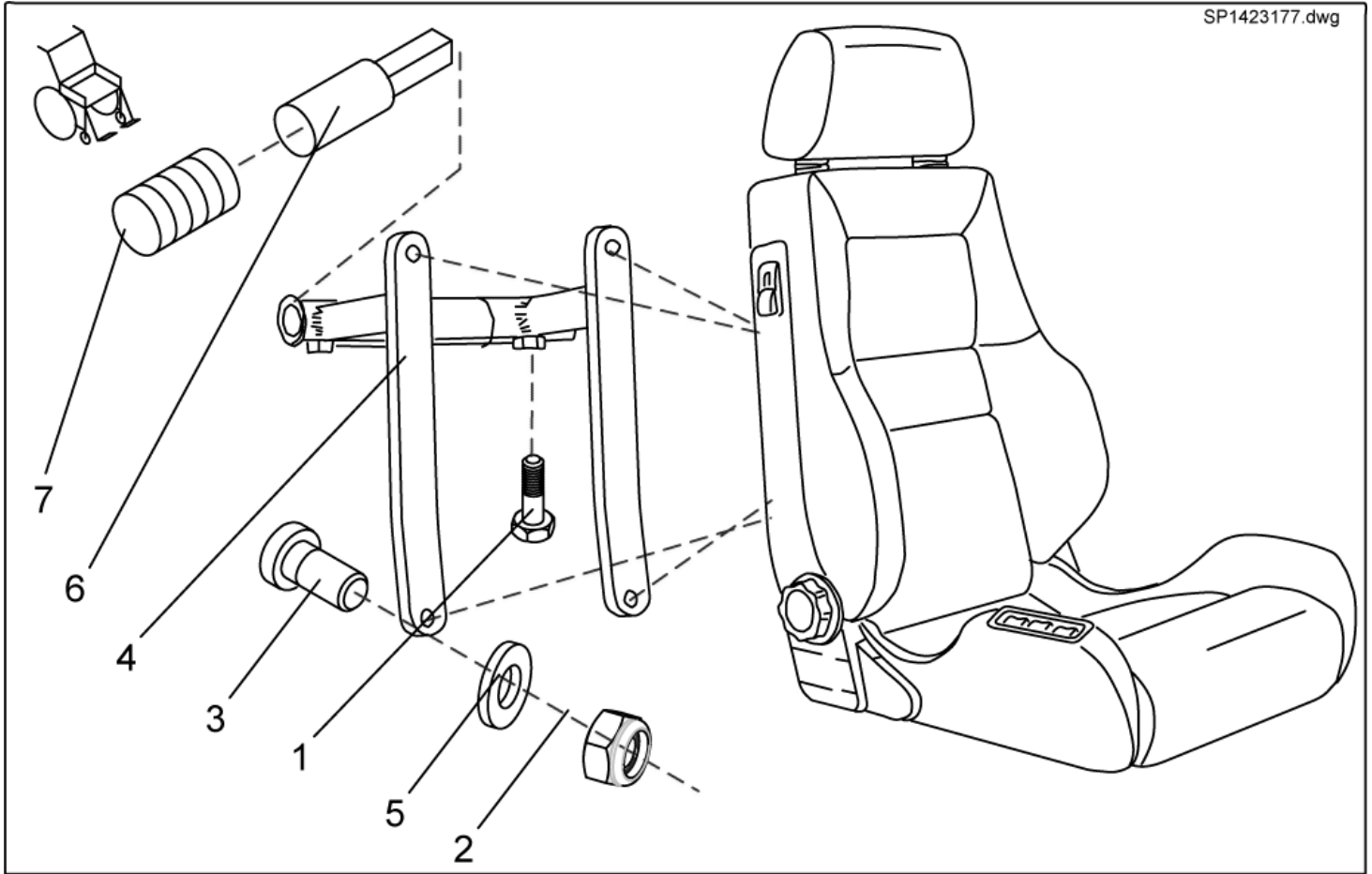

Holder for push handle RECARO cpl.





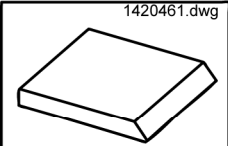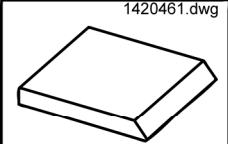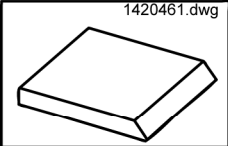






WE RECOMMEND RETROFITTING AT THE FACTORY !

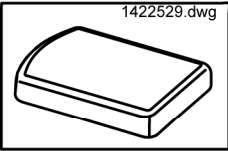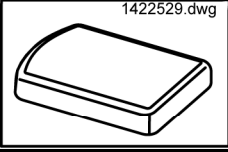





item	ordercode	description	quantity
-	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	1
-	NA	SEAT SETTING STRAP	1
-	NA	TUBE END PLUG SL 22	2
-	NA	TUBE END PLUG E 15 X 15	2
1	NA	HEX HEAD SCREW M8 X 20	2
2	NA	DIN 936 M8 17H BLACK GALVANISED HEX NUT THIN	2
2	NA	NYLOC NUT M6	2
3	NA	HEX HEAD SCREW M6 X 40	4
4	NA	HOLDER FOR PUSH HANDLE RECARO; RAL 9005, DEEP BLACK	1
5	NA	WASHER 6,4 X 12,5 X 1,6	4
6	NA	HANDLE; RAL 9005 DEEP BLACK	2
7	L10009	HANDLE	2

Cushion




Picture	Seat system	Seat width cm	Seat depth cm	color	ordercode
 1420461.dwg	Standard	n.a.	41	black	1420460
 1420461.dwg	Standard	n.a.	46	black	1420461
 1420461.dwg	Standard	n.a.	51	black	1420462
 1422529.dwg	Flex	39	n.a.	Dartex grey	5252582-TR23
 1422529.dwg	Flex	39	n.a.	Vinyl grey	5314542
 1422529.dwg	Flex	39	n.a.	blue	1422529
 1422529.dwg	Flex	39	n.a.	black	1436561
 1422529.dwg	Flex	43	n.a.	Dartex grey	5252583-TR23
 1422529.dwg	Flex	43	n.a.	Vinyl grey	5314525
 1422529.dwg	Flex	41	n.a.	blue	1422530
 1422529.dwg	Flex	41	n.a.	black	1436562

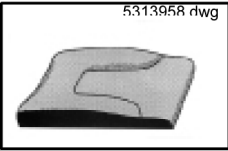Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
	Flex	48	n.a.	Dartex grey	5252584-TR23
	Flex	48	n.a.	Vinyl grey	5314526
	Flex	48	n.a.	blue	1422532
	Flex	48	n.a.	black	1436563
	Flex Vic Air	43	n.a.	VicAir	1422531
	Kontur	43	n.a.	black	1422072
	Kontur	48	n.a.	black	1422074
	Sitzschutz Comfort Flex	39	n.a.	blue	5253078-TR25
	Sitzschutz Comfort Flex	43	n.a.	blue	5253079-TR25
	Sitzschutz Comfort Flex	48	n.a.	blue	5253080-TR25
	positiv	45	n.a.	black	1419632
	positiv	45	n.a.	black	5313953
	positiv	45	n.a.	black	5314038
	negativ	45	n.a.	black	1419633

Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
 <p>5313958 r/wg</p>	negativ	45	n.a.	black	5313958
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	1419634
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	5313962
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	5314044

Arne Jensen-Seats

Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
	1	46	38	-	1419630
	1	46	45	-	5313888
	1	60	45	-	5313889
	1	60	50	-	1419625
	2	46	38	-	1419631
	2	46	38	X	5313867
	2	46	45	-	5313882
	2	60	45	-	5313883
	2	60	45	X	5313895
	2	60	50	-	1419628
	3	60	45	-	5313878

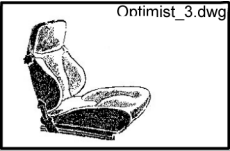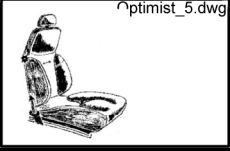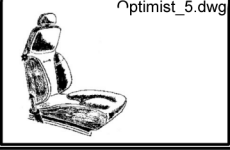Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
	3	70	50	-	5313877
	5	60	45	-	5313872
	5	70	50	-	5313871