
Invacare® Robin®

Robin®, Robin® Mover

Robin®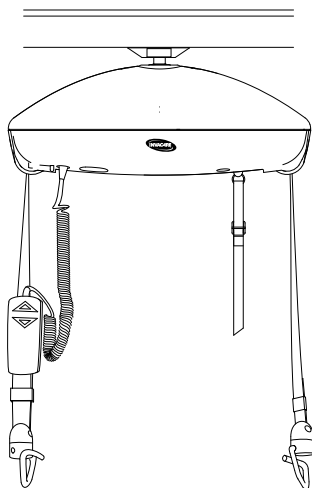

Robin® Mover

en	Ceiling Hoist Service Manual	3
da	Løftemotor Servicemanual	27
fr	Lève-personne sur rail Manuel de maintenance	51
nn	Takmontert løfter Servicehåndbok	77
sv	Taklyft Servicemanual	101

PROVIDER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

©2023 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

1 General	4
1.1 Introduction	4
1.1.1 Symbols in this Manual	4
2 Safety	5
2.1 General Safety Information	5
2.2 Main parts of the ceiling hoist.	5
3 Setup	6
3.1 Installing the ceiling hoist to the rail system.	6
3.1.1 Installing Robin®	6
3.1.2 Installing Robin® Mover	6
4 Maintenance	8
4.1 Charging the Battery	8
4.2 Battery Maintenance	8
4.2.1 Storage Characteristics	8
4.2.2 Storage Temperature	8
4.2.3 Storage Time	8
4.2.4 Storage Humidity	9
4.2.5 Capacity Recovery	9
5 Servicing	10
5.1 Service Interval	10
5.1.1 LOLER Statement	10
5.2 Tightening torques	10
5.3 Periodic Inspection Checklist EC-Track System, Gantry and Robin.	11
6 Instructions	14
6.1 Dismounting the ceiling hoist.	14
6.1.1 Dismounting Robin®	14
6.1.2 Dismounting Robin® Mover	14
6.2 Replacing the hand control	15
6.3 Removing the hoist cover	15
6.4 Replacing Strap and Hook Assemblies	15
6.5 Replacing the battery	16
6.6 Replacing the PCB (Printed Circuit Board).	16
6.7 Replacing the fuse.	16
6.8 Adjusting the solenoid until the hoist traverses	17
6.9 Adjusting the Strap Switches	17
6.10 Replacing the Twin Speed Switch.	20
6.11 Replace Contact Disc for Robin® Mover	20
7 Troubleshooting	21
7.1 Troubleshooting Table	21
7.2 Indicator light and audio signal	22
7.3 Checking Loose Strap Function	22
7.4 Checking Emergency Lowering End Stop Strap Function.	22
7.5 Checking Twin Speed Functionality.	22
7.6 Taking Voltage Readings	22
8 Technical Data	24
8.1 The rail system	24
8.2 Dimensions and weight	24
8.3 Electrical System	24
8.4 Environmental conditions	25

1 General

1.1 Introduction

This document contains important information about assembly, adjustment and advanced maintenance of the product. To ensure safety when handling the product, read this document and the user manual carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative. See addresses at the end of this document.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

For pre-sale and user information, see the user manual.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

1.1.1 Symbols in this Manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

DANGER

Indicates a hazardous situation that will result in serious injury or death if it is not avoided.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Identifies required tools, components and items which are needed to carry out certain work.

Other Symbols

(Not applicable for all manuals)

UK Responsible Person

Indicates if a product is not manufactured in the UK.

Triman

Indicates recycling and sorting rules (only relevant for France).

2 Safety

2.1 General Safety Information

WARNING!

Risk of injury or damage to property

- The procedures in this manual must only be performed by a qualified technician.
- Use only original accessories and spare parts.
- Do not handle this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, installation manuals or instruction sheets supplied with this product or optional equipment.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.

WARNING!

Risk of contamination

- Clean and disinfect the product before servicing.

NOTICE!

Assembly of accessories might not be described in this service manual. Refer to the manual, delivered with the accessory.

- Additional manuals can be ordered from Invacare. See addresses at the end of this document.

NOTICE!

Some replacement parts are only available as a kit. Always use the complete new kit when replacing a part.

- Spare parts can be ordered from Invacare. Refer to your local Invacare website to access the electronic spare parts catalogue (ESPC).

NOTICE!


Refer to the user manual of this product for information on

- Technical data
- Product components
- Labels
- Additional safety instructions
- Cleaning and disinfection instructions

The information contained in this document is subject to change without notice.

2.2 Main parts of the ceiling hoist

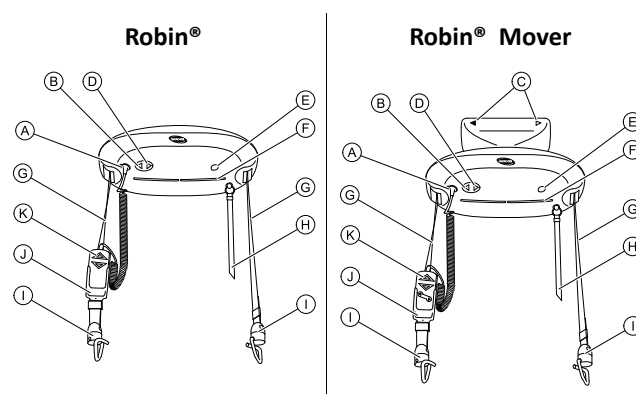(A)	Connection for hand control
(B)	Control button UP
(C)	Directional arrows on Robin® Mover
(D)	Control button DOWN
(E)	Connection for external back-up battery
(F)	Indicator light
(G)	Lifting straps
(H)	Red emergency strap
(I)	Strap hooks
(J)	Retainer for hand control (accessory)
(K)	Hand control

3 Setup

3.1 Installing the ceiling hoist to the rail system


Robin® requires a trolley, guiding the ceiling hoist in the rail system. Robin® Mover has the trolley built-in and can be installed directly to the rail system.

3.1.1 Installing Robin®

 Tools: 4 mm Allen key; 17 mm socket wrench

1. To install the trolley, remove the end stop that is attached to the rail system.


 In some countries/ local areas, there is a demand for double securing of the end stops. If an extra securing screw is installed, remove this screw first.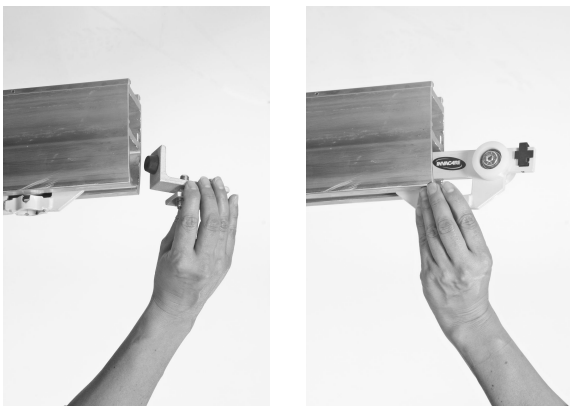


2. Insert the trolley into the rail (no matter which way, as the trolley is symmetric).
3. Re-insert the end stop.

 In some countries/ local areas, there is a demand for double securing of the end stops. If an extra securing screw is required, install this screw. See also *End Stop*.

4. Locate the hoist trolley at the Ø 32 mm round recess in the rail.

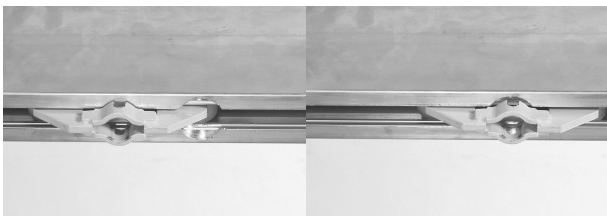
5. Lift the hoist and push the suspension pin through the trolley opening.

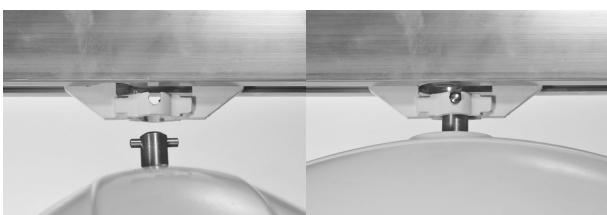6. Turn the hoist by 90 degrees to fix it in the trolley.
7. Connect the hand control.
8. Adjust the length of the emergency stop/lowering cord to within the user's reach.

 If there is excessive resistance, or if the trolley unintentionally rolls by itself due to imbalance in the rail suspensions, adjust the friction brake using a 4 mm Allen key.

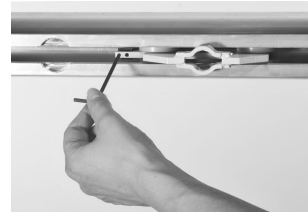3.1.2 Installing Robin® Mover

The trolley is built-in on the hoist, and can therefore not be pre installed in the rail.

 Tools: 4 mm Allen key; 17 mm socket wrench

1. Remove the end stop.

2. Insert the wheels of the hoist into the rail.


 The direction the hoist faces does not matter during installation.

3. Install the end stop.

 Tighten the bolt to 30 Nm.

4. If too much resistance is experienced when the hoist is pulled, or the hoist moves too freely and unintentionally, adjust the friction brake.

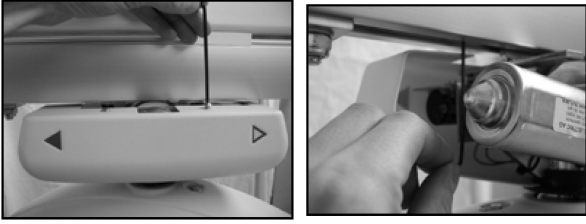5. Connect the hand control.
6. Adjust the length of the emergency stop/lowering cord to within the user's reach.

4 Maintenance

4.1 Charging the Battery

If the charge status becomes low, an audible indicator sounds and the indicator lamp on the bottom of the ceiling hoist flashes (see also *7.2 Indicator light and audio signal, page 22*). When this happens, there is usually sufficient power to lower the patient.

 It is not possible to use the hoist while the battery is charging.

1. Finish the current lift in progress
If the current lift in process discharges the battery completely, an external backup battery may be used to temporarily power the lift. See *Using the external back-up battery*.
2. Move the ceiling hoist to the charging station.
3. Assure that the hand control and charging station are clean and dry before charging
4. Place the hand control into the charging station.
A yellow indicator lamp on the charger indicates the hand control is connected. The indicator lamp illuminates red while charging.
5. When fully charged, the indicator lamp on the charger turns green.
As long as the ceiling hoist is not in use, leave the hand control in the charging station.
6. To use the ceiling hoist, remove the hand control from the charger.

 Keeping the batteries charged guarantees hoist functionality and maintains the battery to ensure a long lifetime.

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need exercise or replacement. See *4.2 Battery Maintenance, page 8*.

If the ceiling hoist will not be used for more than 4 weeks, Invacare recommends disconnecting the charger and pulling the emergency stop.

The charger is connected to the mains with a mains plug. Make sure that the mains plug is accessible and can be unplugged if required.

4.2 Battery Maintenance

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need to be exercised.

To enable the battery to charge fully, the battery should be exercised from time to time.


Following a long period of storage, it could also be necessary to exercise the battery.

It is also suggested that the battery is exercised in situations where the hoist is used infrequently, and/or where the hand control is placed in the charger for long periods of time. In this situation, there will always be

capacity enough for a few lifts, but we recommend to exercise the battery as soon as possible afterwards.

Exercise of the battery can be done by charging and discharging the hoist completely several times.


1. Charge the battery until the LED turns green.
2. Remove the hand control from the charger.
3. Raise and lower the straps with a load for one minute.
4. Wait ten minutes.
5. Repeat Steps 3–4 until the battery is completely discharged.

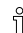 Continue this process after the audible alert sounds and the blue light flashes, indicating the battery is low.

6. Repeat Steps 1–5 two to three times.

4.2.1 Storage Characteristics

Essentially all rechargeable battery cells gradually discharge over time whether they are used or not. This capacity loss is typically due to slow parasitic reactions occurring within the cell. As such, the loss rate (self-discharge rate) is a function of the cell chemistry and the temperature environment experienced by the cell. Due to the temperature sensitivity of the self-discharge reaction, relatively small differences in storage temperature may result in large discharges which are difficult or impossible to reverse. Cell and battery storage issues of concern to most application designers relate either to the speed with which the cells lose their capacity after being charged or the ability of the cells to charge and discharge “normally” after storage for some period of time. In both situations, general guidelines developed for nickel-cadmium cells work acceptably for nickel-metal hydride cells.

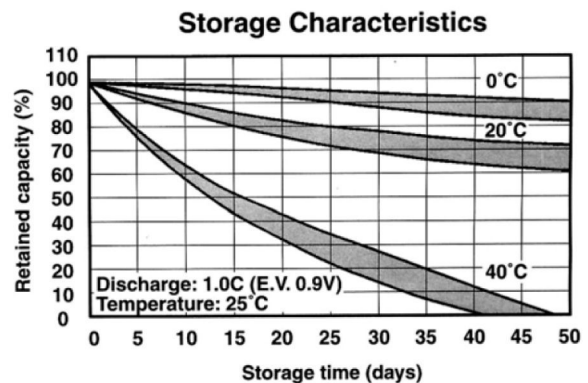4.2.2 Storage Temperature

As already mentioned, the self-discharge reaction rate increases with higher temperatures. Prolonged storage of the battery material deteriorating faster; leakage performance will also deteriorate, resulting in a reduced battery lifetime. It is recommended that, for long storage, batteries should be kept at room temperature or below (0–30 °C or 32–86 °F).

4.2.3 Storage Time

As the battery loses energy during storage, the voltage also drops. In general, the battery capacity loss due to self-discharge during storage can be recovered by recharging. If the battery is stored for over six months, it

is advisable to cycle the battery several times to resume the battery capacity. Use good inventory practices (first in, first out) to reduce time cells spend in storage.

4.2.4 Storage Humidity

Leakage and rusting of metal parts are accelerated in high humidity environments, especially those with correspondingly high temperatures. Storage in high humidity and high temperatures environment is not recommended.

4.2.5 Capacity Recovery

In normal practice, stored cells will provide full capacity on the first discharge after removal from storage and charging with standard methods. Cells stored for an extended period or at elevated temperatures may require more than one cycle to attain pre-storage capacities. Consultation with the manufacturer is recommended if prolonged storage and rapid restoration of capacity is requested.

5 Servicing

5.1 Service Interval

Unless otherwise stated in local requirements, service must be performed at least every 12 months and according to the Service Inspection Checklist. When performing annual or regular service, all parts designed to carry load must be, as a minimum, tested with maximum load. All safety features must be checked according to ISO 10535.

The frequency of inspection must be increased, if the product is continuously exposed to high humidity, high condensation and corrosives (e.g. chlorine or ammonia gases), to prevent product impairment.

5.1.1 LOLER Statement

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998, require any equipment that is used in the workplace to lift a load be subject to safety inspection on a six monthly basis. Please refer to the HSE web site for guidance www.hse.gov.uk.

The person responsible for the equipment must ensure adherence to LOLER regulations.

5.2 Tightening torques

NOTICE!

Do not overtighten the mounting hardware as this might cause damage to the product.

If not otherwise stated in the specific instructions the following guidelines apply:

Thread	Tightening torque in Nm \pm 10%
M4	3 Nm
M5	6 Nm
M6	10 Nm
M8	25 Nm
M10	49 Nm
M12	80 Nm
M14	120 Nm
M16	180 Nm

5.3 Periodic Inspection Checklist | EC-Track System, Gantry and Robin

In accordance with ISO 10535.

CAUTION!

Risk of injury and damage to property

Failure to ensure an adequate **Periodic Inspection** of the product can compromise the safety of people and property.

- A periodic inspection of the product must be performed at least every **12 months** unless otherwise stated in local requirements.
- The frequency of inspection must be increased if the product is continuously exposed to high humidity, high condensation and corrosives (e.g., chlorine or ammonia gases), to prevent product impairment.
- Inspections **MUST** be performed by a qualified person who is acquainted with the design, use and care of the product.
- The inspection record **MUST** be recorded in the logbook and reported to the owner.

Is the EC-Track on the installation?

Yes (Complete the following section):

The Invacare EC-Track System installation label shows the following information:			
Maximum SWL		Installation Date	
Label Number (LN)			

No (Proceed to **2. Checkpoints | Robin**).

1. Checkpoints EC-Track and Gantry	✓	✗	Comments
1.1 The installation label is present, visible and readable.			
1.2 All connection points, brackets to structure are tightened with recommended torque. Retighten, if necessary (if NOT accessible please see point 1.21 on load test).			
1.3 All connection points, brackets to rail system are tightened with recommended torque. If the system is the Gantry, all the connections are tightened with recommended torque. Retighten, if necessary (if NOT accessible please see point 1.21 on load test).			
1.4 All rail ends are secured with end stops, sufficiently tightened. Retighten, if necessary.			
1.5 End stops don't show any sign of damage, wear and/or deformation.			
1.6 Brackets don't show any sign of damage, wear and/or deformation. If the system is Gantry, connections don't show any sign of damage, wear and/or deformation.			
1.7 Trolleys don't show any sign of damage, wear and/or deformation.			
1.8 The trolley (and if applicable, traverse trolleys) run smoothly and silently in the entire track system.			
1.9 Check and adjust if required the trolley friction brake.			
1.10 End Covers are assembled (If applicable).			
1.11 Rail systems with track connections: The connections are aligned. Re-align, if needed.			
Transit Coupling / Track Switch (if applicable)			
1.12 Lock fixtures all OK (secure to tracking, each claw moves independently, correctly aligned with tracking slots, no obstructions to prevent correct function, safety plate is fitted).			
1.13 Check traverse trolley mounting fixtures.			

1. Checkpoints EC-Track and Gantry	✓	✗	Comments		
1.14 The tracking is aligned. Re-align, if needed.					
1.15 The gap between the track and the transit coupling/track switch is max. 3 mm.					
1.16 Function of transit coupling / lock fixture OK (10 times — disable left / right claw x 5).					
1.17 Function test performed under load through the track switch of the system (5 cycles with the maximum SWL).					
1.18 Inlay plate is fitted (L and M rail).					
Load test					
1.19 The track system is leveled and all fixings are sufficiently tightened before carrying out a load test.					
1.20 If all wall and ceiling brackets are accessible, and sufficiently secured a load test must be performed with the maximum SWL (200 kg). The load must be passed through the entire rail system.					
1.21 If all wall and ceiling brackets are NOT accessible a load test must be performed with 300 kg – 1,5 × maximum SWL (200 kg). This test must take a minimum of 20 minutes. The load must be suspended under each bracket/fixing point and passed through the entire rail system.					
1.22 The rail system remains the same height after the load test.					
1.23 The rail system remains leveled after the load test.					
1.24 Place a label with the date for next inspection (MM-YYYY) on the Tracks. Label Example: <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="padding: 2px;">Date for Next Inspection</td> </tr> <tr> <td style="height: 20px;"> </td> </tr> </table>	Date for Next Inspection				
Date for Next Inspection					

Is Robin hoist assembled on the EC-Track?

Yes (Complete the following section).

No (Skip to the **3. Final Steps** and make sure to check the hoist’s manufacturer **Instructions / Installation Manual**).

2. Checkpoints Robin	✓	✗	Comments
2.1 Product Label is present and legible (write SN on the comments section).			
2.2 The hoist is marked with the maximum SWL.			
2.3 Up switches work correctly.			
2.4 Down switches work correctly. Loose strap function works correctly. Check adjustment if it is not working.			
2.5 End stop works correctly. Check adjustment if it is not working.			
2.6 The emergency stop and reset work correctly.			
2.7 The emergency lowering function and reset work correctly.			
2.8 The emergency lowering end stop switch operates correctly.			
2.9 The full length of straps (tapes) have no signs of wear, fraying and/or loose stitching.			
2.10 Check if the end stop strap switch is working correctly (via hand control). Adjust if necessary.			

2. Checkpoints Robin	✓	✗	Comments		
2.11 Hooks and hooks' fixation (screws and pins) do NOT have signs of wear/damage that could endanger the lifting.					
2.12 Check suspension and cross pin.					
2.13 Steel and plastic gearwheels do NOT have signs of wear and damage. If gearwheels are dry, apply gearwheel grease.					
2.14 Check tape guide cylinders.					
2.15 The hand control is connected correctly.					
2.16 The hand control is functional and in good conditions.					
Electrical Connections					
2.17 All cables are intact.					
2.18 No signs of rupture on cables and on isolation.					
2.19 Clamping and mechanical function of micro switches work correctly.					
2.20 The charger does not have signs of damage and is functional.					
2.21 The emergency lowering end stop switch works correctly, if applicable.					
Load Test					
2.22 Perform a full lifting cycle with the maximum SWL (200kg).  Must be effortless and without jarring sound.					
2.23 Stick a label with the date for next inspection (MM-YYYY) on the Robin hoist. Label Example: <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="background-color: #cccccc; padding: 2px;">Date for Next Inspection</td> </tr> <tr> <td style="height: 20px;"></td> </tr> </table>	Date for Next Inspection				
Date for Next Inspection					

3. Final Steps	✓	✗	Comments
3.1 The owner has all the necessary User Manuals .			

Overall assessment

<input type="checkbox"/> Pass No safety or functional deficiencies were detected.	<input type="checkbox"/> Fail Safety or functional deficiencies were detected, and corrective actions are required. Comments:
---	--

Date of next inspection: _____

Service Company Name: _____

Inspection done by: _____


Date / Signature: _____

6 Instructions

6.1 Dismounting the ceiling hoist

6.1.1 Dismounting Robin®

Remove the ceiling hoist from the trolley

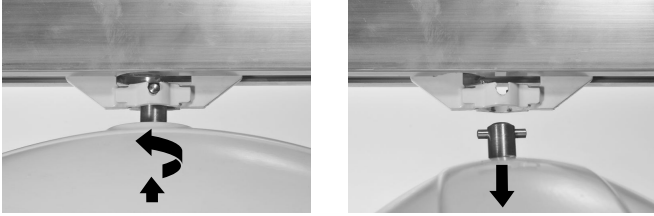1. Lift the hoist slightly and push the suspension pin out of its holder in the trolley.
2. Turn the hoist by 90 degrees and pull it out of the trolley.

Remove the trolley from the rail

1. Remove the end stop.
2. Slide the trolley out of the rail.

6.1.2 Dismounting Robin® Mover

1. Pull the emergency strap to prevent accidental activation of the ceiling hoist after it has been dismantled from the rail.
2. Remove the end stop.

3. Slide the wheels of the hoist out of the rail.

4. Install the end stop.


 Tighten the bolt to 30 Nm.


5. If removing the hoist from the rail motor, perform the following steps:

 When replacing the battery or performing service, it is also necessary to remove the hoist from the rail motor.

- a. Remove the screws securing the plastic shells from the rail motor.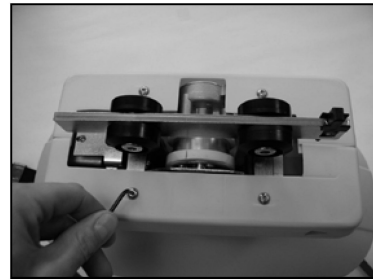


- b. Pull the plastic shells away from each other to remove them from the rail motor.

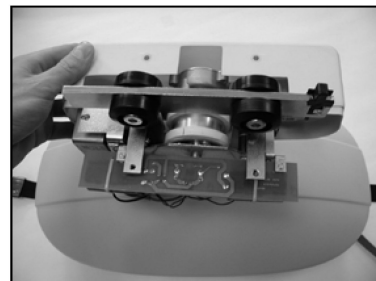
- c. Remove the M6 screw securing the rail motor to the hoist.

- d. Separate the rail motor from the hoist.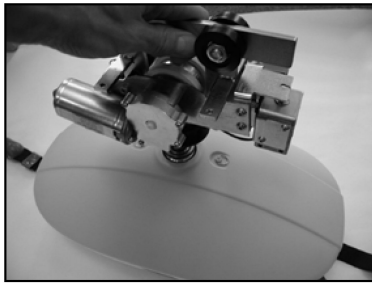


! NOTICE!
 It is possible to remove the hoist from the rail motor without removing the rail motor from the rail system. To avoid damage to the hoist:
 – Always support the hoist when removing the M6 screw.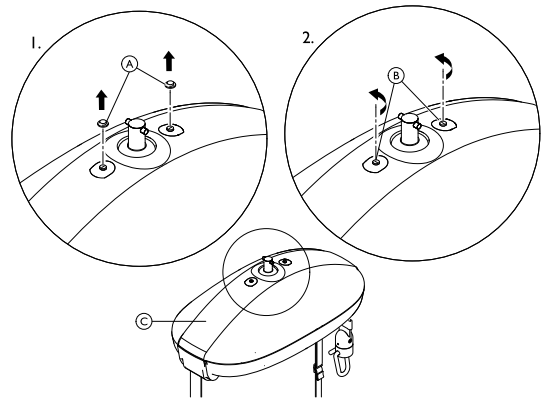

1. Remove protection caps (A). Use fingers or, if required, use a screw driver carefully.
2. Untighten screws (B), using a screw driver.
3. Remove the cover (C) from the hoist.

To close the hoist cover, tighten the screws and attach the protection caps again.

6.2 Replacing the hand control

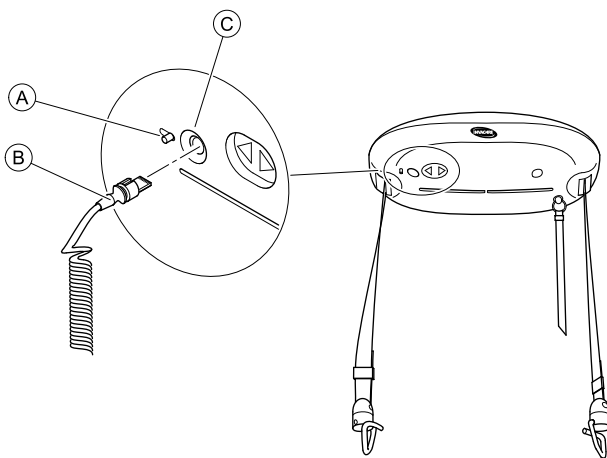1. Slide and hold locking lever (A) away from the aperture (C).
2. Pull the plug (B) out of the aperture.
3. Insert plug of the new hand control into aperture. Make sure, the plug is in the correct orientation with the slot upwards (see image).
4. Release lever (A), so that the gate closes and hold the plug in the aperture.

6.3 Removing the hoist cover

Before removing the hoist cover, pull the red emergency strap to turn off the hoist and remove the hoist from the track system. Refer to:


- 6.1 Dismounting the ceiling hoist, page 14

 Tools: Slotted Screw driver

6.4 Replacing Strap and Hook Assemblies

 Tools: 10 mm socket key; 5 mm hex/Allen key

1. With the ceiling hoist in the track, lower the straps to their lowest point using the hand control.
2. Pull the red emergency strap to perform an emergency stop. Do not perform emergency lowering.
3. Remove hoist from track. Refer to 6.1 Dismounting the ceiling hoist, page 14.
4. Remove the cover. Refer to 6.3 Removing the hoist cover, page 15.
5. Reset the emergency stop by pressing the reset button.
6. Check that the strap (A) is completely unwound as shown in the image. If not, wind the straps out using the UP/DOWN button on the hand control.
7. Align the strap retaining bolts (B) with the aperture in the chassis using the UP/DOWN button on the hand control.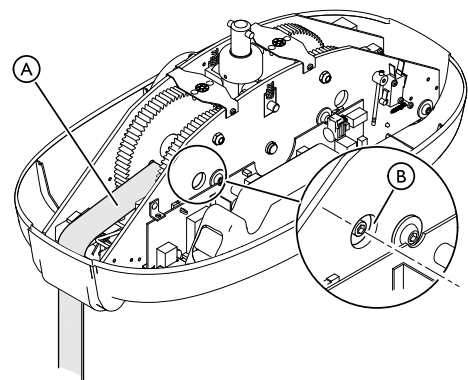

8. Pull the red emergency strap again.
9. Remove strap bolts, using a 10 mm socket key and a 5 mm hex/Allen key.
10. Remove the strap through guide cylinder.
11. Insert a new strap through guide cylinder and assemble using the strap retaining bolts (Torque: 4 Nm). Make sure that the new strap is fed over the slack strap switch wire.
12. Re-mount the cover, re-assemble the hoist to the track and reset the emergency stop.


6.5 Replacing the battery

NOTICE!

Static electricity can damage the PCBs inside the hoist.
 Before working in the inside of the hoist:

- Pull the red emergency strap to turn off the hoist.
- Remove the hoist from the track system. (Refer to 6.1 *Dismounting the ceiling hoist, page 14*)
- Remove the cover from the hoist. (Refer to 6.3 *Removing the hoist cover, page 15*)
- An anti-static mat must be used when working inside the ceiling hoist.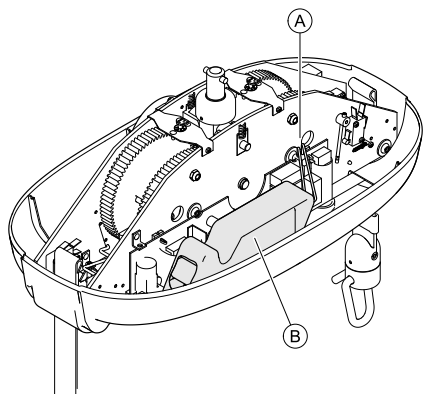

1. Unplug the wires (A).
2. Remove the battery pack (B).
3. Insert a new battery pack.

Use only Invacare recommended batteries.

4. Plug in the wires (A).
5. Re-assemble the hoist and reset the emergency stop.


6.6 Replacing the PCB (Printed Circuit Board)


NOTICE!

Static electricity can damage the PCBs inside the hoist.
 Before working in the inside of the hoist:

- Pull the red emergency strap to turn off the hoist.
- Remove the hoist from the track system. (Refer to 6.1 *Dismounting the ceiling hoist, page 14*)
- Remove the cover from the hoist. (Refer to 6.3 *Removing the hoist cover, page 15*)
- An anti-static mat must be used when working inside the ceiling hoist.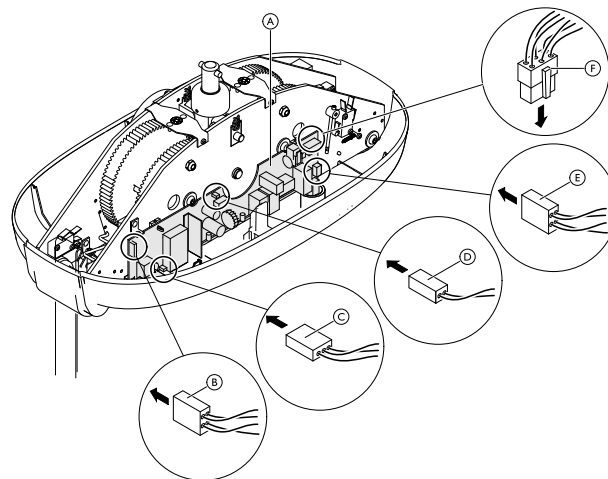

1. Remove the battery. Refer to steps 1 and 2 in chapter 6.5 *Replacing the battery, page 16*.
2. Remove all plugs (B), (C), (D), (E) and (F) from the PCB (A).
3. Release the PCB (A) from the retaining holders by gently pressing down the locking clips and levering the PCB slightly away from the chassis plate.

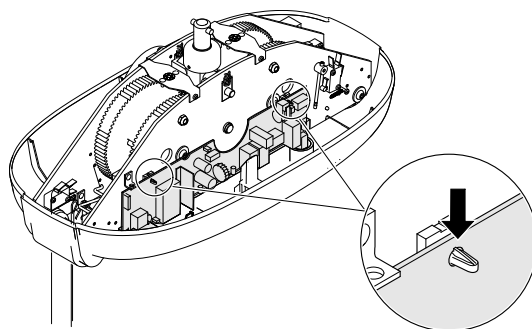4. Lift the PCB up to remove.
5. Place the new PCB into the chassis and press it slightly against the chassis plate until the retaining holders snap into place.
6. Connect the plugs (B), (C), (D), (E) and (F) to the new PCB.
7. Re-insert battery, re-assemble the hoist and reset the emergency stop.

6.7 Replacing the fuse


NOTICE!

Static electricity can damage the PCBs inside the hoist.
 Before working in the inside of the hoist:

- Pull the red emergency strap to turn off the hoist.
- Remove the hoist from the track system. (Refer to 6.1 *Dismounting the ceiling hoist, page 14*)
- Remove the cover from the hoist. (Refer to 6.3 *Removing the hoist cover, page 15*)
- An anti-static mat must be used when working inside the ceiling hoist.

1. Remove the hand control. Refer to chapter 6.2 *Replacing the hand control, page 15*
2. Remove the battery. Refer to steps 1 to 3 in chapter 6.5 *Replacing the battery, page 16*.

- Gently pull the fuse (A) straight out away from the holder (B).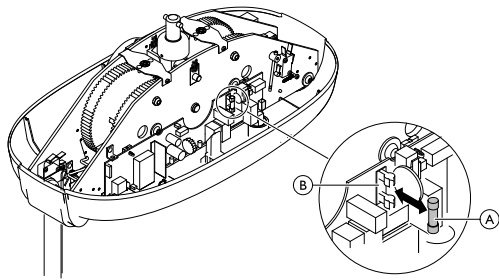


- Insert new fuse.
 - i** Make sure only a fuse with correct rating 10A 250V slow blow is used.
- Re-insert battery, re-assemble the hoist, insert the hand control and reset the emergency stop.

6.8 Adjusting the solenoid until the hoist traverses


Robin® Mover only.

i 4 mm Allen key; 7 mm spanner

- Pull the red emergency strap.
- With the ceiling hoist remaining on the track, remove both plastic shells: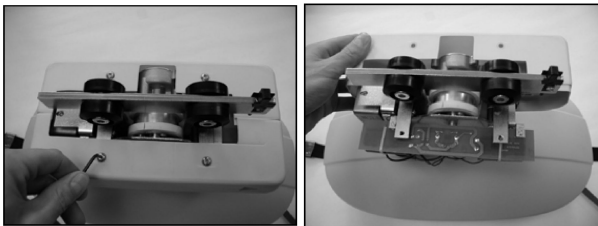

- Remove the screws securing the plastic shells from the rail motor.
 - Pull the plastic shells away from each other to remove them from the rail motor.
- Using a cloth, protect the solenoid plunger whilst clamping with grips at the same time. Using a 7 mm spanner raise the nut (A) by turning anti-clockwise 2 complete turns.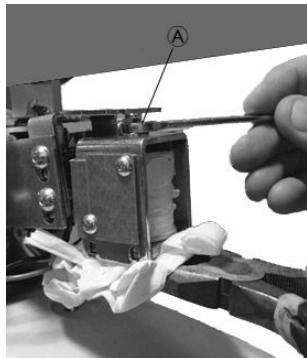

- For testing, restrain the hoist with one hand whilst pressing the Left or Right button on the hand control. There should be good contact between the white drive wheel and the underside of the track. If not, repeat step 3.
- Reassemble the plastic shells. Ensure the correct orientation, with clearance for the plate above the solenoid to move up and down freely. See images in step 2.


6.9 Adjusting the Strap Switches

! **NOTICE!**
 Static electricity can damage the PCBs inside the hoist.
 Before working in the inside of the hoist:
 – Pull the red emergency strap to turn off the hoist.
 – Remove the hoist from the track system. (Refer to 6.1 Dismounting the ceiling hoist, page 14)
 – Remove the cover from the hoist. (Refer to 6.3 Removing the hoist cover, page 15)
 – An anti-static mat must be used when working inside the ceiling hoist.

i 2 mm Allen key


i References to the **emergency lowering end stop switch** are exclusively applicable to newer product models.

- Place the hoist on a surface (such as the corner of a table) so that the hoist is supported yet the straps and hand control are free hanging. Ensure the hoist is safe and cannot slip/fall of the table.
- Locate the loose strap switch (A). Visually check if there is clearance between collet (C) and the chassis.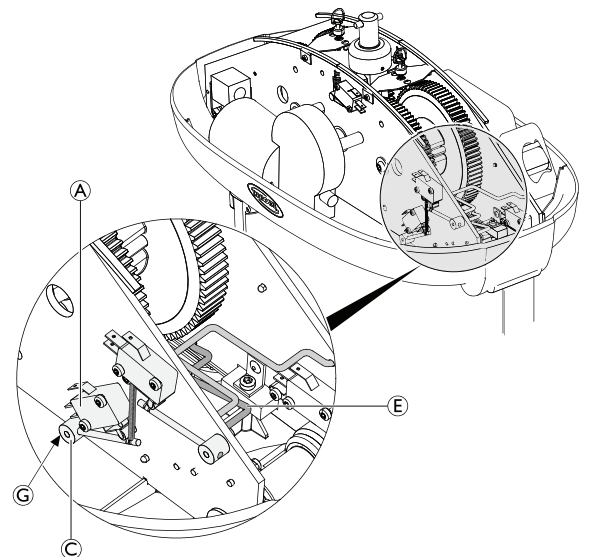


- If not: whilst retaining the geometry of the slack strap wire (E) and the collet arm, loosen the collet's grub screw (G) with a 2 mm Allen key.
- Create clearance between collet and chassis.
- Ensure the slack strap wire (E) is centered between both chassis' plates. Apply thread lock and retighten the collet's grub screw (G).

3. Locate the emergency lowering end stop switch **B**. Visually check if there is clearance between collet **D** and the chassis.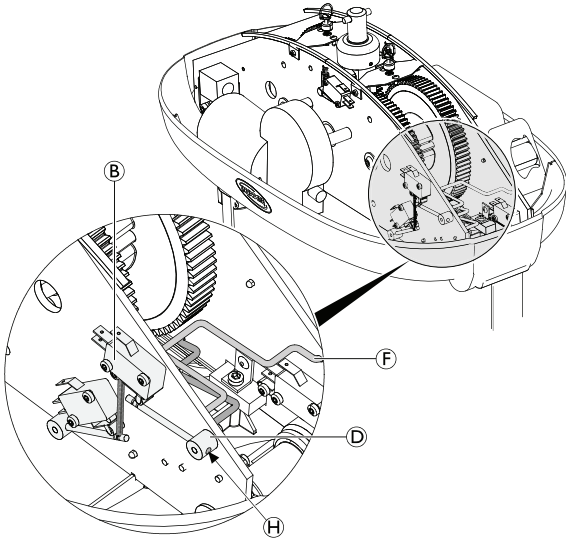


- a. If not: whilst retaining the geometry of the emergency lowering end stop strap wire **F** and the collet arm, loosen the collet's grub screw **H** with a 2 mm Allen key.
 - b. Create clearance between collet and chassis.
 - c. Ensure the emergency lowering end stop strap wire **F** is centered between both chassis' plates. Apply thread lock and retighten the collet's grub screw **H**.
4. Test both the loose strap and emergency lowering end stop strap functions. Refer, respectively, to chapters 7.3 *Checking Loose Strap Function*, page 22 and 7.4 *Checking Emergency Lowering End Stop Strap Function*, page 22. These tests can be carried out with the hoist remaining on the table. If the fault is rectified, re-assemble the hoist. If not, continue with step 5.

5. Adjust tension on the slack strap wire: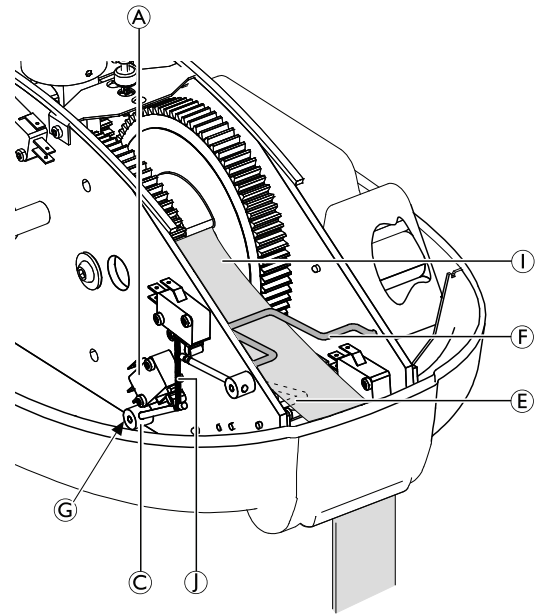

- a. Loosen the collet grub screw **G** with a 2 mm Allen key. Note the spring **I** will now pull the collet arm **C** against the switch so it is open circuit.
 - b. Lift the strap **I** off the slack strap wire **E** and lift up the slack strap wire. Confirm the emergency lowering end stop strap wire **F** is not displaced during this procedure.
 - c. Ensure the slack strap wire **E** is centered between both chassis' plates. Apply thread lock and retighten the collet grub screw **G**.
6. Adjust tension on the emergency lowering end stop strap wire: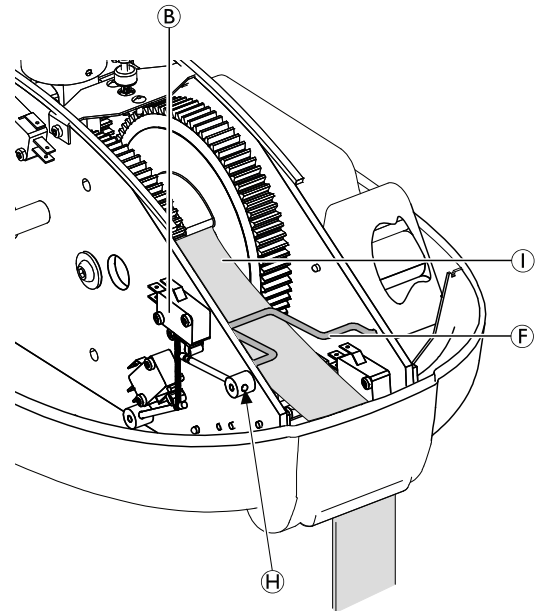

- a. Loosen the collet grub screw **H** with a 2 mm Allen key.
- b. Lift the emergency lowering end stop strap wire **F** off the strap **I**.
- c. Ensure the emergency lowering end stop strap wire **F** is centered between both chassis' plates. Apply thread lock and retighten the collet grub screw **H**.

7. Test both the loose strap and emergency lowering end stop strap functions. Refer, respectively, to chapters *7.3 Checking Loose Strap Function, page 22* and *7.4 Checking Emergency Lowering End Stop Strap Function, page 22*. These tests can be carried out with the hoist remaining on the table.
8. If the fault is rectified, re-assemble the hoist and reset the emergency stop.

6.10 Replacing the Twin Speed Switch


! NOTICE!

- Static electricity can damage the PCBs inside the hoist.
- Before working in the inside of the hoist:
- Pull the red emergency strap to turn off the hoist.
 - Remove the hoist from the track system. (Refer to 6.1 Dismounting the ceiling hoist, page 14)
 - Remove the cover from the hoist. (Refer to 6.3 Removing the hoist cover, page 15)
 - An anti-static mat must be used when working inside the ceiling hoist.

T10 Torx Screwdriver

1. Place the hoist on a surface (such as the corner of a table) so that the hoist is supported yet the straps and hand control are free hanging. Ensure the hoist is safe and cannot slip/fall off the table.
2. Remove the two yellow cables (A) from the twin speed switch (B).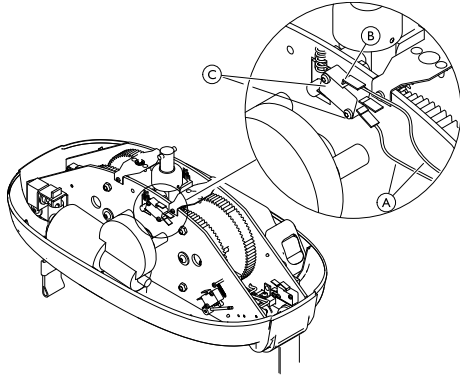

3. Remove the two screws (C).
4. Attach the new twin speed switch with screws (C). Connect the yellow cables (A). Ensure, they are connected to the bottom and top terminals.
5. Re-assemble the hoist and reset the emergency stop.

6.11 Replace Contact Disc for Robin® Mover


! NOTICE!

- Static electricity can damage the PCBs inside the hoist.
- Before working in the inside of the hoist:
- Pull the red emergency strap to turn off the hoist.
 - Remove the hoist from the track system. (Refer to 6.1 Dismounting the ceiling hoist, page 14)
 - Remove the cover from the hoist. (Refer to 6.3 Removing the hoist cover, page 15)
 - An anti-static mat must be used when working inside the ceiling hoist.


Screwdriver

1. Remove the cable tie (A).
2. Remove the contact disc cables (B) from the Mover PCB, using a terminal screwdriver. Note brown cable top and blue cable bottom. Feed the cables through the chassis plate.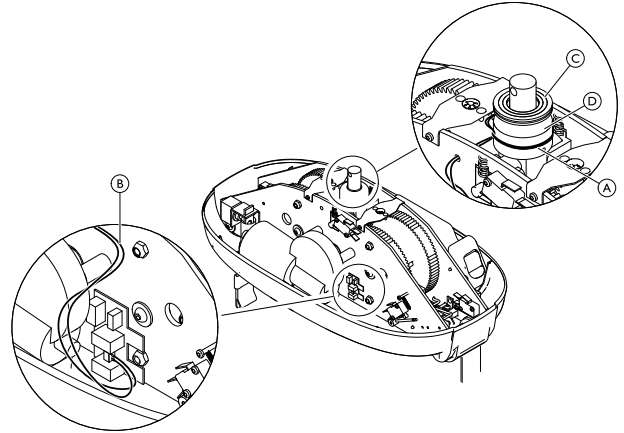

3. Carefully separate and remove the complete contact disc (C) from the suspension base (D) by releasing the self-adhesive tape.
4. Remove, clean off the remains of the self adhesive tape.
5. Fit the new self adhesive tape (E) and the new contact disc (C).

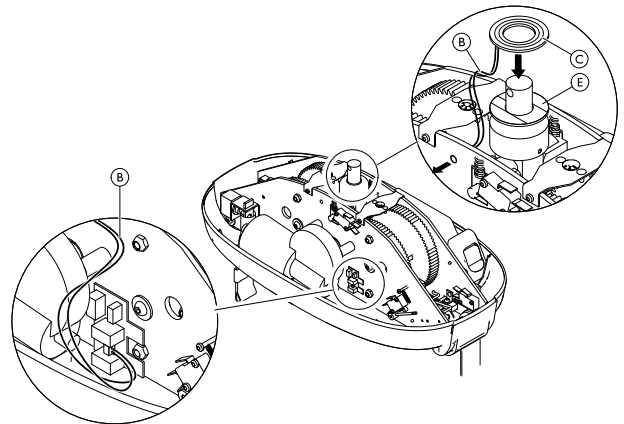6. Feed the cables (B) through the chassis plate and connect them to the Mover PCB.
7. Re-assemble the hoist and reset the emergency stop.

7 Troubleshooting

7.1 Troubleshooting Table

SYMPTOMS	PROBLEM	SOLUTION
Ceiling hoist does not respond to the hand control's buttons and up/down buttons on the ceiling hoist are not responding.	Emergency stop activated	Reset the emergency stop. (Refer to user manual)
	System does not have power or battery is discharged	Charge the battery or use the emergency battery. Refer to 4.1 <i>Charging the Battery, page 8</i> .
	Fuse is blown	Replace the fuse. Refer to 6.7 <i>Replacing the fuse, page 16</i> .
Ceiling hoist does not respond to the hand control's buttons, but up/down buttons on the ceiling hoist are responding.	Hand control not connected properly	Check that the hand control cable is properly plugged into the hoist. Refer to 6.2 <i>Replacing the hand control, page 15</i> .
	Hand control defective	Replace the hand control. Refer to 6.2 <i>Replacing the hand control, page 15</i> .
Lifting straps will only move down and not up.	Battery nearly discharged	Charge the battery or use the emergency battery. Refer to 4.1 <i>Charging the Battery, page 8</i> .
	Load is too high	Reduce the load. The hoist is equipped with an over-current safety system.
Lifting straps will only move up and not down.	Hand control defective	Replace the hand control. Refer to 6.2 <i>Replacing the hand control, page 15</i> .
	Loose strap function activated	Check the loose strap function. Refer to 7.3 <i>Checking Loose Strap Function, page 22</i> .
Ceiling hoist speed is the same with and without load.	Non-load quick movement function activated permanently	Check the quick movement switch. Refer to 7.5 <i>Checking Twin Speed Functionality, page 22</i> .
Lifting straps are twisted and do not straighten out.	Lifting hooks do not turn freely enough	Clean the hooks or replace the straps.
	Straps are frayed	Replace the straps. Refer to 6.4 <i>Replacing Strap and Hook Assemblies, page 15</i> .
Ceiling hoist does not traverse the rails.	Rails require service or cleaning	Inspect the rail system
	Driving wheels worn or damaged (Robin® Mover only)	Adjust driving wheels.
	Hand control is defective	Replace the hand control.
Ceiling hoist emits excessive noise when activated.	Bearings, gearwheels or motor defective	Replace the hoist.
Ceiling hoist will not charge	System has no power	Place the hand control in the charging station and recharge the ceiling hoist
		Check that the power supply is switched on and connected to the charging station.
No indicator light on hoist	System does not have power or battery is discharged.	Check that the ceiling hoist has power supply and that the battery is recharged.
		Check that the power supply is switched on and connected to the charging station.
		Place the hand control in the charging station and recharge the ceiling hoist.
The emergency strap does not turn hoist off	Hoist requires service.	Contact an Invacare representative for repair.
The emergency strap does not lower the hoist.	Hoist requires service.	Contact an Invacare representative for repair.

 If problems are not remedied by the suggested means, contact an Invacare representative.

7.2 Indicator light and audio signal

Hoist

Status	Indicator light	Audio signal
Off / Stand by	Off	No
Hoist traversing (Robin® Mover only)		
Hoist in use (Up/Down)	Blue	No
Low battery	Blue Flashing	Yes

The blue indicator light on the underside of the hoist turns off automatically after 30 seconds.

Wall Mounted Charger

Status	Indicator light	Duration
Battery not connected	Yellow	n/a
Initializing and analysis of battery	Yellow	12–14 seconds
Fast charge	Red	5 min — 3 hours
Charged (trickle charge)	Green	n/a
Fault condition (battery defect)	Alternating red and green	n/a

7.3 Checking Loose Strap Function

A correctly adjusted loose strap switch ensures that the straps stop lowering when they touch an obstruction.

1. With the ceiling hoist remaining on the track, check there are no obstructions under either strap hook.
2. Place an obstruction under one of the strap hooks, then press and hold the down button on the hand control, Expected result: When the strap hook touches the obstruction, the strap stops lowering.
3. If the strap does NOT stop lowering, adjust the loose strap switch. Refer to chapter 6.9 *Adjusting the Strap Switches*, page 17.
4. Repeat steps 2 and 3 for the second strap.

7.4 Checking Emergency Lowering End Stop Strap Function

 The function described on this section is exclusively applicable to newer product models.

A correctly set emergency lowering end stop strap switch ensures the straps stop lowering when the end course is reached, the movement is not reversed and the straps do not start an upward motion.


1. Pull the red emergency lower cord. Expected result: Both straps lower until the end of their course and stop.
2. If the straps do NOT stop lowering, adjust the emergency lowering end stop switch. Refer to chapter 6.9 *Adjusting the Strap Switches*, page 17.

7.5 Checking Twin Speed Functionality

1. With the hoist on the track and both straps free from obstructions, press and hold the up button with no load applied. Note speed (40 mm/s) and sound of motor.
2. Whilst continuing to hold the up button, apply a load of at least 10 kg (e.g. by pulling down on both straps). Note the speed reduction (30 mm/s) and motor sound change.
3. If there is no speed reduction, replace the twin speed switch. Refer to chapter 6.10 *Replacing the Twin Speed Switch*, page 20.

7.6 Taking Voltage Readings
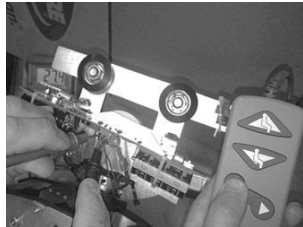
<p>Read Feeding Voltage</p> <p>Take battery voltage feeding PCB.</p> <p>Expected result: 23 to 29 V</p>	
<p>Take voltage feeding main switch (two middle pins)</p> <p>Expected result: 23 to 29 V</p>	
<p>Take voltage feeding the lift motor whilst pressing the UP or DOWN button on the hand control.</p> <p>Expected result: 23 to 29 V</p>	
<p>Take voltage feeding the hoist mover PCB. (Robin® Mover only)</p> <p>Expected result: 23 to 29 V</p>	
<p>Take voltage feeding the copper rings whilst pressing the LEFT or RIGHT button on the hand control. (Robin® Mover only)</p> <p>Expected result: 23 to 29 V</p>	

<p>Read Feeding Voltage</p>	
<p>Take voltage feeding the trolley mover PCB whilst pressing the LEFT or RIGHT button on the hand control. (Robin® Mover only)</p> <p>Expected result: 23 to 29 V</p>	

<p>Read Feeding Voltage</p>	
<p>Take voltage feeding the trolley mover motor whilst pressing the LEFT or RIGHT button on the hand control. (Robin® Mover only)</p> <p>Expected result: 23 to 29 V</p>	
<p>Take voltage feeding the solenoid whilst pressing the LEFT or RIGHT button on the hand control. (Robin® Mover only)</p> <p>Expected result: 23 to 29 V</p>	

8 Technical Data

8.1 The rail system

All rail profiles are manufactured in extruded aluminium.

The rails are available with white powder coating.

The rails are available in three different heights; S= 68 mm, M=114 mm and L=183 mm, providing a free span of 2 m, 4 m and 8 m, respectively.

The profile width of all three profiles is 58 mm. The top of the profiles are shaped to permit the coupling of the rails to the installation brackets.

On delivery, the rail for the hoist will feature a milled groove on its bottom. This groove is used for hoist installation and is called keyhole.

Possible installations

Single rail can be installed on the ceiling or onto the wall and floor. The rail position in the room can be either parallel to a wall or angular in relation to a wall.

The traverse system can be installed on the ceiling or onto the wall and floor. The free span between the two parallel rails can be up to 8000 mm.


Curves are available in the S-profile and can be installed on the ceiling. They are available in angles of 30°, 45°, 60° and 90°.

Special solutions

Installation with pendling brackets is used in the following cases: When the height of the ceiling varies in the lengthwise direction of the rail, or when the rail is to be countersunk.

Angular installation is used when the rail has to be installed in other angles than 90°.

8.2 Dimensions and weight

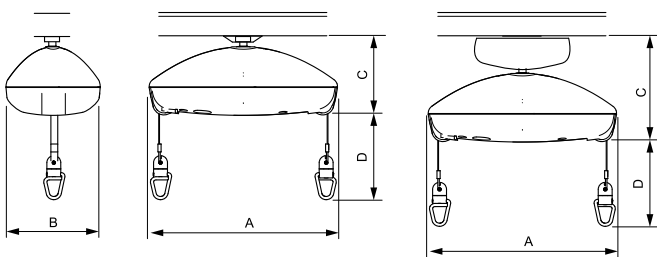Dimensions

	Robin®	Robin® Mover
Hoist length (A)	470 mm	470 mm
Hoist width (B)	250 mm	250 mm
Hoist height (C)	203 mm	266 mm
Maximum lifting range (D)	2.5 m	2.5 m

Weights

	Robin®	Robin® Mover
Maximum lifting capacity (Safe Working Load)	200 kg	200 kg
Total weight without sling	7.3 kg	10 kg

8.3 Electrical System

	Robin®	Robin® Mover
Maximum current input	0.9 A	
Protection class ¹	Entire device: IPx4 (ceiling hoist: IP24; hand control: IP44)	
Insulation class	Class II equipment	
	Type B applied part Applied part complying with the specified requirements for protection against electrical shock according to IEC60601-1.	
Sound level	50 – 55 dB (A)	
Number of lifts per charge	65 lifts of 0.5 m with 80 kg	
Intermittent (periodic motor operation)	10%, max. 2 min. / 18 min.	
Battery	NiMH Battery 24 V / 1.9 Ah	
Charging time	Up to 3 hours at 20°C	
Lifting speed with 200 kg load	2.1 cm/s	2.1 cm/s
Traverse speed in the tracks	n/a	14 cm/s

Charger

Voltage output	During charging: 25 – 36 V DC No-load: 41 V DC
Voltage supply	100 – 240 V AC, 50 – 60 Hz
Protection class ¹	IP41 or IP67 (depending on version)

¹ See product label and label on each electric device for correct protection class. The lowest IP classification decides the overall classification of the device.

- IPx4: Protected against water splashed from any direction.
- IP24: Protected against objects larger than 12.5 mm and protected against water splashed from any direction.
- IP44: Protected against objects larger than 1 mm and protected against water splashed from any direction.
- IP41: Protected against objects larger than 1 mm and protected against vertically dripping water.
- IP67: Dust-tight and protected against temporary immersion into water.

8.4 Environmental conditions

	Storage and transportation	Operation
Temperature	-30 °C to +50 °C	+5 °C to +40 °C
Relative humidity	10 % to 70 %	20 % to 90 % at 30 °C, not condensing
Atmospheric pressure	700 hPa to 1060 hPa	

Allow the product to reach operation temperature before usage:

- Warming up from minimum storage temperature might take up to 24 hours.
- Cooling down from maximum storage temperature might take up to 24 hours.

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Invacare AB
Invacare UK Operations Limited