Invacare® EC-Track®

Installation & Technical Description, Installation og teknisk beskrivelse, Installation et description technique, Montering og teknisk beskrivelse, Montering och teknisk beskrivning

| | | |
|----|--|-----|
| en | Rail system for ceiling hoist Robin™/ Robin Mover™ Service Manual | 3 |
| da | Skinnesystem til Robin™/Robin Mover™ løftemotor Servicemanual | 35 |
| fr | Système de rails pour lève-personne sur rail Robin™/Robin Mover™ Manuel de maintenance | 65 |
| nn | Skinnesystem for takmontert Robin™-/ Robin Mover™-løfter Servicehåndbok | 99 |
| sv | Skensystem för taklyften Robin™/Robin Mover™ Servicemanual | 131 |

©2023 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™]and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

| | |
|---|-----------|
| 1 General | 4 |
| 1.1 Symbols in this Manual | 4 |
| 1.2 General Safety | 4 |
| 1.3 Warranty Information | 4 |
| 2 Rail system | 5 |
| 2.1 Single rail system | 5 |
| 2.2 Traverse system | 6 |
| 2.3 Room Transfer | 7 |
| 2.4 Rails, Brackets & Accessories | 8 |
| 3 Installation | 14 |
| 3.1 General Installation Instructions | 14 |
| 3.2 Ceiling Mounting System | 14 |
| 3.2.1 Concrete Ceiling | 14 |
| 3.2.2 Concrete Ceiling with Suspended Ceiling | 15 |
| 3.2.3 Wooden Ceiling | 17 |
| 3.3 Wall installation – end hung rail | 18 |
| 3.3.1 Rail profile M and L | 18 |
| 3.4 Wall installation – side hung bracket or side/end hung bracket | 19 |
| 3.5 Curves | 19 |
| 3.6 Rail coupling | 20 |
| 3.6.1 Rail coupling for profile M and L | 20 |
| 3.6.2 Rail coupling for profile S | 20 |
| 3.7 Wall support legs | 20 |
| 3.8 Mounting traverse system | 21 |
| 3.9 Installation of track switches | 22 |
| 3.10 Transit coupling | 22 |
| 3.10.1 Adjusting the hole in the wall/door opening | 22 |
| 3.10.2 Installing the transit coupling | 23 |
| 3.10.3 Installing and connecting control button and power supply | 24 |
| 3.10.4 Installing a traverse system with lock fixture for transit coupling | 24 |
| 3.10.5 Installation of a single rail for a coupling system | 24 |
| 3.10.6 Mounting the positioning fixtures on parallel rail S-profile and M/L-profiles | 24 |
| 3.11 Installing the ceiling hoist to the rail system | 25 |
| 3.11.1 Installing Robin® | 25 |
| 3.11.2 Installing Robin® Mover | 26 |
| 3.12 Inspection / check of the rail system | 26 |
| 3.13 Installation Checklist EC-Track System, Gantry and Robin | 27 |
| 3.14 Periodic Inspection Checklist EC-Track System, Gantry and Robin | 30 |
| 4 Technical Data | 33 |
| 4.1 The rail system | 33 |
| 4.2 Dimensions for different track installations | 33 |

1 General

1.1 Symbols in this Manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

DANGER
Indicates a hazardous situation that will result in serious injury or death if it is not avoided.

WARNING
Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION
Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE
Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Identifies required tools, components and items which are needed to carry out certain work.

Other Symbols

(Not applicable for all manuals)

UKRP UK Responsible Person
Indicates if a product is not manufactured in the UK.

Triman
Indicates recycling and sorting rules (only relevant for France).

1.2 General Safety

WARNING!

Safety rules before installation

- All installations must comply with the national rules and standards.
- Use only approved installation components.
- At every point of suspension, the roof/ceiling structure must be able to withstand a static load of at least 300 kg.
- Before mounting a rail system, the roof, walls, and floor must be examined carefully. Determine the exact materials involved in the roof and walls. This examination must only be carried out by a suitably qualified person.
- The rail system must be mounted only by suitably qualified persons.

Stress Test

In accordance with the European standard EN 10535, the system shall be stress tested with 300 kg. This should be done at each fixing point. This test is to be carried out as a testing of the substratum's ability to keep the rail system in place. This test shall be carried out by a suitably qualified person.

1.3 Warranty Information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located inside the back cover of this manual.

2 Rail system

2.1 Single rail system

A single rail system is suited in situations where the lifting is done at specific or limited places (e.g. underneath the track).

A single rail system consists of one straight rail. This rail may be mounted parallel to the walls or diagonally in the room.

The rail system can be optionally extended with curves with angles of 30°, 45°, 60° and 90°.

The rail system can be installed on the wall or the ceiling.

With a single rail system it is possible to hide the installation in the ceiling so that only the opening of the rail is visible.

2.2 Traverse system

A traverse system is suited for cases when an unlimited number of lifting places is required in the room.

A traverse system consists of two parallel rails mounted on the ceiling or the wall. In each parallel rail, a traverse trolley is inserted which can travel the full length of the parallel rails, from end stop to end stop.

Across the parallel rails a traverse rail is mounted to the two traverse trolleys. This installation can be carried out in different ways, as shown in the illustrations below. With a hoist trolley inserted in the traverse rail, the hoist can travel the full length of the traverse rail.

The system permits an unlimited number of lifting places, since the traverse system covers the entire room.

Combinations of traverse systems

| Rails | Combination | Possible combinations |
|--|---|-----------------------|
| Parallel: S-Profile Traverse: S-Profile | Under-hung traverse: Between-hung traverse: | |
| Parallel: S-Profile Traverse: M-Profile | Under-hung traverse: Partly between-hung traverse: | |
| Parallel: S-Profile Traverse: L-Profile | Under-hung traverse: | |
| Parallel: M-Profile Traverse: M-Profile | Under-hung traverse: Between-hung traverse: Partly between-hung traverse: | |
| Parallel: M-Profile Traverse: L-Profile | Under-hung traverse: | |

| Rails | Combination | Possible combinations |
|--|---|-----------------------|
| Parallel: L-Profile Traverse: L-Profile | Under-hung traverse: Between-hung traverse: Partly between-hung traverse: | |

2.3 Room Transfer

Room transfer with two hoists

Room transfer can be done by using two Robin hoists; one in each room. To ease this operation, an Invacare room-to-room trolley is used.

Room transfer is possible in both single- or room covering installations. It is important that the rails are located as close to the doorway as possible. When installing a room covering rail system intended for room transfer, the traverse rail must be installed perpendicular to the wall with the doorway. The distance between the two end stop buffers must not exceed 600 mm. To obtain the best possible result during room transfer, there must be a minimum distance of 1.85 m between the floor and the

underside of the hoist. This must be kept in mind during installation of the rail system.

Room transfer with transit (transit coupling)

As an alternative the room transfer can be achieved by an installed transit coupling.

The coupling can be used to connect either two traverse systems or one traverse system and one single rail system.

The coupling is operated electrically by a control button mounted on the wall. The coupling is delivered with a standard length of 800 mm.

For further information see chapters 2.4 *Rails, Brackets & Accessories*, page 8 and 3.10 *Transit coupling*, page 22.

2.4 Rails, Brackets & Accessories

Rail profiles S, M, L

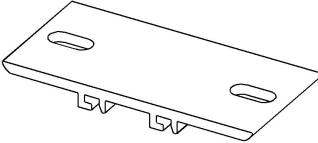
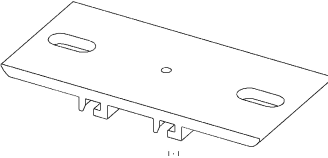
| Type | | Description | Info |
|----------------|--|---|--------------|
| Rail profile S | | Max. length = 7.8 m Max. span without support: 2 m | color: white |
| Rail profile M | | Max. length = 7.8 m Max. span without support: 4 m | |
| Rail profile L | | Max. length = 7.8 m Max. span without support: 8 m | |

Curves 30°, 45°, 60°, 90°


| Type | | Description | Info |
|-----------|--|--|--|
| Curve 30° | | Rail profile S with curve 30° and 2 x 200 mm straight rail | All curves have a radius of 400 mm. All curves are supplied as right or left curves. Characteristics for right and left, see chapter 3.5 Curves, page 19 |
| Curve 45° | | Rail profile S with curve 45° and 2 x 200 mm straight rail | |
| Curve 60° | | Rail profile S with curve 60° and 2 x 200 mm straight rail | |
| Curve 90° | | Rail profile S with curve 90° and 2 x 600 mm straight rail | |

Ceiling brackets




| Type | | Description | Info |
|----------------------------------|--|---|---|
| Ceiling bracket, Quick | | Ceiling brackets with hidden fixing parts. To be used for ceiling installation of single-rail and traverse systems. Sizes (WxDxH): <ul style="list-style-type: none"> • 30x68x14 mm (for curves) • 70x68x14 mm (for ordinary ceiling suspension) • 120x68x14 mm (for rail coupling) | Installation: See chapter 3.2 Ceiling Mounting System, page 14. |
| Ceiling bracket, Quick, Traverse | | Brackets for traverse system and side hung wall installation The bracket is mounted beneath the traverse trolley, at the underhung and partly between-hung traverse assembly. | |

| Type | | Description | Info |
|------------------------------------|---|--|------|
| Ceiling bracket, Standard |  | <p>Ceiling brackets with visible fixing parts</p> <p>To be used for ceiling installation of single-rail and traverse systems.</p> <p>Sizes (WxDxH):</p> <ul style="list-style-type: none"> • 30x146x14 mm (for curves) • 70x146x14 mm (for ordinary ceiling suspension) • 120x146x14 mm (for rail coupling) | |
| Standard ceiling bracket with lock |  | <p>Ceiling bracket with lock screw measuring (70x146). Used for ceiling installation, with min. 1 per straight rail. This ensures that the rail is not displaced after installation.</p> <p>Sizes (WxDxH):</p> <ul style="list-style-type: none"> • 70x146x14 mm | |

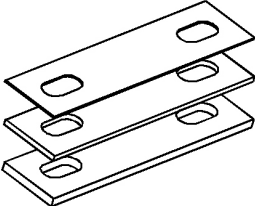
Shims

| Type | | Description | Info |
|-----------------------------------|---|--|--|
| Shim for standard ceiling bracket |  | <p>Available for ceiling brackets in 30, 70, 120 mm width.</p> <p>Height:</p> <ul style="list-style-type: none"> • 1 mm • 3 mm • 5 mm | <p>Shims are to be used in installations to compensate possible differences of the height in the roof/ceiling structure.</p> |

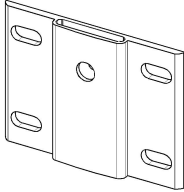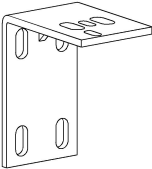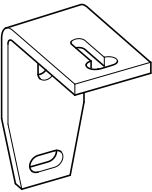
Wall brackets

| Type | | Description | Info |
|---------------------------------|---|---|--|
| Wall bracket, end hung |  | To be used when installing tracks to a wall. They can be used when the walls have sufficiently high carrying capacity when mounting straight and diagonally to a wall. To be used with insert tubes and console. | Installation: See chapter 3.3 <i>Wall installation – end hung rail, page 18.</i> |
| Wall brackets, side hung |  | To be used when installing tracks side hung. Can be applied if the wall has sufficient strength. | Installation: See chapter 3.4 <i>Wall installation – side hung bracket or side/end hung bracket, page 19</i> |
| Wall bracket, side and end hung |  | To be used when installing tracks either side or end hung. Can be applied if the wall has sufficient strength. | Installation: See chapter 3.4 <i>Wall installation – side hung bracket or side/end hung bracket, page 19</i> |


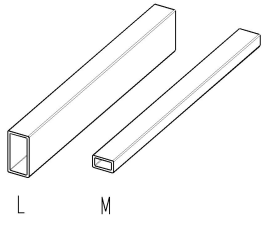
Insert tubes

| Type | | Description | Info |
|-------------------------------------|---|--|---|
| Insert tube for rail profile M or L |  | To be used when mounting rail end hung to wall, when coupling two rails, or when mounting rails diagonally in a room. <ul style="list-style-type: none"> • Insert tube for rail profile M: Length 800 mm. • Insert tube for rail profile L: Length 1 m. | Insert tube is supplied depending on the particular mounting situation. |

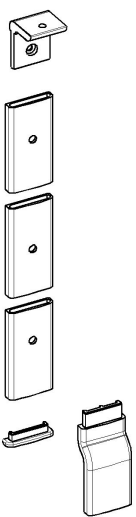Wall support leg, console and wall support foot




| Type | | Description | Info |
|----------------------|---|--|---|
| Wall support leg |  | To be used if the wall does not have sufficient carrying capacity when mounting the rail to the wall. May be used for the mono rail systems as well as the traverse systems. Leg length: 2.6 m / 3 m | Installation: See chapter 3.7 <i>Wall support legs, page 20</i> |
| Wall support console | | The console is also used together with the end hung wall bracket when mounting straight or diagonal. | |
| Wall support foot | | Low foot for rooms without skirting board High foot for rooms with skirting board pan along the floor | |

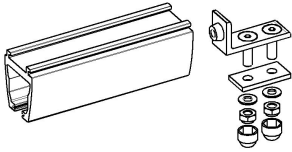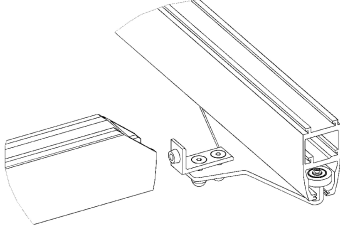End Stop


| Type | | Description | Info |
|----------------------------|---|--|------|
| End stop |  | <p>The end stop is mounted at all rail outlets, which ensures that the trolleys do not leave the rail system.</p> <p>When the end stop is mounted, the rubber plug is to face towards the trolley.</p> | |
| end stop for traverse rail |  | <p>When the traverse system is between-hung, the traverse rails end stop is to be mounted in the traverse trolley.</p> <p>When the rails are mounted to a wall, the end stop is to be placed so far into the rail that the hoist does not hit the wall</p> | |

- i** In some countries/local areas there is a demand for double securing of the end stops. Such extra securing can be achieved by installing one of the following:
- M5x16 mm thread forming screw (e.g. Bufab/Swebolt MRX or equal), 10 mm from each end of the rail, 25 mm from the rail underside (the screw can be installed in a pre-drilled hole Ø4,3 mm).
 - Ø6,3x22 mm thread forming screw (e.g. Bufab/Swebolt R6B or equal), 10 mm from each end of the rail, 18 mm from the rail underside.

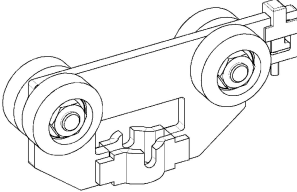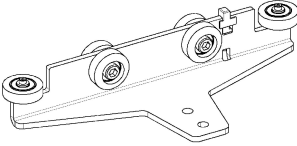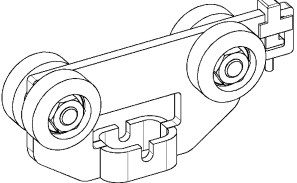
Trolleys

| Type | | Description | Info |
|----------------------|---|--|------|
| Hoist Trolley |  | <p>The hoist trolley is used for individual mono rail systems as well as traverse systems. The hoist trolley is used when a Robin™ ceiling hoist is to be mounted in the track system.</p> | |
| Traverse Trolley |  | <p>The traverse trolley is used when mounting a traverse system. For a traverse system, 2 traverse trolleys are always used.</p> | |
| Room-To-Room Trolley |  | <p>The room-to-room trolley is used when a transfer from one room to another is required, typically through a doorway.</p> | |

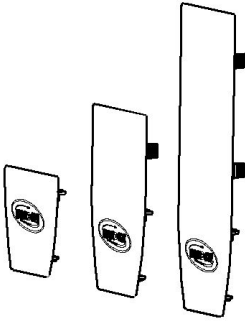
Rail End Plugs

| Type | | Description | Info |
|-----------------------|---|---|------|
| Rail end plug S, M, L |  | <p>Rail end plugs are supplied for rail profiles of 3 sizes. Color: white.</p> <p>To be used at the closure of both ends of all rails.</p> <p>Mounting: Press into the rail profile after the rail system is mounted.</p> | |

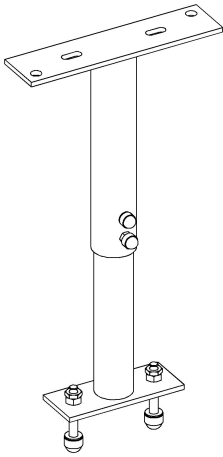
Pendles





| Type | | Description | Info |
|---------------------|--|---|---|
| Pendles, adjustable |  | <p>Pendles are applied if there is a need for installing a lower rail system than the original ceiling level. Can be used in hidden installation as well as visible installations.</p> <p>Sizes (adjustable length):</p> <ul style="list-style-type: none"> • 200 – 300 mm • 280 – 460 mm • 440 – 740 mm | <p>Installation: See 3.2.2 <i>Concrete Ceiling with Suspended Ceiling, page 15.</i></p> |

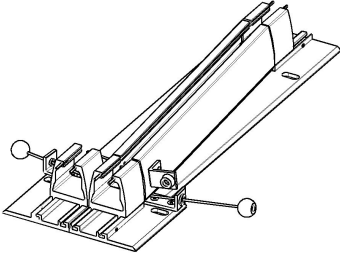Track Switch

| Type | | Description | Info |
|----------------------|---|--|---|
| Track switch, manual |  | <p>The track switch is manually operated and is installed with a standard ceiling bracket. The track switch is used in combination with a ceiling installed single rail system with S-profile, where a change in direction is required during movement of the patient.</p> | <p>Installation: See chapter 3.9 <i>Installation of track switches, page 22</i></p> |

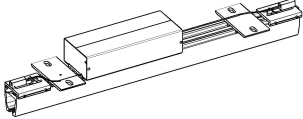Transit Coupling

| Type | | Description | Info |
|------------------|---|---|---|
| Transit coupling |  | <p>The transit coupling is operated electrically and mounted with pendles in the ceiling on both sides of the wall. Length: 800 mm.</p> <p>The coupling is used for movement of a patient from one room to another.</p> <ul style="list-style-type: none"> • in traverse systems with S-, M,- and L-rails. | <p>Control button (for the wall), and a transformer are enclosed when delivered.</p> <p>Installation: See chapter 3.6 <i>Rail coupling, page 20</i></p> |
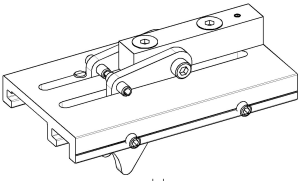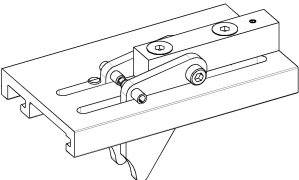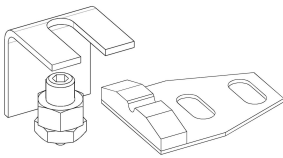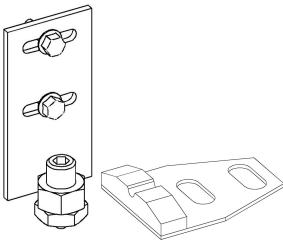
| Type | | Description | Info |
|--|--|---|------|
| | | <ul style="list-style-type: none"> in a single rail system with only S-profile rails. | |
| Lock fixture for traverse profile S |  | To be mounted on the traverse profile in the end that is coupled. It is mounted to ensure that the traverse rail and coupling are locked and connected correctly before room transfer can be done. | |
| Lock fixture for traverse profile M/L |  | To be mounted on the traverse profile in the end that is coupled. It is mounted to ensure that the traverse rail and coupling are locked and connected correctly before room transfer can be carried out. | |
| Positioning fixtures for parallel rail S-profile |  | To be mounted on the parallel rail, close to the coupling, to position the traverse rail correctly in proportion to the coupling. | |
| Positioning fixtures for parallel rail M/L-profile |  | To be mounted on the parallel rail, close to the coupling, to position the traverse rail correctly in proportion to the coupling. | |

3 Installation

3.1 General Installation Instructions

Before installing a rail system, a careful and accurate measuring of the room is essential. It is equally important to determine the condition of the ceiling, the walls and the floor in order to decide whether to mount the rails on the ceiling or on the walls. Next, you need to decide how many lifting places are needed in the room.

When defining the lifting places, an individual assessment is required, which takes into consideration the build of the individual user and number of nurses.

The following placement of typical lifting places may be used as a recommended guideline:

| | |
|----------------|--|
| Bed | 1 m from inside of the bed head end |
| Toilet | 150 mm from the front edge |
| Bathtub | 1/3 way up (sitting position), ½ way up (reclining position) |
| Changing table | 1 m from the head end of the table |

When measuring for the selected rail system, take into consideration a suitable parking place for the hoist when it is not in use. A suitable spot could be near a wall, where the hoist charger is mounted on the wall.

Check the customized rail as well as the brackets and other accessories against the drawing.

Check that any burrs and metal shavings have been removed from the rails to prevent this debris from getting stuck in the trolley wheels producing noise and erratic operation.

In case of single rails and curved rails needing coupling, use the enclosed pipe pins.

1. Push the pipe pins halfway into the one rail. Holes are drilled to $\varnothing 3.2\text{mm}$ in the rail that the previous mentioned rail is to be connected to. Then the next rail is inserted over the pipe pins.
2. Push the next rail into the protruding pins. Do not drill further holes in the first rail.
3. Mount the bracket for rail coupling.

When installing a traverse system where the parallel rails are running from wall to wall, the minimum distance from rail to wall must be 60 mm, insert the traverse trolley and the rail end stop into the parallel rails before installation. If not, the parallel rails must end min. 350 mm from the wall to permit subsequently installation of the trolley.

When installing a mono rail system or a traverse rail in a traverse system, it is important that the rail at one end stops min. 160 mm from the wall to permit subsequently installation of the hoist trolley.

At secondary installation of Robin Mover™ the distance must be minimum 300 mm.

When installing rails with brackets directly on the ceiling, make sure that the ceiling is even and horizontal. (Shims of respectively 1 mm, 3 mm and 5 mm thickness, may be used to level out any unevenness.)

Keyhole for the installation point

Any EC-Track system must be prepared with an installation point for the hoist.

This installation point appears as a 32 mm round milled groove on the bottom of the rail in which the hoist is to be installed. This milled groove is called keyhole.

When installing the rail system, it is crucial that this installation point is placed according to the instructions below.

1. Generally, the keyhole must always be placed as close as possible to the end of a rail. However, it must be at least 250 mm away from the rail end.
2. If the rail goes from wall to wall, it should be placed min. 400 mm from the rail profile end.
3. If a traverse system is mounted, the keyhole should be placed as close as possible across the end of the rail. However, it must be at least 250 mm away from the rail end.
If the traverse rail continues all the way out to the walls, it should be placed min. 400 mm from the end of the rail.

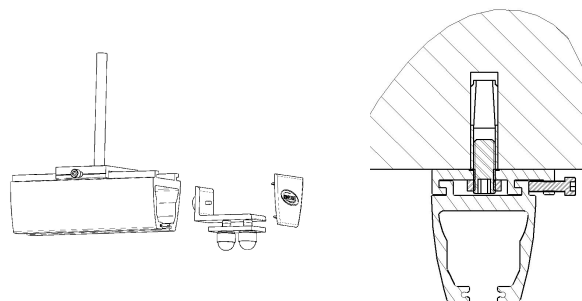
3.2 Ceiling Mounting System

3.2.1 Concrete Ceiling

When referring to mounting in concrete ceilings, we differentiate between massive concrete (factory moulded concrete elements, prestressed concrete surfaces, armored concrete) and hollow surfaces.

Mounting with Quick Ceiling Bracket

When using quick ceiling fittings, all ceiling fittings are to be mounted in the ceiling. Then the rail may be mounted in these. At least 3 ceiling fittings shall be used per rail. There must subsequently be max. 700 mm between each fitting. There must be max. 200 mm from the end of the rail to the first fitting.

Measure up to all fittings at the same time. It is important that the drilled holes are on a straight line. The precise marking for the holes is to be carried out using laser technique, chalk line, etc.

Mounting in massive concrete and hollow surfaces:

1. Drill holes with a 15 mm hammer drill to a depth of 54 mm. Blow holes clean.
2. Insert M12 stroke anchors into the hole (e.g. Hilti HKD-S M12 or equal).
When using a setting tool (e.g. Hilti HSD-G M12x50 or equal), the anchor will be correctly fastened in the concrete.
3. Mount the quick fitting by screwing the pre-mounted threaded rod in the stroke anchor.
It is important that an adhesive is used on a fitting and nut when the threaded rod is mounted to fitting.

 In order to ensure the achievement of max. strength, the threaded rod should be screwed min. 14 mm into the anchor.

When using a stroke anchor in combination with the quick bracket, optimum accuracy is a requirement. As an alternative, two component adhesive mortar for concrete can be recommended.

When all fittings are mounted, push the hoist trolley into the rail. The hoist trolley may face both ways. If the traverse trolley is to be mounted into the rail, it is important that this faces correctly in connection to the following mounting of the traverse rail.

The end stop is mounted in both ends of the rail. The end stop is pushed into the rail with the rubber plug first, and fastened initially loosely with your fingers.

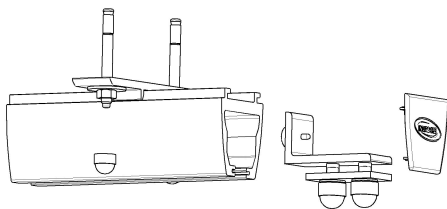
Lift up the rail, adjust the length of it and fasten it into all ceiling fittings.

Place the adjustable end stops correctly and tighten appropriately. It is important that the end stop is placed so that the hoist does not hit the wall.

For parallel rails in a traverse system, it is important that the end stop is placed so that the traverse trolleys hit the end stop at the same time.

Mounting with Standard Ceiling Bracket

The ceiling brackets are pushed onto the rail and distributed with max. 2 m between each bracket and 200 mm from the last bracket at the end of the rail. Use at least three brackets per rail. It is recommended to use an uneven number of brackets, if only one person is installing the rail. Thus, one bracket is placed on the center creating balance when the rail is hung in the first bolt.

 In at least one end of the rail, use a bracket with locking screw to be secured to the rail.

1. Measure for the hole for the one expansion bolt of the center bracket.
Drill the hole to a depth of approx. 80 mm with an 8 mm. impact drill bit. The hole is deeper than the expansion bolt length to facilitate dismounting later on. For dismounting, you can then drive the bolt fully in instead of pulling it out.
2. Drive an expansion bolt M8x75 (e.g. Hilti HSA or equal) in, leaving sufficient room for the bracket thickness (6 mm), the locking washer (2 mm) and the nut (7 mm) plus a little allowance. When you have driven in the bolt, remove the nut and the washer.
3. Insert the hoist trolley in the rail. The trolley can be oriented in both directions.
4. If you are installing the traverse trolley, make sure that it is correctly oriented for the subsequent installation of the traverse rail.
5. Install end stops at both ends of the rail. Insert the adjustable end stops in the rail with the rubber plug side first and the steel plate towards to the bottom of the rail. Finger tighten the screws.
6. Lift the rail in place, mount the washers and nuts and tighten them lightly. Adjust the rail lengthwise and angle wise, and tighten the nut.
7. Drill the hole for the bracket in the one end of the rail but on the opposite side of the rail. Drive in the expansion bolt, make the final adjustments and tighten.
Next, drill the remaining holes at one end of the bracket slot. If the drill bit hits a reinforcement bar before the hole is (50 mm) deep, simply drill at the other end of the slot. If the drill hits the reinforcement bars again, move the bracket approx. 20-25 mm along the rail. The bracket will then cover the failed holes.
 If the hole depth is between 50-70 mm, use an extension bolt M8x57.
8. Drive in the remaining expansion bolts, place the washers and nuts and tighten.
9. Position the adjustable rail stops correctly and tighten. Make sure to position the end stops so that the hoist does not hit the wall.
When installing parallel rail in a traverse system, be sure to position the end stops at the exact same points to ensure that the traverse trolley hit the end stops simultaneously.

3.2.2 Concrete Ceiling with Suspended Ceiling

For rooms with suspended ceilings which are difficult to dismount, it is recommended to choose a solution with wall-mounted or wall support mounted rails.

If the false ceiling can be easily removed or in case of a new mounting of a fixed suspended ceiling, we recommend an installation with lowering brackets for aesthetic reasons. For this solution, use the lowest rail profile S.

Mounting with Pending Brackets

In the case of ceiling mounting, it may often be necessary to pendle the rail downward. This need may arise in the even of mounting in a room with a suspended ceiling, where you have to pendle between the original ceiling and the suspended ceiling, or if the ceiling is so high that you have to pendle the rail down in order to ensure that the hoist's lifting intervals are used as well as possible.

There are more possibilities when carrying out a mounting with pendles:

A – Pending using threaded rods and round spacer pipes, cut on-site at the installation place. Mounting by using this method is described in detail in this section.

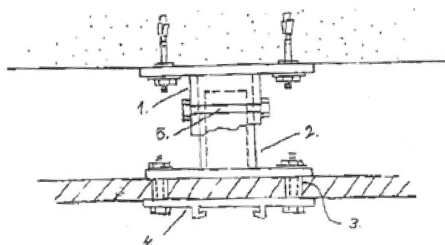
B – Mounting using adjustable pendles. This pendle consists of an upper and lower part. By pushing the upper part onto the outside of the lower part, the adjustable pendle may cover a given interval. As standard, a pendle can be supplied with the following intervals:

- 200 – 300 mm
- 280 – 460 mm
- 440 – 740 mm

When the correct pendle spacing has been determined in connection with the assembly, the pendle is pulled out to this length, and fastened with a screw. Then a hole is drilled all the way through to an M8x65 bolt to lock the pendle in the desired length.

The upper part of the pendle should be mounted in the ceiling as described in chapters 3.2.1 *Concrete Ceiling*, page 14 and 3.2.3 *Wooden Ceiling*, page 17 as when mounting with standard bracket.

The lower part of the pendle should be attached to the final standard ceiling fitting, as shown in the sketch below:


- 1. Pendle, upper part
- 2. Pendle underpart
- 3. Spacer pipe through lowered ceiling, Ø15
- 4. Standard ceiling bracket
- 5. Through locking bolt, M8x65

The pendle may also be supplied with predetermined dimensions, avoiding adjustment on-site.

 Remember also to order ceiling brackets when ordering pendle.

Removable Suspended Ceiling

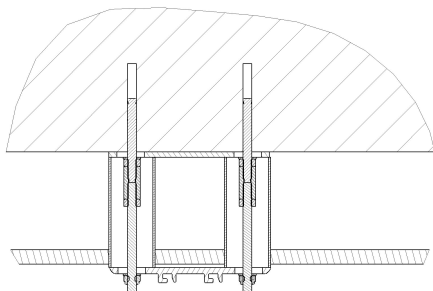In case of a removable suspended ceiling, the space above the suspended ceiling is accessible and pendle brackets and rail can be installed at the same time.


- Concrete ceiling.
- Expansion bolts M8x75, 2 pcs.
- Yoke (Steel plate for even installation surface), 1 pcs. 3mm.
- M8 nuts, 4 pcs.
- M8 coupling nuts, 2 pcs. (Collecting sleeve).

- M8 locked nuts, 2 pcs.
- Aluminium pipes Ø40, 2 pcs.
- Ceiling bracket (standard), 1 pcs.
- Removable suspended ceiling.
- M8 threaded rods, 2 pcs.
- Locking washers, 2 pcs. 11x35x2.


(An HKD anchor can be used as an alternative, which the rods are fixed directly into).

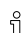1. To get the right length of the pending brackets, use the distance between the concrete ceiling and the lower face of the suspended ceiling as your starting point. This distance is used to determine the length of the pendle bracket pipe and the threaded rod. Their length must equal the previously measured distance described above.
2. When installing, first install the yoke for the center ceiling bracket. To do this, follow the instructions for the installation directly in a concrete ceiling. In this case, however, the holes are drilled for both expansions bolts at the same time.
3. When driving in the expansion bolts, leave approx. 15 mm protruding below the nut, so that you can screw the coupling nut onto it to half its length. Counter-tighten the coupling nut against the expansion bolt nut. Measure for the hole in a suspended ceiling and drill/cut this hole to Ø40 mm.
4. Attach an M8 nut onto the threaded rod (approx. 20 mm onto the rod) and screw the rod into the coupling nut until it hits the expansion bolt and counter-tighten the nut.
5. After completion, push the two aluminium pipes into the holes and fasten the rail as described in the usual instruction instructions.

 The distance between the suspended ceiling and the ceiling bracket must be min. 1-5 mm to prevent damages to the false ceiling.

Fixed Suspended Ceiling

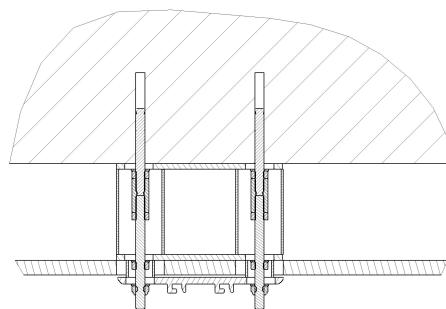In case of a fixed suspended ceiling, the rail installation is carried out in two steps. The lowering bracket therefore needs to be secured without the support from the ceiling bracket.

- Concrete ceiling.
- Expansion bolts M8x75, 2 pcs.
- Yokes, (Steel plate for even installation surface), 2 pcs. 3mm.
- M8 nuts, 6 pcs.
- M8 coupling nuts, 2 pcs.
- M8 locking nuts, 2 pcs.
- Aluminium pipes Ø40, 2 pcs.
- Distance pipes Ø20 (or nuts and washers), 2 pcs.
- Ceiling bracket (standard), 1 pcs.
- Suspended ceiling.

- M8 threaded rods, 2 pcs.
- Locking washers, 2 pcs.

(An HKD anchor can be used as an alternative, which the rods are fixed directly into).

1. Cut the pendle bracket pipe to a length equalling the distance measured between the concrete ceiling and the lower face of the suspended ceiling minus 50 mm. This length is appropriate for a double gypsum layer in the ceiling and an extra allowance of 12 mm.
2. Cut the threaded rod to a length equalling the distance between the concrete ceiling and the lower face of the suspended ceiling minus 10 mm.
3. When installing, first install the top yoke for the center bracket of the rail. Install the yoke as described in the installation instructions for direct installation in a concrete ceiling. In this case, however, drill both holes for the expansion bolts at the same time.
4. When driving in the expansion bolts, leave approx. 15 mm to protruding below the nut, so that you can screw the coupling nut onto it to its half length. Counter-tighten the coupling nut against the expansion bolt nut.
5. Attach a nut approx. 20 mm onto the threaded rod and screw the threaded rod into the coupling nut, until it touches the expansion bolt, and counter-tighten the nut.
6. Attach the bottom yoke and secure it with nuts.

Avoid paint on the bottom section of the threaded rods until the installation is complete.

7. Prepare the rest of the installation points after a careful and accurate measuring.
First step of the installation is now complete and the construction of the suspended ceiling can be completed.

In the final installation, position the distance pipe in the suspended ceiling. This pipe will protect the suspended ceiling. Cut the distance pipe to a length creating a distance of 1-5 mm between the lower face of the suspended ceiling and the upper face of the installation bracket; instead of pipes, nuts and washers can be used to ensure the recommended distance.

3.2.3 Wooden Ceiling

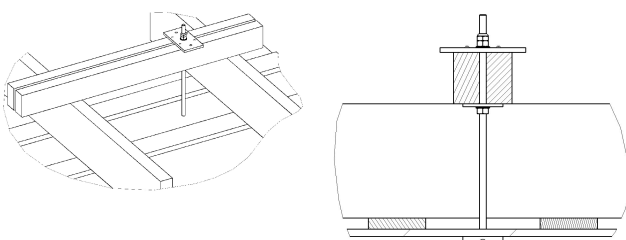
CAUTION!

- Do not install rails to a wooden ceiling with tension screws, such as coach screws.
- Do not install rails to a wooden ceiling with plain wood screws, without having performed a risk assessment.

Quick Bracket

In some cases, mounting to a ceiling timber work may be an alternative to mounting to wall or wall support.

It is provided that the ceiling timber work is accessible from above. When being mounted in the ceiling timber work, exchange should be placed as reinforcement. When reinforcing, it is important to select exchange wood that is dimensioned to tolerate the distance between two ceiling beams, so sufficient carrying capacity is achieved. When having marked up to all ceiling brackets in the ceiling, you should examine whether there is free space above the ceiling material. When this is done, holes with a diameter of $\varnothing 12.5$ mm are drilled through the ceiling material. The holes may now be localized from above, they can be prepared for the exchanges, and you may measure how long the threaded rods should be. The exchange should cover at least two ceiling beams on each side of the suspension point. When using exchanges, two 45x95 mm exchanges should be placed parallel to each other, at a distance of 14 mm, providing a passage for an M12 threaded rod. For determining the length of the threaded rod, determine the ceiling beam's height + length of exchange wood + ceiling material + 60 mm for washer and nuts.

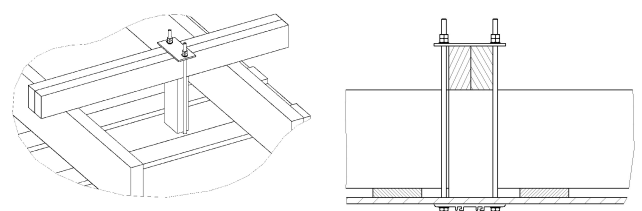
Place the adapted threaded rod between the exchange wood and put in a washer and nut before the threaded rod is guided further down through the ceiling and position it with an aluminium plate, a tightening washer and nut over the exchange wood. The aluminium plate is screwed into the exchange wood from the top. Let approximately 25 mm threaded rod protrude down under the ceiling material. A quick ceiling bracket may now be screwed onto the threaded rod beneath the ceiling material. The ceiling bracket is fastened with an M12 low nut. The threaded rod is to be screwed entirely through the nut, but must protrude further down than below the fitting.

When mounting the bracket and nut, it is important to use adhesive on bracket as well as nut. When all brackets are mounted, lift up the rail, adjust the length and secure it. Before the rail is fastened up to the ceiling, which is done from above the ceiling, it should be checked that the exchange wood is resting on both ceiling beams. If not, something should be placed between.

The nut over the exchange wood is now secured carefully until the bracket lies evenly with the ceiling material. It is important not to tighten so much that the ceiling material is pulled up between the ceiling beams. When this is done, a steel plate is placed in around the threaded rod below the exchange wood, and the nut is secured up into the steel plate, thus locking the position.

Finally, a lock nut is secured against the nut on the upside of the exchange wood. Trolleys, end stop and rail end plugs are mounted as described in 3.2.1 *Concrete Ceiling*, page 14.

Standard Bracket

- Threaded rods M10, 2 pcs.
- M10 nuts 4 pcs.
- Yoke 1 pcs.

- Braces 45x90x1500 mm, 2 pcs.
- Beam layer.
- Under roof/ceiling.
- Ceiling bracket, 1 pcs.
- Locking nuts M10, 2 pcs.
- Washer, 2 pcs. 11x35x2.

In some cases, installation with passing screws may be an alternative to wall or wall support installation.

This requires that the roof beam layer can be accessed from above. This is often the case in one-storied houses. To reinforce the roof beam layer, exchanges of 45x95 mm are placed edgewise.

Instead of the expansion bolts, a M10 threaded rod of appropriate length is used. An installation plate (yoke) rests on top of the braces.

The exchange must rest on the roof beams on both sides of the suspension. If this is not possible, the exchange must be secured to prevent them from tilting when loaded.

The minimum number of three mounting points per rail for concrete ceiling does not apply to wooden ceilings. Here, the max. free span determines the number of mounting points.

The rest of the installation process is similar to the process for concrete ceilings. It is, however, necessary to drill both holes for the center bracket (the standard bracket) at the same time.

Cut the threaded rods to appropriate length and attach the washers and M10 nuts locking nuts.

The threaded rods are to be screwed from below and up through the yoke. The yoke is fastened from above to the exchanges with nails and screws.

Before the rail is fastened to the ceiling, which is done from above, you should double-check that the exchange wood is resting on two ceiling beams. If not, something should be placed between.

Be careful when tightening the nuts to prevent damage to the suspended ceiling.

3.3 Wall installation – end hung rail

The following are recommendations only. Alternatively, expansion bolts with the similar thickness and quality can be applied. Always follow the recommendations from your supplier.

For freely suspended installation, the end of the rails may be attached to the wall with wall brackets or wall support. Choose wall support, if you are in doubt of the load bearing capacity of the wall material. When installing freely suspended rails, the max. rail length must be observed, depending on the selected rail profile. The distance between the walls must be max. 2 m for rail profile S, 4 m for rail profile M and 8 m for rail profile L.

If you need to mount or dismount a hoist trolley after installation, the distance from one wall to the rail must be at least 160 mm. At secondary installation of Robin Mover™, the distance must be minimum 300 mm. If you need to mount or dismount the traverse trolley after installation, the distance between the one wall and the rail must be at least 350 mm. (Minimum distance from rail to wall must always be 60 mm). If you have a wall-mounted rail profile S, you cannot mount or dismount the trolley after installation.

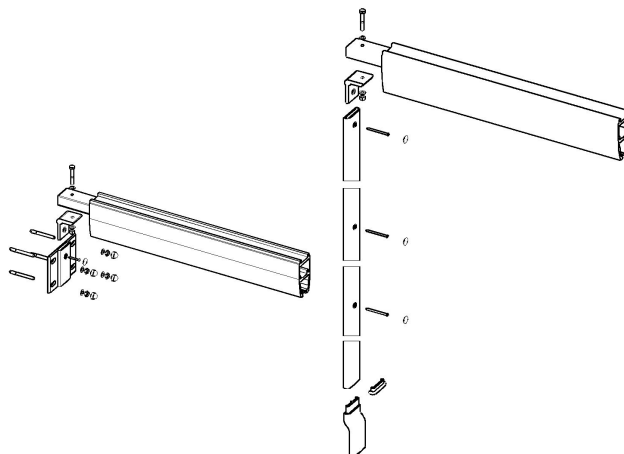
3.3.1 Rail profile M and L

When installing wall brackets or wall support legs, leave a gap from the ceiling to the upper edge of the wall bracket or floor support. The gap should be at least:

- 80 mm for rail profile M
- 150 mm for rail profile L

When using wall support legs, cut the support leg profile to an appropriate length at the floor end.

 Remember to make allowance for the floor support foot.

In case of skirting boards, either use the high wall support foot, or cut away the skirting board.

Always install wall support legs right at the wall in their full length.

Mark for holes using a level, and drill the holes.

Concrete wall

When mounting a freely hanging rail into a concrete wall, a wall bracket should be used for mounting.

For mounting, we recommend using either expansion bolts M10x68 (e.g. Hilti HSA or equal), drilling depth min. 60 mm, Ø10, or adhesive mortar (e.g. Hilti HIT-HY 150 and HAS M10 or equal), drilling depth min. 95 mm, Ø12 (when mounted in wet rooms, use stainless steel HAS quality A4-70).

Brick wall

When mounting into bricks, ordinary wall brackets should be used. However, we recommend using floor supports if the brick wall is very porous.

For mounting into massive bricks, we recommend using adhesive mortar (e.g. Hilti HIT-HY 50 and HAS M10 or equal), drilling depth min. 90 mm in ½ brick and 170 mm in 1/1 brick, Ø12., (when mounted in wet rooms, use stainless steel HAS quality A4-70).

For mounting in hollow bricks, we recommend using either a frame shear connector (e.g. Hilti HRDUGT or equal), drilling depth min. 80 mm, Ø10, or adhesive mortar (e.g. Hilti HIT-HY 20, and HAS M10 or equal), drilling depth min. 95 mm, Ø16.

For mounting into porous brickwork, we recommend using adhesive mortar (e.g. Hilti HIT-HY 50 and HAS M10 or equal), drilling depth min. 90 mm in ½ brick and 170 mm in 1/1 brick, Ø12. If wall supports are used when mounting, use HAS M8, drilling depth min. 80 mm, Ø10.

Lightweight concrete

When mounting in light concrete, you may either use wall brackets or wall supports, depending on the condition of the wall. For mounting, we recommend using adhesive mortar (e.g. Hilti HIT-HY 50 and HAS M10 or equal), drilling depth min. 90 mm, Ø12 (M8 for mounting of wall supports).

You may also use a light concrete sheer connector (e.g. Hilti HGN, HUD-1 10L or equal), drilling depth min. 90 mm, Ø10 / HUD-1 8L, drilling depth min. 80 mm, Ø8.

Gypsum wall

Refer also to chapter 3.7 *Wall support legs, page 20*.

1. Complete the installation of the wall bracket or the wall support leg with console, wall support foot and plugs for covering the screw holes.
2. Insert the hoist trolley or the traverse trolley in the rail. The hoist trolley can be oriented in both directions, whereas the orientation of the traverse trolley must correspond to the layout of the traverse system.
3. Insert end stops at both ends of the rail and finger-tighten them at first.
4. Insert the insertion tube in the rail. The hole in the tube must face outwards.
5. Lift the rail while holding on to the insertion tube to prevent it from sliding into the rail.
6. Mount the insertion tube on the console at the wall bracket or the wall support leg.
7. Adjust the rail lengthwise to achieve the correct distance between the wall and the rail.
8. There must be an overlap of at least 300 mm between the rail and the insertion tube. Check to verify by looking through the inspection hole on the side of the rail. The inspection hole is located exactly 300 mm from the end of the rail.
9. When the rail is properly positioned, drill a Ø7 mm hole through the rail and the insertion tube, approx. 100 mm from the end of the rail. Secure the rail in position with passing bolts M6 with locking nuts.
10. After positioning the rail, adjust the end stop to its proper position. When positioning the end stop, make sure that the hoist will not hit the wall. With a traverse system, ensure that the two traverse trolleys hit the end stop simultaneously.

Angular, wall-mounted

The rail can be mounted diagonally in the room. The rail must be angled max. 45°.

The mounting is carried out as described for ordinary wall mounting with or without wall supports. However, the tube is delivered prepared for angled mounting.

3.4 Wall installation – side hung bracket or side/end hung bracket

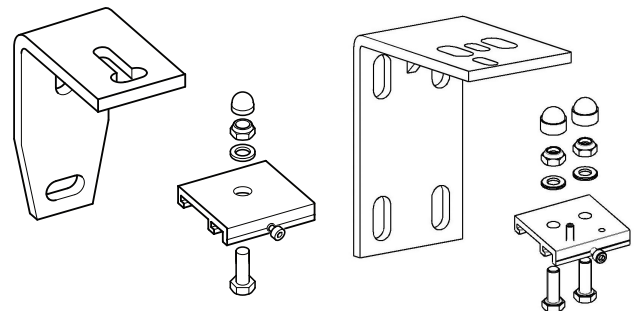
When installing a rail system side hung, the parallel rails can be installed close to the wall.

When installing a rail system side hung, the wall must have such strength, that wall support legs are unnecessary.

When installing **S** profile, 2000 mm is the maximum distance between each fixation point, but at least 3 points are necessary.

When installing **M** profile, 4000 mm is the maximum distance between each fixation point, but at least 3 points are necessary.

When installing **L** profile, 8000 mm is the maximum distance between each fixation point, but at least 3 points are necessary.

Always measure the distance between both walls accurately, in the correct height and location of the system. The wall bracket can be adjusted +/- 6mm. If more is required, use a plate of 6 mm between bracket and wall.

 Final appearance is influenced by the parallelism between wall and rail. Be accurate.

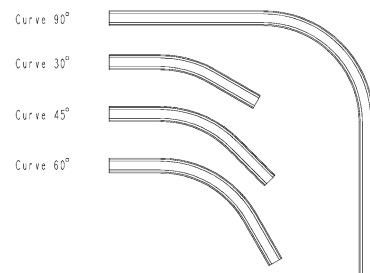
The traverse system measurements here, is the room size measured, deducted 206 mm (if between-hung installed), and 62 mm (if under-hung installed).

Again, this requires an accurate measurement as well as some considering of the adjustability due to wall unevenness.

Wall installation depends on the structure of the wall. Always follow recommendations in chapter 3.3.1 *Rail profile M and L, page 18*. After installing the wall brackets and the two parallel rails, follow instructions in section 3.8 *Mounting traverse system, page 21*.

3.5 Curves


Following curves are available:

- Angle 30° 0.2 + 0.2 m straight ends
- Angle 45° 0.2 + 0.2 m straight ends
- Angle 60° 0.2 + 0.2 m straight ends
- Angle 90° 0.6 + 0.6 m straight ends

All curves have a radius of 400 mm.

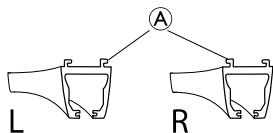The curves are available with right-hand bends and left-hand bends. It is especially important to distinguish between right and left curves, when several curves are included in the same rail system.

- Left curves (L): Mounting channels (A) facing out from the curve.
- Right curves (R): Mounting channels (A) facing inward into the curve.

Pipe pins and coupling brackets with lock screws are used to couple curves with straight rails.

Bracket positions on the curves

Angle 30° 0.2 + 0.2 m straight ends: A coupling bracket at both ends, if the curve is connected to straight rails.

Angle 45° 0.2 + 0.2 m straight ends: A coupling bracket at both ends, if the curve is connected to straight rails and a narrow bracket at the middle of the curve.

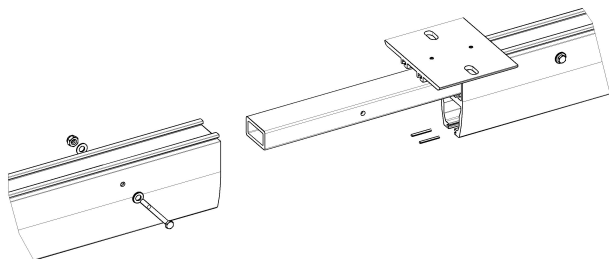
Angle 60° 0.2 + 0.2 m straight ends: One coupling bracket, at both ends if the curve is connected to straight rails and a narrow bracket at the middle of the curve.

Angle 90° 0.6 + 0.6 m straight ends: One coupling bracket, if the curve is connected to straight rails and a narrow bracket at the middle of the curve.

For installation instructions for curves, see 3.2.1 *Concrete Ceiling, page 14* and 3.2.2 *Concrete Ceiling with Suspended Ceiling, page 15*.

3.6 Rail coupling

3.6.1 Rail coupling for profile M and L

The following items are used for rail coupling:

- Rail profile M no. 1.
- Rail profile M no. 2.
- Insertion tube (for profile M).
- Coupling bracket (quick or standard).
- Pipe pins.
- Stop screws.
- Bolt M6.
- Washer.
- M6 lock nuts.

Push the insertion tube halfway into rail profile no. 1 and secure it with a passing bolt. Furthermore, drill the holes for the pipe pins in rail profile no. 2 up to 3.2 mm in a depth corresponding to half the length of the pipe pins. Guide the pipe pins halfway into rail profile no. 1 and squeeze them together in the end.

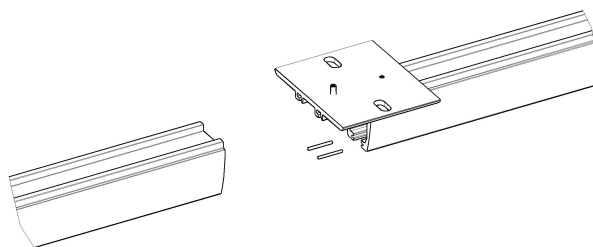
Push the coupling bracket halfway onto rail profile no. 1 and secure it with the stop screw.

The coupling bracket is only required when mounting the rail on the ceiling. If you are coupling a traverse rail, the coupling bracket is not required. Push together rail profile no. 2 and no. 1 and secure them with the locking screw in the coupling bracket and the passing bolt.

Install the coupled rail using the instructions for installation of straight rail profiles, see 3.2 *Ceiling Mounting System, page 14*.

When coupling long rails, it can be difficult to lift the coupled rail in one piece. In this case, first install the first rail on the ceiling. Then lift up the next rail which has been prepared for coupling and push the rails together.

3.6.2 Rail coupling for profile S

The following items are used for rail coupling:

- Rail profile no. 1
 - Rail profile no. 2
 - Coupling bracket (quick or standard)
 - Pipe pins
 - Stop screws
1. Drill the holes for the pipe pins in rail profile no. 2 up to 3.2 mm in a depth corresponding to half the length of the pipe pins.
 2. Drive the pipe pins halfway into rail profile no. 1 and squeeze them together in the end.
 3. Push the coupling bracket halfway onto rail profile no. 1 and secure it with the stop screw.
 4. Insert rail profile no. 2 into the coupling bracket and drive the two rails together.
 5. Tighten the stop screw on rail profile no. 2.
 6. Install the coupled rails using the instructions for installation of straight rail profiles.

NOTICE!

When coupling long rails, it can be difficult to lift up the coupled rail in one piece.
– In this case, first install the first rail on the ceiling. Then lift the next rail which has been prepared for coupling and push the rails together.

3.7 Wall support legs

Wall support legs are used for installation on gypsum walls and walls of other lightweight plate material.

- Wall support leg profile (adjust length).
- Support console.
- Wall support foot.
- Plastic cover plugs.
- Screws M6.

The wall support leg is cut in the right length, in the end facing against the floor.

When measuring the profile length, include a minimum distance between the top of the bracket and the ceiling. Minimum distance to be observed:

- M-profile: min. 80 mm
- L-profile: min. 150 mm

Mount the wall support leg on the wall with three screws. Use Ø6 mm screws. The wall material determines the choice of screws.

 For mounting into gypsum, we recommend using (e.g. Hilti HGA or equal) shear connector for cavities (e.g. Hilti HGA or equal).

In case of skirting boards, either use the high wall support foot or cut the skirting boards.

! **NOTICE!**

- Remember to make allowance for the wall support foot.
- Always install wall support leg directly on the wall in their full length.

3.8 Mounting traverse system

The parallel rails are mounted in ceiling or wall as described in the previous section on the current mounting conditions.

Either if the two parallel rails are mounted using quick or standard fittings, it is important that any deviation of parallelism of the two rails does not exceed +/- 2 mm.

When mounting parallel rails, it is important to take into account whether it must be possible to insert a traverse trolley later on or not. If not, the rails may run all the way out to the wall (minimum distance from rail to wall must always be 60 mm). In this case it is important that the traverse trolley and end stop are pushed into the rail before the rail is mounted. However, if it is possible to insert the trolley later on, it is important that there is a minimum distance between the one end of the rail and the wall of 350 mm.

Remember that the end stop in the parallel rails is to be placed so that the two traverse trolleys hit the end stops simultaneously.

The distance between two parallel rails is always measured from center to center of the two rails.

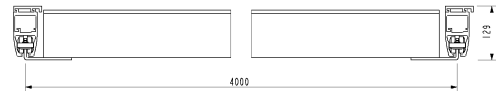
Generally, the following applies to the traverse rail with reference to the free span between the parallel rails as well as the max. protrusion a rail can handle.

- Rail profile S: max. free span 2 m, max. protrusion 200 mm
- Rail profile M: max. free span 4 m, max. protrusion 400 mm
- Rail profile L: max. free span 8 m, max. protrusion 500 mm

 When a rail is mounted as partially between-hung, it is always the max. protrusion for the S-profile that is applicable.

When mounting the traverse rail, there are three possible methods:


The traverse rail is between-hung


The measurement for the traverse rail is always the distance between the two parallel rails minus 86 mm.

When the traverse rail is mounted between-hung, the hoist trolley is inserted in the traverse rail and the end stops are placed loosely at both ends of the traverse rail. Remember that the rubber plug on the end stop always should face inward toward the hoist trolley. Lift the rail on top of the traverse trolley, so that the end stop's 2 bolts align with the two holes in the traverse trolley, then secure it with washers and lock nuts. Before the rail is securely fastened, it is important that the distance between the traverse rail and each of the parallel rails is equal in both ends. When the rail is secured, install the cover plugs on lock nuts and in the rails.

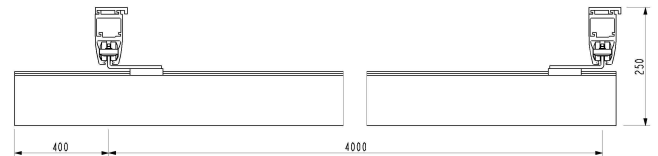The traverse rail is under-hung

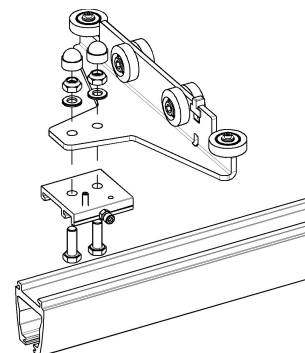When the parallel rails are mounted and the traverse trolleys and end stops are positioned and secured, install quick traverse brackets underneath the traverse trolley, as shown on the drawing.


Regardless whether the two traverse trolleys are facing opposite each other or the same way, it is important that the two quick traverse brackets face the same way, or else there will be problems when mounting the rail.

When the traverse rail is correctly placed lengthwise and secured in the quick traverse bracket with the Allen screw, the rail is locked using a lathe center screw, which is

screwed into the quick traverse bracket from above until it has been screwed all the way down.

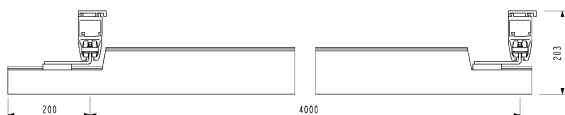The traverse rail is partly between-hung


When the traverse rail is partly between-hung, the same mounting principle is used as with under-hung traverse rail. Here, the rail will be cut upon delivery, so that the quick traverse bracket may be mounted to the rail's middle suspension point.

Remember before mounting the traverse rail to determine whether it should be possible to mount the hoist trolley later or not.

3.9 Installation of track switches

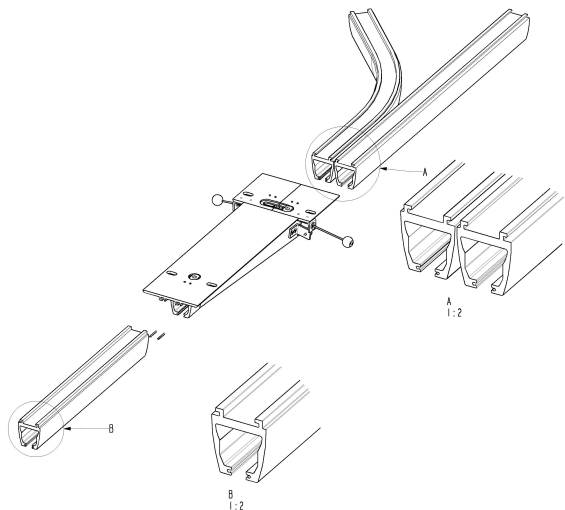The track switch is used where a change in direction is required during movement of the patient. The track switch is manually operated. By coupling of the track switch with both straight rails as well as curved rails, different combinations can be obtained.

The track switch can only be used in combination with ceiling installation and S-profile rails. The track switch is designed for standard ceiling bracket.

During ordering or installation of the straight or curved single rail system, it is important to bear in mind in which direction the mounting dowels of the track switch are facing. Only one standard track switch is available - refer to the illustration.

By installation in concrete ceiling, refer to chapter 3.2.1 *Concrete Ceiling, page 14* (standard brackets), and chapter 3.6.2 *Rail coupling for profile S, page 20*.

A re-bore in the rails is performed when coupling the rails. This re-bore is done in the track switch at delivery, and the necessary tube pins are supplied. When installing the rails to the switch, remember to tighten the pre-installed locking screws on the track switch - refer to the illustration. In connection with installation of a track switch onto different ceiling types or in combinations with pendles, contact Invacare® for further information.

NOTICE!

– After installation, the functions of the track switch and the fastening of the end stops must be inspected.

3.10 Transit coupling

The transit coupling is used in situations where transportation from one room to another is required, using only one Robin motor. The coupling is operated electrically.

The transit coupling can be installed in different rail system combinations. The coupling can be used to connect either two traverse systems or one traverse system and one single rail system.

The coupling is delivered with a standard length of 800 mm, and can be used where the wall is, up to 170 mm thick. The coupling is delivered with one control button, one power supply with 2 meters of wire (which must be adjusted during installation), and a user manual.

If a transit coupling is needed in situations where the wall is thicker than 170 mm, contact Invacare.

When ordering the transit coupling, it is important to order lock fixtures for the rails in both room 1 and 2. When coupling to a single rail system, please see 3.6 *Rail coupling, page 20*. Pendles for the installation are not delivered with the coupling. Order pendles separately in required length for the coupling.

Recommended installation:

1. Transit coupling
2. Traverse system/single rail system
3. Positioning fixtures
4. End test – Function and load test

3.10.1 Adjusting the hole in the wall/door opening

CAUTION!

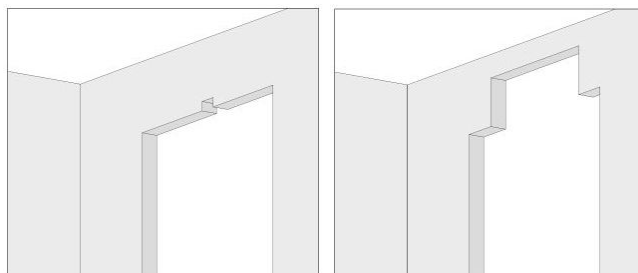
– Before making hole in wall, ensure wall is not load bearing.

The hole must always be placed exactly in the center of the door opening:

If it is required that only the rail must pass through the wall, the hole needs to be minimum 73 mm high and minimum 65 mm wide.

If increased lifting height is required, the hole must be bigger.


For a hole height of more than 80 mm, the width has to be at least 600 mm for the ceiling hoist to pass through.

i The best result is obtained, if the transit coupling has been taken into consideration in the very beginning of the house construction. This way the door opening can be constructed with the optimum lifting height.

3.10.2 Installing the transit coupling

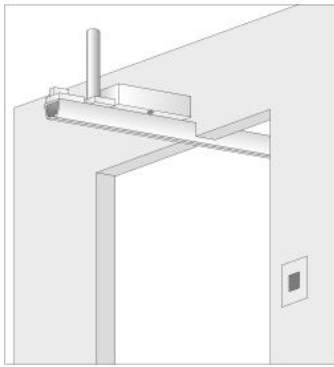
The coupling is delivered assembled with motor box, fixtures for pendles and lock fixtures. The delivered transit coupling is prepared so that the whole motor box is placed in one of the two rooms after installation.

The motor box and lock fixtures, have to be placed as the transit is delivered, to ensure functionality and safety. Fixtures for pendles can be placed freely as long as it complies with the construction of the system. See measure sketch below.


i It is very important that the thickness of the wall is controlled in advance.

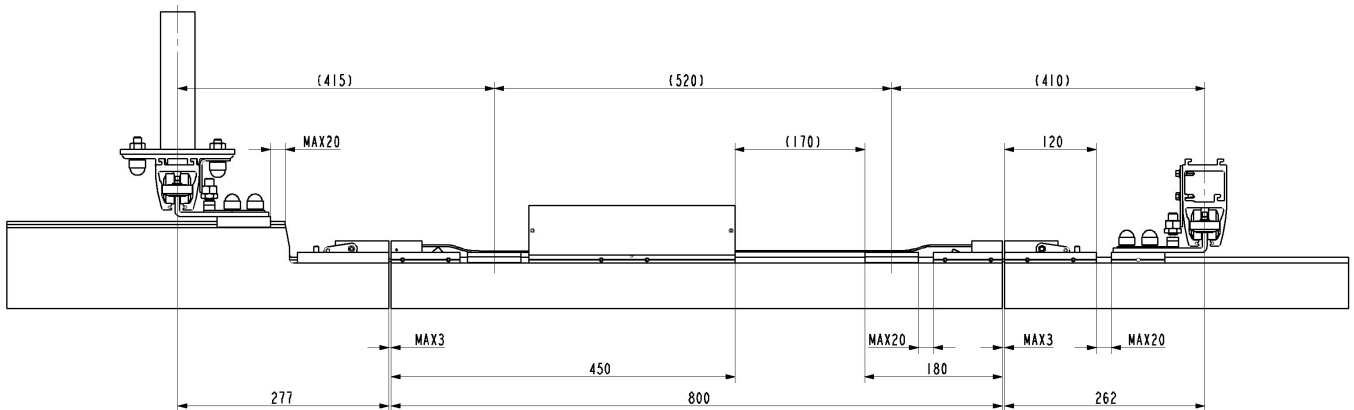Measure sketch for transit coupling

When installing the transit coupling it is very important that the motor box is placed in the room where it was intended when it was ordered, because the entire system is constructed under consideration of where the motor box is placed.

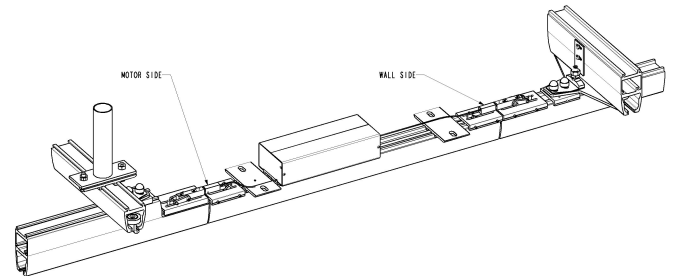Installation with pendles is described in chapter 3.2.2 *Concrete Ceiling with Suspended Ceiling, page 15.*

See the measure sketch below for further information.

Installation of the transit coupling is only possible using pendles. The coupling can therefore only be installed where ceiling mounting is possible.

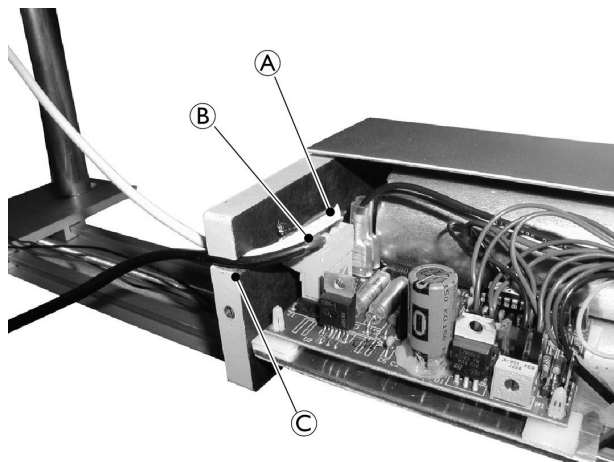
The best result is obtained in a concrete ceiling.

When installing in wooden ceilings, it is very important that the traverse system and coupling are installed in a way that they bend down equally and with minimum bending.

If it is necessary to strengthen the ceiling in a wooden ceiling construction, make sure to strengthen with a connected construction between rail system and transit coupling.

Do not install the pendles in different rafters because this way the coupling and traverse system will not bend down equally and the coupling will not work correctly.

3.10.3 Installing and connecting control button and power supply

Before connection of the control button and power supply, it is necessary to dismantle the covers on the motor box. The covers are dismantled by loosening the two screws, which holds each cover and pulling the covers horizontally to the side.

The control switch is mounted in the room and at the height, the user prefers. The control switch is opened with tools, and the back side is mounted on the wall using two screws $\varnothing 3.5 \times 30$ mm.

The wire is shortened to a suitable length and connected to the motor box. See drawing pos. A.

The power supply is mounted on the wall, visible or hidden, depending on what is most suitable. The power supply is mounted on a $\varnothing 3.5 \times 30$ mm screw on the wall, using the "key" hole on the back side of the power supply.

At the bottom of the power supply there is hole for a $\varnothing 3.5 \times 30$ mm screw which makes it possible to lock the power supply on the wall. The wire is shortened to a suitable length and connected to the motor box. See drawing pos. B.

The cover is re-attached, when the wires are connected correctly and the control switch is tested. Remember to mount the wires in the end-plate, in order to avoid that the wire conflicts with the cover when it is re-attached. See drawing pos. C.

CAUTION!

It is very important that there are no loose wires hanging from the coupling.

3.10.4 Installing a traverse system with lock fixture for transit coupling

In general a traverse system is installed as shown in chapter 3.8 *Mounting traverse system*, page 21. We recommend that the parallel rail of the traverse system is mounted in the ceiling in order to make the function of the transit coupling as stable as possible.

When installing the parallel rail, remember always to place a ceiling bracket/pendle in front of the transit coupling, especially on the rail closest to the transit coupling. See chapter 3.10.6 *Mounting the positioning fixtures on parallel rail S-profile and M/L-profiles*, page 24.

When the parallel rail is mounted, it is important to ensure a correct distance to the coupling. See image in chapter *Measure sketch for transit coupling*, page 23.

The traverse rail is adjusted to the specific order when it is delivered. If however it is necessary to shorten the rail further, it must be shortened in the end opposite to where the coupling is situated.

It is possible to use the following traverse rails together with a transit coupling system:

- S, M and L- profiles, under-hung
- M and L- profiles partly between-hung

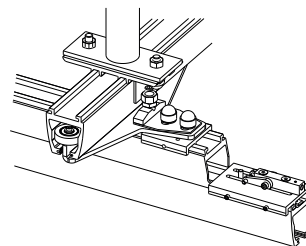
It is very important to ensure the correct free overhang for the traverse rail spacing between the pendles and free span from the rails to ensure that the rails can couple correctly. See drawing above.

When the traverse system and the coupling are installed it is very important to control the height between these and adjust if necessary.

The lower edge of the traverse rail must not differ more than 0.5 mm from the coupling rail. If there is a difference in height, the coupling track must be the lowest.

It may be necessary to adjust the height differently according to the user weight or the condition of the ceiling, to optimize the function of the coupling.

When the lock fixtures are installed, it is very important that they are in line with the end of the traverse rail and that they are turned correctly compared to the fixture on the coupling, before they are locked. See drawing.

Lock fixtures for M- and L-profiles are delivered with a little plate, which is important to install.

3.10.5 Installation of a single rail for a coupling system

When the coupling is installed with a single rail in one of the rooms, it is prepared for this, without the lock fixture, when delivered.

When the coupling is installed with an S-profile rail, the rail is coupled as shown in chapter 3.6.2 *Rail coupling for profile S*, page 20.

The pendle is mounted in the coupling fixture.

The rest of the single rail system is installed as shown in chapters 3.2 *Ceiling Mounting System*, page 14 and 3.6 *Rail coupling*, page 20.

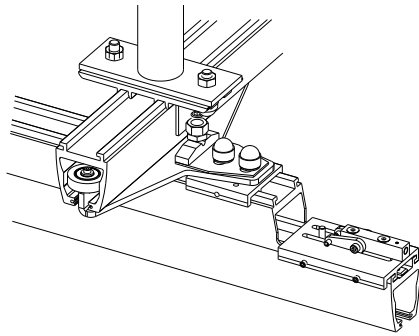
3.10.6 Mounting the positioning fixtures on parallel rail S-profile and M/L-profiles

CAUTION!


It is very important that everything is adjusted correctly and carefully fastened due to safety and functionality.
– Test the system thoroughly after installation.

The positioning fixture is mounted to easily position the traverse rail in proportion to the coupling. The fixture is mounted on the rail closest to the coupling and is turned towards the coupling. See drawing.


The positioning fixture for the S-profile rails is partly mounted in the standard fixture. It is therefore important to place a standard fixture right in front of the coupling rail.

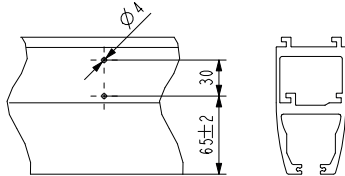The one part of the positioning fixture is mounted in the M10 bolt at the standard ceiling bracket in the same direction as the quick fixture on the traverse trolley. The traverse part of the positioning fixture is mounted on the Quick fixture of the traverse trolley. See drawing.

The positioning fixture for M-/L-profile rails is mounted on respectively the parallel and the traverse rail. The traverse part of the positioning fixture is mounted on the quick fixture of the traverse trolley.

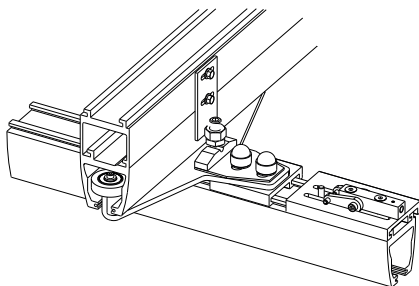
The traverse rail and transit coupling must be coupled when mounting the parallel rail part of the positioning fixture. This way it is possible to tick up for correct position of the fixtures before they are mounted. After the right position is ticked up, the rails are separated again before the $\text{Ø}4$ mm holes for the screws are drilled. See drawing below.

Adjustment of hardness

The positioning fixtures must be adjusted to a hardness that makes it easy for the coupling to catch the traverse rail, but at the same time, makes it possible for the rail to pass the coupling without getting caught by the coupling.

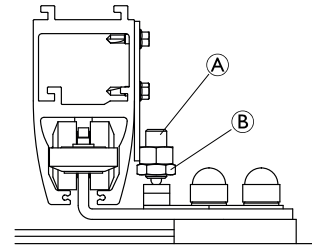1. If the positioning fixture is adjusted too hard, loosen the resilient black screw (A).
2. If the positioning fixture is too soft, tighten screw (A).
3. Tighten lock nut (B) (M12) after any adjustment.

Adjustment of positioning fixtures

The parallel rail part of the positioning fixture can be adjusted further by loosening the screws and moving the fixture horizontally until the rail is right in front of the coupling.

After adjustment the fixture can be fastened.

Note that there is some play, which must be divided between both sides of the rail.

3.11 Installing the ceiling hoist to the rail system

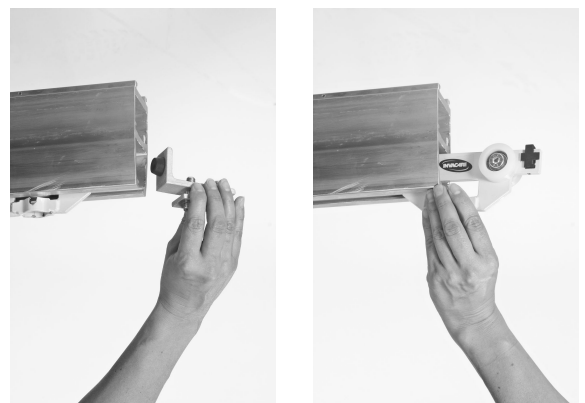
Robin® requires a trolley, guiding the ceiling hoist in the rail system. Robin® Mover has the trolley built-in and can be installed directly to the rail system.

3.11.1 Installing Robin®

Tools: 4 mm Allen key; 17 mm socket wrench

1. To install the trolley, remove the end stop that is attached to the rail system.
 -  In some countries/ local areas, there is a demand for double securing of the end stops. If an extra securing screw is installed, remove this screw first.


2. Insert the trolley into the rail (no matter which way, as the trolley is symmetric).
3. Re-insert the end stop.


In some countries/ local areas, there is a demand for double securing of the end stops. If an extra securing screw is required, install this screw. See also *End Stop*, page 11.

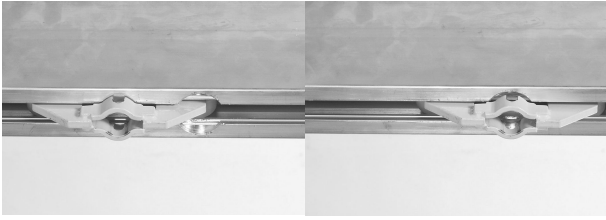4. Locate the hoist trolley at the \varnothing 32 mm round recess in the rail.

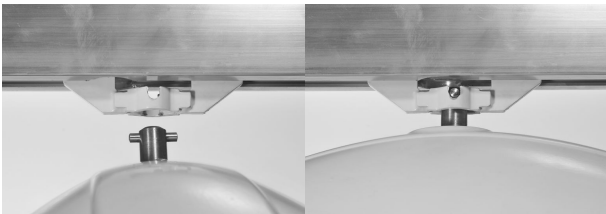
5. Lift the hoist and push the suspension pin through the trolley opening.

6. Turn the hoist by 90 degrees to fix it in the trolley.
7. Connect the hand control.
8. Adjust the length of the emergency stop/lowering cord to within the user's reach.

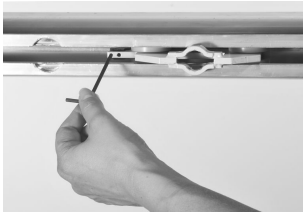 If there is excessive resistance, or if the trolley unintentionally rolls by itself due to imbalance in the rail suspensions, adjust the friction brake using a 4 mm Allen key.

3.11.2 Installing Robin® Mover

The trolley is built-in on the hoist, and can therefore not be pre installed in the rail.

 Tools: 4 mm Allen key; 17 mm socket wrench

1. Remove the end stop.

2. Insert the wheels of the hoist into the rail.

 The direction the hoist faces does not matter during installation.

3. Install the end stop.

 Tighten the bolt to 30 Nm.

4. If too much resistance is experienced when the hoist is pulled, or the hoist moves too freely and unintentionally, adjust the friction brake.

5. Connect the hand control.
6. Adjust the length of the emergency stop/lowering cord to within the user's reach.

3.12 Inspection / check of the rail system

CAUTION!

After the installation of a rail system, the entire system must be inspected.

- The inspection of a rail system must only be carried out by a suitably qualified person.
- The inspection must follow 3.13 *Installation Checklist | EC-Track System, Gantry and Robin, page 27* and 3.14 *Periodic Inspection Checklist | EC-Track System, Gantry and Robin, page 30*.

3.13 Installation Checklist | EC-Track System, Gantry and Robin

The rail system and/or hoist has been checked and tested for the following according to the **Instructions / Installation Manual**:

Is the EC-Track on the installation?

Yes (Complete the following section).

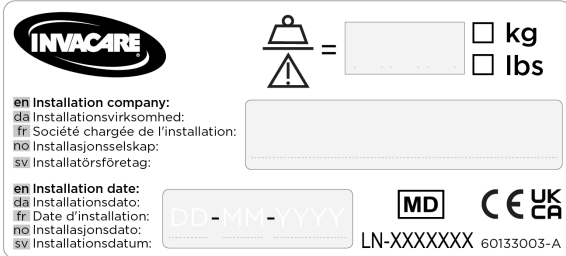
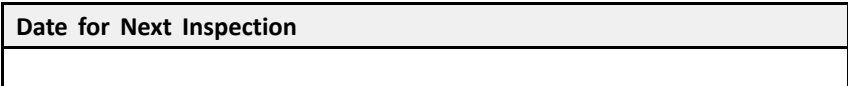
| The Invacare EC-Track System installation label shows the following information: | | | |
|--|--|-------------------|--|
| Maximum SWL | | Installation Date | |
| Label Number (LN) | | | |

Type of installation

| | | | |
|-------------------------------|---|--|---|
| Type of track assembly | <input type="checkbox"/> Straight/ Bend rail(s) | <input type="checkbox"/> Traverse (X-Y System) | <input type="checkbox"/> Room-to-Room |
| Room type | <input type="checkbox"/> Bedroom | <input type="checkbox"/> Bathroom | <input type="checkbox"/> Living Room |
| Room type | <input type="checkbox"/> Other: _____ | | |
| Fixation | <input type="checkbox"/> Ceiling | <input type="checkbox"/> Wall | <input type="checkbox"/> Floor to Ceiling (Wall Post) |
| Fixation | <input type="checkbox"/> Free Standing / Gantry | | |
| Reinforcements | <input type="checkbox"/> No | Yes, performed by: <input type="checkbox"/> Installer <input type="checkbox"/> Another Contractor: _____ | |

No (Proceed to **2. Checkpoints | Robin** and make sure to check the Track's manufacturer **Instructions / Installation Manual**).

| 1. Checkpoints EC-Track and Gantry | ✓ | ✗ | Comments |
|--|---|---|----------|
| 1.1 All necessary reinforcements are securely fastened (if applicable). | | | |
| 1.2 The rail system components are chosen and mounted according to the Installation Manual . | | | |
| 1.3 The brackets are positioned below their rated maximum distance spacings. If the system is the Gantry, the poles are positioned below their rated maximum distance spacing. | | | |
| 1.4 All brackets are tightened correctly to the ceiling/wall. | | | |
| 1.5 The bracket to rail fixations are securely fastened. If the system is the Gantry, all the connections are securely fastened. | | | |
| 1.6 All rail ends are fitted with end stops, sufficiently tightened and, have end covers (if applicable). | | | |
| 1.7 The rails are clean and free from chips or other waste from the installation. | | | |
| 1.8 The trolley (and if applicable, traverse trolleys) run smoothly and silently in the entire track system. | | | |
| 1.9 Adjust the friction brake on trolley, if needed. | | | |
| 1.10 Check traverse trolley mounting fixtures. | | | |
| Transit Coupling / Track Switch (if applicable) | | | |
| 1.11 Lock fixtures are all OK (secure to tracking, each claw moves independently, correctly aligned with tracking slots, no obstructions to prevent correct function, safety plate is fitted). | | | |
| 1.12 Check tracking alignment. Re-align if necessary. | | | |
| 1.13 Gap between track and transit coupling/track switch is max. 3 mm. | | | |
| 1.14 Function of transit coupling / lock fixture OK (10 times: disable left / right claw x5). | | | |

| 1. Checkpoints EC-Track and Gantry | ✓ | ✗ | Comments |
|---|---|---|-----------|
| 1.15 Function test performed under load through the track switch of the system (5 cycles with the maximum SWL). | | | |
| 1.16 Inlay plate is fitted (L and M rail). | | | |
| Load test | | | |
| 1.17 Check if the rail system is leveled (before testing the rail system). | | | |
| 1.18 The rail shall then be loaded with 300 kg – 1,5 × maximum SWL (200 kg) – for 20 minutes.  The load must be suspended under each fixing point for 20 minutes. | | | ISO 10535 |
| 1.19 The rail system remains the same height after the load test. | | | |
| 1.20 The rail system remains leveled after the load test. | | | |
| 1.21 The rail system has been marked with the installation label showing the maximum SWL (200 kg), the installation date and installation company. | | | |
| 1.22 Fill out the required fields on the Installation Label and place it on the track. Cover it with the protective laminate.  <p>The label template includes the INVACARE logo, a weight icon with a box for SWL, and checkboxes for kg and lbs. It also has fields for installation company name in four languages (en, da, fr, no, sv), installation date in four languages (en, da, fr, no, sv) with a date format MM-YY, and a box for the serial number LN-XXXXXXX 60133003-A. It also features CE, UK, and CA certification marks.</p> | | | |
| 1.23 Stick a label with the date for next inspection (MM-YYYY) on the Tracks. Label Example:  <p>The label example shows a box with the text "Date for Next Inspection" and a blank space for the date.</p> | | | |

Is Robin hoist assembled on the EC-Track?

Yes (Complete the following section).

No (Skip to the **3. Final Steps** Section and make sure to check the hoist’s manufacturer **Instructions / Installation Manual**).

| 2. Checkpoints Robin | ✓ | ✗ | Comments |
|---|---|---|----------|
| 2.1 Robin hoist Serial Number (write it on the comments section). | | | |
| 2.2 The hoist is marked with the maximum SWL (200 kg). | | | |
| 2.3 Up switches work correctly. | | | |
| 2.4 Down switches work correctly. Loose strap function works correctly. Check adjustment if it is not working | | | |
| 2.5 End stop works correctly. Check adjustment if it is not working. | | | |
| 2.6 The emergency stop and reset work correctly. | | | |
| 2.7 The emergency lowering function and reset work correctly. | | | |
| 2.8 The emergency lowering end stop switch operates correctly. | | | |
| 2.9 The full length of straps (tapes) have no signs of wear, fraying and/or loose stitching. | | | |
| 2.10 Hooks and hooks’ fixations do not have signs of wear/damage that could endanger the lifting. | | | |
| 2.11 The hand control is connected correctly. | | | |

| 2. Checkpoints Robin | ✓ | ✗ | Comments |
|--|---|---|----------|
| 2.12 The hand control is functional and in good conditions. | | | |
| 2.13 Check suspension and cross pin. | | | |
| 2.14 Perform a full lifting cycle with the maximum SWL (200kg). ⓘ Must be effortless and without jarring sound. | | | |

| 3. Final Steps | ✓ | ✗ | Comments |
|--|---|---|----------|
| 3.1 The owner has all the necessary User Manuals . | | | |
| 3.2 Installation waste has been gathered and adequately disposed of. | | | |

Installation Performed by:

| | |
|---|--|
| Entity responsible for the System: (e.g., Provider, Technical Aid Center, Invacare, etc). | |
| Installation company name: | |
| Installer Name [block letters]: | |
| Installer Signature and Date: | |

ⓘ **Two** signed copies of this Installation Checklist must be completed. One for the Installation Company and one for the manufacturer (Invacare). Please send a copy (scanned document, PDF or legible photo) to ectrackinst@invacare.com with the following name code: **“Installation company name” + “Label Number” + “Installation date”**,

e.g.: **“Invacare LN0000001 18-Oct-2022”**.

3.14 Periodic Inspection Checklist | EC-Track System, Gantry and Robin

In accordance with ISO 10535.

CAUTION!

Risk of injury and damage to property

Failure to ensure an adequate **Periodic Inspection** of the product can compromise the safety of people and property.

- A periodic inspection of the product must be performed at least every **12 months** unless otherwise stated in local requirements.
- The frequency of inspection must be increased if the product is continuously exposed to high humidity, high condensation and corrosives (e.g., chlorine or ammonia gases), to prevent product impairment.
- Inspections **MUST** be performed by a qualified person who is acquainted with the design, use and care of the product.
- The inspection record **MUST** be recorded in the logbook and reported to the owner.

Is the EC-Track on the installation?

Yes (Complete the following section):

| The Invacare EC-Track System installation label shows the following information: | | | |
|--|--|-------------------|--|
| Maximum SWL | | Installation Date | |
| Label Number (LN) | | | |

No (Proceed to **2. Checkpoints | Robin**).

| 1. Checkpoints EC-Track and Gantry | ✓ | ✗ | Comments |
|--|---|---|----------|
| 1.1 The installation label is present, visible and readable. | | | |
| 1.2 All connection points, brackets to structure are tightened with recommended torque. Retighten, if necessary (if NOT accessible please see point 1.21 on load test). | | | |
| 1.3 All connection points, brackets to rail system are tightened with recommended torque. If the system is the Gantry, all the connections are tightened with recommended torque. Retighten, if necessary (if NOT accessible please see point 1.21 on load test). | | | |
| 1.4 All rail ends are secured with end stops, sufficiently tightened. Retighten, if necessary. | | | |
| 1.5 End stops don't show any sign of damage, wear and/or deformation. | | | |
| 1.6 Brackets don't show any sign of damage, wear and/or deformation. If the system is Gantry, connections don't show any sign of damage, wear and/or deformation. | | | |
| 1.7 Trolleys don't show any sign of damage, wear and/or deformation. | | | |
| 1.8 The trolley (and if applicable, traverse trolleys) run smoothly and silently in the entire track system. | | | |
| 1.9 Check and adjust if required the trolley friction brake. | | | |
| 1.10 End Covers are assembled (If applicable). | | | |
| 1.11 Rail systems with track connections: The connections are aligned. Re-align, if needed. | | | |
| Transit Coupling / Track Switch (if applicable) | | | |
| 1.12 Lock fixtures all OK (secure to tracking, each claw moves independently, correctly aligned with tracking slots, no obstructions to prevent correct function, safety plate is fitted). | | | |
| 1.13 Check traverse trolley mounting fixtures. | | | |

| 1. Checkpoints EC-Track and Gantry | ✓ | ✗ | Comments |
|---|---|---|----------|
| 1.14 The tracking is aligned. Re-align, if needed. | | | |
| 1.15 The gap between the track and the transit coupling/track switch is max. 3 mm. | | | |
| 1.16 Function of transit coupling / lock fixture OK (10 times — disable left / right claw x 5). | | | |
| 1.17 Function test performed under load through the track switch of the system (5 cycles with the maximum SWL). | | | |
| 1.18 Inlay plate is fitted (L and M rail). | | | |
| Load test | | | |
| 1.19 The track system is leveled and all fixings are sufficiently tightened before carrying out a load test. | | | |
| 1.20 If all wall and ceiling brackets are accessible, and sufficiently secured a load test must be performed with the maximum SWL (200 kg). The load must be passed through the entire rail system. | | | |
| 1.21 If all wall and ceiling brackets are NOT accessible a load test must be performed with 300 kg – 1,5 × maximum SWL (200 kg). This test must take a minimum of 20 minutes. The load must be suspended under each bracket/fixing point and passed through the entire rail system. | | | |
| 1.22 The rail system remains the same height after the load test. | | | |
| 1.23 The rail system remains leveled after the load test. | | | |
| 1.24 Place a label with the date for next inspection (MM-YYYY) on the Tracks. Label Example: | | | |
| Date for Next Inspection | | | |
| | | | |

Is Robin hoist assembled on the EC-Track?

Yes (Complete the following section).

No (Skip to the **3. Final Steps** and make sure to check the hoist's manufacturer **Instructions / Installation Manual**).

| 2. Checkpoints Robin | ✓ | ✗ | Comments |
|---|---|---|----------|
| 2.1 Product Label is present and legible (write SN on the comments section). | | | |
| 2.2 The hoist is marked with the maximum SWL. | | | |
| 2.3 Up switches work correctly. | | | |
| 2.4 Down switches work correctly. Loose strap function works correctly. Check adjustment if it is not working. | | | |
| 2.5 End stop works correctly. Check adjustment if it is not working. | | | |
| 2.6 The emergency stop and reset work correctly. | | | |
| 2.7 The emergency lowering function and reset work correctly. | | | |
| 2.8 The emergency lowering end stop switch operates correctly. | | | |
| 2.9 The full length of straps (tapes) have no signs of wear, fraying and/or loose stitching. | | | |
| 2.10 Check if the end stop strap switch is working correctly (via hand control). Adjust if necessary. | | | |

| 2. Checkpoints Robin | ✓ | ✗ | Comments | | |
|---|---------------------------------|---|----------|--|--|
| 2.11 Hooks and hooks' fixation (screws and pins) do NOT have signs of wear/damage that could endanger the lifting. | | | | | |
| 2.12 Check suspension and cross pin. | | | | | |
| 2.13 Steel and plastic gearwheels do NOT have signs of wear and damage. If gearwheels are dry, apply gearwheel grease. | | | | | |
| 2.14 Check tape guide cylinders. | | | | | |
| 2.15 The hand control is connected correctly. | | | | | |
| 2.16 The hand control is functional and in good conditions. | | | | | |
| Electrical Connections | | | | | |
| 2.17 All cables are intact. | | | | | |
| 2.18 No signs of rupture on cables and on isolation. | | | | | |
| 2.19 Clamping and mechanical function of micro switches work correctly. | | | | | |
| 2.20 The charger does not have signs of damage and is functional. | | | | | |
| 2.21 The emergency lowering end stop switch works correctly, if applicable. | | | | | |
| Load Test | | | | | |
| 2.22 Perform a full lifting cycle with the maximum SWL (200kg).  Must be effortless and without jarring sound. | | | | | |
| 2.23 Stick a label with the date for next inspection (MM-YYYY) on the Robin hoist. Label Example: <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="background-color: #cccccc; padding: 2px;">Date for Next Inspection</td> </tr> <tr> <td style="height: 20px;"></td> </tr> </table> | Date for Next Inspection | | | | |
| Date for Next Inspection | | | | | |
| | | | | | |

| 3. Final Steps | ✓ | ✗ | Comments |
|--|---|---|----------|
| 3.1 The owner has all the necessary User Manuals . | | | |

Overall assessment

| | |
|---|--|
| <input type="checkbox"/> Pass No safety or functional deficiencies were detected. | <input type="checkbox"/> Fail Safety or functional deficiencies were detected, and corrective actions are required. Comments: |
|---|--|

Date of next inspection: _____

Service Company Name: _____

Inspection done by: _____

Date / Signature: _____

4 Technical Data

4.1 The rail system

All rail profiles are manufactured in extruded aluminium.

The rails are available with white powder coating.

The rails are available in three different heights; S= 68 mm, M=114 mm and L=183 mm, providing a free span of 2 m, 4 m and 8 m, respectively.

The profile width of all three profiles is 58 mm. The top of the profiles are shaped to permit the coupling of the rails to the installation brackets.

On delivery, the rail for the hoist will feature a milled groove on its bottom. This groove is used for hoist installation and is called keyhole.

Possible installations

Single rail can be installed on the ceiling or onto the wall and floor. The rail position in the room can be either parallel to a wall or angular in relation to a wall.

The traverse system can be installed on the ceiling or onto the wall and floor. The free span between the two parallel rails can be up to 8000 mm.

Curves are available in the S-profile and can be installed on the ceiling. They are available in angles of 30°, 45°, 60° and 90°.

Special solutions

Installation with pendling brackets is used in the following cases: When the height of the ceiling varies in the lengthwise direction of the rail, or when the rail is to be countersunk.

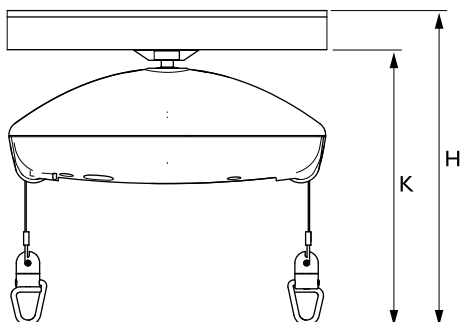
Angular installation is used when the rail has to be installed in other angles than 90°.

4.2 Dimensions for different track installations

The building dimensions of an installation is defined as the distance between the ceiling underside and the hooks of the hoist.


Single rail height

For the EC-Track™ system and Robin™ ceiling hoist, the dimensions for being built-in, measured from the ceiling with the rail mounted as close as possible to the ceiling, are:
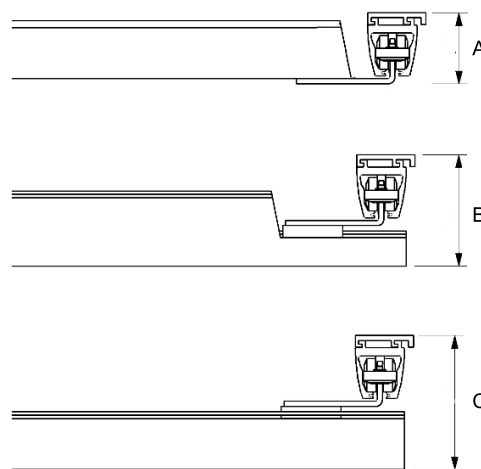
| | | Robin™ | Robin Mover™ |
|---|--|--------|--------------|
| K | Measured from beneath the rail to the sling hook | 336 mm | 396 mm |
| H | Ceiling mounted Rail profile S | 410 mm | 470 mm |
| | Wall/floor mounted Rail profile M | 450 mm | 510 mm |
| | Wall/floor mounted Rail profile L | 519 mm | 579 mm |

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Invacare AB
Invacare UK Operations Limited