Invacare® Alizé

H2080

en	Mobile Shower Chair User Manual	3
de	Mobiler Duschstuhl Gebrauchsanweisung	5
fr	Chaise de douche mobile Manuel d'utilisation	8
it	Sedia da doccia mobile Manuale d'uso	11
es	Silla de ducha móvil Manual del usuario	14
pt	Cadeira de duche móvel Manual de utilização	17
nl	Mobiele douchestoel Gebruiksaanwijzing	20

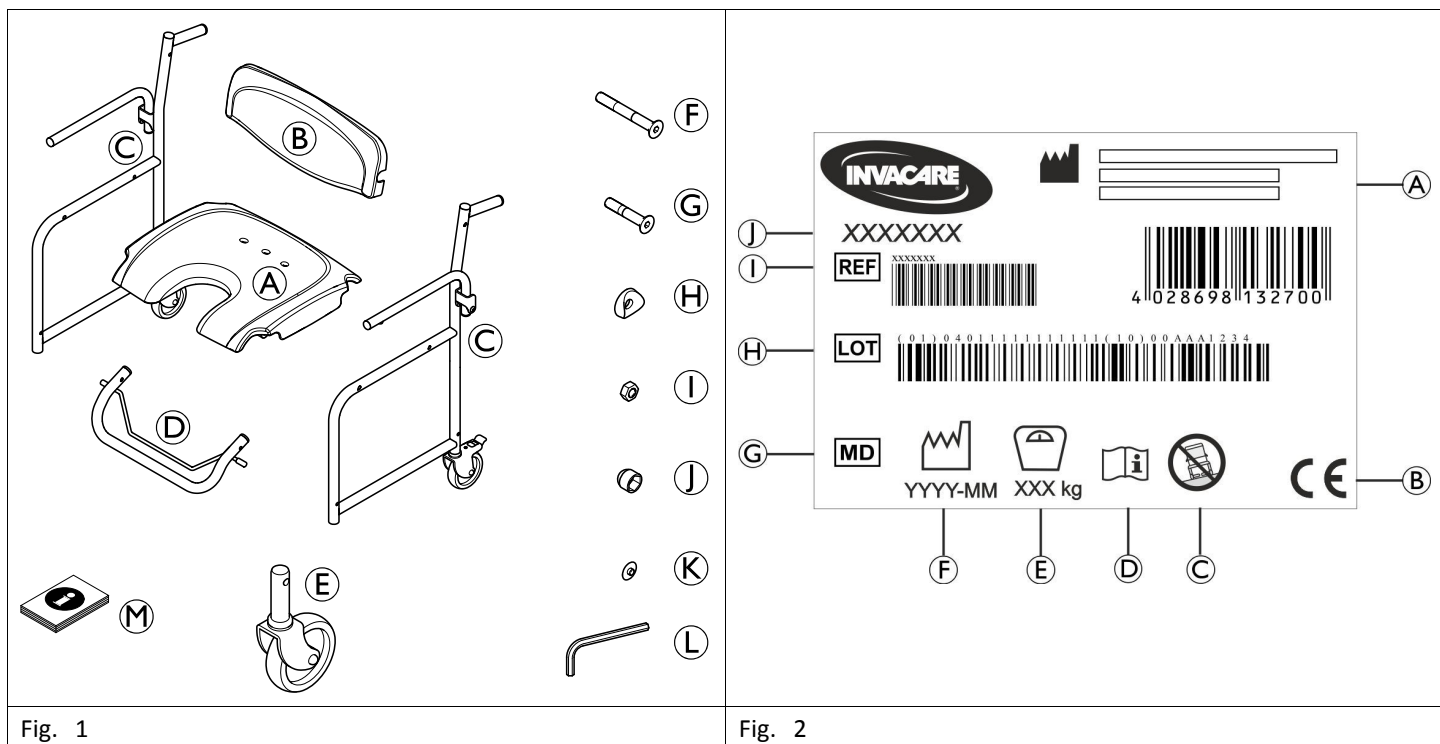
Fig. 1

Fig. 2

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

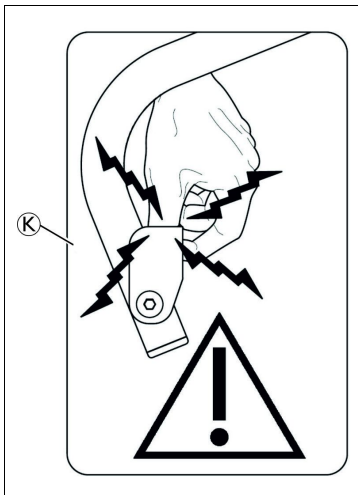
Fig. 3

Fig. 4

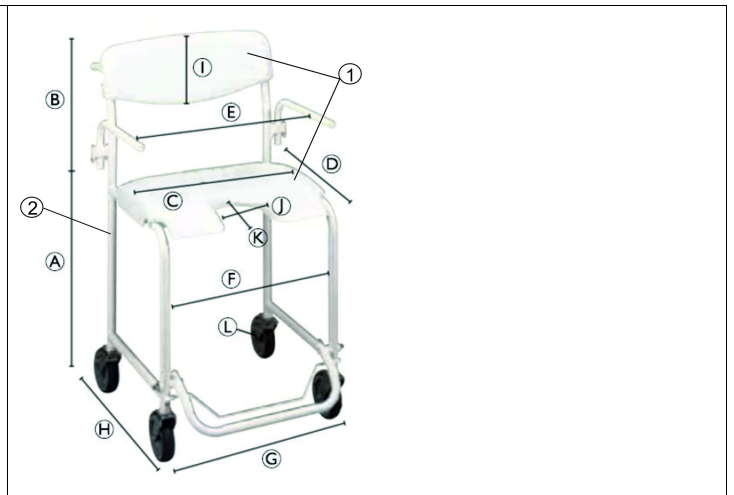
Fig. 5

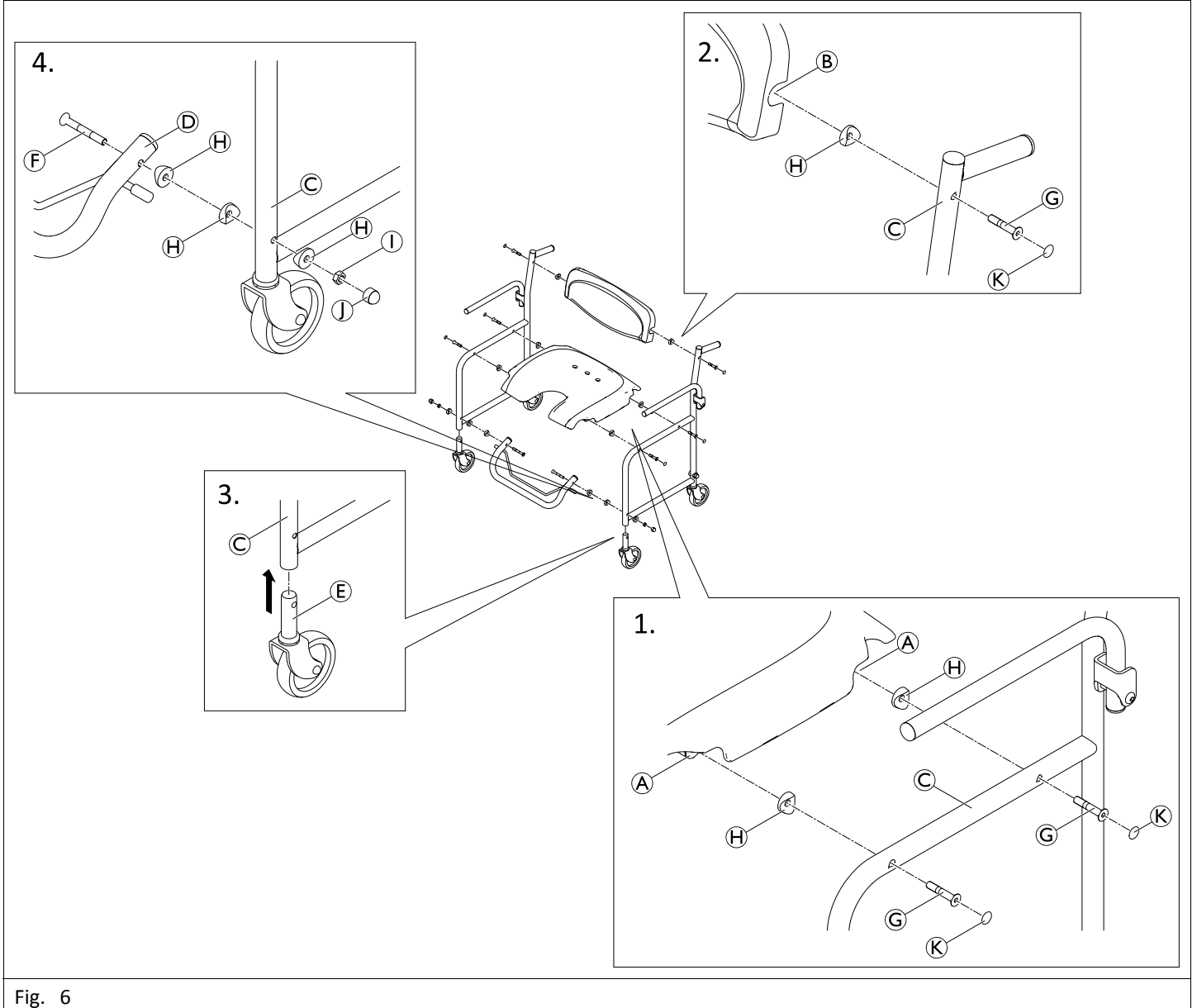
Fig. 6

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not stand on this product.
- Do not stand on the footrest.
- Do not use as a climbing aid.
- Do not lift the chair on the armrest / footrest.
- Use only on a flat surface.
- Do not use or store the product outdoors.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 2, 3)

(A)	Manufacturer	(G)	Medical Device
(B)	European Conformity	(H)	Lot number
(C)	Use only on level ground	(I)	Reference number
(D)	Read user manual	(J)	Product designation
(E)	Maximum user weight	(K)	Warning label: Risk of pinching when folding the armrest (Fig. 3)
(F)	Production date		

The identification label is attached to the frame of the product.

The warning label "Risk of Pinching" is located on the inner side of the rear frame under the armrest on each side. (Fig. 3)

7 Scope of Delivery (Fig. 1)

- (A) Seat
- (B) Backrest
- (C) Frame with armrest and rear castor with castor lock (left and right)
- (D) Footrest
- (E) Front castor (2x)
- (F) Screw (M6x65) (2x)
- (G) Screw (M6x50) (6x)
- (H) Saddle spacer (12x)
- (I) Nut (M6) (2x)
- (J) Covering cap for nut (2x)
- (K) Covering cap for screw (6x)
- (L) Allen key
- (M) User manual

8 Intended Use

This product is intended to be used as a shower aid providing stable seating position while showering.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips). If used without an assistant, users must be able to keep their own balance.

The product is intended exclusively for home use and is not intended for intensive use in institutions.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 120 kg.

9 Product Description

Invacare® Alizé H2080 is a shower chair composed of flip-up armrests, backrest, four castors (two with brakes), fold-up footrest and a seat plate with hygiene cut-out.

10 Assembling the Product (Fig. 6)

CAUTION!

Risk of Pinching

– Take care during assembly.

Perform each of the following steps on both sides of the shower chair. Refer to fig. 6.

1. Mount both tubes of the seat (A) to the frame (C) using screws (G) and saddle spacers (H). Cover the screw heads with caps (K).
2. Mount the tube of the backrest (B) to the frame (C) using screws (G) and saddle spacers (H). Cover the screw heads with caps (K).
3. Insert the front castor (E) into the frame tube (C).
4. Mount the footrest (D) to the frame (C) using screws (F), saddle spacers (H) and nuts (I). Cover the nuts with caps (J).

11 Usage (Fig. 4)

Transfer to and from the Shower Chair

WARNING!

Risk of Injury

- Only use the shower chair on level ground.
- Before sitting down or standing up from the shower chair engage the castor locks of both the rear castors.

CAUTION!

Risk of Trapping Fingers

- Do not keep your hand on the handle when raising the armrest. When lowering the armrest, ensure that your fingers are away from the hinges to avoid catching them.

1. Place the shower chair in a suitable position.
2. Engage both castor locks on the rear castor (N).
3. Fold up the footrest (D).
4. Fold up the armrest (O) for easier transfer if necessary.
5. Transfer to the seat of the shower chair.
6. Fold down the armrest (O).
7. Fold down the footrest (D) and place your feet on it.

Moving the Shower Chair

1. Hold the shower chair at the push handles.
2. Release both castor locks on the rear castors.
3. Move the shower chair to the desired position.
4. Engage both castor locks on the rear castors.

12 Maintenance

12.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

12.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

Cleaning Intervals

- !** **IMPORTANT!**
Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

Cleaning Instructions

- !** **IMPORTANT!**
- Clean by hand
 - Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *12 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

14 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

15 Technical Data (Fig. 5)

Dimensions			
Ⓐ	530 mm	Ⓒ	590 mm
Ⓑ	380 mm	Ⓓ	690 mm
Ⓒ	410 mm	Ⓔ	200 mm
Ⓓ	400 mm	Ⓕ	170 mm
Ⓔ	530 mm	Ⓖ	140 mm
Ⓕ	420 mm	Ⓖ	∅ 100 mm

Weights	
Alizé H2080 mobile shower chair	10.0 kg
Maximum user weight	120 kg

Materials			
①	ABS	②	Steel, powder-coated

All product components are corrosion resistant and do not contain natural rubber latex.

16 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Limited