Aquatec® product range

9630E Styxo, Aquatec 90 / Aquatec 900 / Aquatec 90000,
Pico / Pico TS, Tellus / Pluto, Sorrento range

en	Hygiene medical devices Service Manual	3
de	Medizinische Hygieneprodukte Servicehandbuch	15
fr	Dispositifs médicaux d'hygiène Manuel de maintenance	27
da	Medicinsk hygiejneudstyr Servicemanual	39
nn	Medisinsk hygieneutstyr Servicehåndbok	51
sv	Hygientekniska medicinska produkter Servicemanual	63
fi	Lääkinnälliset hygienialaitteet Huoltokäsikirja	75

PROVIDER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

AQUATEC®

Yes, you can.®

©2021 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

1 General	4
1.1 Introduction	4
1.2 Symbols in this Document	4
2 Safety	5
2.1 General safety information	5
2.2 Personal safety information	5
3 Overview	6
4 Servicing	10
4.1 Inspection checklist	10
4.2 Spare parts	10
4.3 Colour codes	10
5 Reconditioning	11
5.1 Cleaning and disinfection	11
5.2 Material overview	11
5.3 Reconditioning scheme	11
6 General repair information	14

1 General

1.1 Introduction

This document contains important information about assembly, adjustment and advanced maintenance of the product. To ensure safety when handling the product, read this document and the user manual carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative. See addresses at the end of this document.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

For pre-sale and user information, see the user manual.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

2 Safety

2.1 General safety information

WARNING!

Risk of injury or damage to property

- The procedures in this manual must only be performed by a qualified technician.
- Use only original accessories and spare parts.
- Do not handle this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, installation manuals or instruction sheets supplied with this product or optional equipment.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.

WARNING!

Risk of contamination

- Clean and disinfect the product before servicing.

IMPORTANT!

- Some replacement parts are only available as a kit. Always use the complete new kit when replacing a part.
- Spare parts can be ordered from Invacare. Refer to your local Invacare website to access the electronic spare parts catalogue (ESPC).

IMPORTANT!

- Refer to the user manual of this product for information on
- Technical data
 - Product components
 - Labels
 - Additional safety instructions
 - Cleaning and disinfection instructions

The information contained in this document is subject to change without notice.

2.2 Personal safety information

These safety instructions are intended to help avoid accidents during work and must be observed under all circumstances.

All employees coming into contact with contaminated products must regularly consult a company doctor. Work clothing and personal protective equipment must be available in necessary quantities and be in proper condition. Reliable hand and surface disinfection must be ensured.

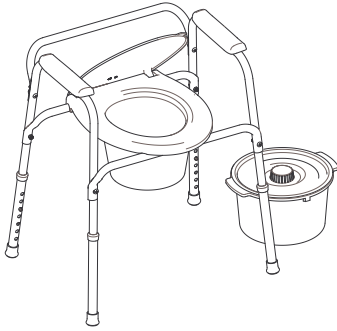
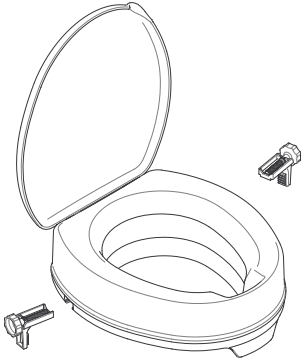
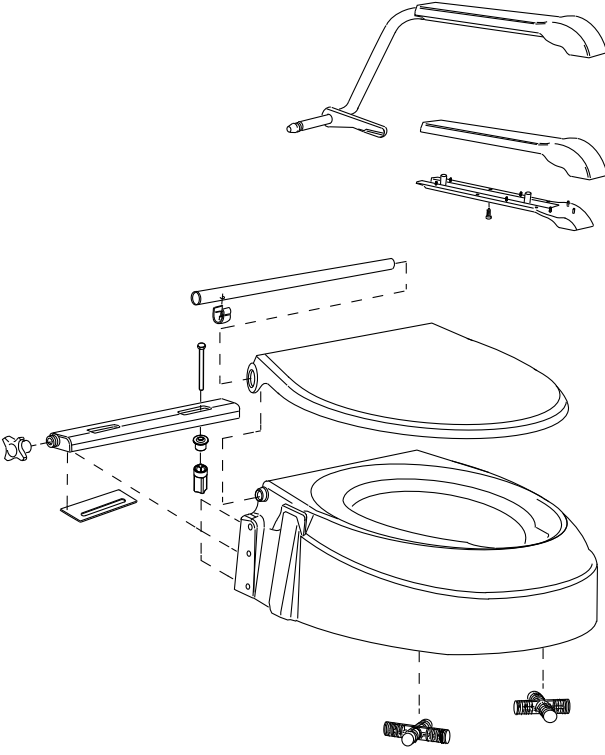
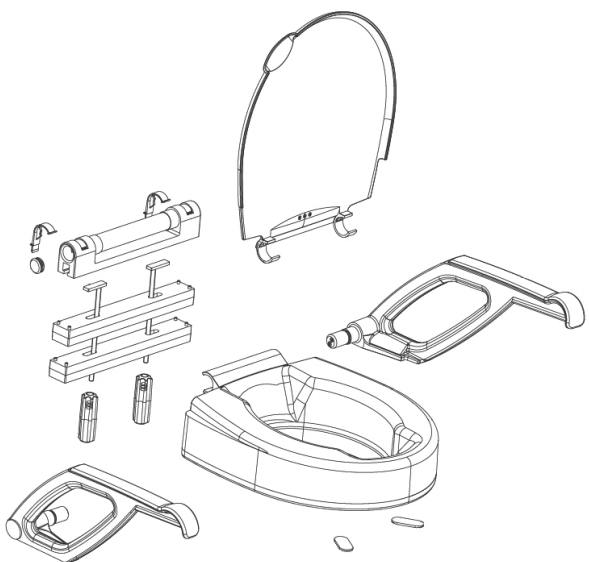
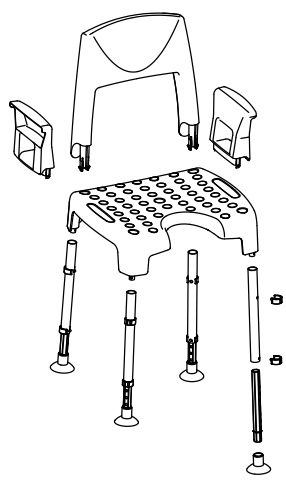
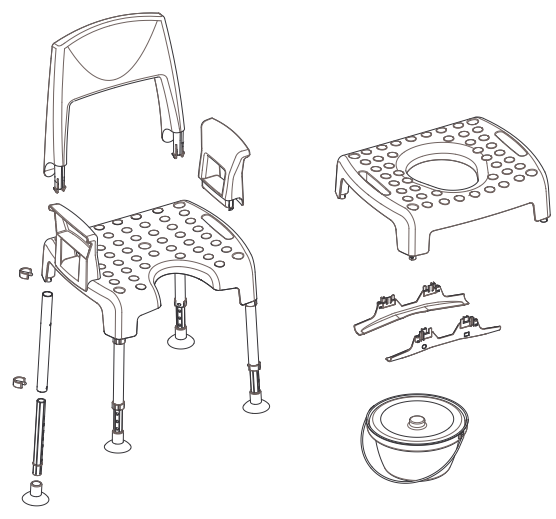
WARNING!

Risk of contamination

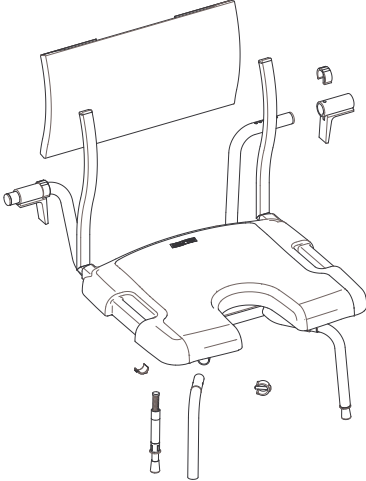
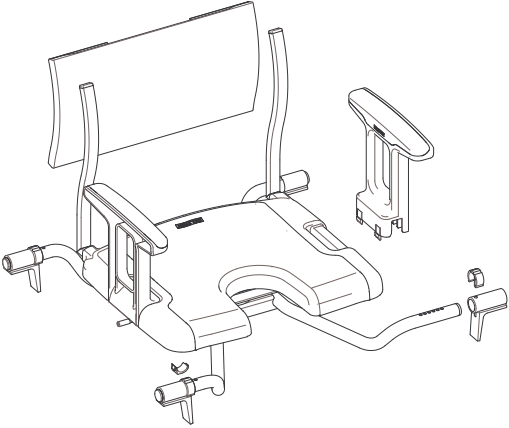
- Clean and disinfect the product before carrying out repairs.

3 Overview

Product name	Product view
9630E Styxo	
Aquatec 90	
Aquatec 900	

Product name	Product view
Aquatec 90000	 <p>An exploded view diagram of the Aquatec 90000 shower seat. The components shown include a curved backrest, a seat with a central opening, two armrests, and various mounting brackets and screws. The seat and backrest are shown in their relative positions as they would be assembled.</p>
Pico	 <p>An exploded view diagram of the Pico shower chair. The components include a seat with a perforated surface, a backrest, two armrests, and four adjustable legs with foot caps. The diagram shows how the legs attach to the seat and how the backrest and armrests are positioned.</p>
Pico TS	 <p>An exploded view diagram of the Pico TS shower chair. It shows the main chair components: seat, backrest, armrests, and legs. Additionally, it includes a separate tray with a central opening and a circular lid, which are likely accessories for the chair.</p>

Product name	Product view
Pluto / Tellus	
Sorrento shower chair	

Product name	Product view
Sorrento bathtub seat	
Sorrento bathtub swiveling seat	

4 Servicing

4.1 Inspection checklist

General inspection	☺	☹
Has the product been cleaned and disinfected?	<input type="checkbox"/>	<input type="checkbox"/>
Is the product in good condition and is it complete (product and accessories)?	<input type="checkbox"/>	<input type="checkbox"/>
Is the product free from damage or weaknesses of any kind?	<input type="checkbox"/>	<input type="checkbox"/>
Does the product operate correctly under nominal load?	<input type="checkbox"/>	<input type="checkbox"/>
Is the product fully functional in accordance with the user manual?	<input type="checkbox"/>	<input type="checkbox"/>
Eradication of faults	☺	☹
Have all the faults found been eliminated and have faulty components been replaced?	<input type="checkbox"/>	<input type="checkbox"/>
Completion of checks	☺	☹
Has the product been cleaned and disinfected?	<input type="checkbox"/>	<input type="checkbox"/>
Are all screws/bolts firmly fitted and is the product securely assembled	<input type="checkbox"/>	<input type="checkbox"/>
Is the product technically and functionally safe?	<input type="checkbox"/>	<input type="checkbox"/>

Is the identification label easily readable and is it securely mounted on the product?	<input type="checkbox"/>	<input type="checkbox"/>
Is the product accompanied by the latest revision of the user manual?	<input type="checkbox"/>	<input type="checkbox"/>

4.2 Spare parts

WARNING!

Original spare parts must be used in all repairs. Otherwise the warranty and product declaration of conformity shall be rendered invalid.

All spare parts must be obtained from the Invacare customer service department. An electronic spare parts catalogue can be found at www.invacare.co.uk.

4.3 Colour codes

Item	Colour code (RAL)
Styxo – frame	Cool grey, Pantone 7C
Aquatec 900 – armrest tubes	White, RAL9003
Tellus/Pluto – frame	White, RAL9003
Tellus/Pluto – Clamp for backrest	Blue, RAL 5002

5 Reconditioning

5.1 Cleaning and disinfection

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to your local guidelines on infection control and your local infection control policy.

Preparation at the place of use

1. Thorough pre-clean the product with hot tap water.

- Pre-cleaning storage time must be as short as possible.
- Products to be cleaned must be protectively packed, e.g. sealed with adhesive tape in such a way that no more openings are visible on the packaging.
- Pathogens with which the product has been infected (even potentially) must be noted and visibly specified on the outer packaging.

Preparation before cleaning

1. Dismantle the product into individual components and create an optimum item for cleaning and disinfection.
2. If present, cover electrical connections with protective caps and seal any sites where water may leak in.
3. Remove superficial contamination.

Cleaning

1. Clean the product with decalcifying sanitizing detergent. Manual brushing is permitted.
2. Rinse with tap water until the detergent is completely removed.
3. Mechanically cleaning of washable components (e.g. covers) in industrial washing machine.

4. Rinse with tap water until the detergent is completely removed.
5. Manually clean the product with universal detergent. Supplementary manual brushing is also permitted.
6. Rinse with tap water until the detergent is completely removed.
7. Air dry until there is no more superficial moisture on the product.

WARNING!

Risk of contamination

- Do not continue to use the product in case of water penetration or if it shows discolourations which cannot be removed.

Disinfection

1. Disinfect the product by wiping generally accessible surfaces with disinfectant.
2. Air dry until there is no more superficial moisture on the product.

5.2 Material overview

Products	Material(s)
Styxo	PP, Coated steel
Aquatec 90	PP
Aquatec 900	PP, Coated steel
Aquatec 90000	PP
Pico / Pico TS	PP, Aluminum
Tellus / Pluto	PP, Coated steel
Sorrento range	PP (glass fiber reinforced)/TPE, Aluminum, Nylon (PVC coated)

5.3 Reconditioning scheme

For replacement parts refer to the electronic spare parts catalogue (ESPC).

The following items must be reviewed and checked when reconditioning is required:

Toilet frames

Products	Issue	Troubleshooting action	OK
Styxo 9630E	Frame damaged (cracks, fractures)	Discard the product.	<input type="checkbox"/>
	Frame coating damaged or scratched.	Use touch-up colours to repair damages. See chapter 4.3 <i>Colour codes, page 10</i> .	<input type="checkbox"/>
	Toilet seat damaged (cracks, sharp edges)	Replace with new one. See ESPC.	<input type="checkbox"/>
	Toilet pan and/or lid damaged		<input type="checkbox"/>
	Splash guard damaged		<input type="checkbox"/>
Armrest pad damaged	Replace with new one. See ESPC.	<input type="checkbox"/>	

Products	Issue	Troubleshooting action	OK
	Damage to parts of the height adjustable legs	Replace plastic cap. See ESPC. Replace locking knob. See ESPC. Replace rubber foot. See ESPC.	<input type="checkbox"/>
	Screws loose	Re-tighten all screws	<input type="checkbox"/>

Toilet seat raisers

Products	Issue	Troubleshooting action	OK
Aquatec 90	Lid and/or hinge damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
Aquatec 900 Aquatec 90000	Installation kit/ clamping elements damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
Aquatec 90 Aquatec 900	Body damaged	Discard the product.	<input type="checkbox"/>
Aquatec 900	Crossbar/ attachment cross/ clips/ rear tube/ adhesive tapes/ star screw damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Armrests/ armrest overlays damaged		<input type="checkbox"/>
Aquatec 90000	Armrests damaged	Replace with new ones. See ESPC.	<input type="checkbox"/>
	Body damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Slip protection/ base/ clip/ height adapter/ plug damaged	Replace with new one. See ESPC.	<input type="checkbox"/>

Shower stools / Shower chairs / Commodes

Products	Issue	Troubleshooting action	OK
Pico Pico TS	Backrest/ armrest/ seat plate damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Leg damaged	Replace complete leg or affected part with new one. See ESPC.	<input type="checkbox"/>
Pico TS	Toilet pan/ holder damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Toilet pan holder loose	Re-tighten the screws.	<input type="checkbox"/>
Pluto / Tellus	Frame tube / backrest tube damaged	Discard the product.	<input type="checkbox"/>
	Frame end cap damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Seat plate damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Backrest cover damaged (Pluto only)	Replace with new one. See ESPC.	<input type="checkbox"/>
	Backrest fixation kit loose	Re-tighten screws.	<input type="checkbox"/>
	Backrest fixation kit broken	Replace complete fixation kit. See ESPC.	<input type="checkbox"/>
	Telescope leg damaged	Replace complete telescope leg. See ESPC.	<input type="checkbox"/>
	Transition cap and/or clip damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Rubber tip damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Frame coating damaged or scratched.	Use touch-up colours to repair damages. See chapter 4.3 <i>Colour codes, page 10.</i>	<input type="checkbox"/>

Products	Issue	Troubleshooting action	OK
Sorrento shower chair Sorrento bathtub (swivel) seat	Seat damaged	Discard the product.	<input type="checkbox"/>
	Armrest damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Backrest tube damaged		<input type="checkbox"/>
	Backrest cover damaged		<input type="checkbox"/>
	Clamps broken/damaged		<input type="checkbox"/>
Sorrento shower chair	Telescope leg/ clip fixation/ clip for height adjustment/ rubber foot damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
Sorrento bathtub seat	Height adjustment leg damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Leg tube damaged		<input type="checkbox"/>
	Clip for height adjustment damaged.		<input type="checkbox"/>
Sorrento bathtub (swivel) seat	Footrest damaged	Replace with new one. See ESPC.	<input type="checkbox"/>
	Clip for footrest damaged		<input type="checkbox"/>

6 General repair information

IMPORTANT!

- If possible, continue to use the old identification label; if this is not possible, the new identification label must contain the same information and the old serial number. (Replacement of spare parts with serial numbers).
- When components are replaced it is necessary to ensure the traceability of the components replaced.
- If screws with thread locker are loosened, these must be replaced with new screws with thread locker. Alternatively, new thread locker must be applied (e.g. Loctite 2701).
- Parts that become damaged during removal must be replaced with new ones.

For instructions on how to replace the spare parts refer to the graphics in the ESPC.

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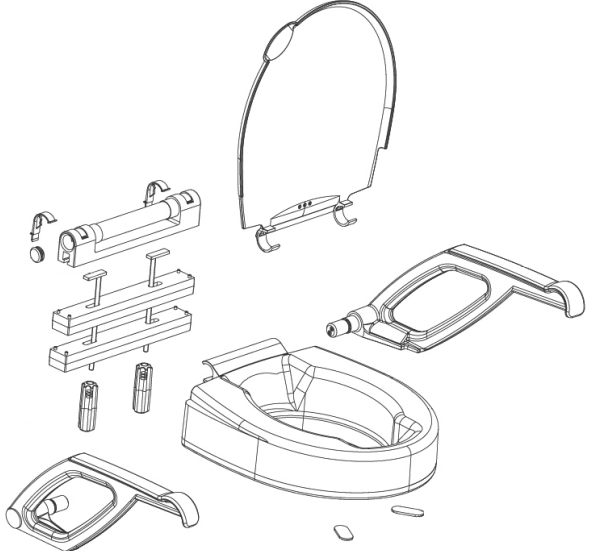
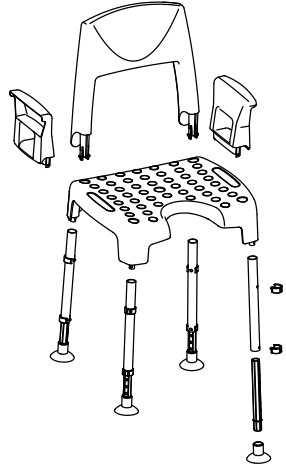
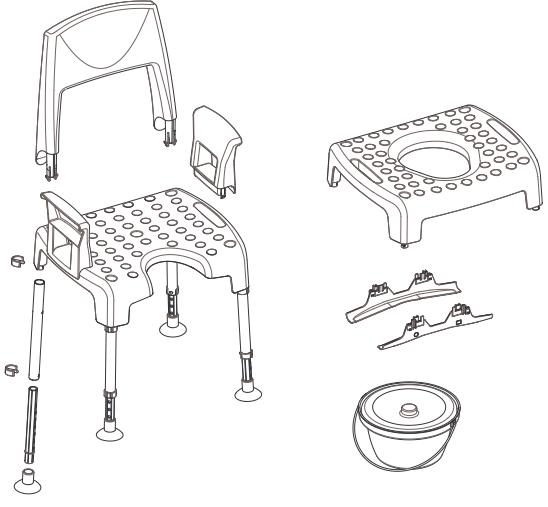
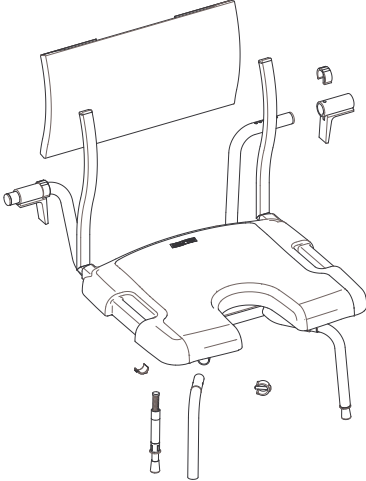
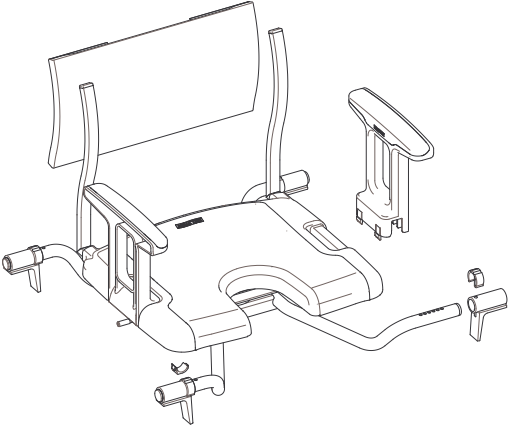
Produkt navn	Produktvisning
Aquatec 90000	 <p>An exploded view diagram of the Aquatec 90000 shower seat. The components shown include a large, curved backrest, a seat base with a central opening, two side handles, a mounting bracket with two screws, two vertical support posts with caps, and a small circular cap.</p>
Pico	 <p>An exploded view diagram of the Pico shower chair. The components include a high-back chair with armrests, a perforated seat, four adjustable legs with caps, and two small rectangular side panels.</p>
Pico TS	 <p>An exploded view diagram of the Pico TS shower chair. The components include a high-back chair with armrests, a perforated seat, four adjustable legs with caps, a separate perforated seat base, two side panels, and a circular tray with a central cap.</p>

Produkt navn	Produktvisning
Pluto/Tellus	<p>The diagram illustrates the assembly of the Pluto/Tellus shower chair. It includes a seat with a central opening, a backrest, and four legs. The backrest is shown being attached to the seat. The legs are shown being attached to the seat and backrest. Hardware such as screws and caps are shown being inserted into the legs. A small inset diagram shows the fully assembled chair.</p>
Sorrento dusjstol	<p>The diagram illustrates the assembly of the Sorrento shower chair. It includes a seat with a central opening, a backrest, and four legs. The backrest is shown being attached to the seat. The legs are shown being attached to the seat and backrest. Hardware such as screws and caps are shown being inserted into the legs. A small inset diagram shows the fully assembled chair.</p>

Produkt navn	Produktvisning
Sorrento badekarsete	 Exploded view diagram of the Sorrento bathtub seat. The main assembly includes a seat with a backrest, two armrests, and a central cutout. A separate leg is shown below the seat. To the right, a separate component, possibly a footrest or a different style of armrest, is shown with its own leg.
Sorrento svingsete for badekar	 Exploded view diagram of the Sorrento swing seat for bathtub. The main assembly includes a seat with a backrest, two armrests, and a central cutout. A separate leg is shown below the seat. To the right, a separate component, possibly a footrest or a different style of armrest, is shown with its own leg.

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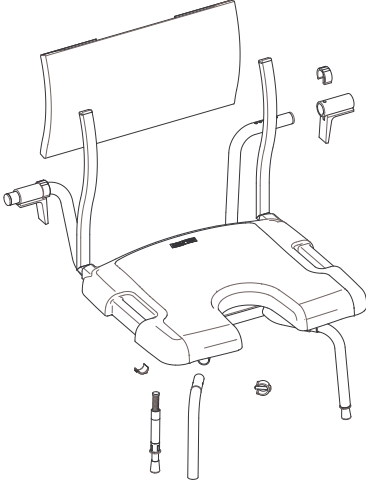
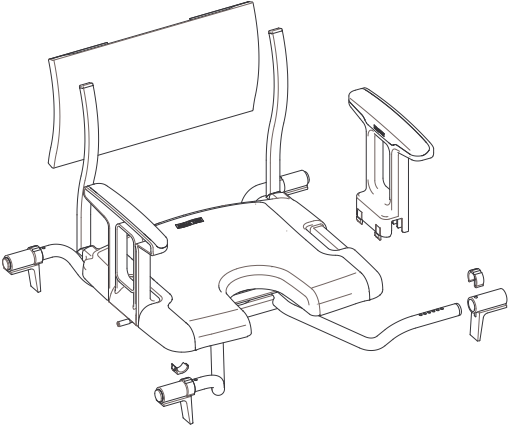
Tuotteen nimi	Tuotteen kuva
Pluto / Tellus	<p>The diagram illustrates the assembly of the Pluto / Tellus shower chair. It includes the seat, backrest, and four legs. Key components shown include casters for the front legs, height extenders for the rear legs, and various adjustment pins and caps. A small inset shows the fully assembled chair.</p>
Sorrento-suihkutuoli	<p>The diagram illustrates the assembly of the Sorrento shower chair. It includes the seat, backrest, and four legs. Key components shown include casters for the front legs, height extenders for the rear legs, and various adjustment pins and caps. A small inset shows the fully assembled chair.</p>

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Invacare New Zealand Ltd
Invacare GmbH