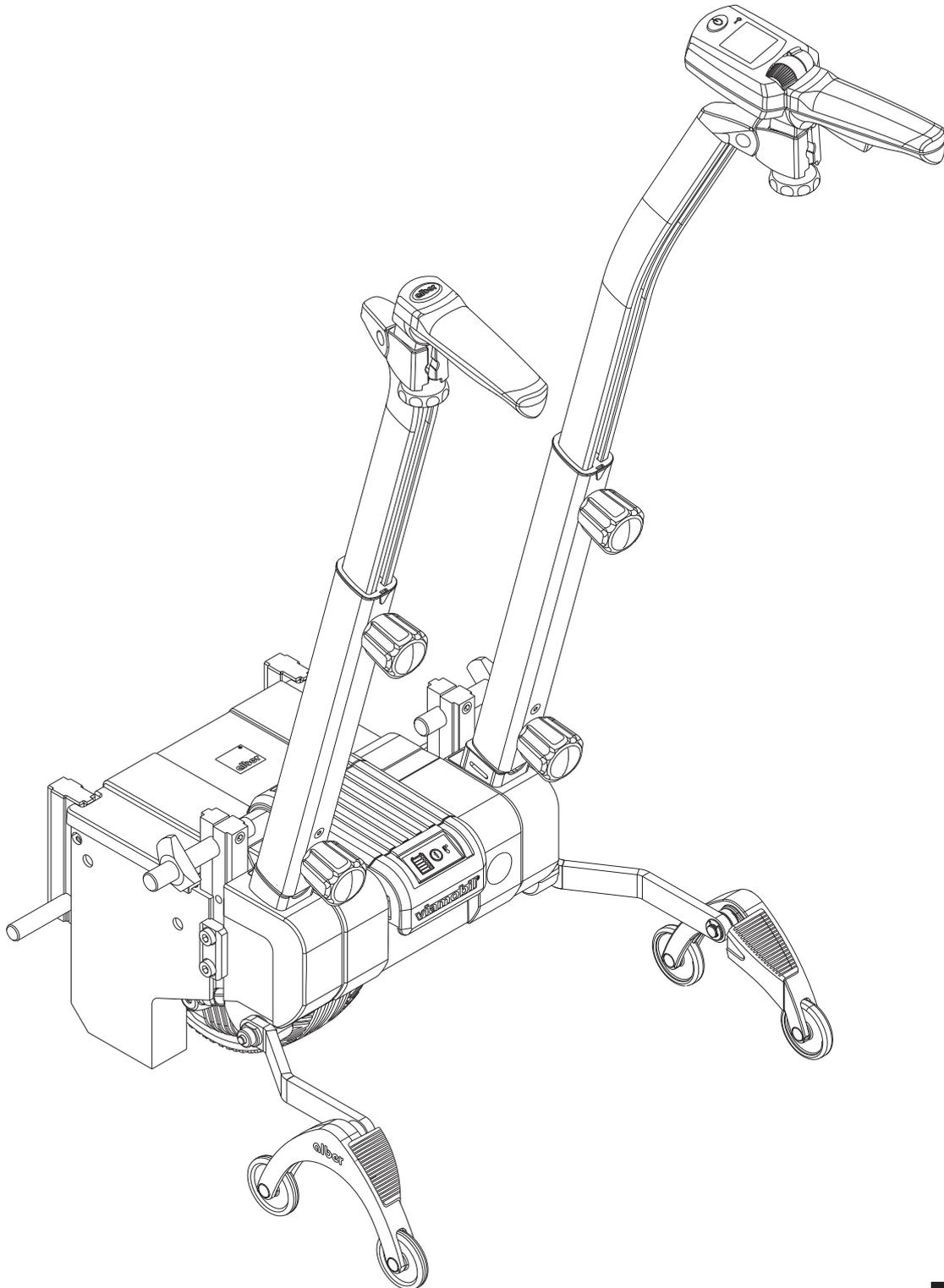
# viamobil V25<sup>®</sup>

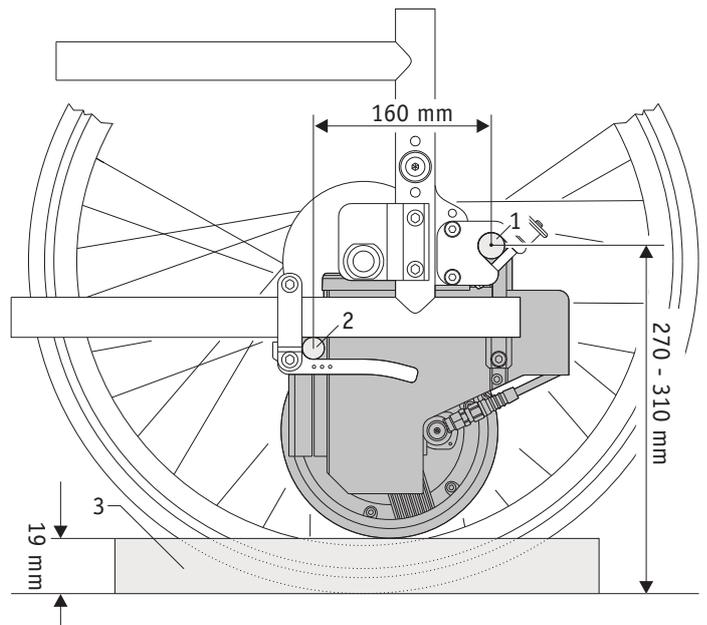
Anbauanweisung  
*Mounting instruction*

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### Information about the Mounting of the viamobil at the wheelchair

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
1. Mount the bracket on the wheelchair before you begin the installation work on the viamobil. Please take care of the distances indicated on the drawing of the upper [1] and lower bolts [2] which apply to most manual wheelchairs. The values might differ from the graph depending on the wheelchair model.
2. The manual wheels need to be fixed on the chair to adjust the viamobil. If applicable, please check the air pressure of the tyres.
3. The drive wheel of the viamobil must be retracted during the assembly; the distance of the retracted wheel to the ground must be 15 mm.
4. Please use a 19 mm thick wooden plate [3] to compensate the contact pressure of the wheel.

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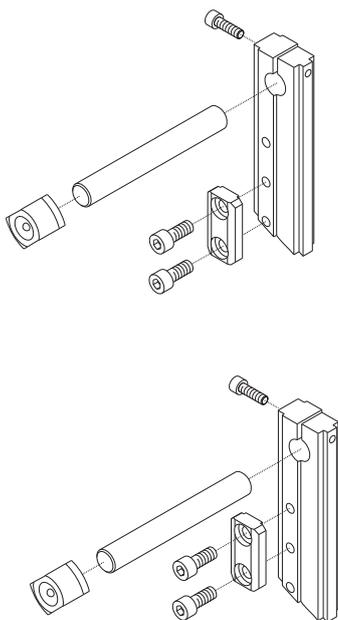
### Information about the possible mounting positions of the upper and lower bolts

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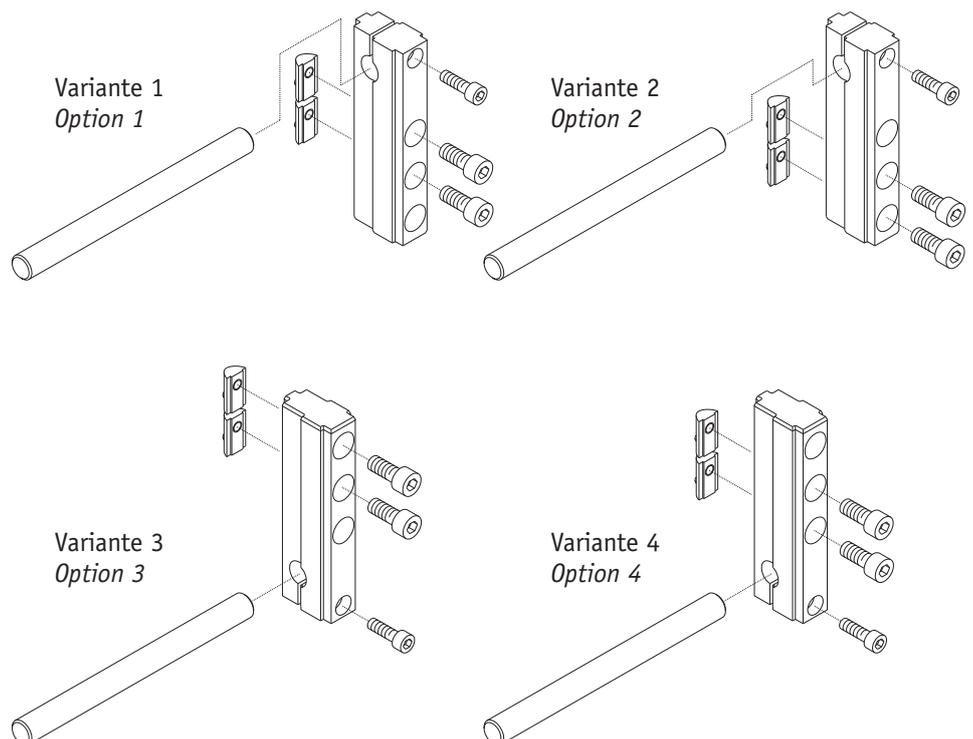
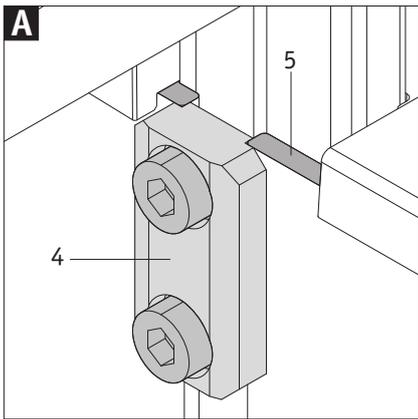
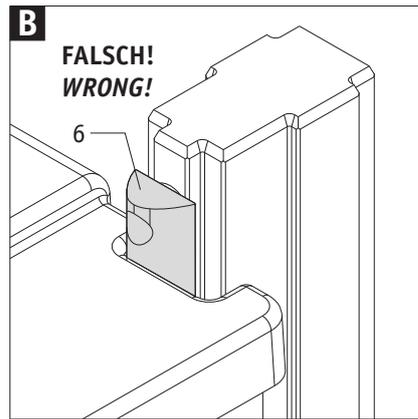
*The counterpart [4] shall not exceed the edge of the guide rod [5] on the drive unit, but must completely touch the housing of the viamobil.*

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
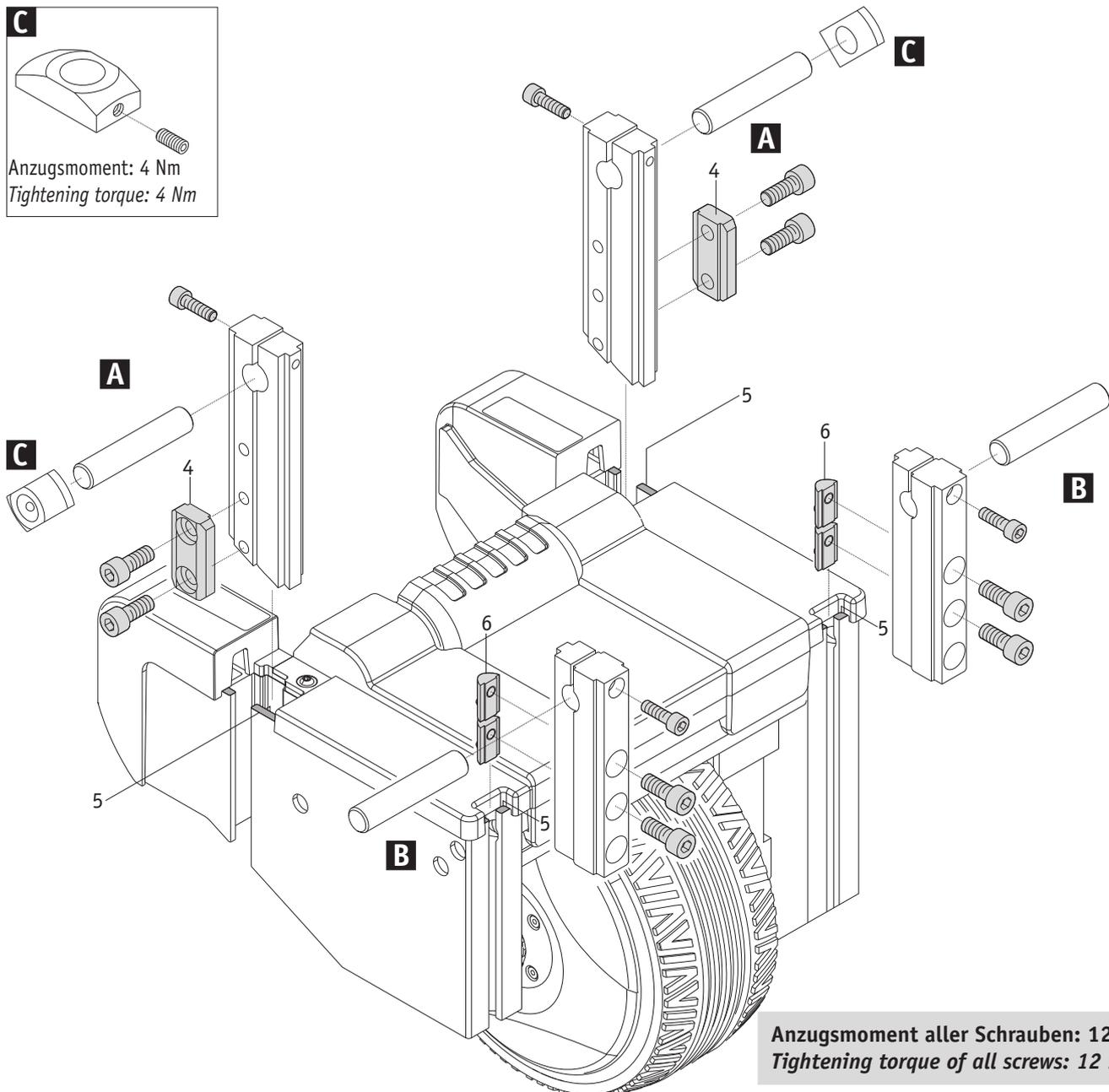
*The screw nut [6] shall not exceed the edge of the guide rod [5] on the drive unit, but must stay completely within the guide rod [5].*

**Anzugsmoment aller Schrauben: 12 Nm**  
**Tightening torque of all screws: 12 Nm**

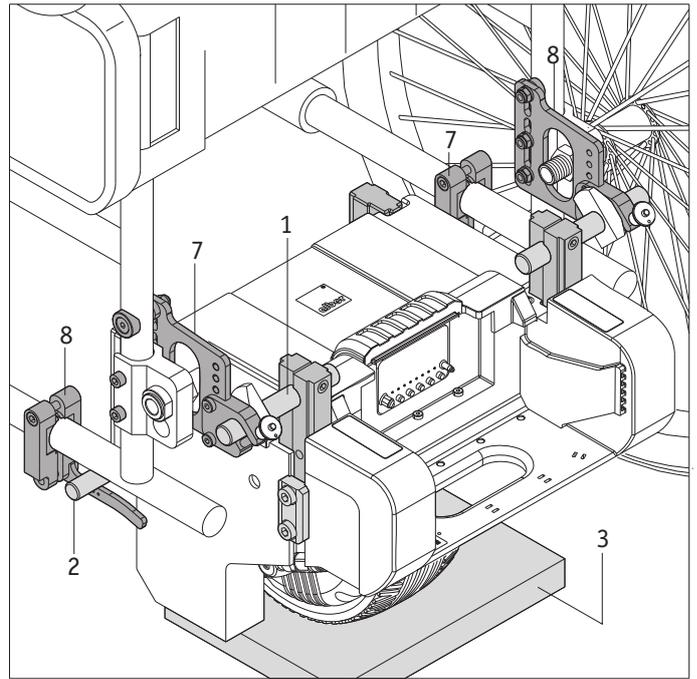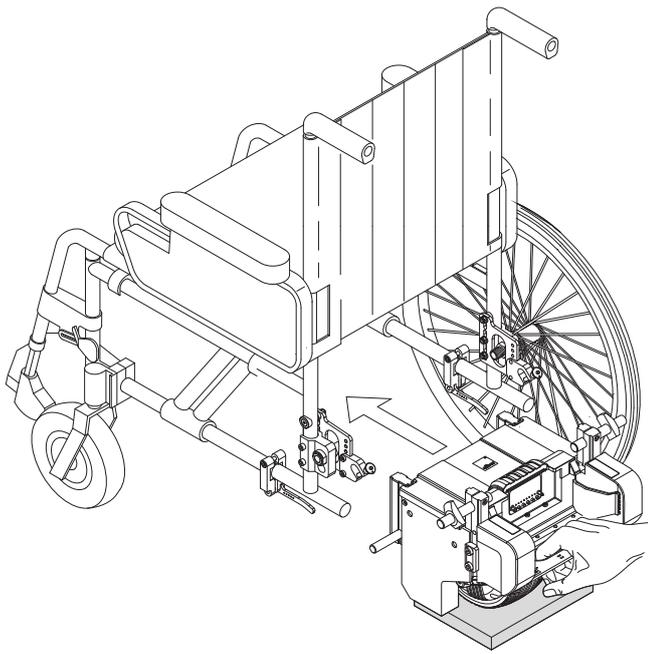
**Montageschritt 2 - Montage des viamobil am Rollstuhl**  
**Mounting step 2 - Mounting of the viamobil at the wheelchair**

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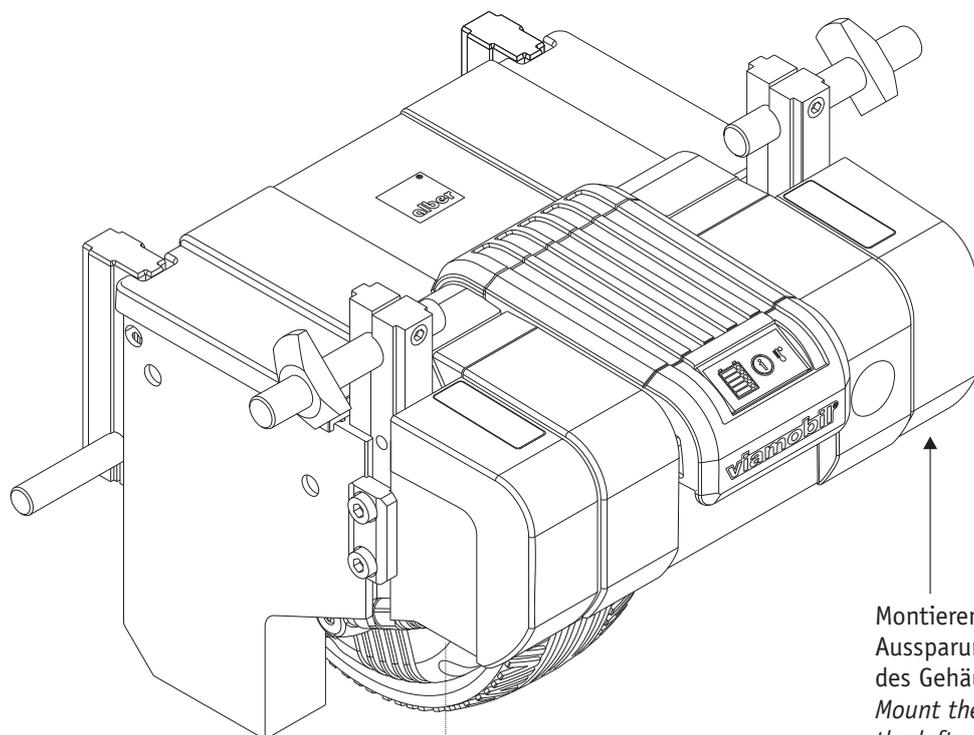
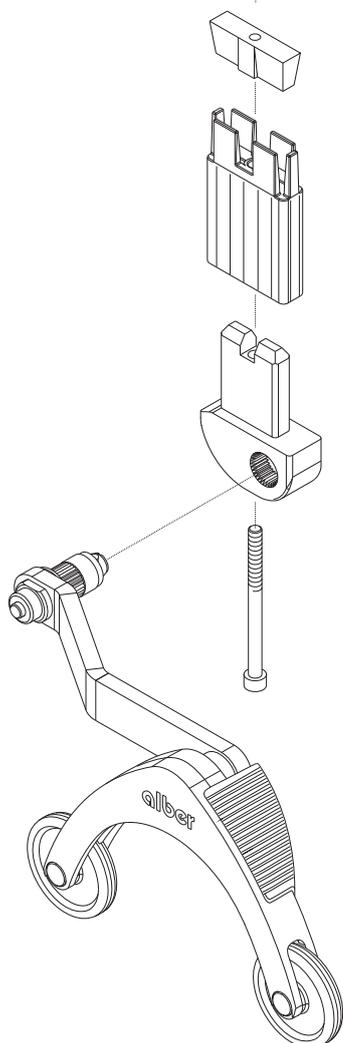
*Mount the anti-tippers in the outside notch at the left respectively right side of the housing.*

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
1. Add all parts and screw the screw (no more than three turns).
2. Insert the parts in the housing of the Viamobil.
3. Tighten the screw, tightening torque: 4 Nm

## Montage des Bediengeräts am Rollstuhl Mounting of the control unit at the wheelchair

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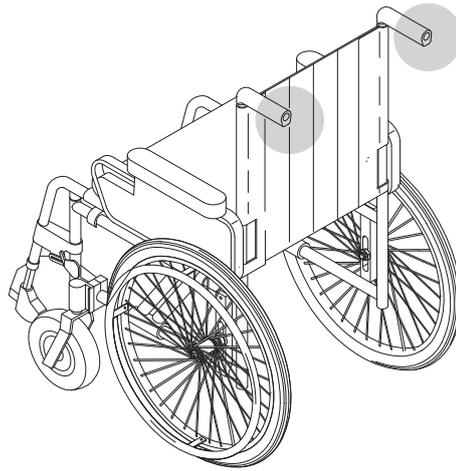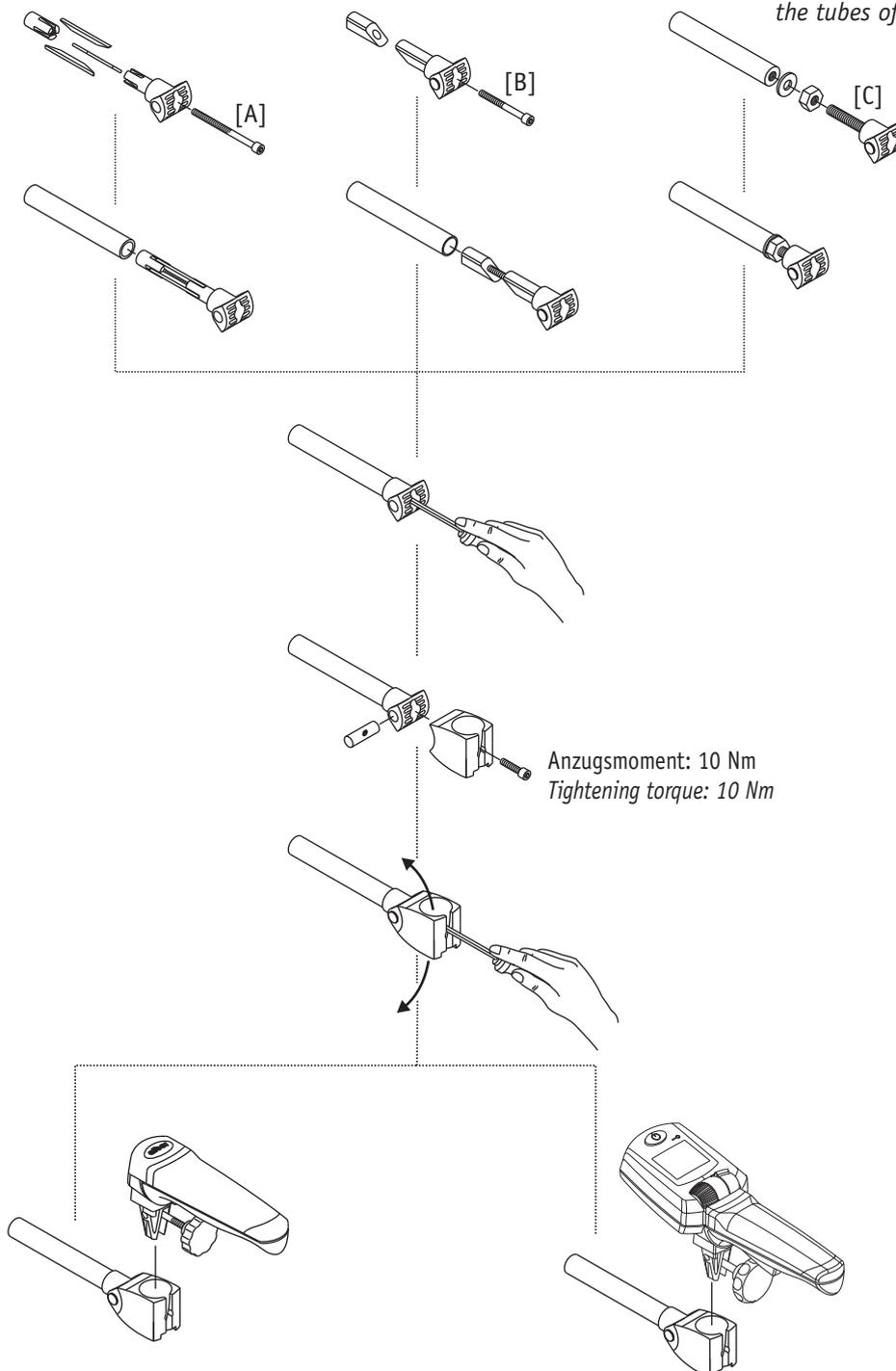*[A], [B] and [C]:  
The tightening torque depends on  
the material and the thickness of  
the tubes of the wheelchair.*

## Montage der Zusatzschiebegriffe (optionales Zubehör) Mounting of the additional handgrips (optional accessory)

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**Ulrich Alber GmbH**