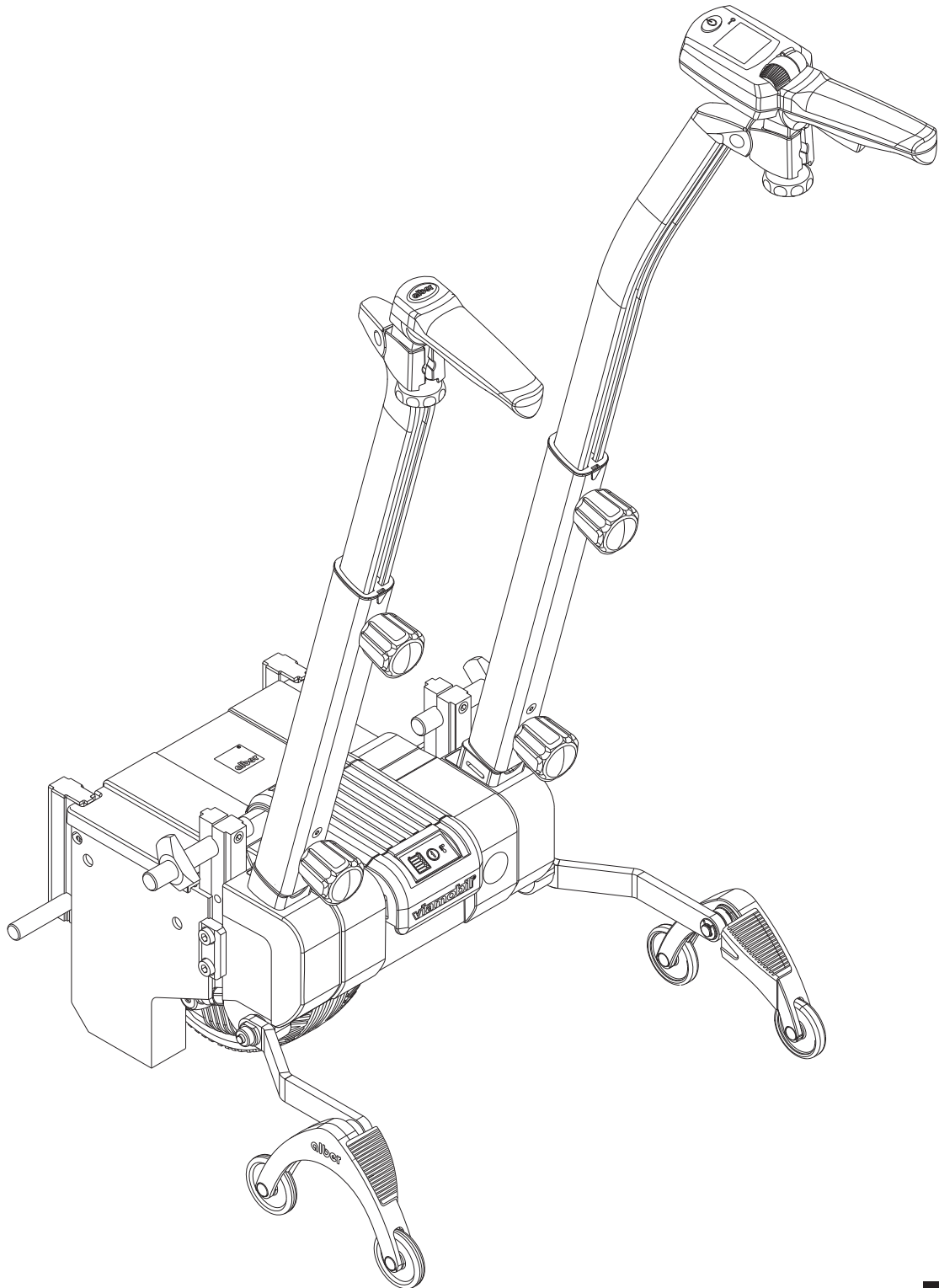
# viamobil V25<sup>®</sup>

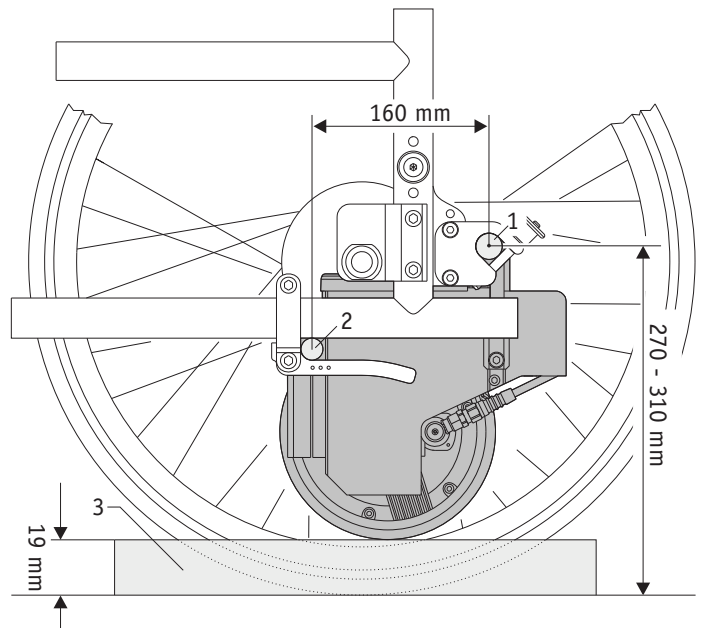
Anbauanweisung  
*Mounting instruction*

## Hinweis zur Anpassung des viamobil an den Rollstuhl *Information about the Mounting of the viamobil at the wheelchair*

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
1. Mount the bracket on the wheelchair before you begin the installation work on the viamobil. Please take care of the distances indicated on the drawing of the upper [1] and lower bolts [2] which apply to most manual wheelchairs. The values might differ from the graph depending on the wheelchair model.


**!** Pay attention to the mounting information of the viamobil at page 3.

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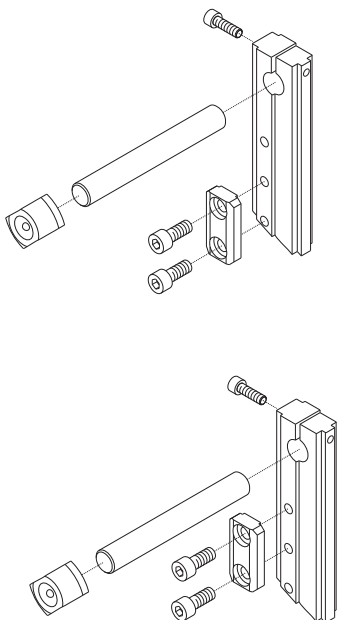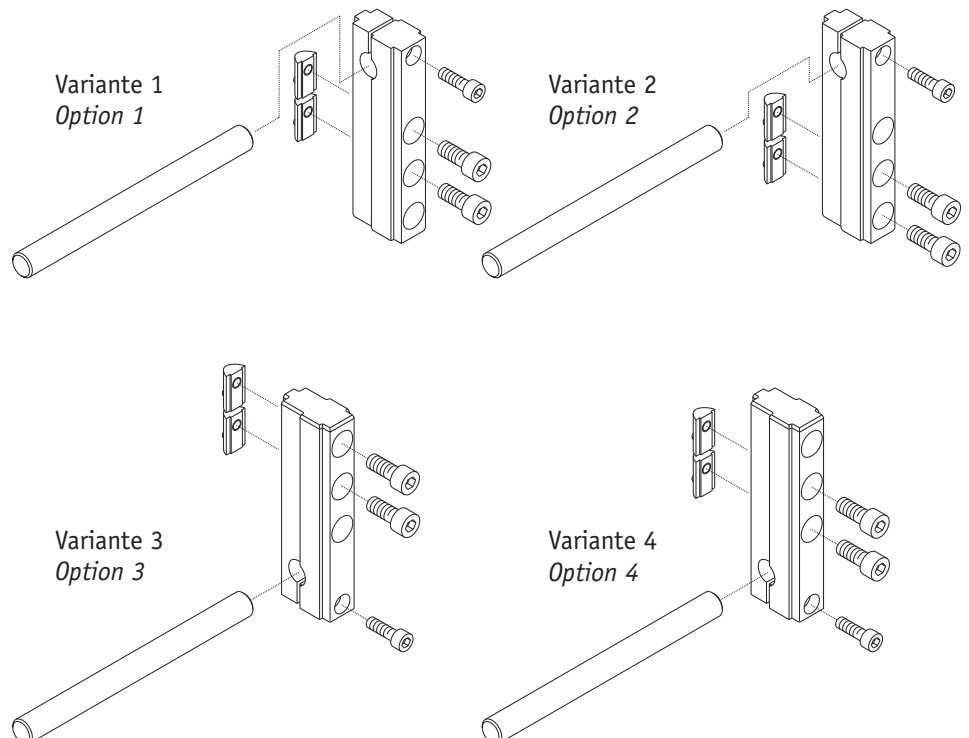
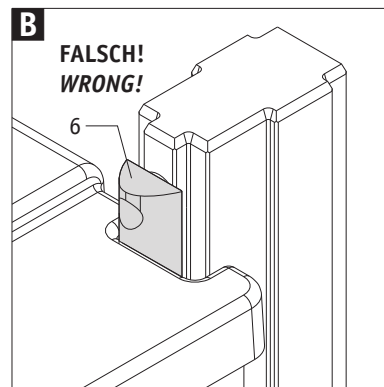
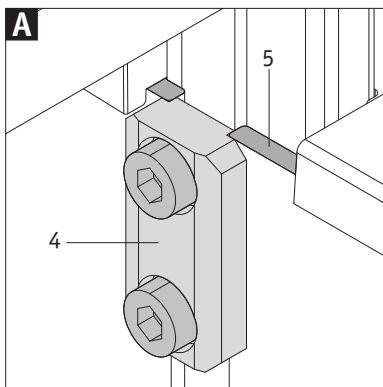
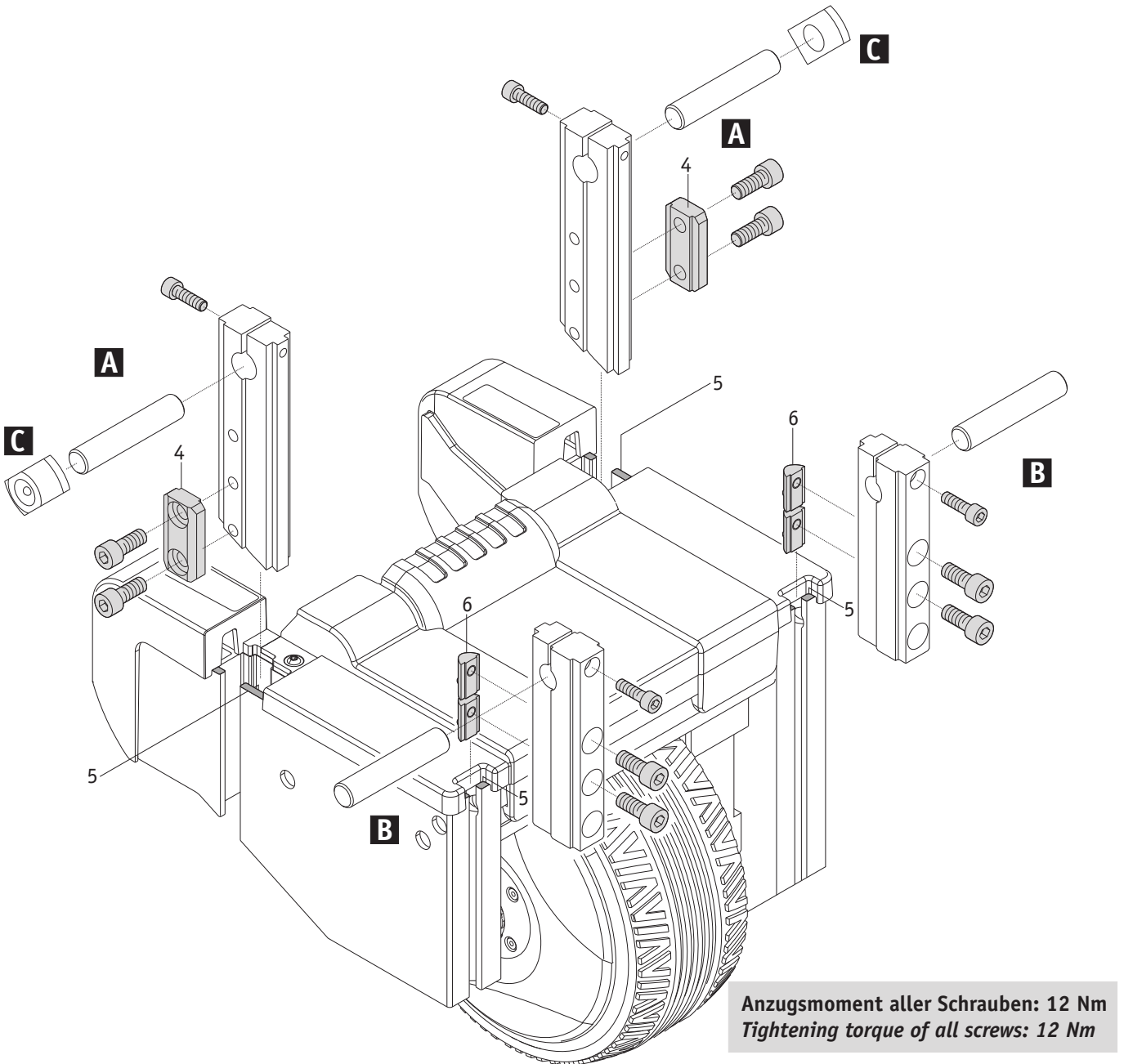
## Montageschritt 2 - Montage des viamobil am Rollstuhl Mounting step 2 - Mounting of the viamobil at the wheelchair

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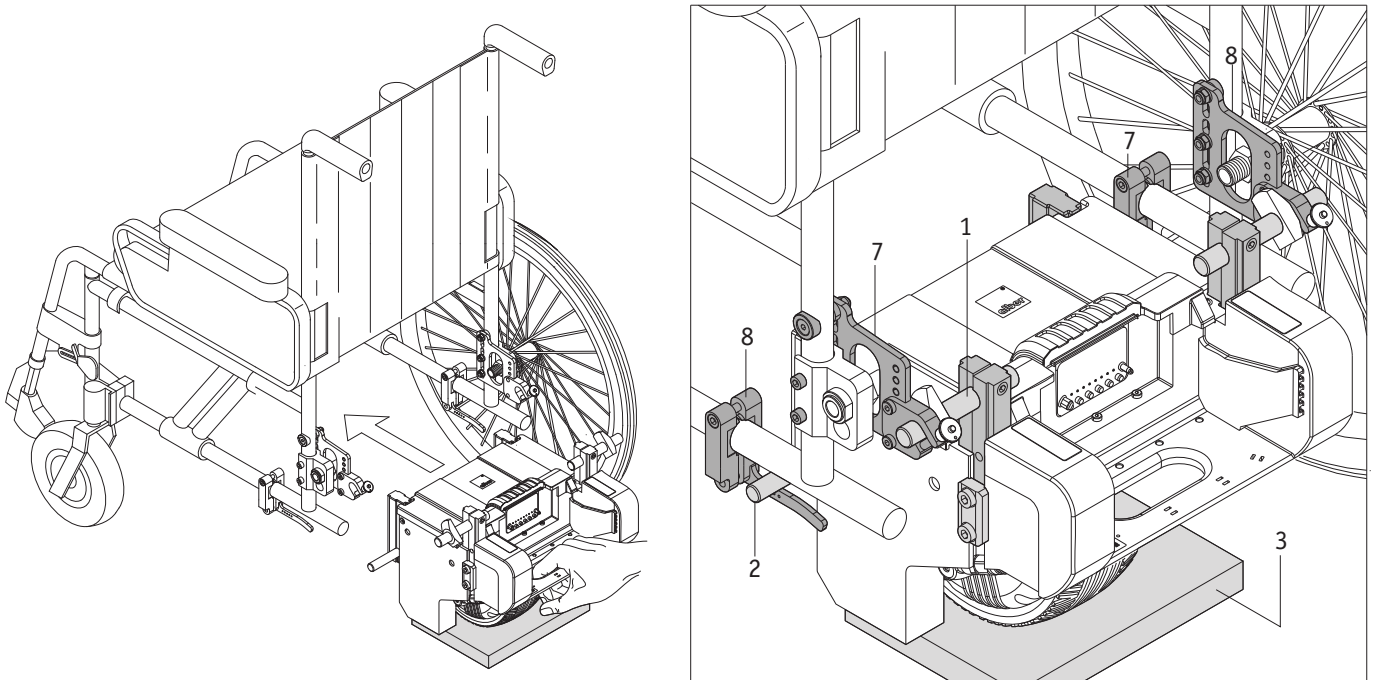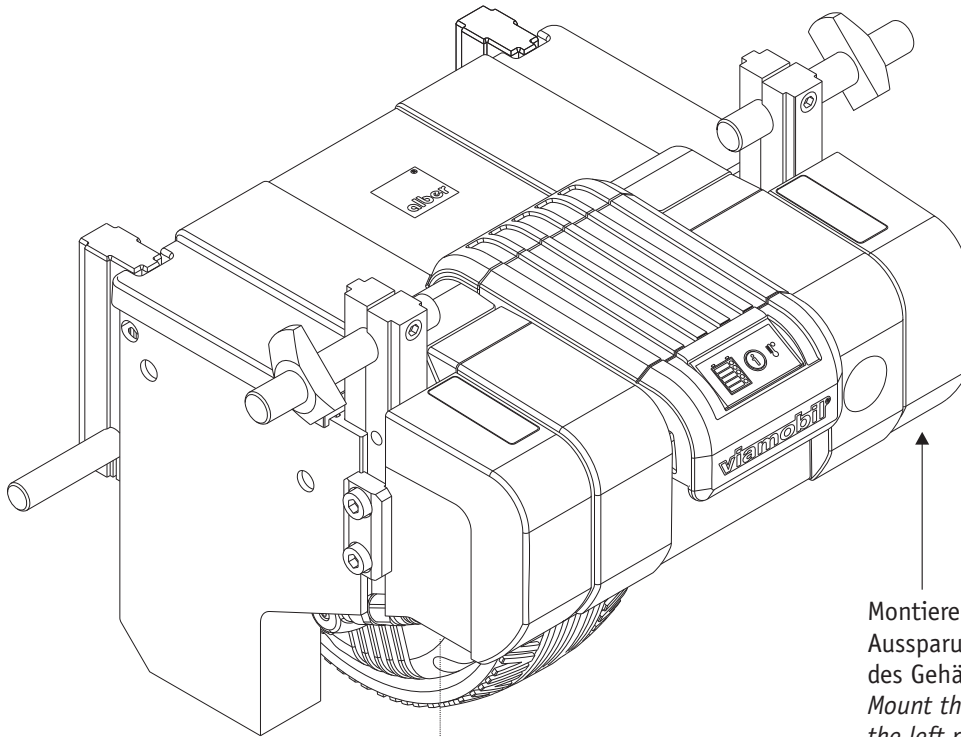
### ***Please pay attention to the following instructions when mounting the viamobil:***

- *The manual wheels of the wheelchair need to be fixed on the chair to adjust the viamobil. Please check the air pressure of the tyres.*
- *While adapting the viamobil to the chair, the profiles should still be height adjustable so the screws should not be tightened too much.*
- *Push the drive unit underneath the wheelchair. viamobil has to be switched off, thus the drive wheel must not be pulled out.*
- *Insert the lower bolts in the respective part of the mounting bracket [8].*
- *Insert the upper bolts in the fork of the upper mounting bracket [7] and fasten them by using the Quickpins.*
- *Put a 19 mm thick plate [3] (e. g. made of wood or steel] under the viamobil drive wheel. Alternatively, you can use the Alber mounting aid (part no. 1521404).*
- *Align the viamobil drive unit using a spirit level.*
- *Fix the bolt receiving profiles in this position. Tighten the screws to a torque of 12 Nm.*
- *Remove the plate [3]. The drive wheel decompresses a little bit. The distance from the wheel to the ground must now be 15 mm to ensure safe traction when pushing and braking the wheelchair.*

## Montage der Kippstützen (optionales Zubehör) Mounting of the anti-tippers (optional accessory)

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



1. Add all parts and screw the screw (no more than three turns).
2. Insert the parts in the housing of the viamobil.
3. Tighten the screw, tightening torque: 4 Nm

## Montage des Bediengeräts am Rollstuhl Mounting of the control unit at the wheelchair

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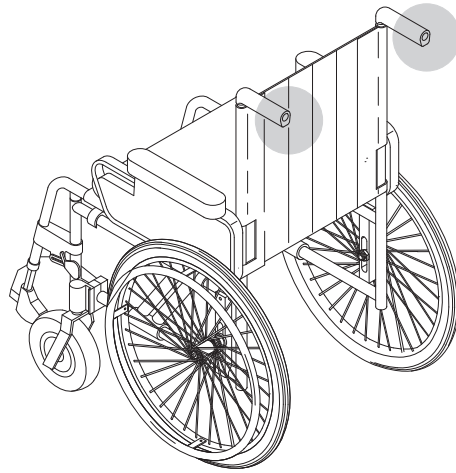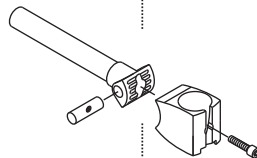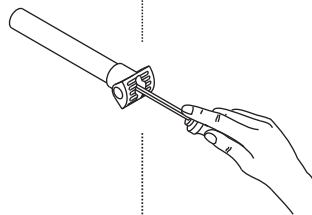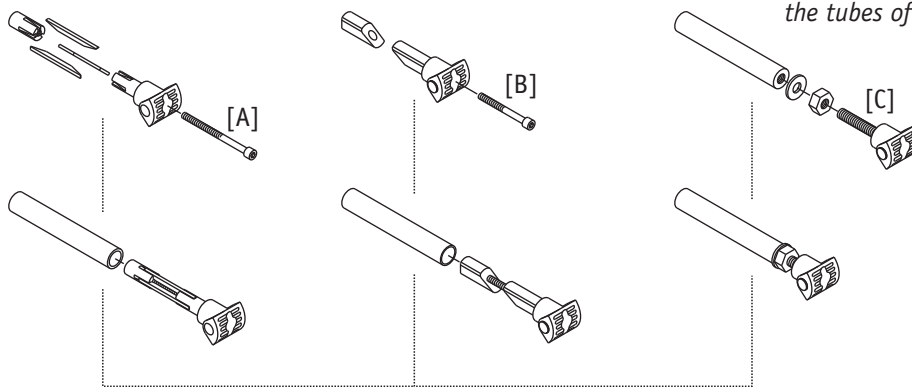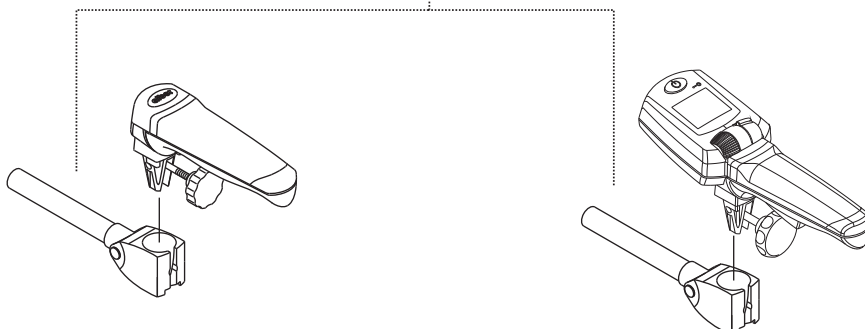
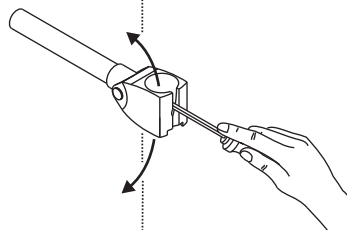
[A], [B] and [C]:  
The tightening torque depends on  
the material and the thickness of  
the tubes of the wheelchair.

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**Ulrich Alber GmbH**