

Adjusting Seat Fore/Aft, Backrest Angle – Force R

NOTE: For this procedur, refer to FIGURE 6.7R.

NOTE: Seat should be adjusted so that when seated with feet in the footrests, there is a slight bend at the knee and a slight bend at the elbow when the handpedal is furthest away.

1. Remove the seat upholstery. Refer to Replacing Seat Uphostery on page 61 of the XLT Owner's Manual (Part No. 1146024).
2. Loosen, but DO NOT remove the four mounting bolts (denoted 'A') securing the two back clamps to the seat rails. Loosen, but DO NOT remove the two bolts (denoted 'B') securing the backrest adjustment tube to the backrest and the camber tube.
3. Slide back clamps forward or rearward along the seat rails to desired seat position. Slide the backrest adjustment tube to desired backrest angle.
4. Tighten the four mounting bolts (denoted 'A') securing the two back clamps to the seat rails. Tighten the two bolts (denoted 'B') securing the backrest adjustment tube to the backrest and the camber tube.
5. Reinstall the seat upholstery. Refer to Replacing Seat Upholstery on page 61 of the XLT Owner's Manual (Part No. 1146024).

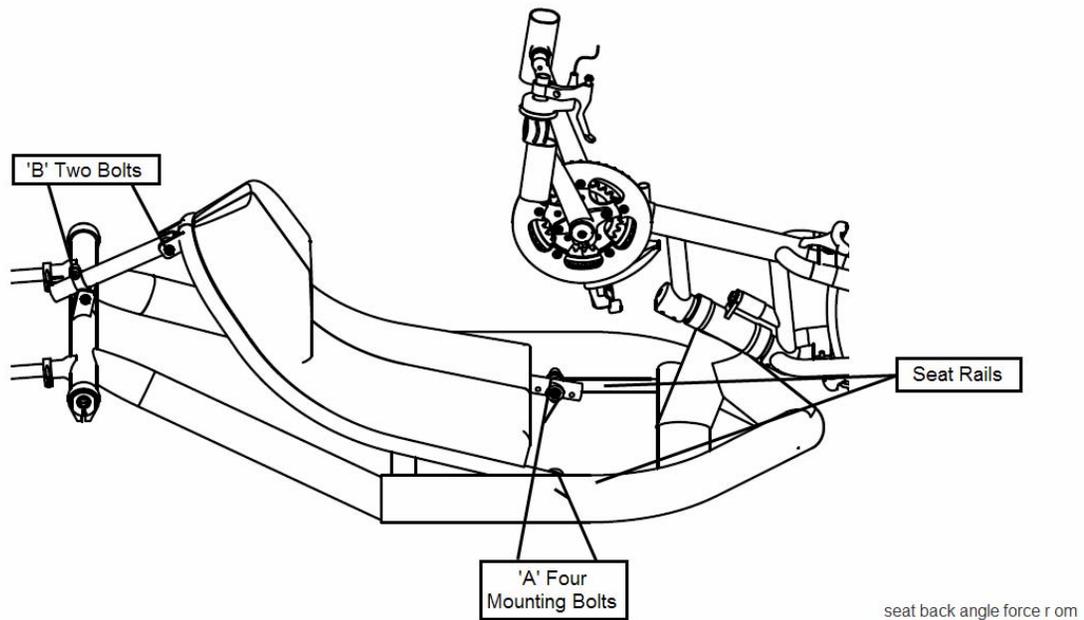


FIGURE 6.7R Adjusting Seat Fore/Aft, Backrest Angle – Force R