

Unit no.: A10 _____

Specialist supplier: _____



Ulrich Alber GmbH
 Vor dem Weißen Stein 21
 72461 Albstadt
 Telefon 07432/2006-0
 Telefax 07432/2006-299

Work	Check	Comments
------	-------	----------

Power-assisted wheels

- Clean stub axle (WD 40 spray oil)
- Check tires for damage and wear
- Replace damaged tires
- Check tire pressure
- Check wheel rims, stub axle for damage
- Replaced damaged wheel rims, stub axle *
- Check plug-in contact for damage
- Send drive with damaged plug-in contact to works
- Check running noises
- Send to works if unidentified running noises are detected
- Check and tighten screws

Steering wheels

- Check stub axle functionality, clean (WD 40 spray oil)
- Check tires for damage and wear
- Replace damaged tires
- Check tire pressure
- Check wheel rims, stub axle for damage
- Replaced damaged wheel rims, stub axle *
- Check and tighten screws

Chassis

- Check and tighten screws
- Check front/rear steering for damage
- Replace upper front left, upper front right rocker arm *
- Replace lower front left, lower front right rocker arm *
- Replace rear left, rear right rocker arm *
- Adjust shock strut pre-tension to driver's weight
- Check wheel ejection mechanism functionality
- Clean wheel ejection mechanism
- Parking brake functionality check
- Check parking brake Bowden cables
- Replace parking brake Bowden cables
- Checking battery adaptation locking mechanism

Adventure A10

Unit no.: A10 _____

Specialist supplier: _____



Ulrich Alber GmbH
 Vor dem Weißen Stein 21
 72461 Albstadt
 Telefon 07432/2006-0
 Telefax 07432/2006-299

Work	Check	Comments
Lighting		
Check functionality	<input type="checkbox"/>	
Replace lamps	<input type="checkbox"/>	
Check glass / housing for damage / replace	<input type="checkbox"/>	
Replace glass / housing	<input type="checkbox"/>	
Interface		
Check wheel mounting plug-in contact for damage	<input type="checkbox"/>	
Replace power cable if the plug-in contact is damaged / send interface to works *	<input type="checkbox"/>	
Check battery contacts for damage	<input type="checkbox"/>	
Send interface to works if battery contacts are damaged	<input type="checkbox"/>	
Check power cable for damage	<input type="checkbox"/>	
Replace power cable if the power cable is damaged / send interface to works *	<input type="checkbox"/>	
Check control unit connection for damage	<input type="checkbox"/>	
Replace connection cable if the control unit connection is damaged / send interface to works *	<input type="checkbox"/>	
Check charger connection for damage	<input type="checkbox"/>	
Replace connection sleeve if the charger connection is damaged / send interface to works *	<input type="checkbox"/>	
Check fuses	<input type="checkbox"/>	
Fuses replaced	<input type="checkbox"/>	
Battery pack		
Check cells	<input type="checkbox"/>	
Check connections for wear	<input type="checkbox"/>	
Check fuses	<input type="checkbox"/>	
Replace fuses	<input type="checkbox"/>	
Check housing for damage	<input type="checkbox"/>	
Replace housing	<input type="checkbox"/>	
Control unit		
Check functionality	<input type="checkbox"/>	
Check cable and plugs	<input type="checkbox"/>	
Replace cable	<input type="checkbox"/>	
Check housing, keypad for damage	<input type="checkbox"/>	
Replace upper housing casing	<input type="checkbox"/>	
Replace lower housing casing	<input type="checkbox"/>	
Check joystick / corrugated retention *	<input type="checkbox"/>	
Replace joystick / corrugated retention *	<input type="checkbox"/>	

* Delete inapplicable

Unit no.: A10 _____

Specialist supplier: _____



Ulrich Alber GmbH
 Vor dem Weißen Stein 21
 72461 Albstadt
 Telefon 07432/2006-0
 Telefax 07432/2006-299

Work	Check	Comments
Seat unit		
Check functionality	<input type="checkbox"/>	
Check and tighten screws	<input type="checkbox"/>	
Replaced damaged components **	<input type="checkbox"/>	
Warning sign (at 12 km/h) should be fitted	<input type="checkbox"/>	
Charger		
Check functionality and cable	<input type="checkbox"/>	
Final test of complete vehicle		
Test drive with complete control unit functionality check	<input type="checkbox"/>	
Test parking brake (with occupant weighing min. 80 kg) on 20 % incline	<input type="checkbox"/>	
Documentation		
Replace "Technical safety check" label on unit	<input type="checkbox"/>	(1)

(1) Labeling with waterproof felt-tip pen.
 Mark date in "Next examination" field (interval 24 months)

! Technical safety checks conducted on the Adventure A10 should only be realized by skilled personnel.

* Delete inapplicable
 ** See spare parts list

Date

Name

Technician's signature

