## L803 L839 L840

<b>F</b>	1.	Guide utilisateur
<b>GB</b>	2.	User's manual
<b>D</b>	3.	Gebrauchsanweisung
<b>E</b>	4.	Manual del usuario
<b>I</b>	5.	Manuale dell'utente
<b>NL</b>	6.	Gebruiksaanwijzing
<b>P</b>	7.	Guia do utilizador
<b>S</b>	8.	Bruksanvisning
<b>DK</b>	9.	Brugsanvisning
<b>FIN</b>	10.	Käyttäjän käsikirja
<b>N</b>	11.	Bruksanvisning

### I. FRANCAIS

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### 2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

#### The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- For reasons of hygiene, the mattress and pillows should always be used by the same person.
- As a primary precaution against bedsores, change the body's support points (as often as possible for the pillows and every three hours for the mattress) to avoid compressing blood vessels for too long at the bony protuberances and to assist circulation.

#### L803 - L839 OVERMATTRESS AND COMPRESSOR

##### Use

- Place the compressor flat or hang it with hooks on the end of the bed.
- Place the overmattress on the mattress, holding it in place by sliding the two 50cm flaps on the edges of the overmattress under the mattress at the head and foot of the bed.
- Connect the compressor to the overmattress using the two hoses. Check that the hoses are not kinked or caught under the mattress.
- Set the pressure to maximum. Connect the compressor to the electrical supply and set the switch to "ON". The green LED will light to indicate that the compressor is working. Wait 20 minutes for the overmattress to inflate; this can be done with the patient on the mattress.
- Adjust the pressure so that you can place two fingers between the patient's buttocks and an uninflated part of the mattress. The following table provides an idea of the pressure settings according to the weight of the patient.

Pressure	Weight (kg)	Pressure	Weight (kg)
1	30	5	70
2	40	6	80
3	50	7	90
4	60	8	100

- Warning: to maintain its efficiency, the overmattress must be covered with a flat or rubber sheet.
- Do not use near inflammable anaesthetics and periodically check the compressor's condition.

##### Technical details

- Electric cable length: 3m
- Airflow: 3.2 l/min.
- Complete cycle every 3 minutes

- Fuse: IA - 250V

The fuse is accessible from under the compressor using a screwdriver. Use a IA - 250 V replacement fuse. Contact your dealer immediately if you encounter any problem in operation. Do not attempt to open the compressor yourself.

#### Maintenance

When using the appliance at home, clean and disinfect the overmattress using normal disinfectant, always following the instructions for use (for example, cleaning with soap and water or alcohol at 70°). For use in hospital environments, follow the instructions of the infection control supervisor.

In the event of a puncture, deflate the overmattress and use the repair kit supplied with the overmattress. Degrease the vinyl cover to assist adhesion.

#### FRAMELESS WATER OVERMATTRESS (L840)

- The water mattress is designed for users weighing from 40 to 120 kg.
- Install the water overmattress on a 15cm foam mattress with a density of 35 kg.
- Lay the overmattress flat and fill all 3 sections with water. The amount of water required depends on the user's weight. It is advisable to add disinfectant to each section (see Maintenance). The water should be changed approximately every two months.

#### Maintenance

After use, rinse each section internally and externally with disinfectant (Actisept, Bactinyl, Sanicid, etc.). Talc the sections before storing them away from light, heat and humidity. A repair kit is provided. To carry out repairs, clean and dry the area around the hole, remove the cover from the adhesive patch and place the patch over the hole, pressing it down firmly for a few seconds. Wait two hours before re-using the mattress.

#### USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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