DE

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ES

Asiento plegable de ducha

Manual de instrucciones

EN

Folding shower seat

Operating instructions

SV

Fällbar duschsits

Bruksanvisning

FR

Strapontin de douche

Mode d'emploi

FI

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Käyttöohje

NL

Opklapbaar douchezitje

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NO

Dusjklappsete

Bruksanvisning

IT

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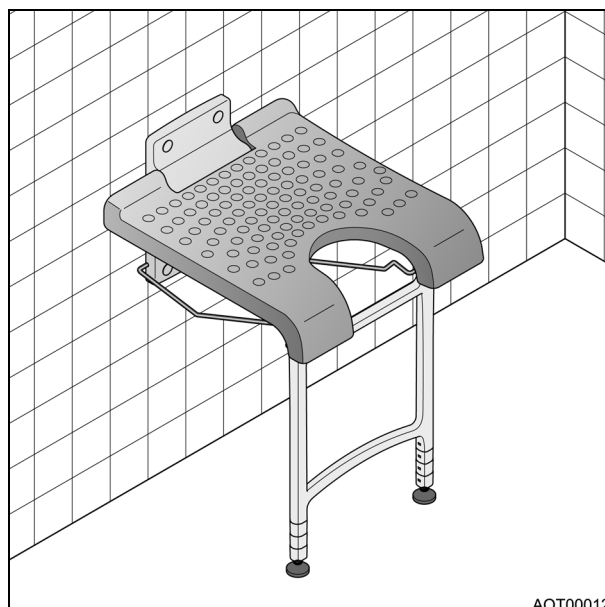
DA

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DE

WICHTIG!

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Contents

1	General instructions	9
1.1	Introduction	9
1.2	Proper use	9
1.3	Warranty	9
1.4	Re-use/Reconditioning	9
1.5	Disposal	9
1.6	CE marking	9
2	Safety instructions	9
2.1	Warnings and symbols	9
2.2	General instructions	9
3	Description	9
3.1	Scope of delivery	9
3.2	Technical data.	10
4	Setting up	10
4.1	Determining the seat height	10
4.2	Attaching the adapters	10
4.3	Fitting the feet.	10
4.4	Mounting the seat on the wall	10
4.5	Adjusting the feet	11
5	Use	11
6	Care	11

1 General instructions

1.1 Introduction

These operating instructions contain information and instructions regarding the safe and proper use of your folding shower seat.

In the text, we refer to figures and individual items within the figures. These references are shown in brackets.

Example: (3, fig. 1) refers to figure 1, item 3.

Keep all documents supplied in a safe place.

1.2 Proper use

The folding shower seat is designed only as a seating option for showers.

Any other use is prohibited.

Do not use the folding shower seat as a climbing aid or stepladder.

The folding shower seat can be safely mounted on walls made of concrete, solid bricks, gas-aerated concrete or hollow bricks. If the walls are dry mortarless constructions, you can only mount the folding shower seat if you use a strengthening plate for wall-hanging loads.

1.3 Warranty

The warranty depends on the legal specifications in the country of destination or on the specifications made by our sales partners.

In the event of damage, please notify your specialist dealer or contact one of the addresses printed in these operating instructions.

The warranty period begins at the delivery date as shown on the delivery note.

1.4 Re-use/Reconditioning

The product is designed for repeated use. The maximum number of times you can re-use the product depends on its general state and condition.

On reconditioning, always follow the **AQUATEC** reconditioning and hygiene instructions. These are available on request from **AQUATEC**.

1.5 Disposal

Important!

The product does not contain any hazardous substances.

Always dispose of the individual parts at your local facility according to the material identification labels on the parts.

Proper and complete disposal can be carried out by the manufacturer.

1.6 CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

2 Safety instructions

2.1 Warnings and symbols

The following warnings and symbols are used in these operating instructions:

*	CAUTION!
	This notice indicates a potential hazard. Not following these instructions can result in injury or damage.

Important!

This symbol indicates additional instructions, information or tips.

2.2 General instructions

Only use the folding shower seat for sitting in the shower.

Do not climb on to the folding shower seat. There is danger of falling.

Observe the information on the label. Do not overload the folding shower seat.

If you detect any cracks or damage, please contact your authorised dealer immediately.

3 Description

3.1 Scope of delivery

The **AQUATEC SANSIBAR** is supplied with the following parts (fig. 2):

- 1 Drilling template
- 2 Operating instructions
- 3 Pre-assembled seat
- 4 Decorative plugs (5 x)
- 5 Adapters (6 x)
- 6 Feet, consisting of base plate and block with thread (2 x)

* In the text, a picture illustrating the type of risk replaces the asterisk.

3.2 Technical data

Seat width:	400 mm
Seat depth:	410 mm
Seat height (with four levels):	460 - 530 mm
Weight:	2.6 kg
Maximum load capacity:	130 kg

The label (fig. 1) provides important technical information.

The label is attached under the seat.

4 Setting up

4.1 Determining the seat height

	CAUTION!
	Danger of slipping due to inclined seat Calculation and setting of the seat height must be carried out by a specialist.

Important!

Both of the user's feet must stand on the ground when seated (figure 3).

1. Choose a normal chair on which the user can sit comfortably.
2. Let the user sit down on the chair.
3. Measure the height of the seat surface (figure 3).
4. Use the following table to determine the number of adapters required for the folding shower seat.

Height of seat surface	Number of adapters
Up to 460 mm	No adapter necessary
461 to 484 mm	1 adapter
485 to 507 mm	2 adapters
508 to 530 mm	3 adapters

4.2 Attaching the adapters

Important!

Prior to assembly, determine the number of adapters as described in section 4.1.

1. Insert the adapter from below into the leg frame or into an adapter already fitted (figure 4) until the lug engages in the catch.
2. Check that the adapter is firmly fitted. To do so pull the adapter.

4.3 Fitting the feet

Important!

- If adapters are used, insert the foot last of all (figure 4).
 - The base plate (2, fig. 4) must be almost entirely screwed into the block with the thread (1, fig. 4).
1. Insert the foot (4, fig. 5) from below into the leg frame (1, fig. 5) or into an adapter already fitted, until the lug (3, fig. 5) engages in the catch (2, fig. 5).
 2. Check that the foot is firmly fitted. To do so pull the foot.

4.4 Mounting the seat on the wall

	CAUTION!
	Danger of slipping due to loose folding shower seat <ul style="list-style-type: none"> ● Mount the folding shower seat on dry mortarless walls only in combination with a strengthening plate for wall-hanging loads. ● The folding shower seat must be mounted by a specialist.

Important!

The choice of fastening material depends on the condition of the wall. No fastening material is supplied.

1. Select five screws and washers made of stainless steel and five wall plugs.
2. Place a rail over the shower basin and determine the basin depth (T) (fig. 6).
3. Calculate where the drilling template needs to be cut off (fig. 7).

To do so use the following formula:

$$H = 291 \text{ mm} + (\text{number of adapters} \times 23 \text{ mm}) - T \text{ (in mm)}$$

H is the calculated cutting height; the drilling template must be cut off here.

291 mm is height of the leg frame with foot.

23 mm is the height of an adapter.

T is the calculated basin depth.

Calculation example with 2 adapters and 140 mm basin depth (T):

$$H = 291 \text{ mm} + (2 \times 23 \text{ mm}) - 140 \text{ mm} = 197 \text{ mm}$$

4. Enter the calculated height H in the measurement table on the drilling template.
5. Cut the drilling template off at the marked line.

Important!

- If possible, use the tile seams for drilling so as not to damage the tiles.
- Ensure horizontal alignment.

6. Use the cut drilling template to mark the position of the four holes for the wall holder. To do so place the drilling template on the edge of the basin with the measurement table facing downwards and set it up against the wall.
7. Drill the holes and insert the wall plugs.

Important!

The bracket must be in the recess between wall holder and wall.

8. Screw the wall holder tight.
9. Insert the decorative plugs (4, fig. 2).
10. Carry out the fine adjustment of the feet as described in section 4.5.
11. Carry out a functional test. To do so fold the folding shower seat up and down several times as described in section 5.

4.5 Adjusting the feet

CAUTION!

Danger of slipping due to inclined seat
Adjust both feet to the same length.

1. Fold the folding shower seat down.
2. Loosen the conternut (2, fig. 8) by turning it in clockwise direction or turn it down.
3. Rotate the base plate (3, fig. 8) in clockwise direction until it touches the ground.
4. Check if the seat surface is in horizontal position.
5. If required, readjust the foot.
6. Turn the conternut in clockwise direction upwards and tighten lightly with screw-wrench (13 mm).

5 Use

CAUTION!

Catching in the folding mechanism

Grasp the seat at the seat surface as far to the front as possible to fold the seat down or up.

Make sure that the seat is snapped open on rear cylinder and leg frame.

Do not burden the seat surface on one side, e.g. by bracing yourself when sitting down.

1. Grasp the seat from the outside at the front round sections (fig. 9).
2. Move the seat down until the feet are on the floor of the shower basin.
3. Sit down in the middle of the seat surface.

After use:

4. Move the seat up until the seat engages in the upper holding position.

6 Care

Important!

- Regularly clean the folding shower seat and keep it clean.
- Do not use abrasive cleaning agents.

1. Wash the individual parts with commercial detergents and disinfectants using a cloth or brush.
2. Rinse the parts under warm water.
3. Dry the parts with a cloth.

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© 2005 AQUATEC GmbH D-88316 Isny
CLARKE HEALTH CARE PRODUCTS Inc.