Invacare® **FUTURA®**

Yes, you can.

Manuel d'utilisation
User guide
Gebruiks en instellingshandleiding
Bedienungsanleitung

Chère Cliente, cher Client

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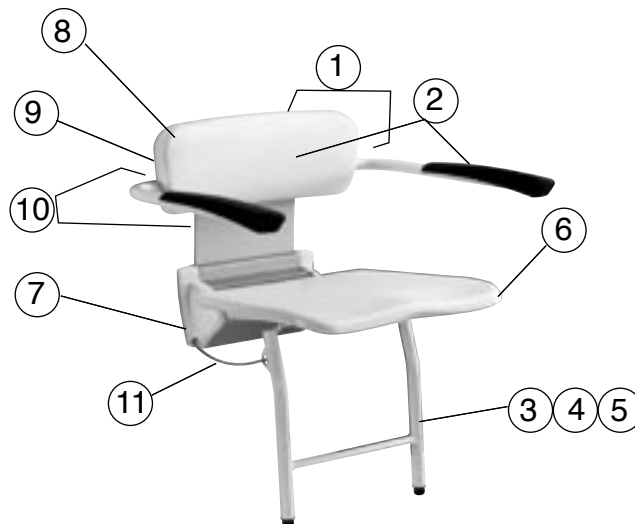
Dear Customer

Thank you for purchasing an Invacare product. This model was designed to provide you with all the benefits and features to meet your needs. Only quality components were selected for your wheelchair based upon rigorous inspections during the entire manufacturing process.

Foreword

For any question or assistance you may need, please apply to your local Invacare[®] Dealer. He has the required qualification, means and knowledge to provide you with the best service with regard to your Invacare[®] product.

8

The information contained in this manual is subject to change without notice. Some information is submitted under copyright – all rights reserved. Any information in this document cannot be photocopied or duplicated without prior written authorization by Invacare[®].

Stamps of the Distributor

SUMMARY

1. General	9
2. Introduction	10
3. Warranty	10
4. General safety	10
5. Maintenance	11
6. Installation	11
7. Technical specifications	14
8. Options	14
9. Spares parts	15

1. General

The shower seat provides a technical aid to make it easier the washing and care of patients (and residents in wet cells).

The following shower seat models are available:

- Basic model (no backrest); type R8802
- Basic model with backrest; type R8803
- Basic model with backrest and armrests; type R8804
- All basic models are available with optional adjustable support leg (fitting accessory R850, R860 or R865)

2. Introduction

Thank you for purchasing an INVACARE quality aid. Our products are designed and manufactured according to the most rigorous safety and quality standards, and meet relevant European Directives (MDD 93/42/EEC).

It is imperative to comply with the following instructions for use and maintenance:

- Check that the product is delivered in good condition before use.
- If any doubt regarding use restrictions, adjustment or maintenance, please contact your local Invacare dealer.
- Do not exceed the maximum user weight as indicated in the table below or the load weight stated on the product label.
- Always make sure that the product meets your needs. If any doubt, please apply to a qualified health advisor.
- Worn, damaged or missing parts should be replaced by a registered repair specialist.
- Do not leave or store the product outside.
- Regularly check if the product is stable and rigid enough and if needed, check screw tightening and rubber cap condition.

3. Warranty

All Invacare toilet aids come with a 2-year manufacturer's warranty.

Normal wear and tear and improper use do not fall under the terms of this warranty.

Invacare does not accept any liability for personal injury which is caused by improper use of the shower seats.

4. General safety

Ensure that the shower seat is installed in keeping with the specifications and drawings attached.

The shower seat should preferably be installed by a qualified workman.

For the installation you should use the R6600 tool set provided by Invacare.

Wall attachment:

The shower seat should be attached to a wall with sufficient load-bearing strength (brickwork, breeze block or similar). If not, you should use an adjustable support leg (fitting accessory R850, R860 or R865); it is also possible to attach the seat by means of a counter plate (type R6630), only if the wall is thick enough (see figure 4)

In the event of queries or defects, please contact your local Invacare dealer.

 **In order to prevent injuries, keep your fingers away from the hinge when you put the seat down.**

5. Maintenance

The shower seat and accessories can be rinsed down after use with water and normal household detergents; but do not use abrasive products or aggressive solvents for this purpose. For disinfection within a hospital, please follow the hygiene specialist's recommendations.

6. Installation

Please follow the steps described below for wall attachment of the Invacare Futura shower seat

- Unscrew the 2 screws which are located under the fixing plate. Hold this plate and then take it off the seat (see figure 1)
- To determine the user's ideal sitting height ("H"), measure the height between the user's foot (without shoe) and back knee (see figure 2). See product options on page 8.

H	R855	R860	R865	R8802	R8803	R8804
$H < 300$				X	X	X
$300 \geq H \geq 350$				X	X	X
$350 \geq H \geq 400$	X			X	X	X
$400 \geq H \geq 450$	X	X		X	X	X
$450 \geq H \geq 650$	X	X	X	X	X	X
$650 \geq H \geq 700$		X	X	X	X	X
$700 \geq H \geq 750$			X	X	X	X
$H > 700$				X	X	X

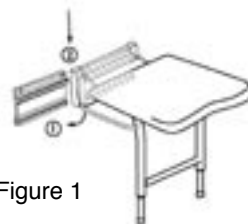
Figure 1

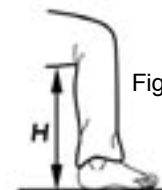
Figure 2

- To attach the shower seat to the wall, take the fixing plate to mark the holes out on the wall.
 - Then place the plate at (H) height that has been determined (see point 2)).
 - Make sure the plate is level : all marks should be at the same height.
 - Drill the holes (drill diameter = 10 mm) in the wall and insert raw plugs.
- Screw the plate on the wall preferably using the screws supplied in the standard tool set.
(except for R6600 model) (see figure 4)

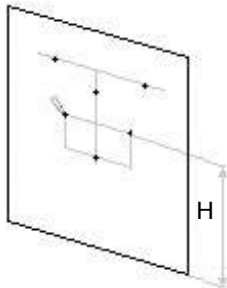
Figure 3

Figure 4

- Install the seat by sliding it into the fixing plate, push the plate downwards and insert the set screws back. (see figure 5)
- Fit the steel section provided with the set to hide the fixing screws. If necessary, install the plate which links the backrest to the seat (see figure 6)
- To attach the backrest, take it out of the fixing plate by pushing it upwards. Then slide the backrest fixing plate into the linking plate rail.
- Use the fixing plate as a template to mark the holes out on the wall.

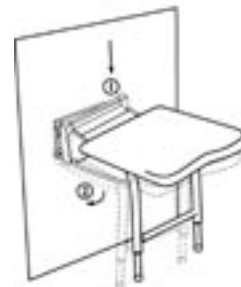
Figure 5

Figure 6

- Make sure they are level (see figure 7).
Cover the seat which you have already installed to protect it against drilling dust.

- Drill the holes in the wall (drilling diameter = 10 mm) and fit the plate preferably using the screws included in the standard tool set (except for R6600 model)

Use the steel sections included in the tool set to hide the fixing screws (see figure 8)

Adjust to the right leg height by unscrewing the adjustable foot supports (if ordered) (see figure 9)

Figure 7

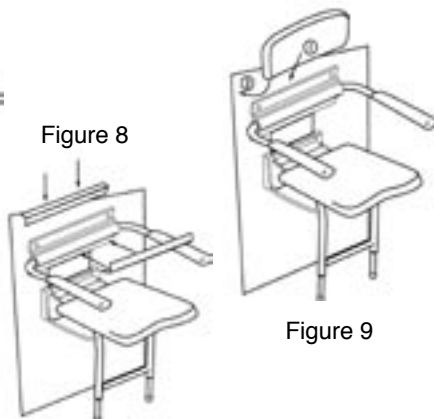
Figure 8

Figure 9

• Fitting the support legs after assembly.

- You have to order the adjustable support legs corresponding to the chosen height (R855, R860, R865)

- Drill in the mark located on the right side of the seat to insert the support rod (see figure 10).

- Take the plastic caps off the seat underside to fit the support legs tubes. (see figure 11)

- Screw the tubes and adjust to the right leg height by screwing / unscrewing.

• Folding the seat

The seat can easily be lowered and raised. When in raised position, the seat should remain firmly in position. If not, please consult your dealer.

Figure 10


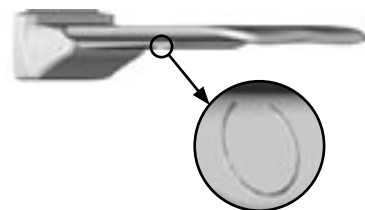
Figure 11

7. Technical specifications

- Armrest tubing with shockproof and impact-resistant Rilsan coating.
- Polyurethane armrests.

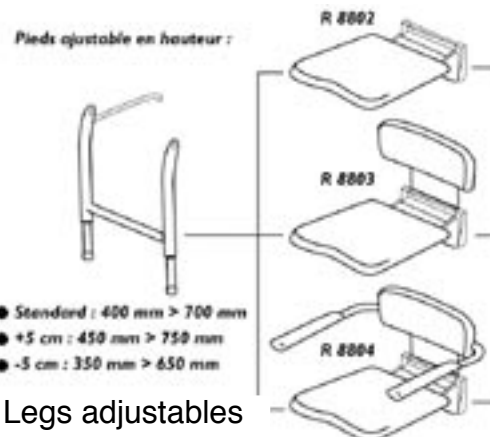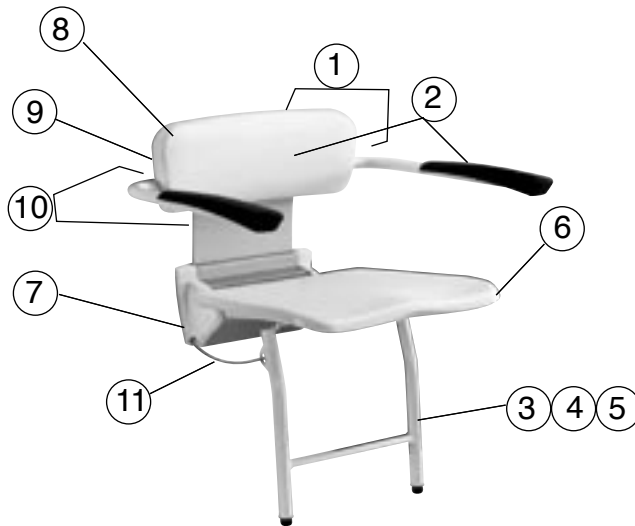
	R8802	R8802,60	R8803	R8803,60	R8804	R8804,60
	4,8 kg	5,9 kg	6,6 kg	7,7 kg	8,0 kg	9,1 kg
	135 kg	135 kg	135 kg	135 kg	135 kg	135 kg

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Invacare® Poirier SAS
Invacare® AB
Invacare® B.V.
Invacare® Ltd