
Invacare® Alizé

H2480/1

en	Shower Chair with Arm Rests User Manual	3
de	Duschstuhl mit Armlehnen Gebrauchsanweisung	5
fr	Chaise de douche avec repose-bras Manuel d'utilisation	8
it	Sedia da doccia con braccioli Manuale d'uso	10
es	Silla de ducha con reposabrazos Manual del usuario	13
pt	Cadeira de duche com apoios de braços Manual de utilização	16
nl	Douchestoel met armsteunen Gebruiksaanwijzing	18
sv	Duschstol med armstöd Bruksanvisning	21
fi	Käsinojallinen suihkutuoli Käyttöohje	24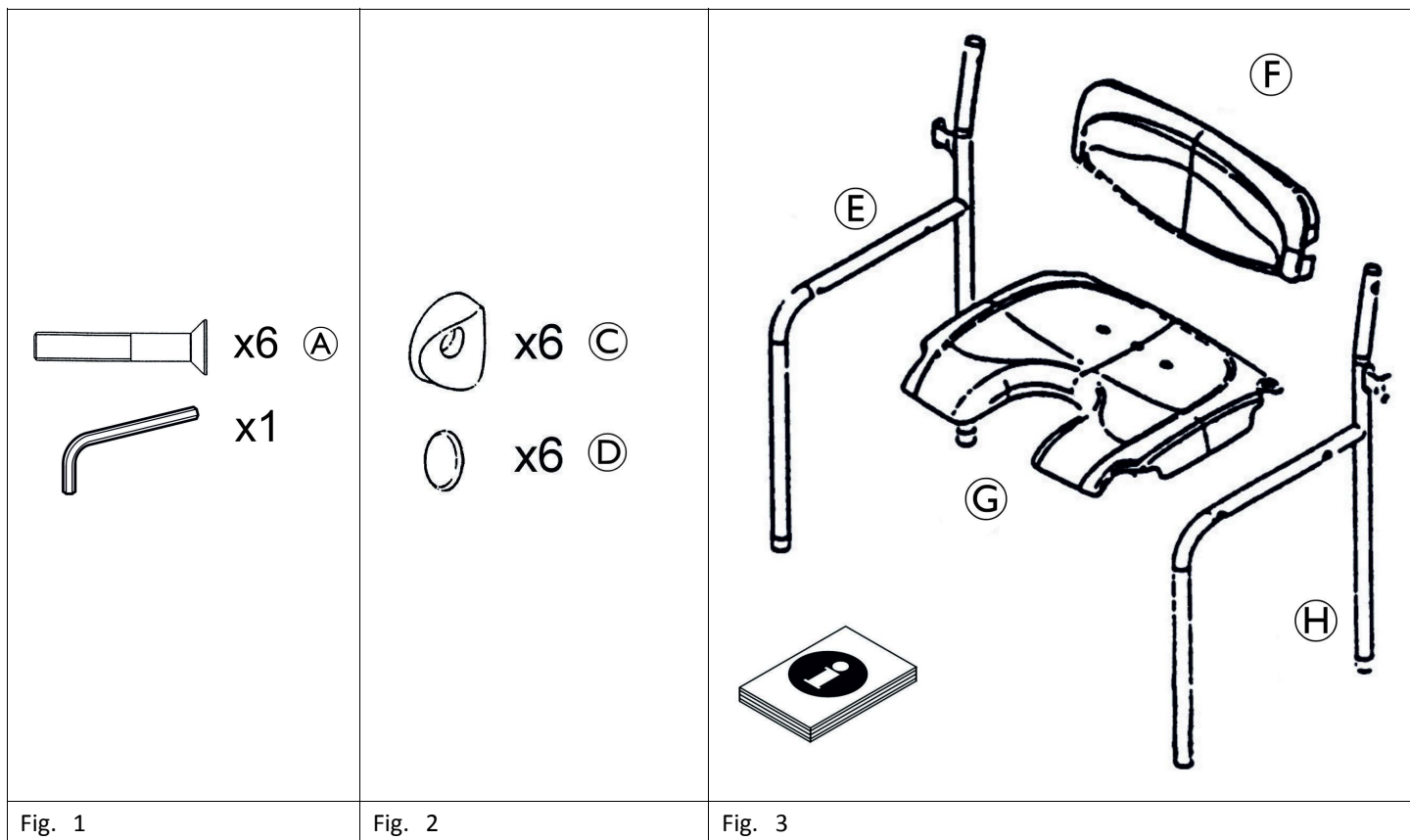

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

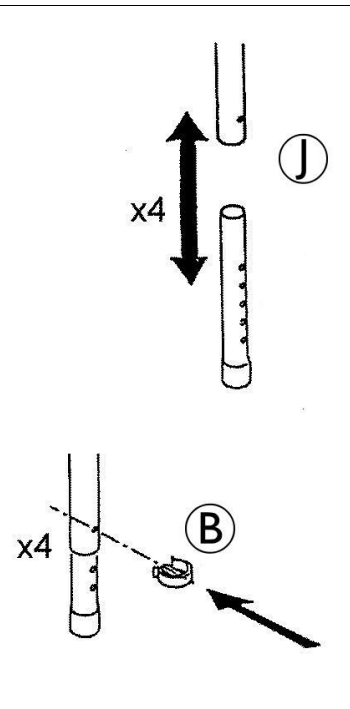
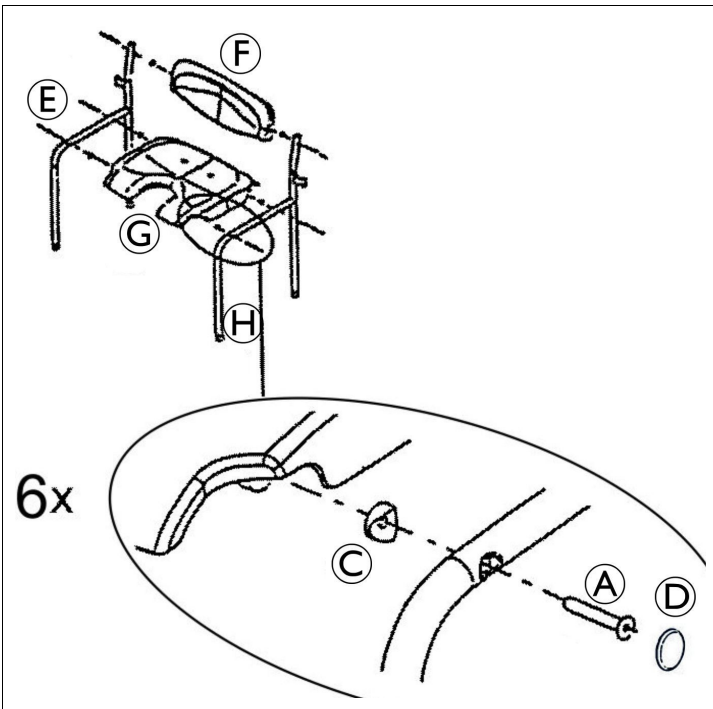
Fig. 4

Fig. 5

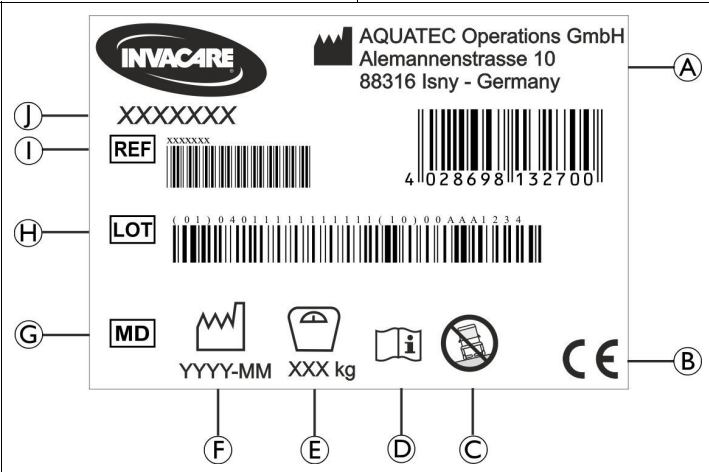
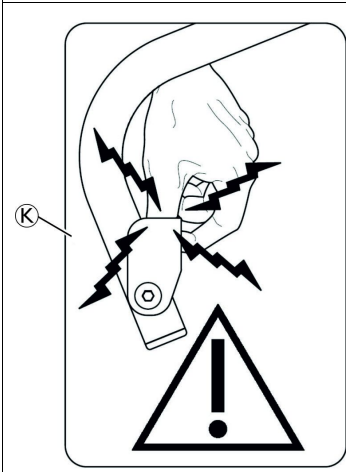
Fig. 6

Fig. 7

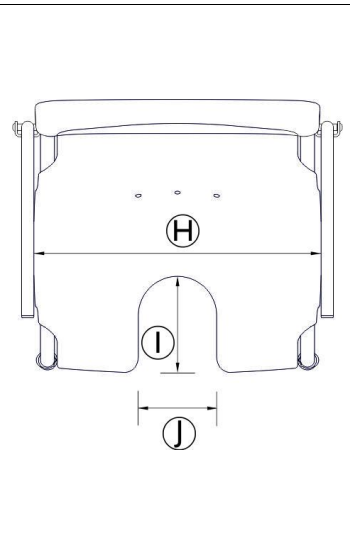
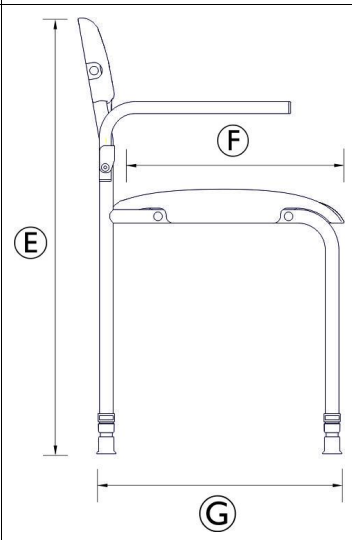
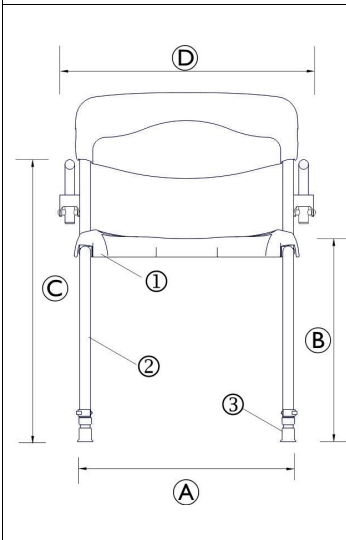
Fig. 8

Fig. 9

Fig. 10

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.
- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use or store the product outdoors.
- All four legs must be adjusted to the same height.
- All four leg tips must be in contact with shower / tub floor at all times and on even surface.

WARNING!

Risk of trapping Fingers!

- Do not keep your hand on the handle when raising the armrest. When lowering the armrests, ensure that your fingers are away from the hinges to avoid trapping them.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

- Non-original or wrong parts may affect the function and safety of the product.
- Only use original parts for the product in use.
 - Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 6, 7)

Ⓐ	Manufacturer	Ⓔ	Medical Device
Ⓑ	European Conformity	Ⓕ	Lot number
Ⓒ	For use on level ground only	Ⓖ	Reference number
Ⓓ	Read user manual	Ⓗ	Product designation

Ⓔ	Maximum safe load	Ⓚ	Warning label: Risk of trapping when folding the armrest
Ⓕ	Production date		

 The identification label is attached to the left rear tube of the product.

 The warning label “Risk of Trapping” is located on the inner side of the rear frame under the armrest on each side.

7 Scope of Delivery (Fig. 1 – 3)

- Ⓐ Screws (M6 x 50 mm) (6x)
- Ⓒ Saddle spacer (6x)
- Ⓓ Covering cap for screw (6x)
- Ⓔ, Ⓕ Frame (left and right)
- Ⓕ Backrest
- Ⓖ Seat plate
- Allen key no. 5
- User manual

8 Intended Use

This product is intended to be used as a shower aid.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips). If used without an assistant, users must be able to keep their own balance.

The product is intended exclusively for home use and is not intended for intensive use in institutions.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 120 kg.

9 Product Description

Invacare® Alizé H2480/1 is a shower chair made of powder-coated steel composed of four height adjustable anti-slip feet, two flip-up armrests and a plastic seat with hygiene opening and plastic backrest.

10 Assembling the Product (Fig. 4, 5)

CAUTION!

Risk of Trapping

- Take care during assembly.

1. Assemble both frames Ⓔ, Ⓕ to the backrest Ⓕ and to the seat Ⓖ (Fig. 4). Tighten the screws Ⓐ by hand.
2. Tighten all screws with the Allen key no. 5. Install the covering caps Ⓓ on the screw heads.
3. Remove the safety clip Ⓑ from the leg.
4. Adjust the desired height by moving the adjustment tube ① into or out of the leg.
5. Reattach the safety clips Ⓑ through the desired holes of the leg and the adjustment tube.
6. All four legs must be adjusted to the same height.

CAUTION!

Risk of Injury or Damage

- Make sure that all four clips are securely fastened.

11 Maintenance

11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

11.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning Instructions

IMPORTANT!

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

12 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection

- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

13 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

14 Technical Data (Fig. 8, 9, 10)

Dimensions			
Ⓐ	500 mm	Ⓕ	400 mm
Ⓑ	465 – 535 mm	Ⓖ	450 mm
Ⓒ	650 – 720 mm	Ⓗ	510 mm
Ⓓ	590 mm	Ⓘ	170 mm
Ⓔ	800 – 870 mm	Ⓣ	140 mm

Weight	
Shower chair	7.00 kg

Materials			
①	Steel, powder-coated	②	ABS
③	TPE		

All product components are corrosion resistant and do not contain natural rubber latex.

15 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Limited