



## Seat Support/ Plate

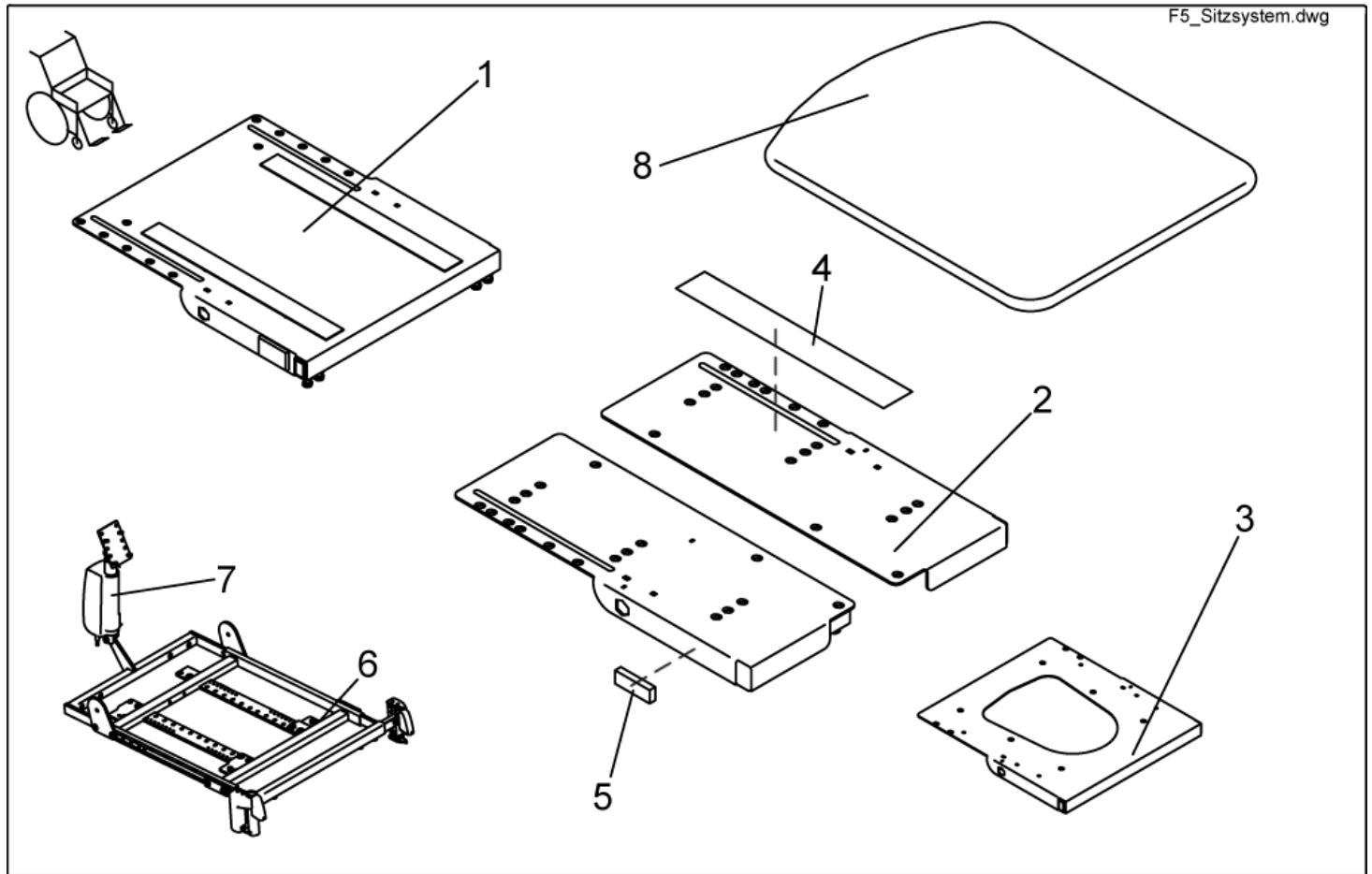

item	ordercode	description	quantity
1	SP1422060	SEAT PLATE (ONE PIECE) COMPLETE	1
1	1419999-9005	SEAT PLATE	1
1	SP1423205	SEAT MOUNTING KIT (ONE-PIECE VERSION)	1
2	SP1422121	SEAT PLATE, FLEX F. SB 39 CPL.	1
2	SP1422064	SPLIT SEAT SB 43 CPL.	1
2	SP1422065	SPLIT SEAT SB 48 CPL.	1
2	1420458-9005	SEAT PLATE LH CPL.	1
2	1420457-9005	SEAT PLATE RH CPL.	1
2	SP1423206	SEAT (SPLIT) ASSEMBLY KIT	1
2	1444732	ASSEMBLY KIT FOR SPLIT SEAT SB 43cm	1
2	1444733	ASSEMBLY KIT FOR SPLIT SEAT SB 48cm	1
2	1436355	INSTALLATION KIT BRACKET SEAT REINFORCEMENT (RH + LH)	1
3	SP1423183	SEAT PLATE RECARO CPL.	1
3	1423174-9005	RECARO SEAT PLATE	1
4	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 400	2
5	P31298	REFLECTOR, YELLOW	2
6	SP1428110	SEAT SUPPORT Optimist, Seat width 45cm, complete	1
7	1419417	MOTOR LA 28.1-100-24-001	1
8	5314532	SEAT PLATE 410 X 520 MM	1
8	5314533	SEAT PLATE 460 X 520 MM	1

# Mounting kit seat plate


1423205.dwg

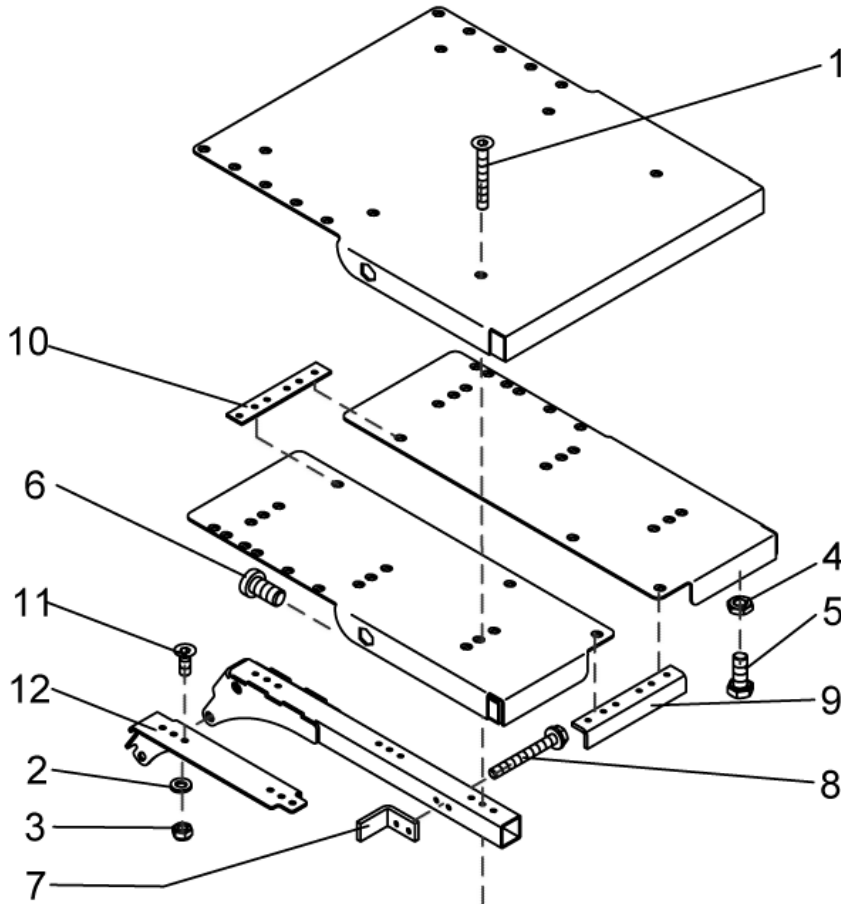

item	ordercode	description	quantity
1-8	SP1423205	SEAT MOUNTING KIT (ONE-PIECE VERSION)	1
1,2,3,4,5,6,7,8,9,10,11	SP1423206	SEAT (SPLIT) ASSEMBLY KIT	1
1,2,3,4,5,6,7,8,9,10,11	1444732	ASSEMBLY KIT FOR SPLIT SEAT SB 43cm	1
1,2,3,4,5,6,7,8,9,10,11	1444733	ASSEMBLY KIT FOR SPLIT SEAT SB 48cm	1
2,3, 11,12	1436355	INSTALLATION KIT BRACKET SEAT REINFORCEMENT (RH + LH)	1
1	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9	6
2	NA	WASHER 6,4 X 12,5 X 1,6	10
3	NA	NYLOC NUT M6	10
4	NA	HEX NUT M8	4
5	NA	HEX HEAD SCREW M8 X 20	4
6	NA	BOLT M10 X 20	2
7	1421700-9005	ANGLE BRACKET RH, SB 43	1
7	1421705-9005	SEAT ANGLE LH SB 48	1
7	1421701-9005	ANGLE BRACKET LH, SB 43	1
7	1421706-9005	SEAT ANGLE RH, SB 48	1
8	NA	BOLT M6 X 50	4
9	NA	ANGLE BRACE	1
10	NA	REINFORCING PLATE	1
11	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 16	4
12	NA	BRACKET SEAT REINFORCEMENT LH	1
12	NA	BRACKET SEAT REINFORCEMENT RH	1

# RECARO-Seat


F3\_RECARO-Sitz.dwg

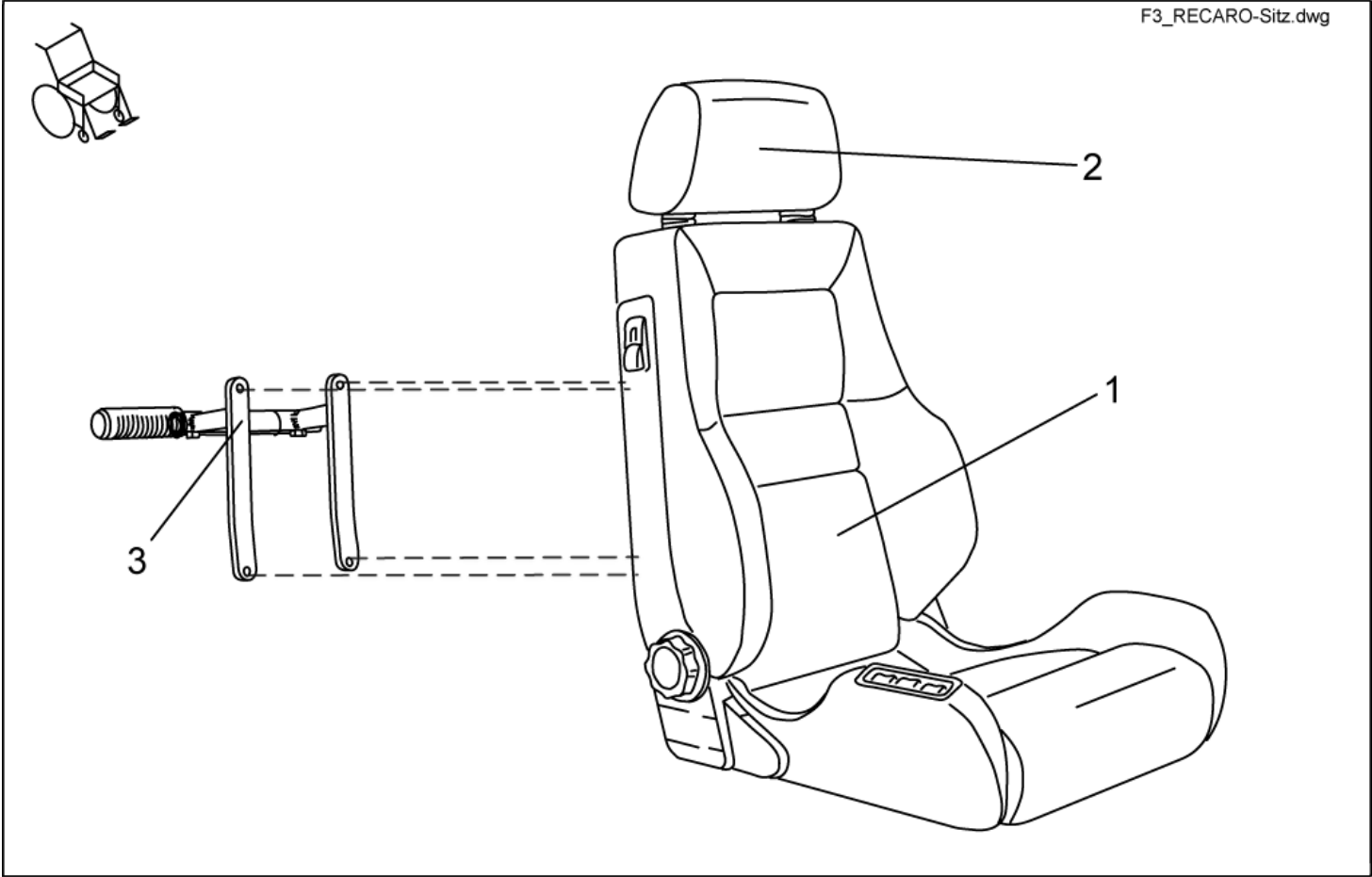

item	ordercode	description	quantity
1	SP1422546	RECARO SEAT N-JOY WITH MOUNTING KIT	1
1	SP1420495	RECARO SEAT MOBILITY WITH MOUNTING KIT	1
-	SP1423173	RECARO SEAT MOUNTING KIT	1
-	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO	1
-	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO	1
2	N89999	HEADREST - SPORT NARDO	1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	





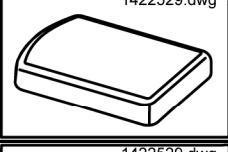
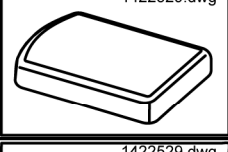
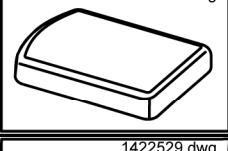
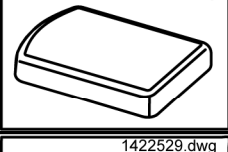
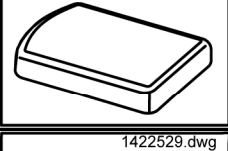
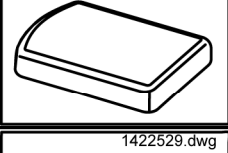
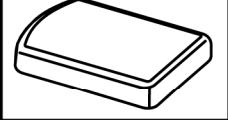
## Holder for push handle RECARO cpl.





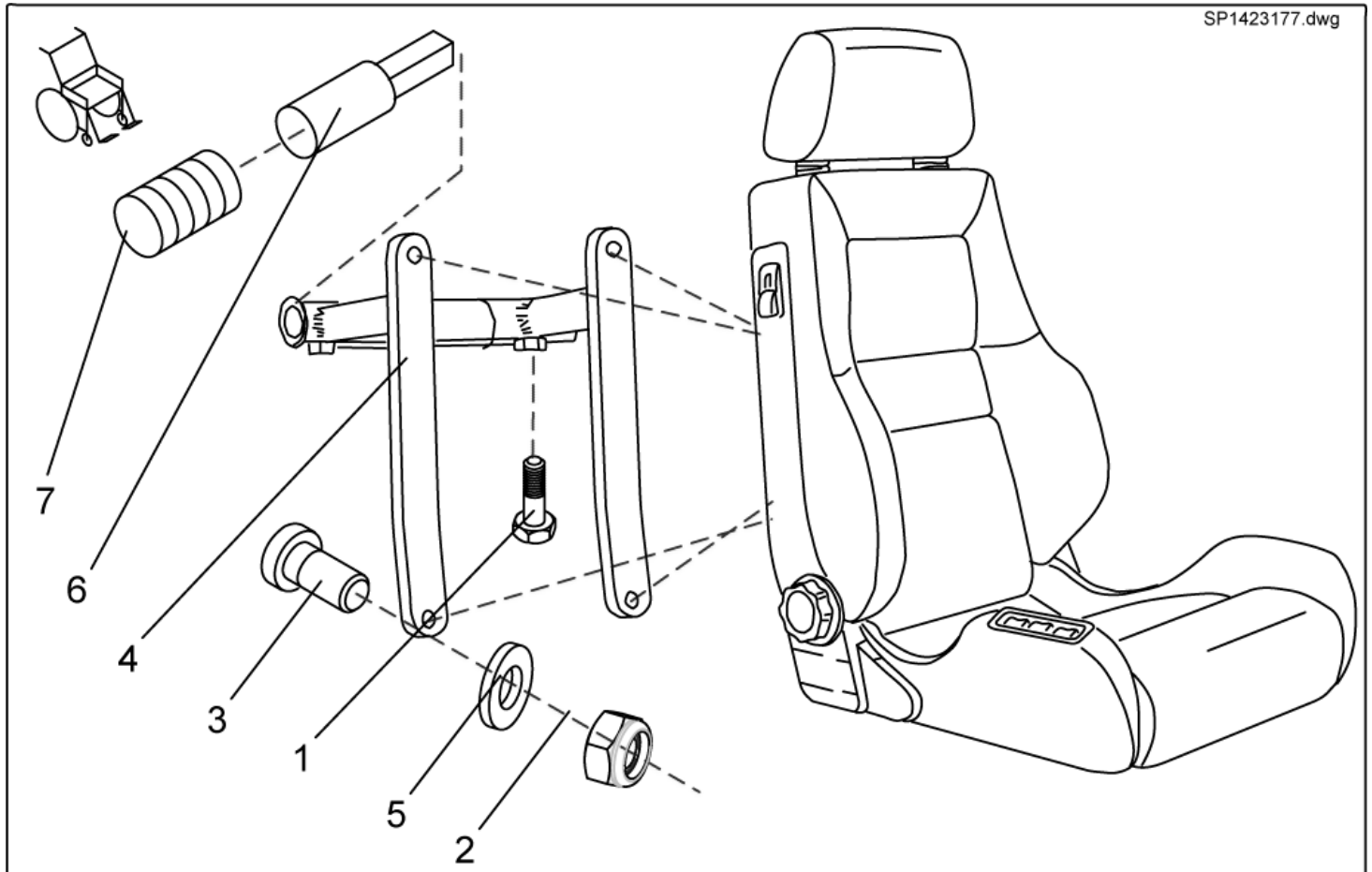



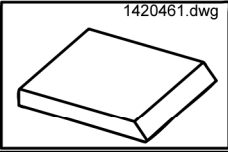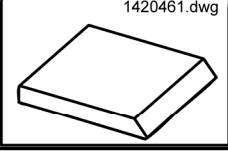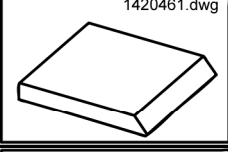


WE RECOMMEND RETROFITTING AT THE FACTORY!

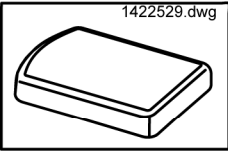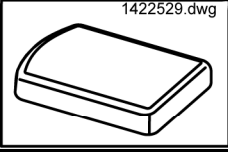





item	ordercode	description	quantity
-	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	1
-	NA	SEAT SETTING STRAP	1
-	NA	TUBE END PLUG SL 22	2
-	NA	TUBE END PLUG E 15 X 15	2
1	NA	HEX HEAD SCREW M8 X 20	2
2	NA	DIN 936 M8 17H BLACK GALVANISED HEX NUT THIN	2
2	NA	NYLOC NUT M6	2
3	NA	HEX HEAD SCREW M6 X 40	4
4	NA	HOLDER FOR PUSH HANDLE RECARO; RAL 9005, DEEP BLACK	1
5	NA	WASHER 6,4 X 12,5 X 1,6	4
6	NA	HANDLE; RAL 9005 DEEP BLACK	2
7	L10009	HANDLE	2

# Chushion

Picture	Seat system	Seat width cm	Seat depth cm	color	ordercode
	Standard	n.a.	41	black	1420460
	Standard	n.a.	46	black	1420461
	Standard	n.a.	51	black	1420462
	Flex	39	n.a.	Dartex grey	5252582-TR23
	Flex	39	n.a.	Vinyl grey	5314542
	Flex	39	n.a.	blue	1422529
	Flex	39	n.a.	black	1436561
	Flex	43	n.a.	Dartex grey	5252583-TR23
	Flex	43	n.a.	Vinyl grey	5314525
	Flex	43	n.a.	blue	1422530
	Flex	43	n.a.	blue	1436562



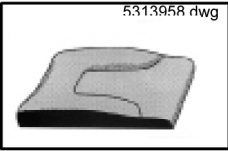
Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
	Flex	48	n.a.	Dartex grey	5252584-TR23
	Flex	48	n.a.	Vinyl grey	5314526
	Flex	48	n.a.	blue	1422532
	Flex	48	n.a.	black	1436563
	Flex Vic Air	43	n.a.	VicAir	1422531
	Kontur	43	n.a.	black	1422072
	Kontur	48	n.a.	black	1422074
	Sitzschutz Comfort Flex	39	n.a.	blue	5253078-TR25
	Sitzschutz Comfort Flex	43	n.a.	blue	5253079-TR25
	Sitzschutz Comfort Flex	48	n.a.	blue	5253080-TR25
	positiv	45	n.a.	black	1419632
	positiv	45	n.a.	black	5313953
	positiv	45	n.a.	black	5314038
	negativ	45	n.a.	black	1419633

Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
 <p>5313958 r/wg</p>	negativ	45	n.a.	black	5313958
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	1419634
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	5313962
 <p>5313962 r/wg</p>	paraplegic	45	n.a.	black	5314044

## Arne Jensen-Seats


Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
	1	46	38	-	1419630
	1	46	45	-	5313888
	1	60	45	-	5313889
	1	60	50	-	1419625
	2	46	38	-	1419631
	2	46	38	X	5313867
	2	46	45	-	5313882
	2	60	45	-	5313883
	2	60	45	X	5313895
	2	60	50	-	1419628
	3	60	45	-	5313878

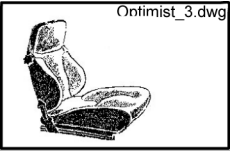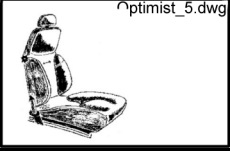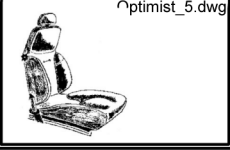

Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
	3	70	50	-	5313877
	5	60	45	-	5313872
	5	70	50	-	5313871

## Side Panels - Euro Side Panels


1429339.dwg

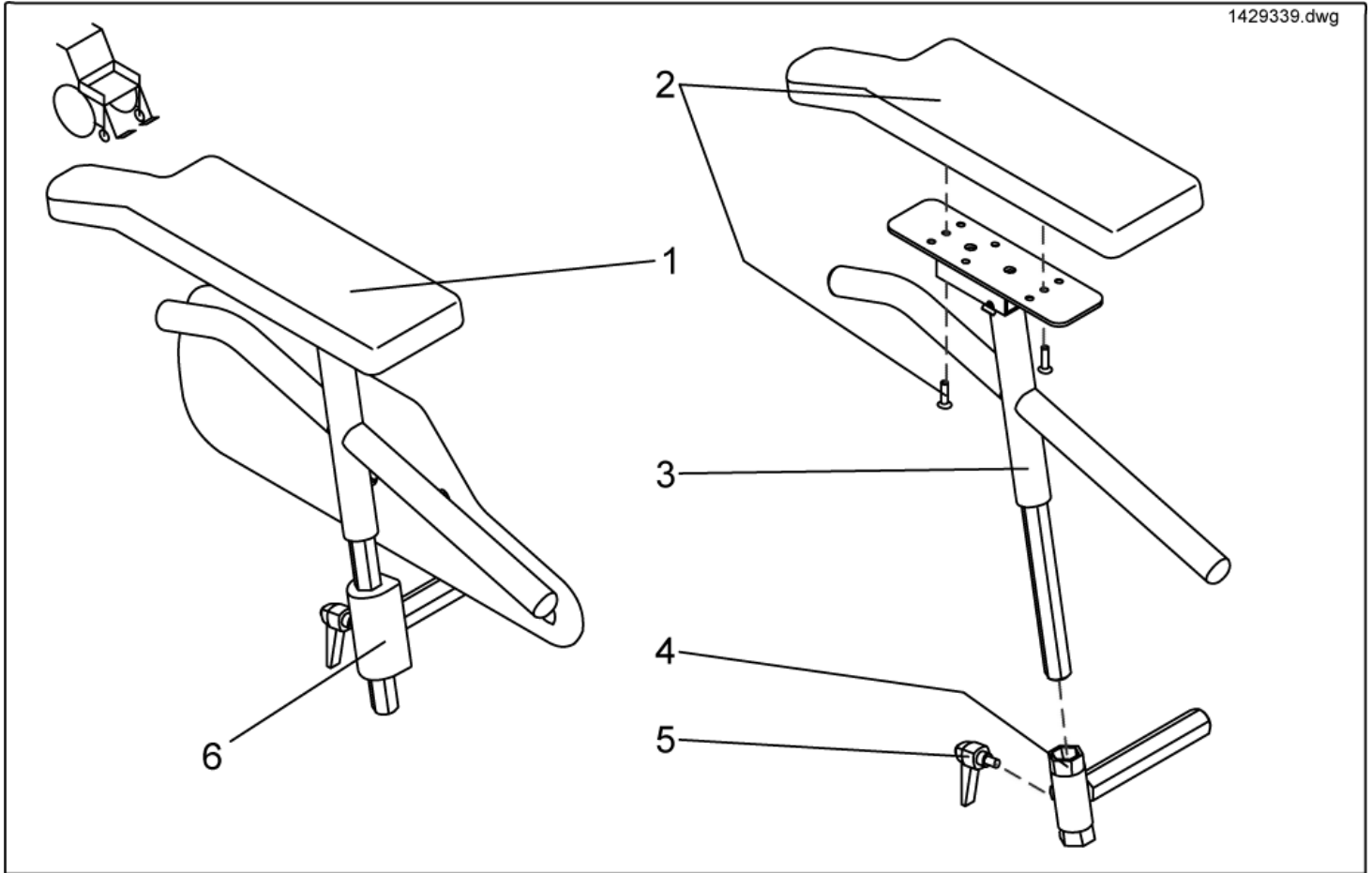

item	ordercode	description	quantity
1	1429339	ARMREST RH CPL. (EURO)	1
1	1429340	ARMREST LH CPL. (EURO)	1
2	SP1423224	ARMREST PAD LH CPL.	1
2	SP1423221	ARMREST PAD RH CPL.	1
3	1428474-9005	ARM SUPPORT LH CPL.	1
3	1428470-9005	ARM SUPPORT RH CPL.	1
4	1428459-9005	SIDE PANEL MOUNTING LH	1
4	1428465-9005	SIDE PANEL MOUNTING RH	1
5	L32902	LEVER M6 X 10; BLACK	2
6	1430359	Abdeckkappe Seitenteil RH, rechts	1
6	1430211	Abdeckkappe Seitenteil LH, links	1




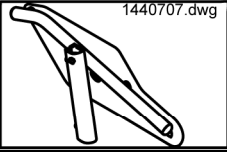
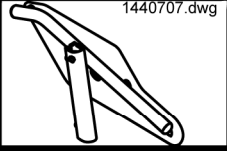

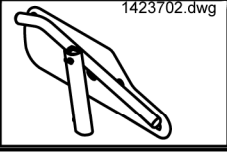
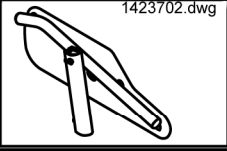

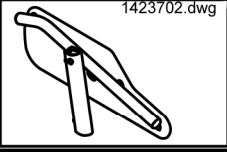
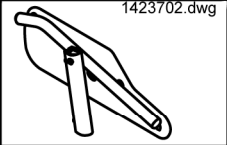

## Side Panels - Standard Side Panels

Seitenteile.dwg

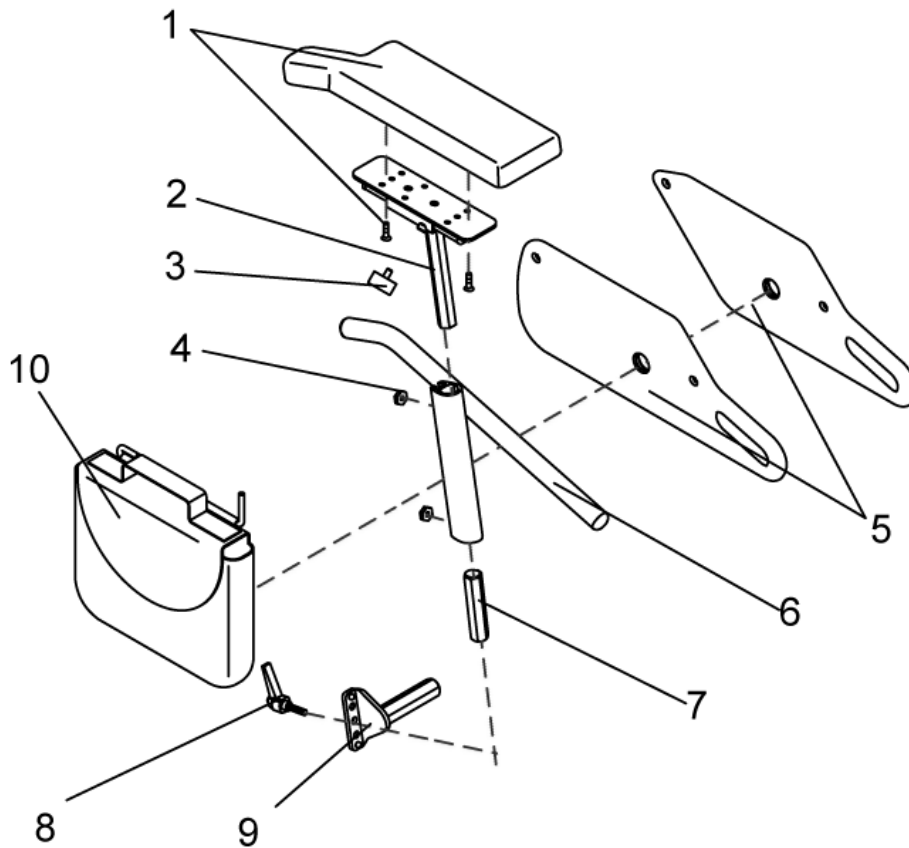





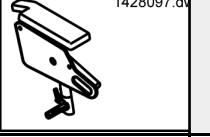
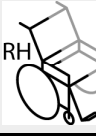
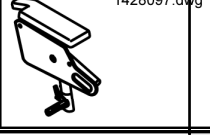

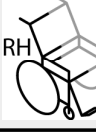
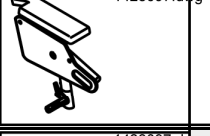

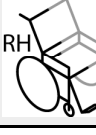

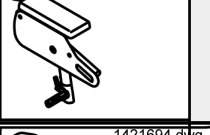
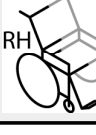



item	ordercode	description	quantity
1,2,3,4,5,6,7,8,9	NA	Armrest assemblies complete »»» see "Standard Armrest assemblies cpl."	2
1	SP1423224	ARMREST PAD LH CPL.	1
1	SP1423221	ARMREST PAD RH CPL.	1
2	1421717-9005	ARM PAD RECEPTACLE LH	1
2	1421716-9005	ARM PAD RECEPTACLE RH	1
3	L31001	HAND KNOB M6 X 10	4
4	Z173189.99	NUT M6 SW-14	4
5	NA	»»» see "SIDE PART PLATES COMPLETE WITH ADJUSTMENT"	
6	1422095-9005	SIDE PART ADJUSTMENT LH CPL. 250-340 MM	1
6	1422096-9005	SIDE PART ADJUSTMENT RH CPL. 250-340 MM	1
6	1422099-9005	SIDE PART ADJUSTMENT LH CPL. 290-380 MM	1
6	1422098-9005	SIDE PART ADJUSTMENT RH CPL. 290-380 MM	1
7	1422092-0002	CONNECTION PIECE	2
8	1415027	CLAMP LEVER M6 X 25	2
9	NA	»»» see "SIDE PART RECEPTACLES COMPLETE"	
9	NA	»»» see "SIDE PART RECEPTACLES SINGLE"	
10	SP1430417	BAG FOR SIDE PANEL LH CPL.	1
10	SP1430416	BAG FOR SIDE PANEL RH CPL.	1








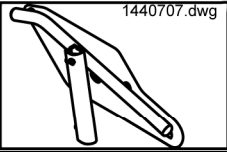
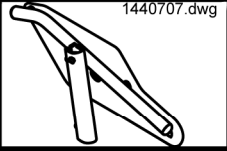
## Side Panels - Side part plates complete with side part adjustment

Picture	Seat depth cm	Height cm	left/ right	ordercode
	41	34		1440707
	41	34		1440708
	41	38		1440705
	41	38		1440706
	41...	34		SP1423702
	41...	34		SP1423222
	41...	38		SP1423703
	41...	38		SP1423701

- ✓ **Information:** The height of the side parts is settable. Preset is 34cm or 38cm.
- Parts with **34cm** are settable between **30cm and 40cm**.
  - Parts with **38cm** are settable between **34cm and 44cm**.

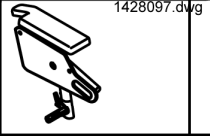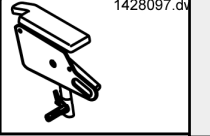
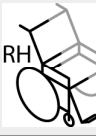


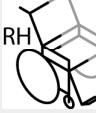

## Side Panels - Side Panels for Standard seatheight

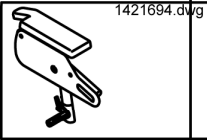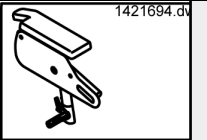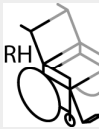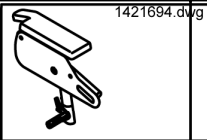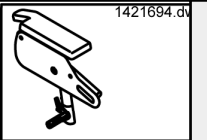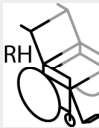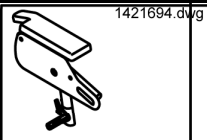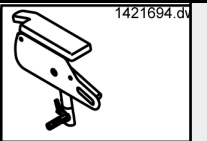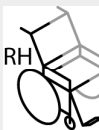Picture	Seat system	Seat depth cm	Height cm	Seat width cm	left/ right	ordercode
	Flex	41	34	39		SP1428101
	Flex	41	34	39		SP1428100
	Standard/Kontur/Flex	41	34	39...		SP1428097
	Standard/Kontur/Fle	41	34	39...		SP1428096
	Flex	41	38	39		SP1428103
	Flex	41	38	39		SP1428102
	Standard/Kontur/Flex	41	38	39...		SP1428099
	Standard/Kontur/Fle	41	38	39...		SP1428098
	Flex	41...	34	39		SP1422543
	Flex	41...	34	39		SP1422542
	Standard/Kontur/Flex	41...	34	39...		SP1421694
	Standard/Kontur/Fle	41...	34	39...		SP1421692




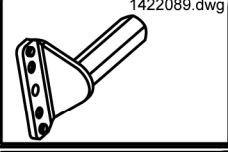
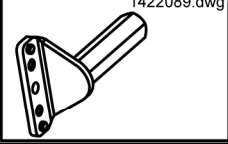

Picture	Seat system	Seat depth	Height	Seat width	left/ right	ordercode
		cm	cm	cm		
	Flex	41...	38	39		SP1422549
	Flex	41...	38	39		SP1422548
	Standard/Kontur/Flex	41...	38	39...		SP1421695
	Standard/Kontur/Fle	41...	38	39...		SP1421693
	RECARO	n.a.	38	n.a.		SP1425846
	RECARO	n.a.	38	n.a.		SP1425845

- ✓ **Information:** The height of the side parts is settable. Preset is 34cm or 38cm.
- Parts with **34cm** are settable between **30cm and 40cm**.
  - Parts with **38cm** are settable between **34cm and 44cm**.

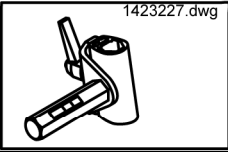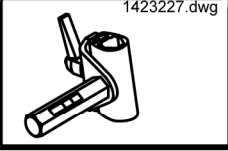**Side Panels - Side part receptacles complete**

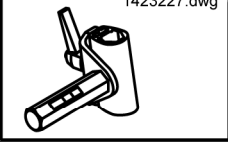




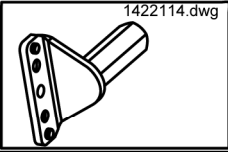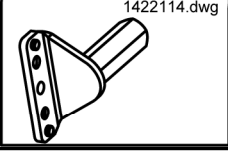





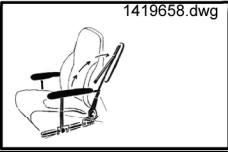




Picture	Seat width cm	left/ right	ordercode
	39		SP1423227
	39		SP1423226
	39...		SP1423225
	39...		SP1423223



**Side Panels - Side part receptacles single**

Picture	Seat width cm	left/ right	ordercode
	39		1422115-9005
	39		1422114-9005
	39...		1422097-9005
	39...		1422089-9005




## Side Panels - Side Panels for Arne Jensen-Seats

Picture	length mm	left/ right	ordercode
	300		1419658
	300		1419657
	350		1419660
	350		1419659
	400		1419662
	400		1419661
	300		1419648
	300		1419647
	350		5314010S
	350		5314011
	400		5314010

Picture	length	left/ right	ordercode
	mm		
	400		5314011L

## Side Panels - Sliding Armrests

SP142585X.dwg

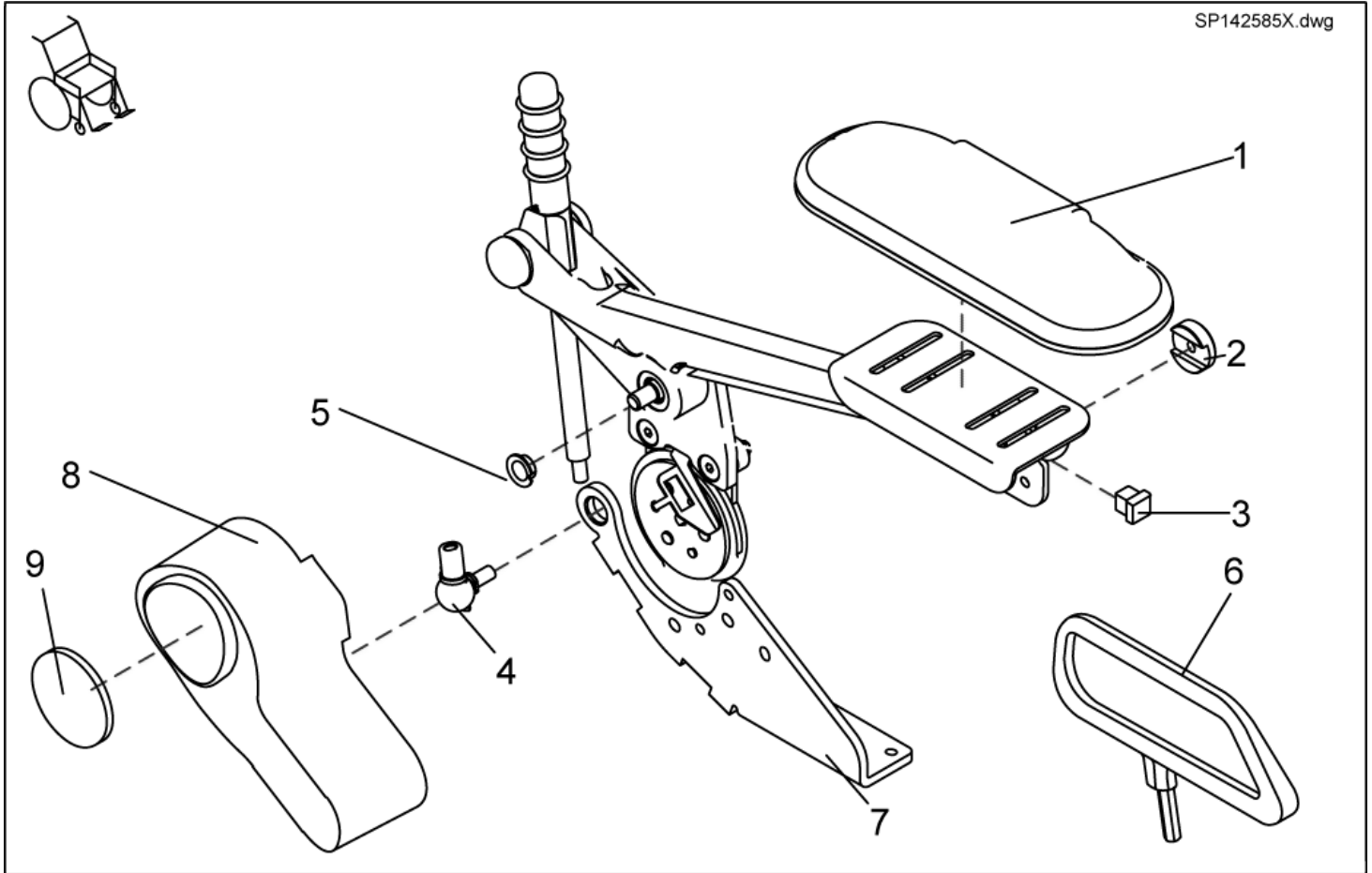

item	ordercode	description	quantity
-	SP1425853	KIT SLIDING ARMRESTS PAIR MANUAL	1
-	SP1425859	KIT SLIDING ARMRESTS PAIR, F. EL. BACKREST ADJUSTMENT	1
1,2,3,4,5	SP5314641	SLIDING ARMREST LH CPL.	1
1,2,3,4,5	SP5311680	SLIDING ARMREST RH CPL.	1
1	5311680	ARMREST UPHOLSTERY 260 MM	1
1	5314641	ARMREST UPHOLSTERY 320 MM	1
2	1421657-0003	RETAINING WASHER BLACK GALVANISED	4
3	L01055	TUBE END PLUG E 15 X 15	1
4	N04109	SWIVEL JOINT AS 13-M8 RH	2
5	N02194	BUSH BB 1209 DU	8
6	SP1436540	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB=39CM	1
6	SP1436539	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB>39CM	1
7	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST	1
7	1429563-9005	FITTING ATTACHMENT RH F. MANUAL ADJUSTMENT	1
8	1428057	MOTOR CAP FOR ELECTRIC BACKREST	1
9	1420482	REFLECTOR	1