


# Lifter


1418527.dwg

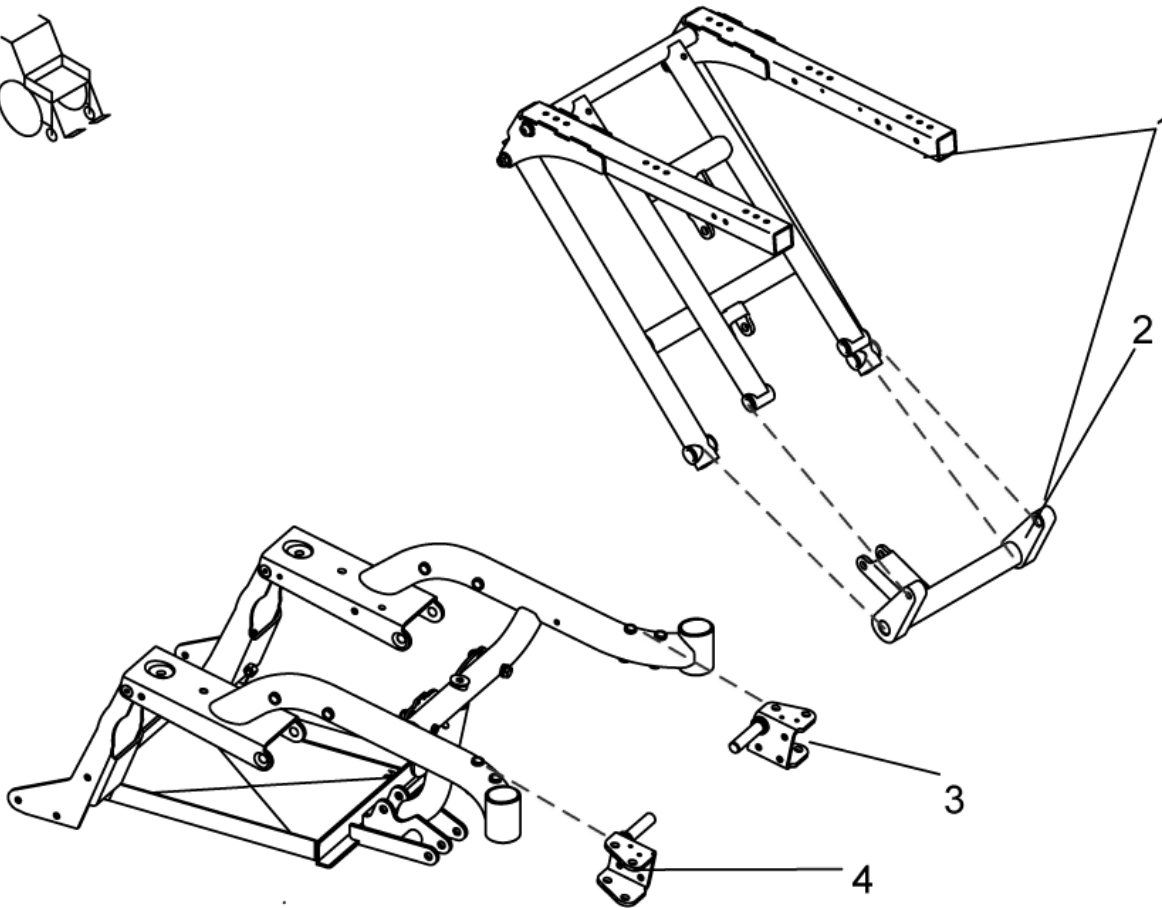


item	ordercode	description	quantity
1	1418527	PREASSEMBLED LIFTER UNIT	1
2	1440506	SWIVEL FORK LIFTER WITH BUSHINGS CPL.	1
3	1418547-9005	BEARING PLATE LH	1
4	1418544-9005	BEARING PLATE RH	1
-	1434934	SEAT HEIGHT ADJUSTMENT MANUAL, PREASSEMBLED	1
-	1434935	SEAT ANGLE ADJUSTMENT MANUAL, PREASSEMBLED	1
-	1434931	SEAT HEIGHT ADJUSTMENT ELECTRICAL, PREASSEMBLED	1
-	1434932	SEAT ANGLE ADJUSTMENT ELECTRICAL, PREASSEMBLED	1

# Seat Height Adjustment Manual

1434934.dwg

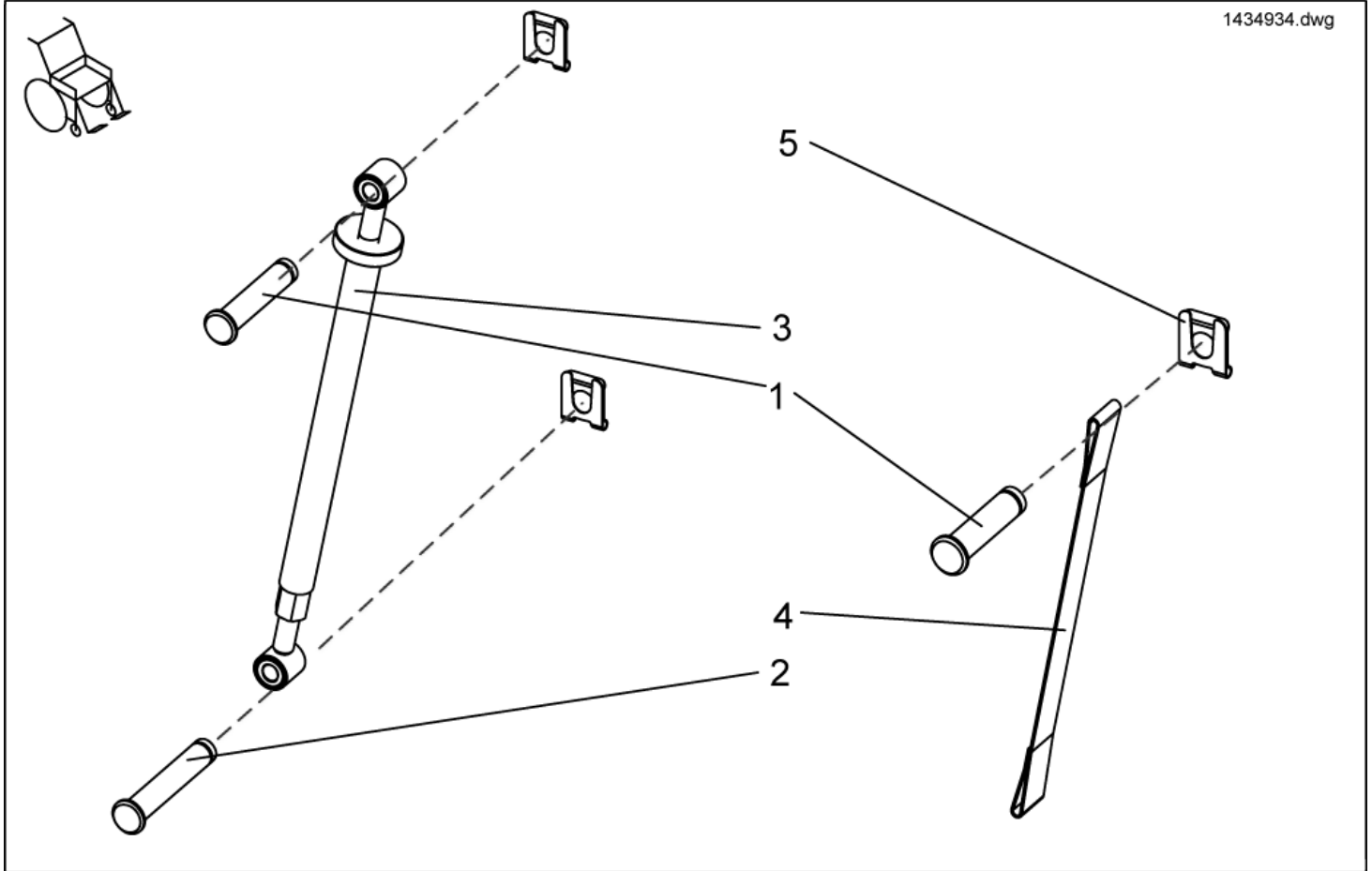


item	ordercode	description	quantity
1-5	1434934	SEAT HEIGHT ADJUSTMENT MANUAL, PREASSEMBLED	1
1	1419941-0003	SECURING BOLT 44.5 MM	2
2	1419943-0003	SECURING BOLT 52.5 MM	1
3	1420003	SPINDLE SEAT HEIGHT ADJUSTMENT MANUAL CPL.	1
4	1430121	RESTRAINT 25 MM	1
5	K32395	SL-LOCKING WASHER 10 MM	3

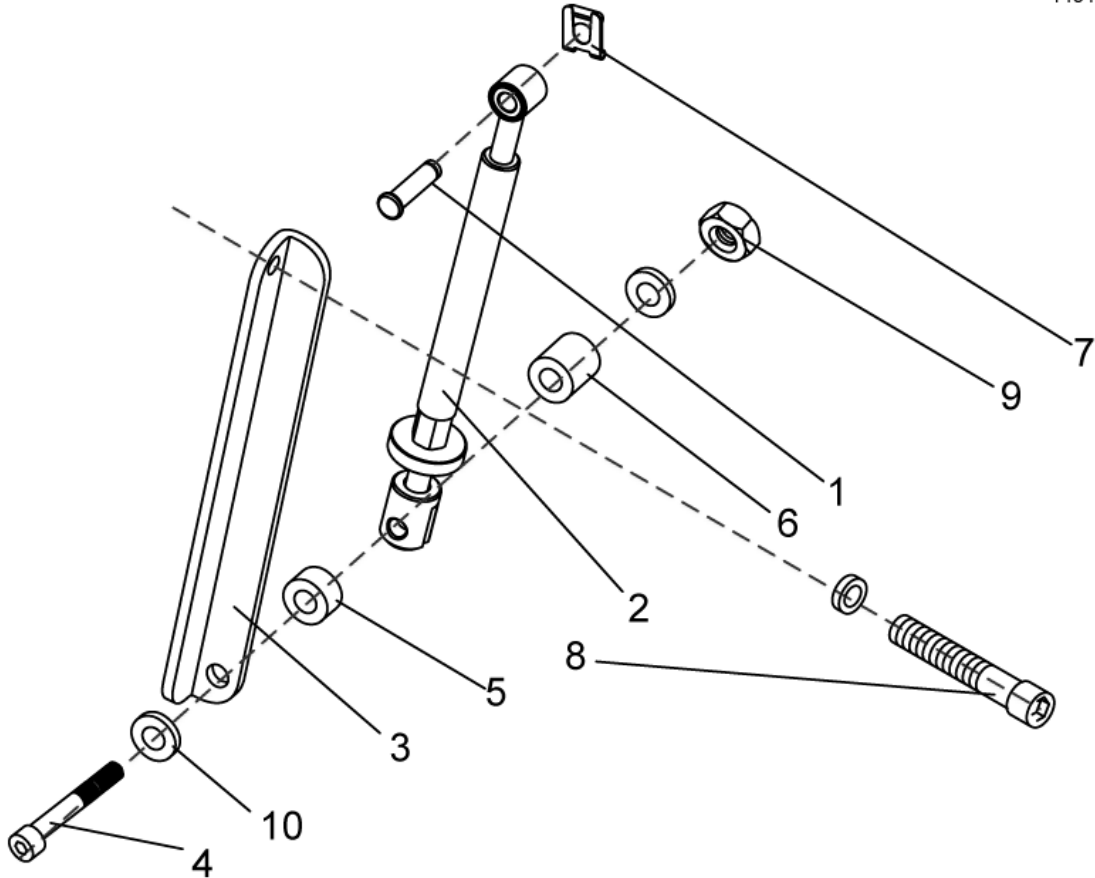

item	ordercode	description	quantity
1-10	1434935	SEAT ANGLE ADJUSTMENT MANUAL, PREASSEMBLED	1
1	1419941-0003	SECURING BOLT 44.5 MM	1
2	1420004	SPINDLE SEAT ANGLE ADJUSTMENT MANUAL CPL.	1
3	1423440-9005	ANGLE BRACE	1
4	1423490	BOLT M10 X 125 DIN 912	1
5	1426296	SPACER 10,5 X 21 X 12	1
6	1426297	SPACER 10,5 X 21 X 16	1
7	K32395	SL-LOCKING WASHER 10 MM	1
8	5824318	SCREW M8 X 65 DIN 912	1
9	NA	NYLOC NUT M10 DIN 985	1
10	NA	WASHER DIN 125 10,5x2	3

## Seat Height Adjustment electrical


1434931.dwg

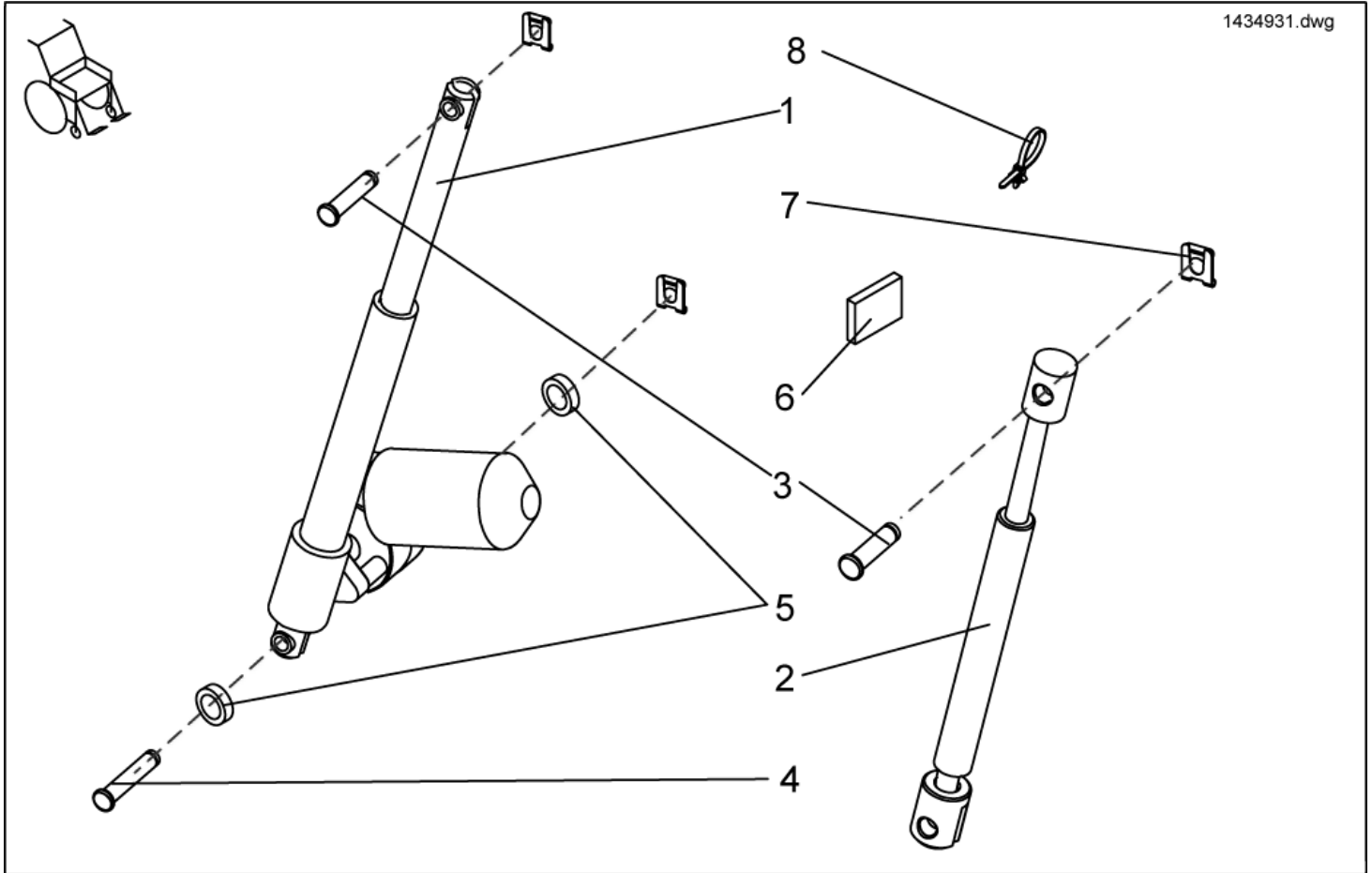

item	ordercode	description	quantity
1-8	1434931	SEAT HEIGHT ADJUSTMENT ELECTRICAL, PREASSEMBLED	1
1	1419400	MOTOR LA30.1S-150-24-002	1
2	SP1419237	GAS PRESSURE SPRING CPL.	1
3	1419941-0003	SECURING BOLT 44.5 MM	2
4	1419943-0003	SECURING BOLT 52.5 MM	1
5	1426295	SPACER 10,5x21x4	2
6	1429559	MAGNET	1
7	K32395	SL-LOCKING WASHER 10 MM	3
8	NA	CABLE TIE L=180 MM ( LONG ); BLACK	2

## Seat Angle Adjustment electrical

1434932.dwg

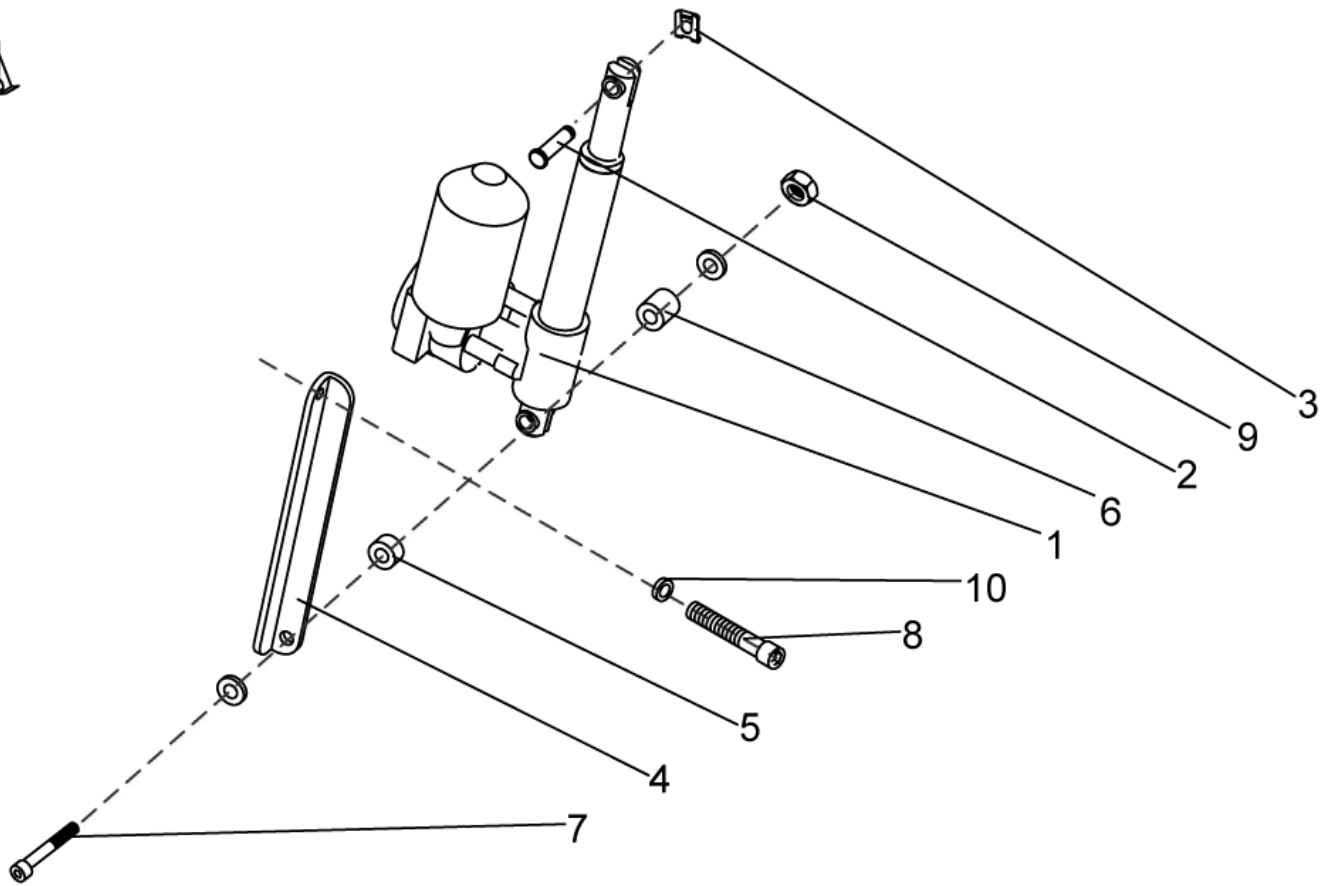

item	ordercode	description	quantity
1-10	1434932	SEAT ANGLE ADJUSTMENT ELECTRICAL, PREASSEMBLED	1
1	1434425	MOTOR LA30.1L-50-24-001 w. PULL BRAKE	1
2	1419941-0003	SECURING BOLT 44.5 MM	1
3	K32395	SL-LOCKING WASHER 10 MM	1
4	1423440-9005	ANGLE BRACE	1
5	1426296	SPACER 10,5 X 21 X 12	1
6	1426297	SPACER 10,5 X 21 X 16	1
7	1423490	BOLT M10 X 125 DIN 912	1
8	5824318	SCREW M8 X 65 DIN 912	1
9	NA	NYLOC NUT M10 DIN 985	1
10	NA	WASHER DIN 125 10,5x2	3