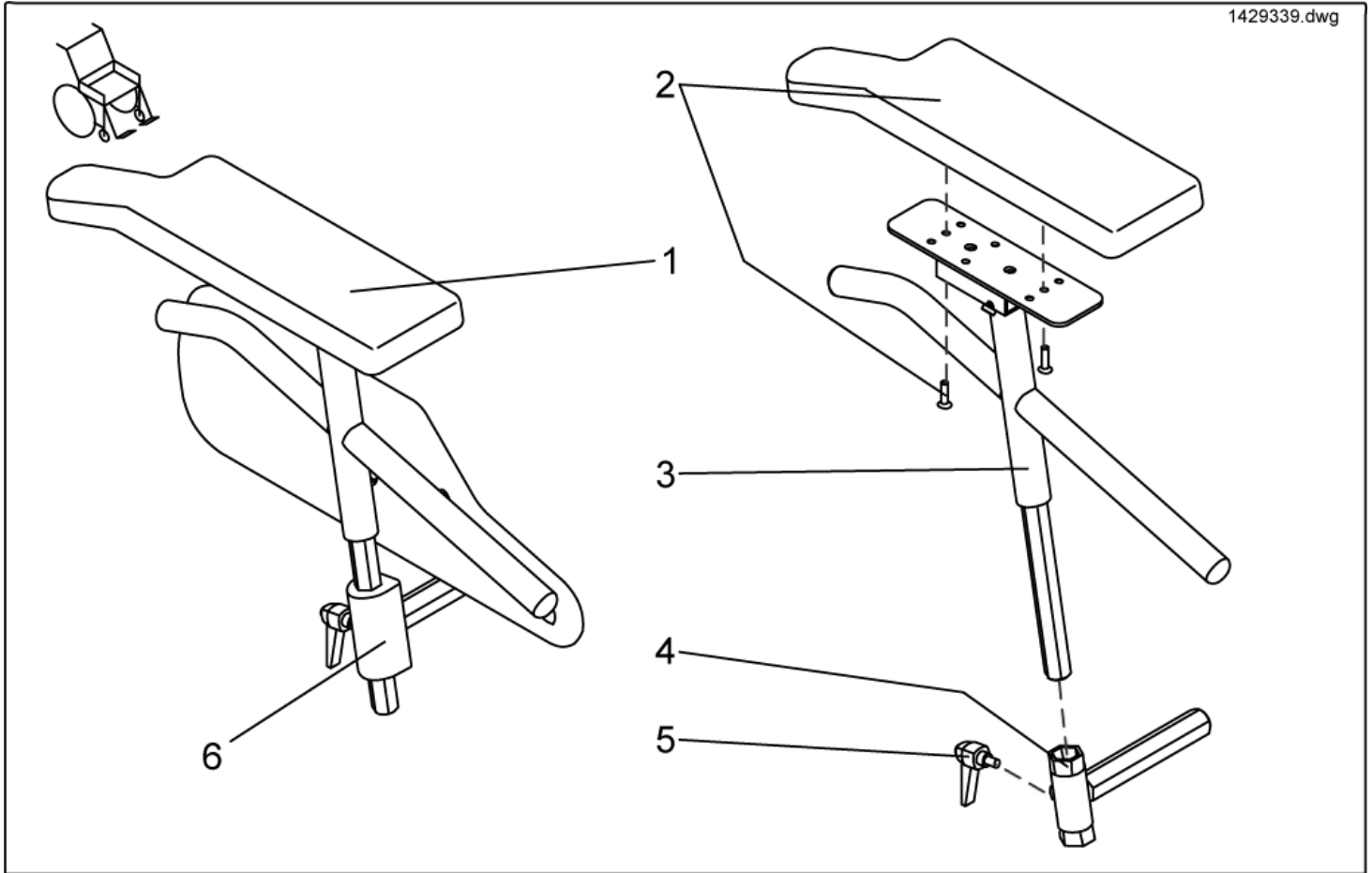


## Side Panels - Euro Side Panels

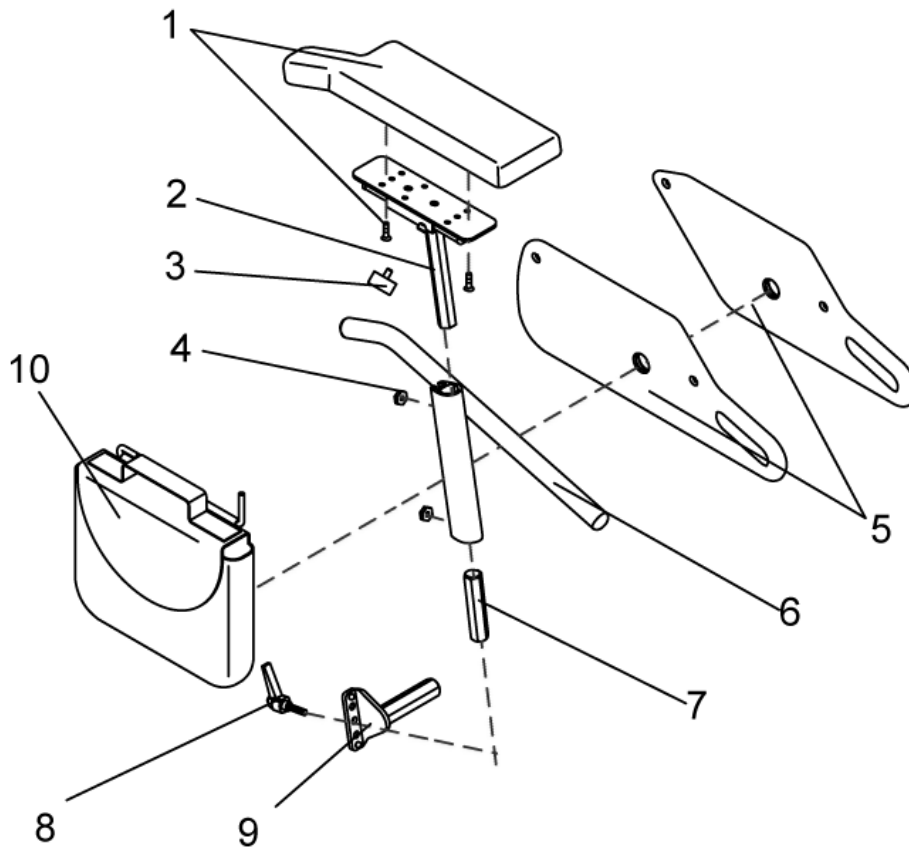
1429339.dwg



item	ordercode	description	quantity
1	1429339	ARMREST RH CPL. (EURO)	1
1	1429340	ARMREST LH CPL. (EURO)	1
2	SP1423224	ARMREST PAD LH CPL.	1
2	SP1423221	ARMREST PAD RH CPL.	1
3	1428474-9005	ARM SUPPORT LH CPL.	1
3	1428470-9005	ARM SUPPORT RH CPL.	1
4	1428459-9005	SIDE PANEL MOUNTING LH	1
4	1428465-9005	SIDE PANEL MOUNTING RH	1
5	L32902	LEVER M6 X 10; BLACK	2
6	1430359	Abdeckkappe Seitenteil RH, rechts	1
6	1430211	Abdeckkappe Seitenteil LH, links	1

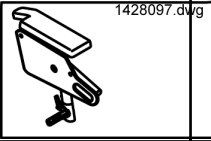

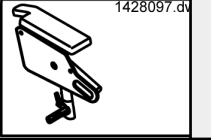

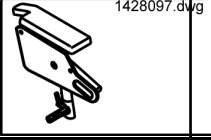

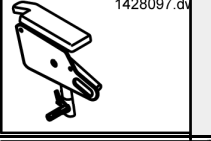





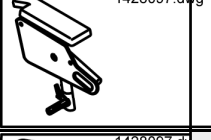

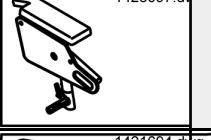

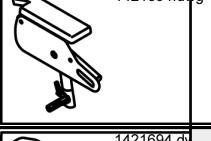

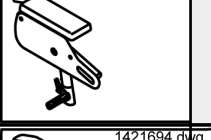
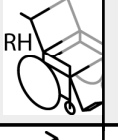
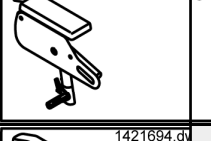



## Side Panels - Standard Side Panels

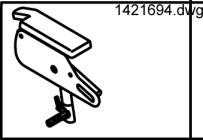

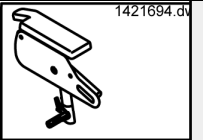
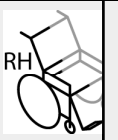
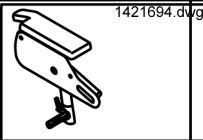

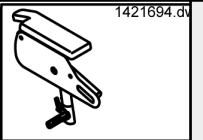
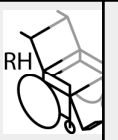
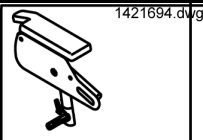

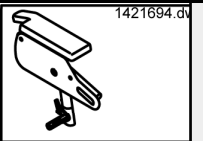

Seitenteile.dwg



item	ordercode	description	quantity
1-9	NA	Armrest assemblies complete »»» see "Standard Armrest assemblies cpl."	2
1	SP1423224	ARMREST PAD LH CPL.	1
1	SP1423221	ARMREST PAD RH CPL.	1
2	1421717-9005	ARM PAD RECEPTACLE LH	1
2	1421716-9005	ARM PAD RECEPTACLE RH	1
3	L31001	HAND KNOB M6 X 10	4
4	Z173189.99	NUT M6 SW-14	4
5	NA	»»» see "SIDE PART PLATES COMPLETE WITH ADJUSTMENT"	2
6	1422095-9005	SIDE PART ADJUSTMENT LH CPL. 250-340 MM	1
6	1422096-9005	SIDE PART ADJUSTMENT RH CPL. 250-340 MM	1
6	1422099-9005	SIDE PART ADJUSTMENT LH CPL. 290-380 MM	1
6	1422098-9005	SIDE PART ADJUSTMENT RH CPL. 290-380 MM	1
7	1422092-0002	CONNECTION PIECE	2
8	1415027	CLAMP LEVER M6 X 25	2
9	NA	»»» see "SIDE PART RECEPTACLES COMPLETE"	2
9	NA	»»» see "SIDE PART RECEPTACLES SINGLE"	2
10	SP1430417	BAG FOR SIDE PANEL LH CPL.	1
10	SP1430416	BAG FOR SIDE PANEL RH CPL.	1

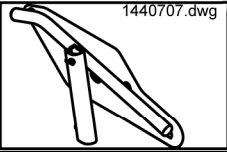

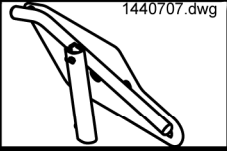

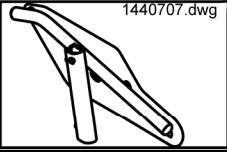

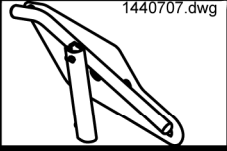

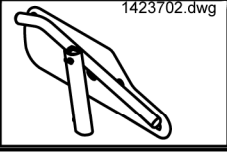

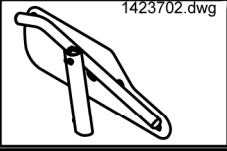

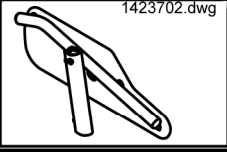

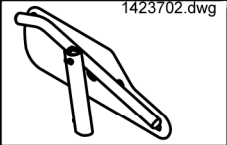

## Side Panels - Side Panels for Standard seatheight

Picture	Seat system	Seat depth cm	Height cm	Seat width cm	left/ right	ordercode
	Flex	41	34	39		SP1428101
	Flex	41	34	39		SP1428100
	Standard/Kontur/Flex	41	34	39...		SP1428097
	Standard/Kontur/Fle	41	34	39...		SP1428096
	Flex	41	38	39		SP1428103
	Flex	41	38	39		SP1428102
	Standard/Kontur/Flex	41	38	39...		SP1428099
	Standard/Kontur/Fle	41	38	39...		SP1428098
	Flex	41...	34	39		SP1422543
	Flex	41...	34	39		SP1422542
	Standard/Kontur/Flex	41...	34	39...		SP1421694
	Standard/Kontur/Fle	41...	34	39...		SP1421692

Picture	Seat system	Seat depth	Height	Seat width	left/ right	ordercode
		cm	cm	cm		
	Flex	41...	38	39		SP1422549
	Flex	41...	38	39		SP1422548
	Standard/Kontur/Flex	41...	38	39...		SP1421695
	Standard/Kontur/Fle	41...	38	39...		SP1421693
	RECARO	n.a.	38	n.a.		SP1425846
	RECARO	n.a.	38	n.a.		SP1425845

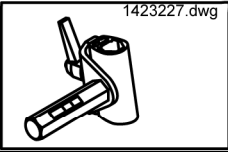

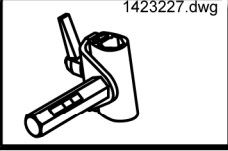





- ✓ **Information:** The height of the side parts is settable. Preset is 34cm or 38cm.
- Parts with **34cm** are settable between **30cm and 40cm**.
  - Parts with **38cm** are settable between **34cm and 44cm**.

## Side Panels - Side part plates complete with side part adjustment

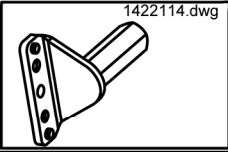

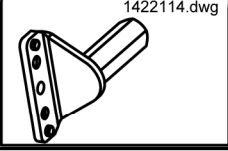

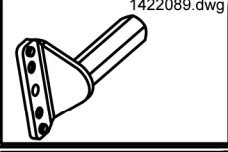

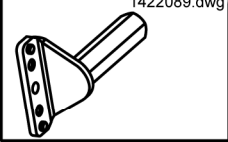

Picture	Seat depth cm	Height cm	left/ right	ordercode
	41	34	LH 	1440707
	41	34	RH 	1440708
	41	38	LH 	1440705
	41	38	RH 	1440706
	41...	34	LH 	SP1423702
	41...	34	RH 	SP1423222
	41...	38	LH 	SP1423703
	41...	38	RH 	SP1423701

- ✓ **Information:** The height of the side parts is settable. Preset is 34cm or 38cm.
- Parts with **34cm** are settable between **30cm and 40cm**.
  - Parts with **38cm** are settable between **34cm and 44cm**.












**Side Panels - Side part receptacles complete**

Picture	Seat width cm	left/ right	ordercode
 1423227.dwg	39	 LH	SP1423227
 1423227.dwg	39	 RH	SP1423226
 1423227.dwg	39...	 LH	SP1423225
 1423227.dwg	39...	 RH	SP1423223



## Side Panels - Side part receptacles single

Picture	Seat width cm	left/ right	ordercode
	39		1422115-9005
	39		1422114-9005
	39...		1422097-9005
	39...		1422089-9005

## Side Panels - Side Panels for Arne Jensen-Seats

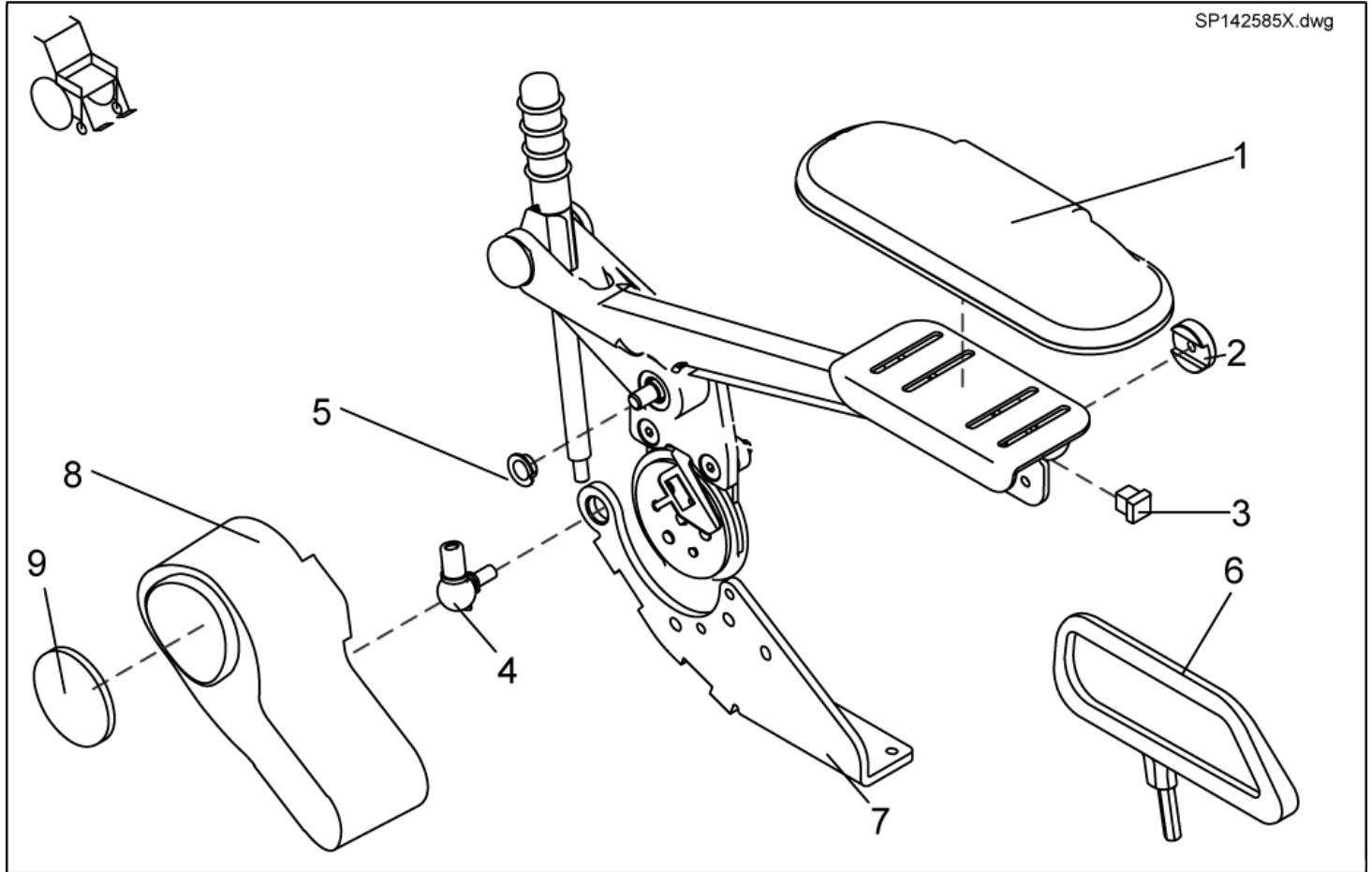
Picture	length mm	left/ right	ordercode
 1419658.dwg	300	LH	1419658
 1419658.dwg	300	RH	1419657
 1419658.dwg	350	LH	1419660
 1419658.dwg	350	RH	1419659
 1419658.dwg	400	LH	1419662
 1419658.dwg	400	RH	1419661
 1419648.dwg	300	LH	1419648
 1419648.dwg	300	RH	1419647
 1419648.dwg	350	LH	5314010S
 1419648.dwg	350	RH	5314011
 1419648.dwg	400	LH	5314010



Picture	length	left/ right	ordercode
	mm		
	400		5314011L

## Side Panels - Sliding Armrests

SP142585X.dwg



item	ordercode	description	quantity
-	SP1425853	KIT SLIDING ARMRESTS PAIR MANUAL	1
-	SP1425859	KIT SLIDING ARMRESTS PAIR, F. EL. BACKREST ADJUSTMENT	1
1-5	SP5314641	SLIDING ARMREST LH CPL.	1
1-5	SP5311680	SLIDING ARMREST RH CPL.	1
1	5311680	ARMREST UPHOLSTERY 260 MM	1
1	5314641	ARMREST UPHOLSTERY 320 MM	1
2	1421657-0003	RETAINING WASHER BLACK GALVANISED	4
3	L01055	TUBE END PLUG E 15 X 15	1
4	N04109	SWIVEL JOINT AS 13-M8 RH	2
5	N02194	BUSH BB 1209 DU	8
6	SP1436540	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB=39CM	1
6	SP1436539	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB>39CM	1
7	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST	1
7	1429563-9005	FITTING ATTACHMENT RH F. MANUAL ADJUSTMENT	1
8	1428057	MOTOR CAP FOR ELECTRIC BACKREST	1
9	1420482	REFLECTOR	1