


Back Unit - Drives


1487397.dwg

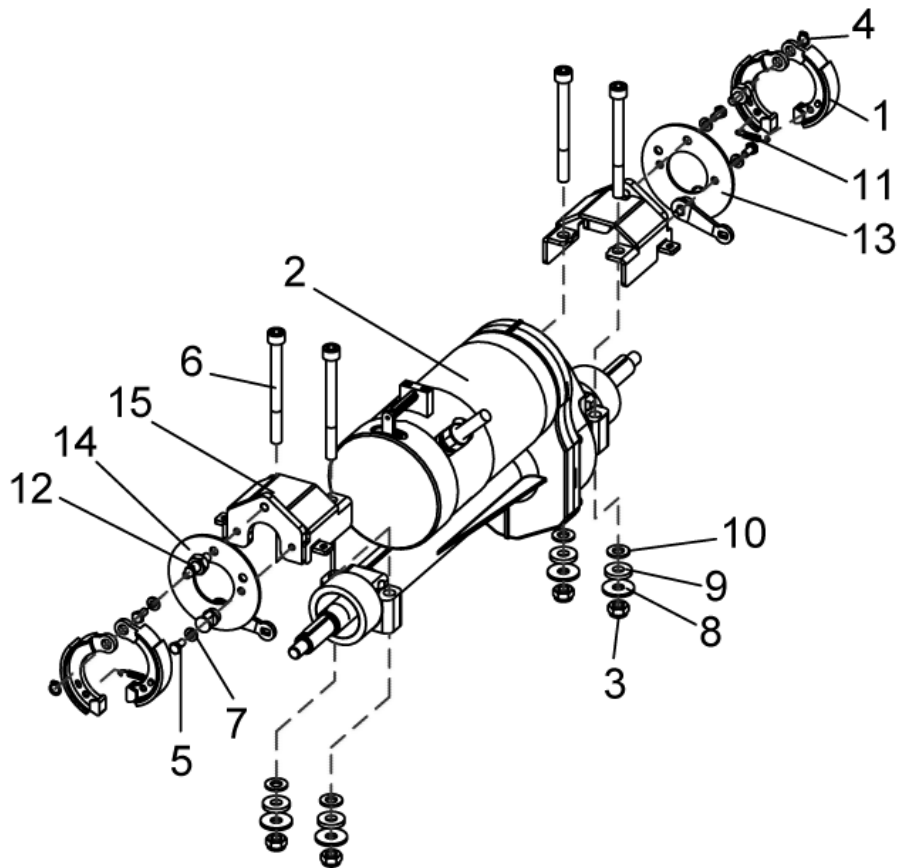

item	ordercode	description	quantity
-	SP1487397	DRIVE KPL 6KM/H G50	1
-	SP1487398	DRIVE KPL 10KM/H G50	1
-	P09294	BRUSH CAP	4
-	1419264	CARBON BRUSHE	4
3,6,8,9,10	1478436	ASSEMBLY KIT DRIVE G50	1
1,4,11,13,15	1478437	ASSEMBLY KIT BRAKE G50, LH	1
5,7,12,14,15	1478438	ASSEMBLY KIT BRAKE G50, RH	1
1	1451357	BRAKE SHOE	4
2	1487944	DRIVE 6 KM/H G50	1
2	1486036	DRIVE 10 KM/H G50	1
3	NA	NYLOC NUT M10 DIN 985	4
4	NA	RETAINER RING 10 MM	2
5	NA	HEX HEAD SCREW M6 X 12	4
6	NA	BOLT DIN 912 M10X100	4
7	NA	WASHER 6,4 X 12,5 X 1,6	4
8	NA	DIN 9021 10,5 MM VZ SCHEIBE M10 GROSS	4
9	NA	WASHER DIN 125 10,5X2	4
10	NA	TENSION SPRING G50 BREAKS	4
11	1491883	BRAKE ANCHOR PLATE LH PRE-MOUNTS	2
12	NA	BRAKE ANCHOR PLATE RH PRE-MOUNTS	2
13	NA	BRAKE GIRDER SHEET METAL, COMPLETE	1
14	NA	BRAKE ANCHOR PLATE RH, PRE-mounted	1
15	NA	BRAKE ANCHOR PLATE, CPL.	2

Back Unit - Uncoupling


1487401.dwg

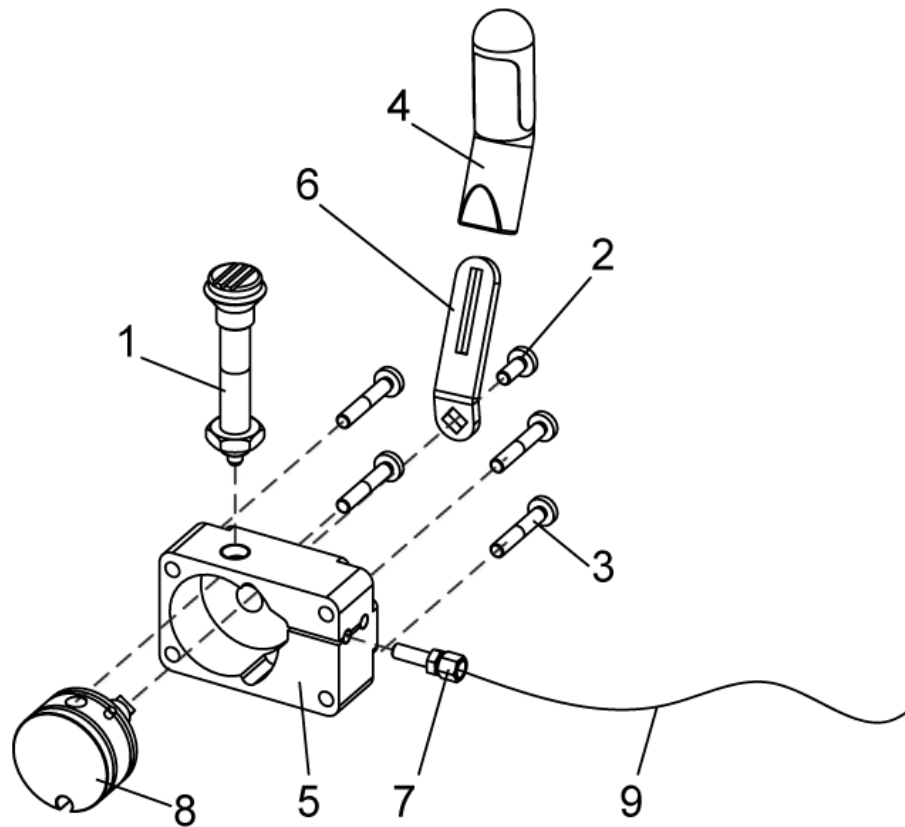

item	ordercode	description	quantity
1	1488803	BRAKE SHOE	1
2	NA	DRIVE 10 KM/H G50	1
2,4,6,	1468386	HAND GRIP UNCOUPLING, CPL.	1
3	NA	NYLOC NUT M10 DIN 985	4
4	NA	RETAINER RING 10 MM	1
5	NA	HEX HEAD SCREW M6 X 12	1
6	NA	BOLT DIN 912 M10X100	1
7	NA	WASHER 6,4 X 12,5 X 1,6	1
8	NA	DIN 9021 10,5 MM VZ SCHEIBE M10 GROSS	1
9	NA	BOWDEN CABLE 1500mm	1
7,9	1478428	ASSEMBLY KIT BOWDEN CABLE, CPL.	1

Back Unit - Parking Brake

1487399.dwg

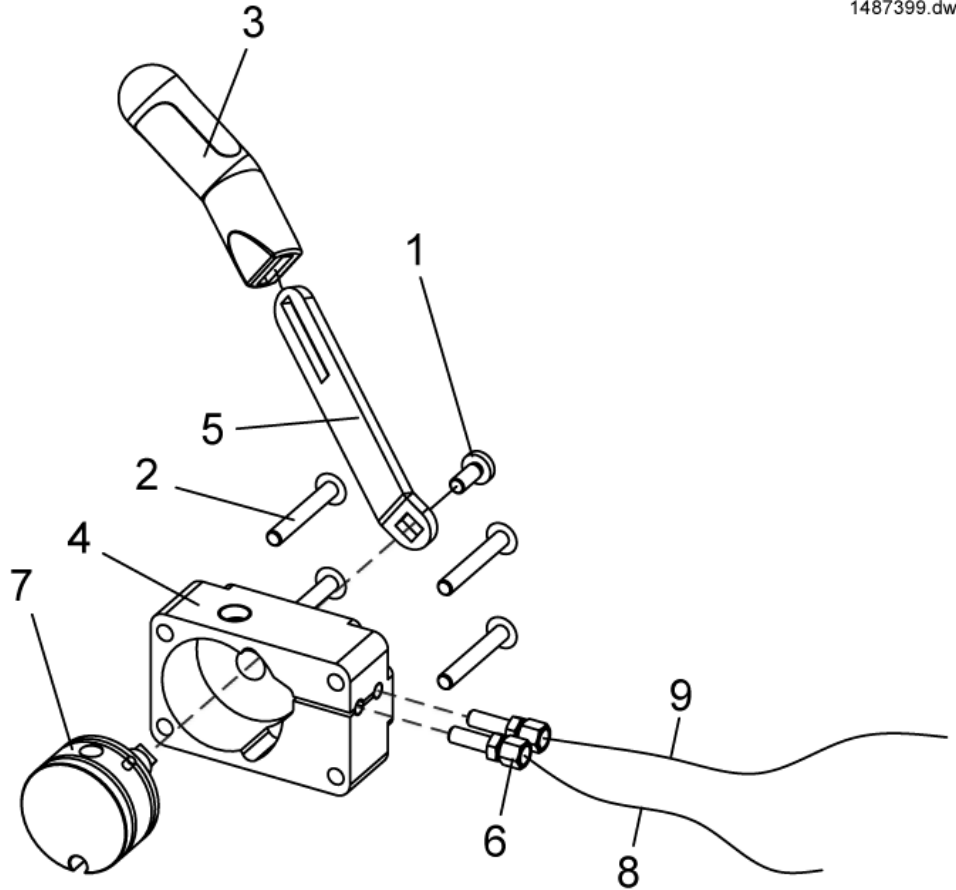

item	ordercode	description	quantity
1,2,3,4,5,6,7	SP1487399	PARKING BRAKE LH G50, CPL.	1
1,3,5	1468296	ASSEMBLY KIT BRAKE LEVER G50	1
1	NA	BRAKE SHOE	1
2	NA	DRIVE 10 KM/H G50	4
3	NA	NYLOC NUT M10 DIN 985	1
4	NA	RETAINER RING 10 MM	1
5	NA	HEX HEAD SCREW M6 X 12	1
6	NA	BOLT DIN 912 M10X100	2
7	NA	WASHER 6,4 X 12,5 X 1,6	1
8	NA	BOWDEN CABLE 1500mm	1
9	NA	BOWDEN CABLE 1500mm	1
6,8,9	1478428	ASSEMBLY KIT BOWDEN CABLE, CPL.	2