**H300
H302
H305T5
H305T10
H305T15
H355
H356
H430/2
H430A/2**

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Brugsanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

1. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.

INSTALLING THE LIFTERS

H355 lifter

- Check that there are 4 non-slip runners supplied.
- Raise the existing seat and place the lifter on the bowl.

H356 lifter

- Raise the existing seat. Adjust the feet to the width of the bowl and tighten them before placing the lifter on the bowl.
- Ensure that the lifter is correctly installed before use.

H300, H305T5, H305T10 and H305T15 lifters

- Screw the two support feet under the lifter and unscrew the adjustment knobs.
- Raise (or remove) the existing lid and place the lifter on the bowl.
- Adjust the feet at the sides and tighten the two setting screws.
- Ensure that the lifter is correctly installed before use.

H430/2 and H430A/2 lifters

- Unscrew the 2 front knurled nuts. Withdraw the stoppers and place the bar at the desired height (6, 10 or 14cm). Tighten using the two nuts. Proceed to set the rear bar in the same way. The two bars should be positioned at the same height.
- Remove the existing toilet lid, replacing it with the lifter by inserting the two threaded rods in the holes. Tighten using the two plastic fixings.
- Ensure that the lifter is correctly installed before use.
- Do not press down on the armrests during transfer.
- **Caution** : Whenever the chair has been transported, always take care to reseat yourself correctly to avoid being pinched as the armrests are repositioned.

POSITIONING THE BIDET (H302)

Place the bidet on a H300, H305, H355 or H356 lifter. Do not place it directly on the toilet bowl.

MAINTENANCE

Clean regularly with a normal household cleaner. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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