9630E H340 C407A H403

| | | |
|-----|-----|---------------------|
| F | 1. | Guide utilisateur |
| GB | 2. | User's manual |
| D | 3. | Gebrauchsanweisung |
| E | 4. | Manual del usuario |
| I | 5. | Manuale dell'utente |
| NL | 6. | Gebruiksaanwijzing |
| P | 7. | Guia do utilizador |
| S | 8. | Bruksanvisning |
| DK | 9. | Brugsanvisning |
| FIN | 10. | Käyttäjän käsikirja |
| N | 11. | Bruksanvisning |

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed :

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

H340 LIFTING COMMODE

- Install the backrest tubing behind the commode using the spring buttons.
- Clip the toilet bowl and lid onto the rear upright and ensure that the bowl is correctly positioned before sitting down.
- Adjust the seat height using the spring buttons on the legs. Ensure that all four legs are set to the same height before use.
- The armrests can be folded away. To lower the armrests press the side buttons.

9630E LIFTING COMMODE

- Fix the backrest tubing between the two side uprights with the armrests using 2 screws and 2 wing nuts.
- Fix the two cross struts to support the seat (one in front, the other at the back) between the side uprights with the armrests using 4 screws and 4 wing nuts. The pinched ends of the cross struts should face downwards and towards the inside of the commode. Warning : before installing the second cross strut, fit the 2 aluminium "Vs" that support the toilet bowl into the notches provided with the points of the "Vs" facing outwards.
- Fit the four legs and set them all to the same height (using the spring buttons).
- Tighten all the screws securely.
- Clip the toilet bowl and lid onto the rear cross strut and ensure that the bowl is correctly positioned before sitting down.

C407A TOILET FRAME

- Fix the U-shaped tube between the side uprights with armrests using four screws in the upper holes.
- Fix the small tube in the same way using the two lower holes.
- Fit the four legs, setting them all to the same height (using the spring buttons).
- Ensure that the whole frame is stable before use.

H403 WOODEN COMMODE

- Insert 3 screws in the commode's side holes and lay this side on a soft surface.
- Insert the front upright, the rear upright and the back over the three screws. Assemble without screwing tight.
- Turn over and repeat the operation for the other side.
- Stand the commode on its legs and fit the plastic seat and the removable seat. Fit the toilet bowl on its holder.
- Tighten all the screws and retighten after 7 to 10 days' use.

MAINTENANCE

Clean regularly with a standard household cleaner. For the H403, wipe all the wooden surfaces using a household cleaner and a soft cloth; avoid scratching the surface. The bowls can be put through an autoclave (1 hour at 80°). For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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