Fig. 1

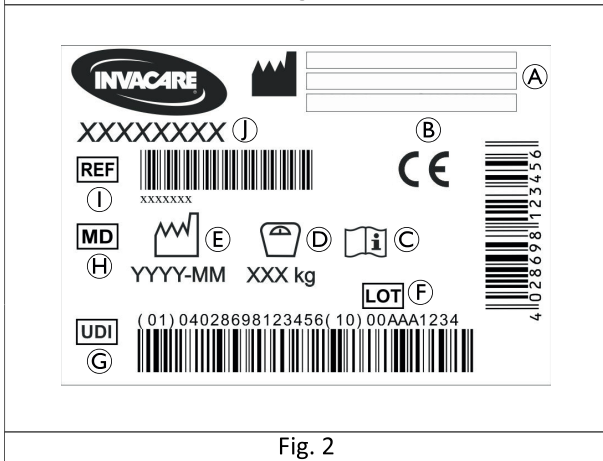
Fig. 2

Invacare® Basica

H330/1

en	Foldable Grab Bar	
	User Manual	1
fr	Barre de maintien relevable	
	Manuel d'utilisation	3
es	Asa de sujeción plegable	
	Manual del usuario	5
pt	Barra de apoio rebatível	
	Manual de utilização	7
de	Klappbarer Haltegriff	
	Gebrauchsanweisung	9
nl	Inklapbare handgreep	
	Gebruiksaanwijzing	11

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

This CE-marked medical device conforms to: EU Medical Device Regulation, ISO 13485, ISO 14971, REACH, ISO 17966 and related standards.

1.4 Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary depending on frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up
- Unauthorised modifications and/or use of unsuitable spare parts

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

2 Safety

2.1 Safety Information

WARNING!
Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material.

WARNING!
Risk of Falling!

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use or store the product outdoors.
- Do not use as sports equipment or climbing aid.

2.2 Labels and Symbols on the Product (Fig. 2)

Ⓐ	Manufacturer	Ⓕ	Lot number
Ⓑ	European Conformity	Ⓖ	Unique Device Identifier
Ⓒ	Read user manual	Ⓗ	Medical Device
Ⓓ	Maximum user weight	Ⓘ	Reference number
Ⓔ	Production date	Ⓚ	Product designation

The identification label is attached to the tube.

3 Product Overview

3.1 Intended Use

Invacare Basica H330/1 is a powder-coated steel grab bar to be installed on wall surfaces with screws. It can be flipped up and includes a toilet paper holder. This product is intended to be used as a transfer and support aid when e.g. sitting down and standing up from a toilet seat.

Indication

If used without an assistant, the users must be able to keep their own balance. There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data*, page 3.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

CAUTION!
Risk of Finger Trapping

- Take care during assembly.

CAUTION!
Risk of Falling

- Mount only to walls with sufficient load bearing capacity.

1. With the user standing upright, without shoes, measure the height of the thumb's upper joint. This height corresponds to the grab bar's highest point.
2. Mark out the location of the holes on the wall taking into account the previously measured height.
3. Drill and insert four plugs appropriate for the type of wall.
4. Fix the grab bar to the wall using the screws and washers.
5. Flip the product up and down as required, grip the bar and carry out the desired body movement.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- before first use,
- regularly while it is in use,
- when it has been in contact with any body fluids.

Cleaning

NOTICE!

Maximum cleaning temperature:
– 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

WARNING!

Risk of Falling

Worn fastening elements can suddenly come loose and cause falling.

- Always use new wall plugs and bolts when repositioning the product.

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 2*.

Make sure that the user manual is handed over with the product.

6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Weights and Materials

Weights	
Basica H330	2.2 kg
Maximum user weight	130 kg

Materials	
①	Steel, powder coated

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare S.A.
Invacare Poirier SAS
Invacare GmbH