Invacare® Banjo™ P452E/3

en	Rollator User Manual	2
de	Rollator Gebrauchsanweisung	7
es	Rollator Manual del usuario	13
fr	Déambulateur Manuel d'utilisation	19
it	Rollator Manuale d'uso	24
nl	Rollator Gebruiksaanwijzing	30
pt	Andarilho Manual de utilização	36

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Serious Injury

Incorrect use can lead to hazardous situations.

- Do not use on stairs or on escalators.
- Do not stand on the product.
- Be careful when walking on sloping or slippery ground or on uneven surfaces.
- Avoid walking against obstacles, both forwards and backwards.
- Do not let children play with the product.
- Do not use the product to transport a person.
- Make sure that no clothes, such as long skirts, can get stuck in the wheels.

**CAUTION!****Risk of Injury**

- Unbalanced or heavy load may lead to overturning.
- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
 - Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
 - The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
 - Refer to chapter 7 *Technical Data, page 7* for maximum user weight and load information.

**CAUTION!****Risk of Injury**

- Non-original or wrong parts may affect the function and safety of the product.
- Only use original parts for the product in use.
 - Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**CAUTION!****Risk of First Degree Burns**


- Parts of the product may heat up, when exposed to high temperatures (above 41° C).
- Be careful if the product is exposed to high temperatures, e.g. direct sunlight.

**IMPORTANT!**

- There is a risk of damage due to wear, e.g. corrosion.
- Do not use the product under water, e.g. in pools.
 - Do not use the product in hot areas like saunas.
 - Do not store the product in wet or humid environments.

2.2 Labels and Symbols on the Product

Identification label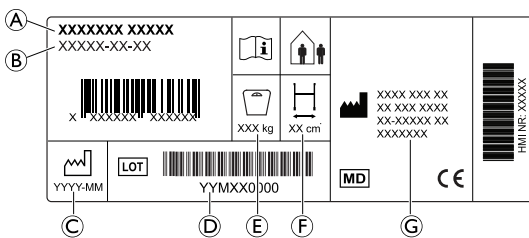

- Ⓐ Product name
- Ⓑ Part number
- Ⓒ Date of manufacture
- Ⓓ LOT number
- Ⓔ Maximum user weight
- Ⓕ Maximum width
- Ⓖ Manufacturer address

Symbols

	European Conformity		Medical Device
	Manufacturer		Read user manual
	Date of manufacture		LOT number

	Maximum width		Maximum user weight
	Indoor / Outdoor use		

3 Product Overview

3.1 Product Description

The four-wheeled walking aid is equipped with a central seat that can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels. The walking aid is available in different sizes and can be equipped with a wide range of accessories.

3.2 Intended Use

The walking aid is intended to support a person during walking.

The walking aid is for indoor use only.

The weight of the user must not exceed the maximum user weight as stated on the product label and in chapter 7 *Technical Data, page 7*.

3.3 Indications for Use


Indications

Persons with impaired muscle strength and/or balance, who need additional support whilst walking. The remaining walking and cognitive ability must be sufficient for safe use of a walking aid.

Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

3.4 Scope of Delivery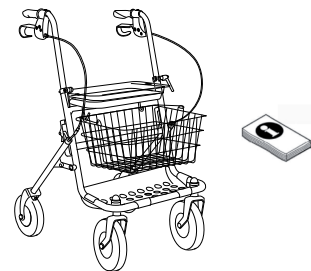

- Rollator, partly assembled
- User manual

3.5 Accessories

Various accessories, like a basket / bag, are available for this product and can be found in the country-specific sales documents. Depending on the configuration accessories may be pre-assembled.

4 Usage

4.1 Before Use


**CAUTION!****Risk of Injury or Damage to Property**

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.

Invacare presume a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

4.2 Assembling the Walking Aid

Before assembly, remove the plastic crossbar between the rear wheels and the plastic caps in the side tubes.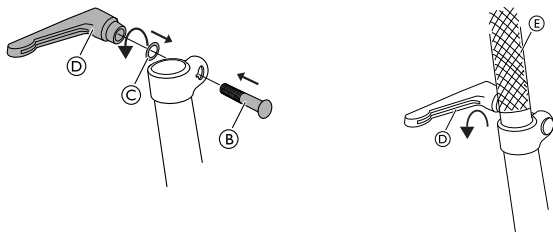

1. Attach the bolt **B** and the washer **C**. Ensure that the bolt is attached from the inside and slots into the recess of the fitting.
2. Attach the locking lever **D** on the bolt **B**, so that it is placed on the outside of the tube.
3. Slide the handle **E** into the tube.
4. Tighten the locking lever **D**.


 The locking levers must be pointing downwards during use.

4.3 Folding and Unfolding

Unfolding to Operating Position

 **WARNING!**
Risk of Injury

– Before use, make sure that the walking aid is fully unfolded and the safety latch is in locked position.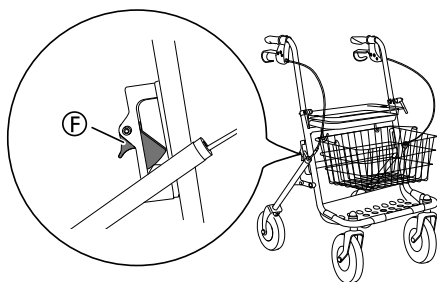


1. Press the walking aid downwards until the safety latch **F** locks in position with an audible “click”.

Folding

 **CAUTION!**
Risk of Pinching

– Be aware of the risk of pinching fingers between the parts of the frame when folding.

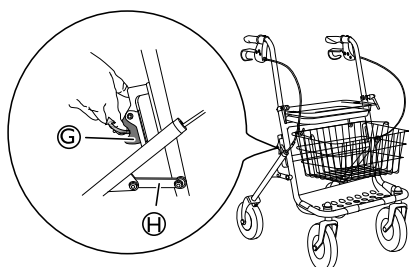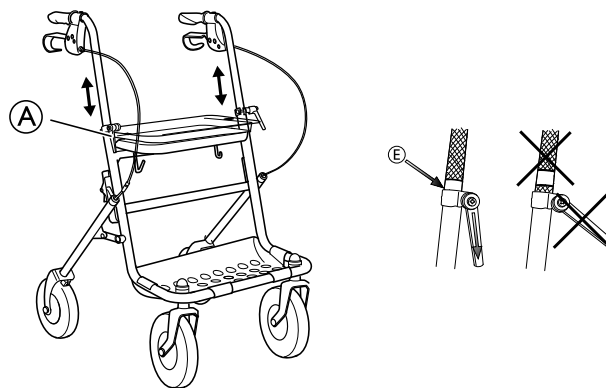
1. Release the safety latch **G** upwards - backwards.
2. Lift the cross tube **H** under the seat upwards.

4.4 Adjusting the Handle Height

 **WARNING!**
Risk of falling

- Always ensure that the levers are tightened securely to prevent the handle from unintended lowering.
- Do not set the handles above their highest allowed position, indicated by the marking on the tubes.


 **Recommended height:** The handles should be positioned at wrist height when standing straight with relaxed arms. Therapist may suggest different heights depending on specific needs of the client.

1. Loosen the lever **A**.
2. Adjust the handle to suitable height.
3. Re-tighten the lever. Press on the handle to check for secure fit.
4. Repeat the procedure on the other side.

For a more comfortable hand position, you may swivel the handles to point inwards. A locking mechanism prevents exceeding the maximum allowed angle.


 **WARNING!**
– Do not swivel the handles to point outwards.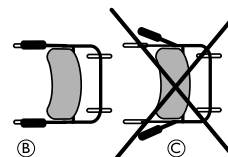

4.5 Walking


 **WARNING!**
Risk of falling

- Be careful if the wheels stop against an obstacle, e.g. a kerb.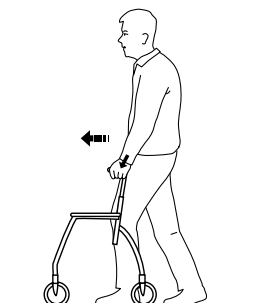

1. Place the walking aid slightly in front of you and take a step towards it while using the hand grips for support.

4.6 Sitting Down on a Chair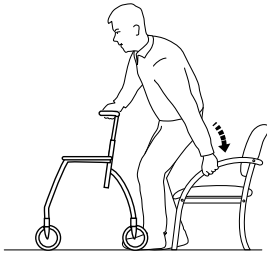

1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Lock both hand brakes of the walking aid.
3. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.


4.7 Rising to a Standing Position

WARNING!

Risk of tipping

- Do not use the walking aid to pull yourself up from a sitting position.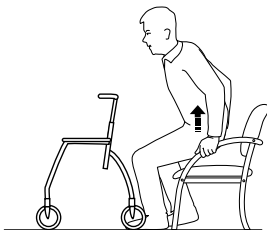

1. Place the walking aid in front of you and lock both hand brakes.
2. Use the armrests of the chair as support and rise carefully to a standing position.
3. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

4.8 Sitting on the Walking Aid


WARNING!


Risk of Falling

- Use the walking aid for stationary seating only.
- Before sitting down, lock both brakes in parking position and make sure the walking aid stands stable.
- Always sit on the walking aid from the rear side. Do not sit down from the front side.
- Do not sit on the walking aid on sloping ground.
- Do not maneuver the walking aid with your feet while sitting on it.

4.9 Using the hand brakes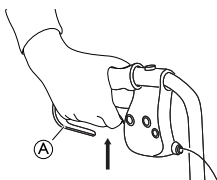

1. Apply the running brakes by pulling both brake handles (A) upwards.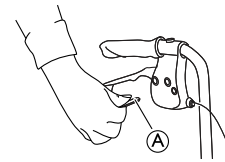

2. To apply the parking brakes, press the brake handles (A) completely downwards.
 - i An audible sound indicates that the brake is in the correct position.
 - i It is important that both brakes are put into parking position.
3. To release the parking brakes, pull the brake handles upwards.


Adjusting the brake effect

WARNING!

Risk of poor brake effect

- The adjustment of the brake must be done by qualified personnel.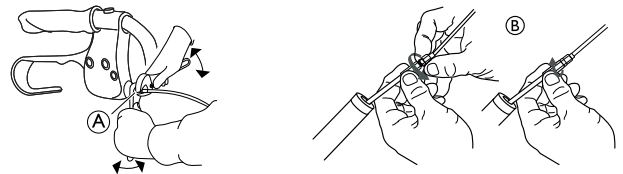

The brake can be adjusted both by the handle (A) or in the middle of the brake cable (B).


Spanners 8 mm (not included)

4.10 Installing the basket and tray

Basket


- 1.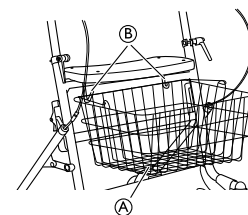

Attach the basket (A) on the attachment hooks (B).

Tray

- 1.

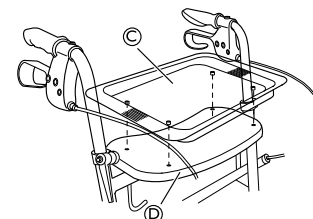Place the tray (C) on the top of the integral shelf (D) and fit the plastic plugs in the holes.

5 Maintenance

5.1 General Maintenance Information

WARNING!
Risk of Injury

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.
- Ensure that hand grips are tight.
If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation.
If brake adjustment is required, see chapter 4 *Usage*, page 3.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels' tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5.3 Cleaning and Disinfection

General Safety Information

CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning Instructions

The product can be washed in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After washing, wipe the product dry.

Disinfection

- Wipe down all generally accessible surfaces with disinfectant.

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection


For detailed information, see 5 *Maintenance*, page 6.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions and Weights

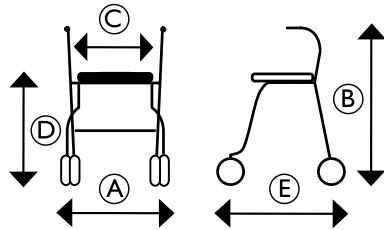	Banjo
Total width A	600 mm
Total height B	710 – 980 mm
Width between handles C	390 mm
Seat height D	610 mm
Total length E	690 mm
User height Min – Max	1600 – 1830 mm
Diameter front and rear wheels	190 mm
Total weight	9 kg
Max. user weight*	130 kg
Max. load basket	9 kg
Max. load tray	5 kg
Max. load shelf	7 kg

* This weight limitation includes the weight of the user, whether standing or sitting, and any additional load, e.g. the content of the basket.

7.2 Material

Frame	Polyester powder coated steel
Handle bars	Chrome plated steel
Handgrips	PVC
Wheels	Polypropylene with treads of TPE
Plastic parts	PET, PA, PP, PVC, PE
Screws	Steel, rust protected

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Invacare Limited
Invacare Poirier SAS
Invacare Austria GmbH
Invacare Rea AB