


# Wheels, Pneumatically


1451074.dwg

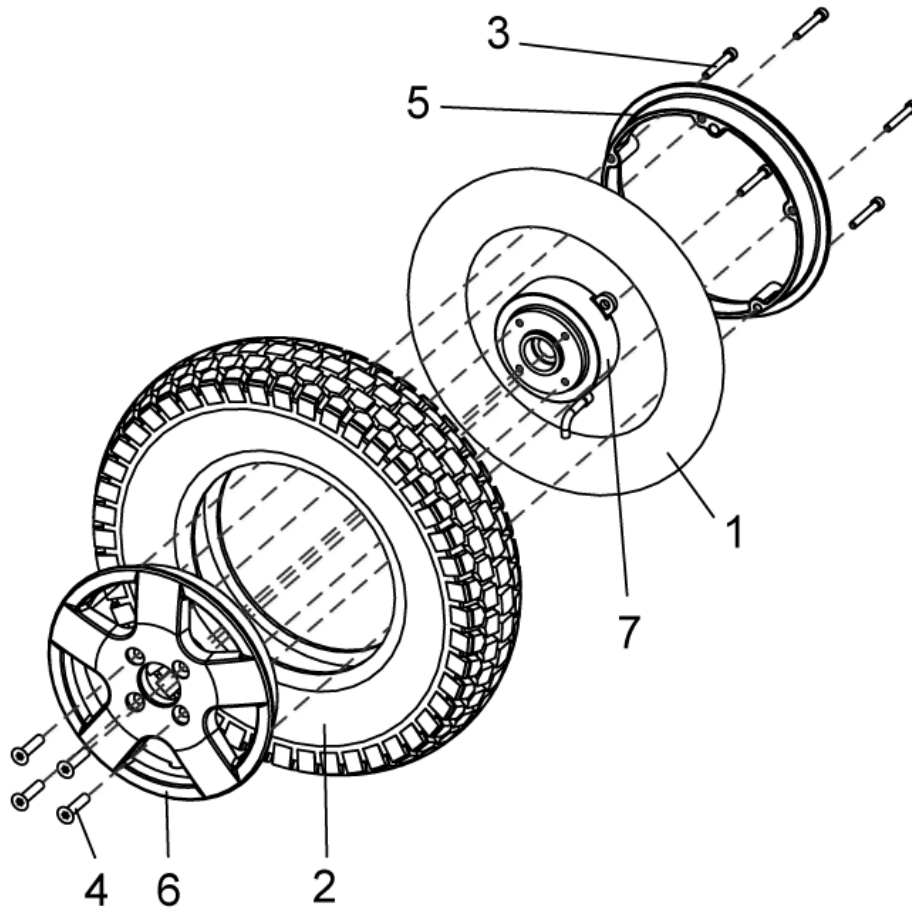

item	ordercode	description	quantity
-	SP1451074	FRONT WHEEL 4.00-8 GREY, PNEUMATICALLY, CPL.	1
-	SP1451075	FRONT WHEEL 4.00-8 BLACK, PNEUMATICALLY, CPL.	1
-	SP1451077	FRONT WHEEL 4.00-8 GREY, BREAKDOWN-protected, CPL.	1
-	SP1451078	FRONT WHEEL 4.00-8 BLACK, BREAKDOWN-protected, CPL.	1
1	1451080	HOSE 3.50-/4.00-8 JS87C G50	1
2	1450981	WHEEL 4.00-8 GREY G50	1
2	1450980	WHEEL 4.00-8 BLACK G50	1
3,4,5,6	1468293	RIM RING, CPL.	1
3	NA	SCREW M6x40	5
4	NA	SCREW M8 X 35 TORX DIN 7991	4
5	NA	RIM RING 8Z IN HI SILBGR STORM 3	1
6	NA	RIM RING 8Z AU HI SILBGR STORM 3glanz	1
7	1415171	WHEEL HUB FOR BRAKE	1


# Wheels, Breakdown-saved

1451076.dwg

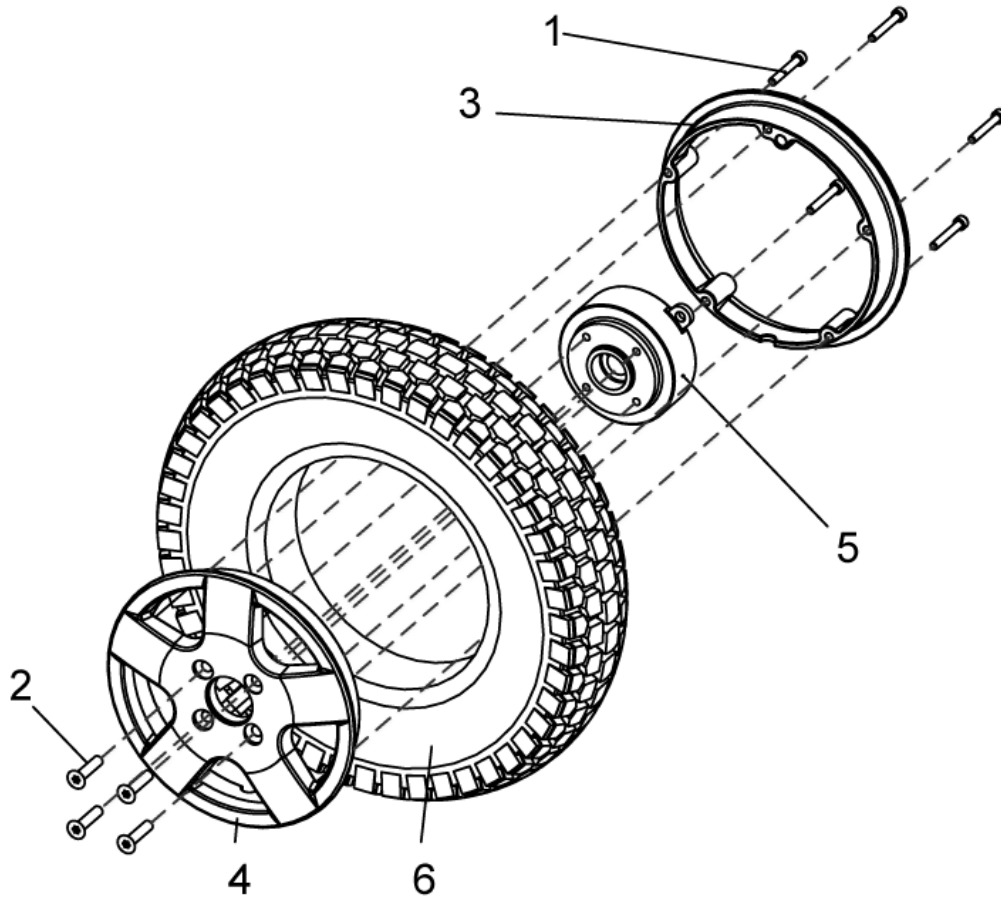

item	ordercode	description	quantity
-	SP1451076	FRONT WHEEL 4.00-8 BREAKDOWN-SAVED, GREY, CPL.	1
1,2,3,4	1468293	RIM RING, CPL.	1
1	NA	GROOVED BALL BEARING 6004-2SRSR	5
2	NA	BOLT M6X35 DIN912	4
3	NA	FRONT WHEEL RIM OUTSIDE	1
4	NA	RIM RING 6 TARIFF INSIDE	1
5	1415171	TIRE 4.10/3.50-6 PNUNCTURE-PROOF SOLID, GREY	1
6	1488416	SPACER SLEEVE	1


# Tire, pneumatically

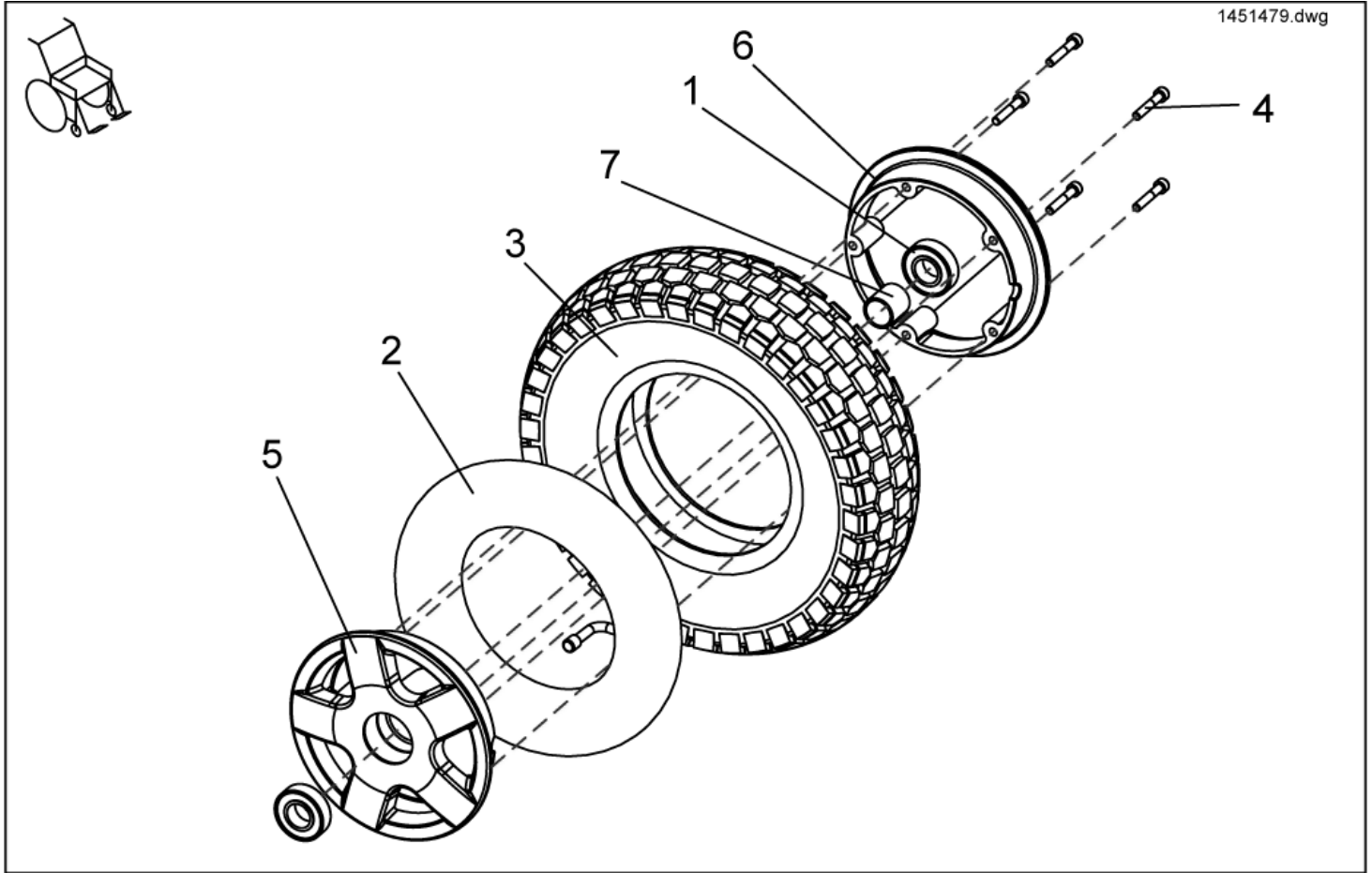

item	ordercode	description	quantity
-	SP1451479	TIRE 4.10/3.50 6, GREY	1
-	SP1451480	TIRE 4.10/3.50 6, BLACK	1
-	SP1487393	TIRE 4.10/3.50 6, GREY, BREAKDOWN-PROTECTED	1
-	SP1487394	TIRE 4.10/3.50 6, BLACK, BREAKDOWN-PROTECTED	1
1	1417943	GROOVED BALL BEARING 6004-2SRSR	2
2	1451081	HOSE 4.10/3.50 6	1
3	1450982	WHEEL 4.10/3.50-6, GREY	1
3	1450983	WHEEL 4.10/3.50-6, BLACK	1
4,5,6,7	1468294	RIM RING, CPL.	1
4	NA	BOLT M6X35, DIN 912	5
5	NA	FRONT WHEEL RIM OUTSIDE	1
6	NA	RIM RING 6 TARIFF INSIDE	1
7	NA	SPACER SLEEVE	1

**Tire, puncture-proof solid inlay**

1451481.dwg

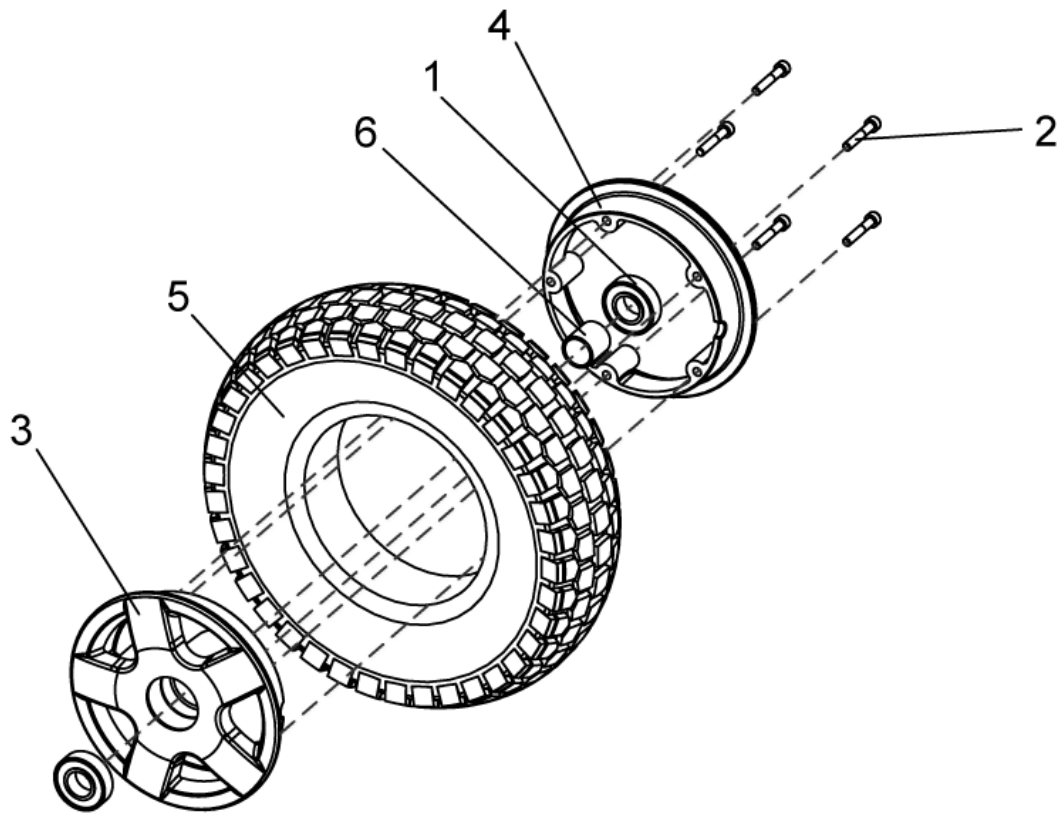

item	ordercode	description	quantity
-	SP1451481	REAR WHEEL 4.10/3.50-6PNUNCTURE-PROOF SOLID, GREY, CPL.	1
1	1417943	GROOVED BALL BEARING 6004-2SRSR	2
2,3,4,6	1468294	REAR RIM RING, CPL.	1
2	NA	BOLT M6X35 DIN912	5
3	NA	FRONT WHEEL RIM OUTSIDE	1
4	NA	RIM RING 6 TARIFF INSIDE	1
5	1485380	TIRE 4.10/3.50-6 PNUNCTURE-PROOF SOLID, GREY	1
6	NA	SPACER SLEEVE	1